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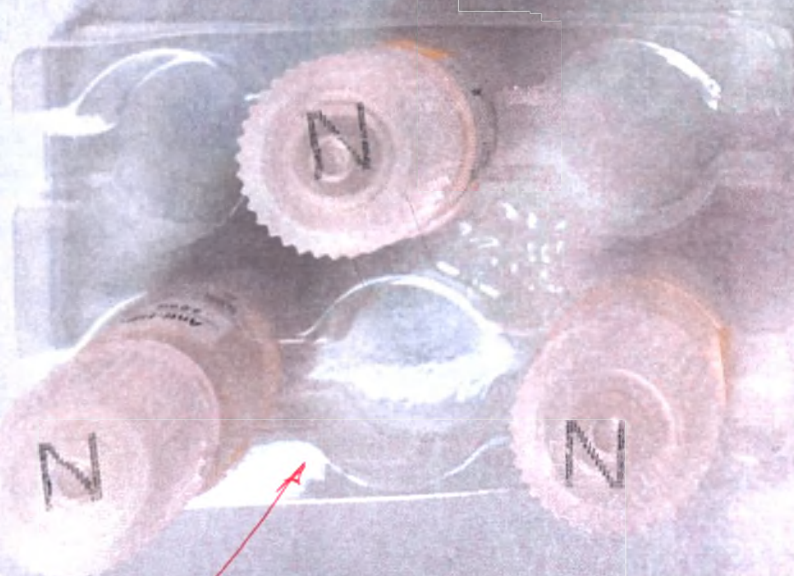
Anti-HBs Negative Control

Order Information

Catalog No.	Package Size
Anti-HBsN311	3×2.0 mL/vial
Anti-HBsN312	6×2.0 mL/vial

Intended Use

Anti-HBs Negative Control is used for monitoring the accuracy and precision in the quantitative measurement of Anti-HBs by Mindray CL-series Chemiluminescence Immunoassay Analyzer. It is also intended for clinical laboratories to assess their capability in quality control of Lab work.

Summary

Anti-HBs Negative Control has similar components as test samples. Therefore, the control can be tested by following the same procedure as test samples, measured values of the control is compared to the target values and ranges to see if the control meets its specification.

Components

Anti-HBs Negative Control is a control from

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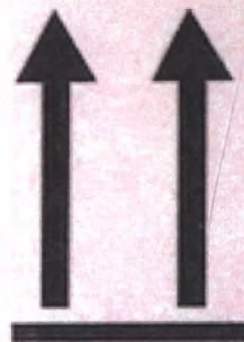
047-007382-00(2.0)

SHENZHEN MINDRAY BIO-MEDICAL ELECTRONICS CO., LTD.

Mindray Building, Keji 12th Road South, Hi-tech Industrial Park,
Nanshan, Shenzhen, 518057, P.R. China

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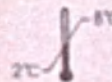
Contents: 6×2.0 mL

mindray

SHENZHEN MINDRAY BIO-MEDICAL ELECTRONICS CO., LTD.
Mindray Building, Keji 12th Road South, High-Tech Industrial Park,
Nanshan, Shenzhen, 518057, P.R. China

Anti-HBs Negative Control

Target Value Sheet

Product	Lot Number and Expiration Date	Instrument Model	Unit
Anti-HBs Negative Control	LOT 2019100100 2021-05-21	Mindray CL-series Chemiluminescence Immunoassay Analyzer	mIU

mindray

Anti-HBs Negative Control

Order Information

Catalog No.	Package Size
Anti-HBs-1	150.0 mL/VOL
Anti-HBs-2	542.0 mL/VOL

Intended Use

Anti-HBs Negative Control is used for monitoring the accuracy and precision in the laboratory measurement of Anti-HBs by Mindray CL-series Chemiluminescence Immunoassay Analyzer. It is also intended for control purposes to assess their capability in quality control of lab work.

Summary

Anti-HBs Negative Control has similar components to test samples. Therefore, the control can be tested on the following the same machine as test samples. Measured value of the control is compared to the target values and ranges to see if the control meets its specification.

Components

1 vial of Anti-HBs Negative Control is enclosed in the box.

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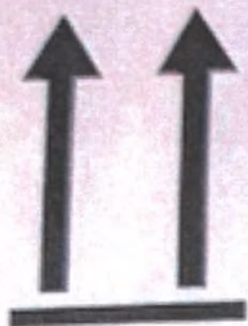
Anti-HBs Positive Control

Order Information

Catalog No.	Package Size
Anti-HBsP311	3×2.0 mL/vial
Anti-HBsP312	6×2.0 mL/vial

Intended Use

Anti-HBs Positive Control is used for monitoring the accuracy and precision in the quantitative measurement of Anti-HBs by Mindray CL-series Chemiluminescence Immunoassay Analyzer. It is also intended for clinical laboratories to assess their capability in quality control of lab work.

Summary

Anti-HBs Positive Control has similar components as the test samples. Therefore, the control can be tested by following the same procedure as the specimen; measured value of the control is compared to the target value and range to see if the control meets its specification.

Components

Anti-HBs Positive Control contains Anti-HBs.

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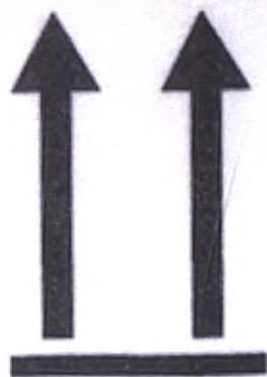
mindray
Information Card
of Anti-HBs
Positive Control

2020040100
2021-11-18

mindray

IVD

CONTROL

Anti-HBs Positive Control

Contents: 6x2.0 mL

mindray

SHENZHEN MINDRAY BIO-MEDICAL OLE
Mindray Building, Key 12th Road South, Hq
Nanshan, Shenzhen, 518057, P.R. China

mindray
Anti-HBs Positive
Control

Order Information

Catalog No.	Package Size
Anti-HBsP311	3x2.0 mL/vial
Anti-HBsP312	6x2.0 mL/vial

Intended Use
Anti-HBs Positive Control is used for monitoring the accuracy and precision in the quantitative measurement of Anti-HBs by Mindray C-Series Chemluminescence Immunoassay Analyzer. It is also intended for clinical laboratories to assess their capability in quality control of lab work.

Summary
Anti-HBs Positive Control has similar components as the test samples. Therefore, the control can be tested by following the same procedure as the specimen; measured value of the control is compared to the target value and range to see if the control meets its specification.

Components

Anti-HBs Positive Control contains Anti-HBs

Anti-HBs Positive Control
Target Value Sheet

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