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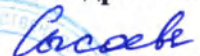
Адрес: Mindray Building, Keji 12th Road South, High-Tech Industrial Park, Nanshan, 518057 Shenzhen, People's Republic of China

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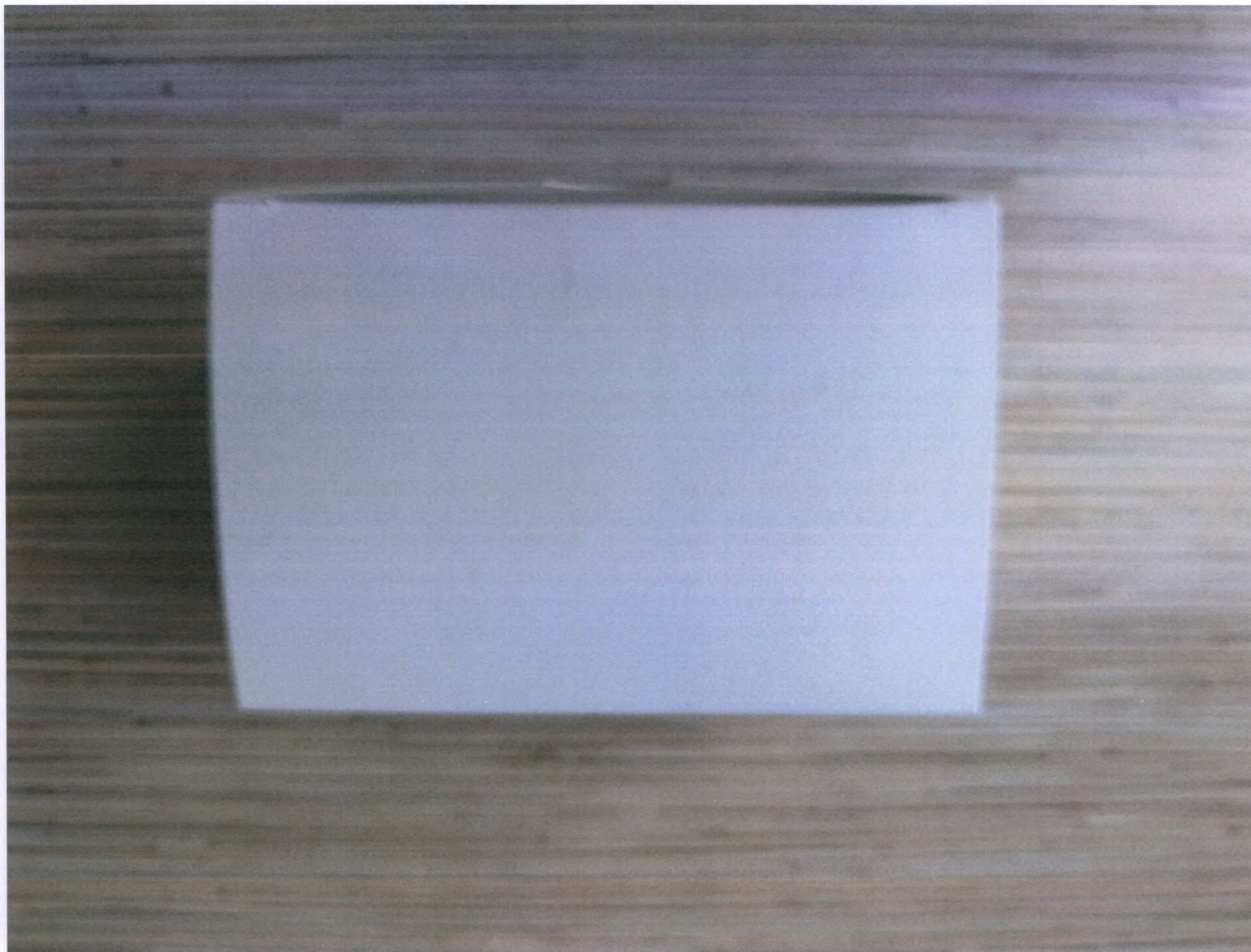
048-002152-00(2.0)

mindray
Anti-HCV

Two-dimensional Barcode of
Calibration Factor

LOT 2021050111

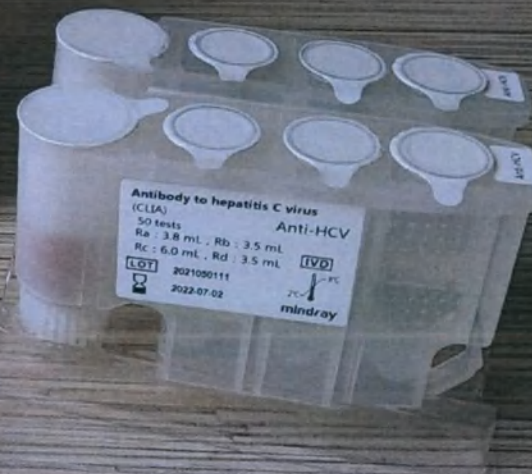
2022-07-02

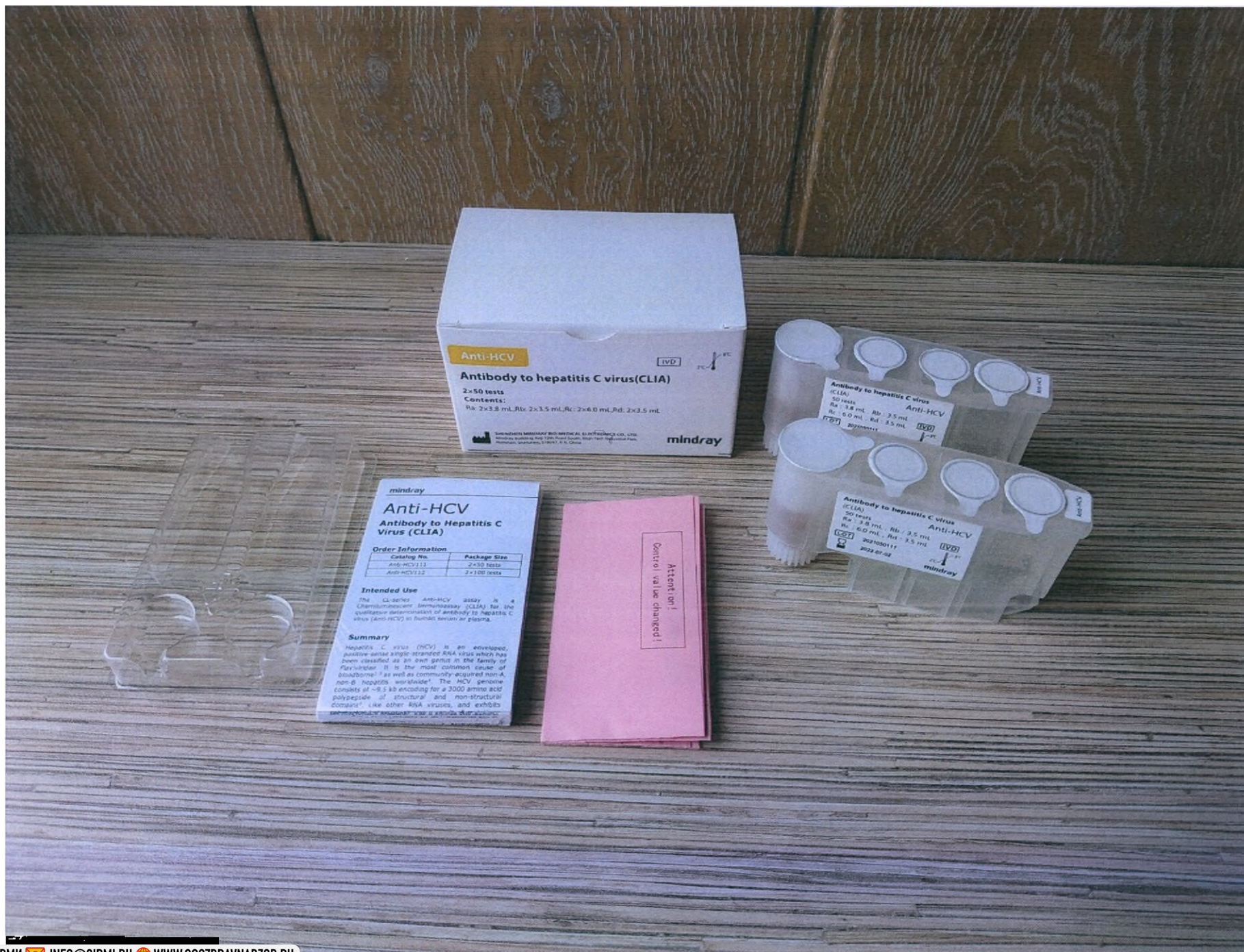
Control values changed ! See
the value sheet in this box !

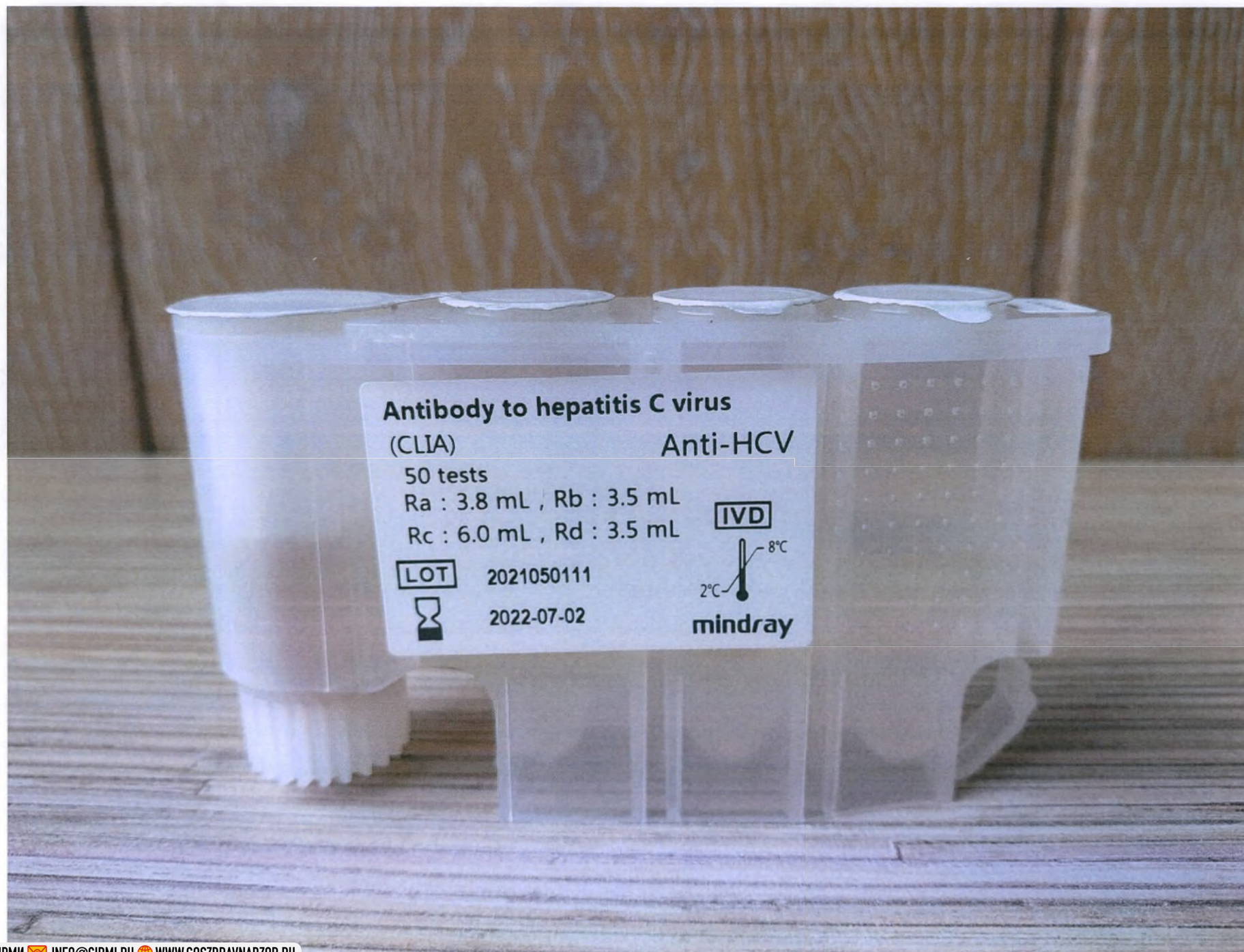
Attention!
Control value changed!

Antibody to hepatitis C virus (CIA)

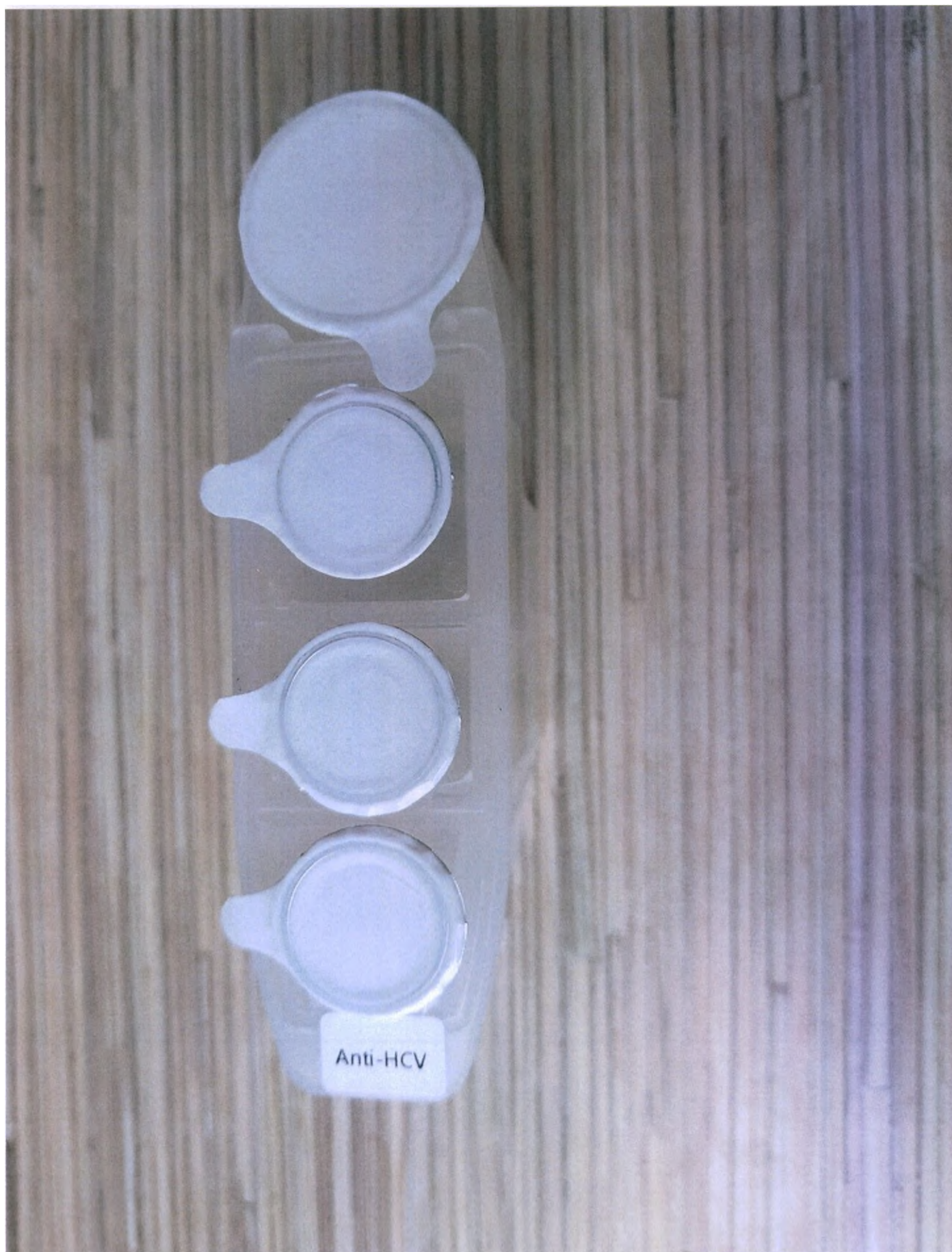
Antibody to hepatitis C virus

(CLIA)

Anti-HCV

50 tests

Ra : 3.8 mL , Rb : 3.5 mL

Rc : 6.0 mL , Rd : 3.5 mL

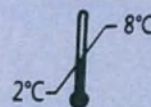
IVD

LOT

2021050111

2022-07-02

mindray

mindray

Anti-HCV

Antibody to Hepatitis C Virus (CLIA)

Order Information

Catalog No.	Package Size
Anti-HCV111	2×50 tests
Anti-HCV112	2×100 tests

Intended Use

The CL-series Anti-HCV assay is a Chemiluminescent Immunoassay (CLIA) for the qualitative determination of antibody to hepatitis C virus (Anti-HCV) in human serum or plasma.

Summary

Hepatitis C virus (HCV) is an enveloped, positive-sense single-stranded RNA virus which has been classified as an own genus in the family of Flaviviridae. It is the most common cause of bloodborne¹⁻³ as well as community-acquired non-A, non-B hepatitis worldwide⁴. The HCV genome consists of ~9.5 kb encoding for a 3000 amino acid polypeptide of structural and non-structural domains⁵. Like other RNA viruses, and exhibits

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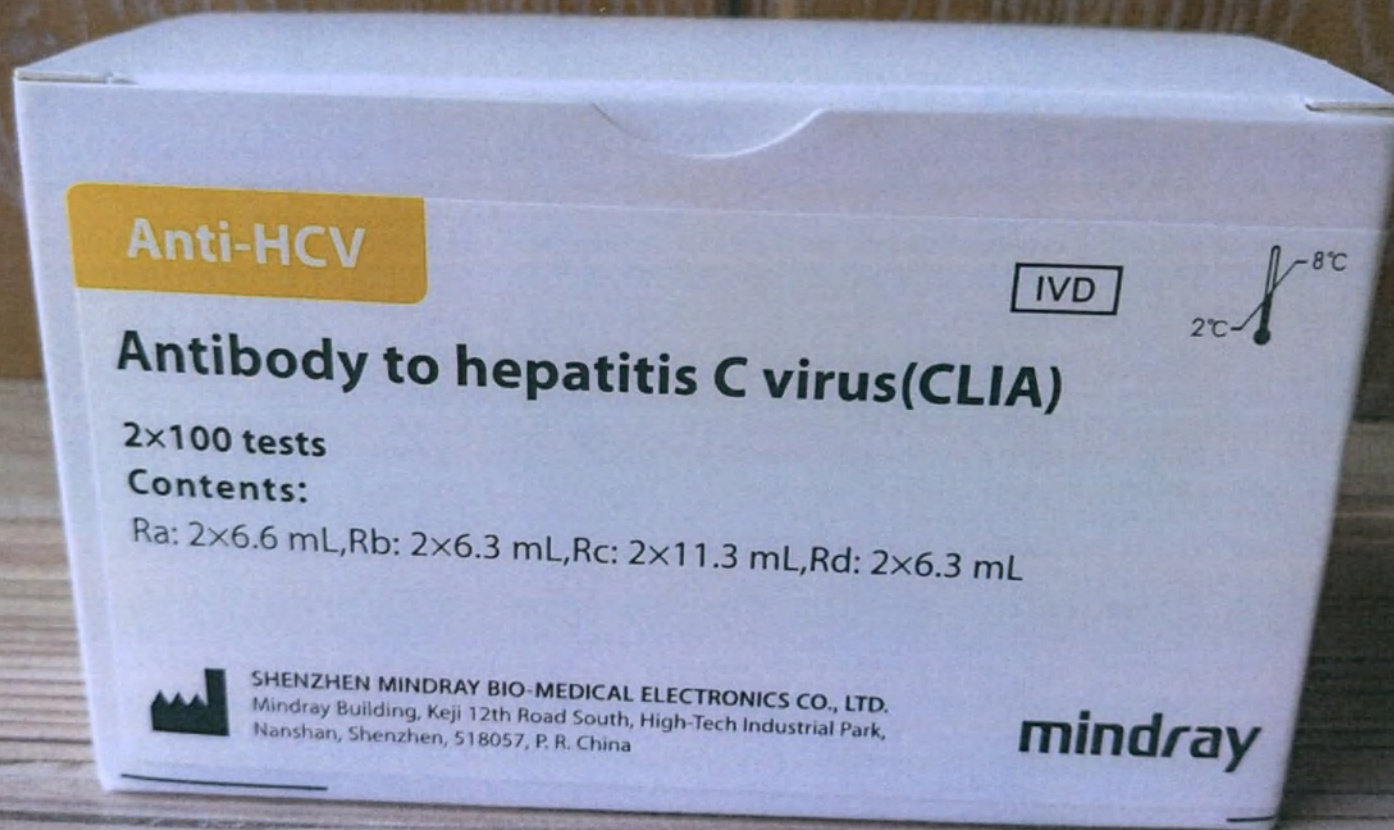
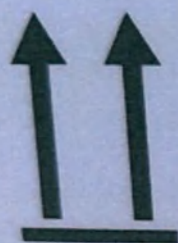
047-013081-00(2.0)

SHENZHEN MINDRAY BIO-MEDICAL ELECTRONICS CO., LTD.

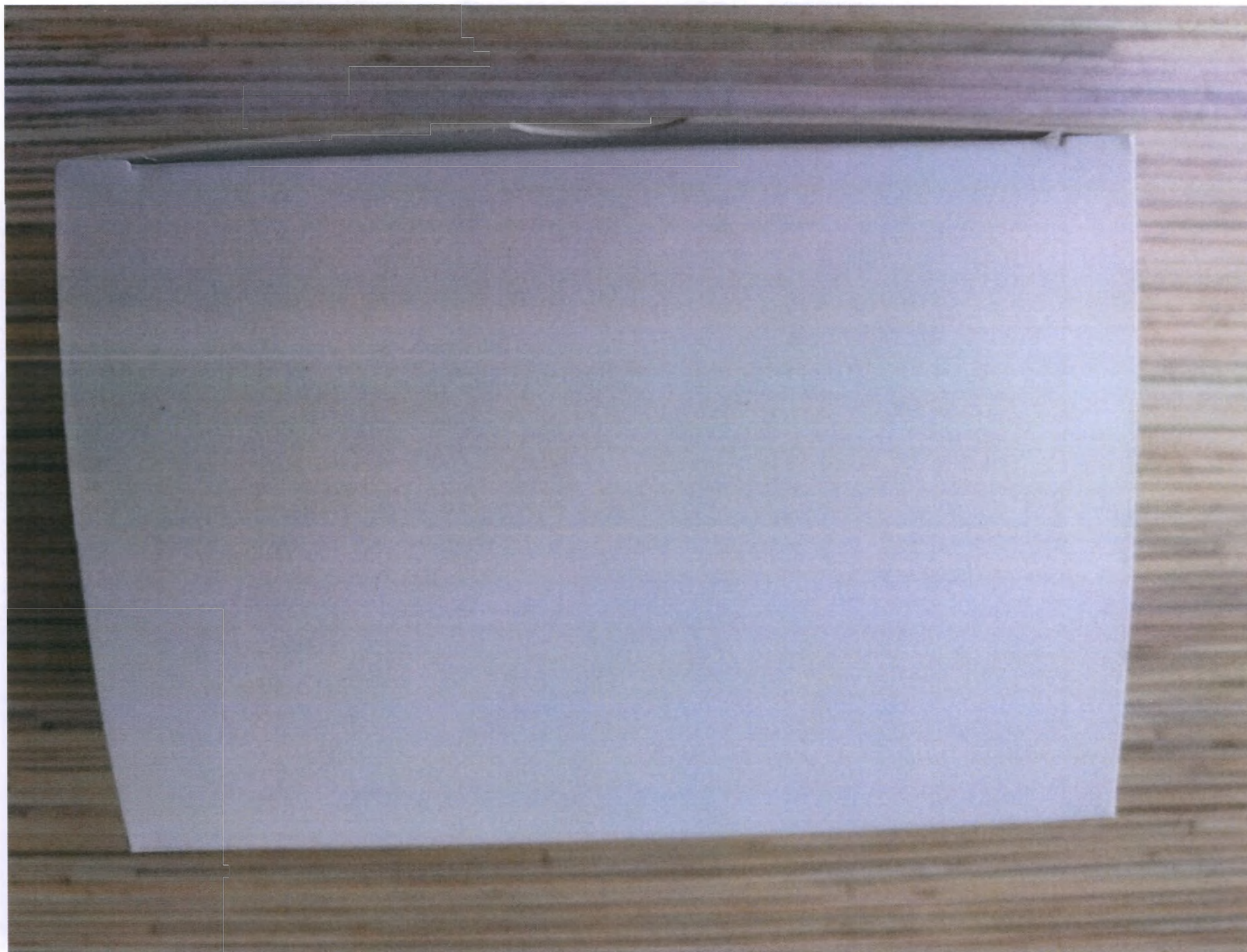
Mindray Building, Keji 12th Road South, Hi-tech Industrial Park,
Nanshan, Shenzhen, 518057, P.R. China

Tel: +86-755-26582888 Fax: +86-755-26582680

Website: <http://www.mindray.com>

048-002152-00(2.0)

mindray
Anti-HCV
Two-dimensional Barcode of
Calibration Factor

LOT 2021010111

2022-04-06

Control values changed ! See
the value sheet in this box !

Attention!
Control value changed!

mindray
Anti-HCV
Antibody to Hepatitis C Virus (CLIA)

Order Information	
Catalog No.	Anti-HCV111
Package Size	2x50 tests
	2x100 tests

Intended Use
The CLIA uses Anti-HCV monoclonal antibody for the qualitative determination of antibody to Hepatitis C virus (Anti-HCV) in human serum or plasma.

Summary
Hepatitis C virus (HCV) is an enveloped, positive-sense single-stranded RNA virus which has been classified as an own genus in the family of Flaviviridae. It is the most common cause of bloodborne virus infection, as well as community-acquired non-A, non-B hepatitis worldwide. The HCV genome consists of ~9.5 kb encoding for a 3000 amino acid polypeptide of structural and non-structural domains. Like other RNA viruses, and exhibits

Control values changed! See the value sheet in this box!

Order No.	Lot No.	Batch No.	Expiry Date
20200101	20200101	20200101	20200101

Control values changed! See the value sheet in this box!

Attention!
Control value changed!

mindray 迈瑞

Target Value and Range Sheet

Control Values that match Lot 2021010111			Anti-HCV		Reagent						
Reagent Kit:	Antibody to Hepatitis CVirus (CLIA)										
Product	Lot Number	Instrument Model	Unit	Target Values	ISD	Target Values and Ranges (Target Values±ISD)	International Unit	Target Values	ISD	Target Values and Ranges (Target Values±ISD)	
Anti-HCV Negative Control	20200401	Mindray CL-series Chemiluminescence Immunoassay Analyzer	COI	0.05	0.13	0.00 ~ 0.44	/	/	/	/ ~ /	
	/			/	/ ~ /	/		/	/ ~ /		
	/			/	/ ~ /	/		/	/ ~ /		
	/			/	/ ~ /	/		/	/ ~ /		
	/			/	/ ~ /	/		/	/ ~ /		
Anti-HCV Positive Control	20200401	Mindray CL-series Chemiluminescence Immunoassay Analyzer	COI	3.12	0.52	1.56 ~ 4.68	/	/	/	/ ~ /	
	/			/	/ ~ /	/		/	/ ~ /		
	/			/	/ ~ /	/		/	/ ~ /		
	/			/	/ ~ /	/		/	/ ~ /		
	/			/	/ ~ /	/		/	/ ~ /		

Remark: /

Antibody to hepatitis C virus

(CLIA)

Anti-HCV

100 tests

Ra : 6.6 mL , Rb : 6.3 mL

Rc : 11.3 mL , Rd : 6.3 mL

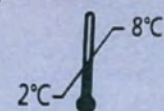
IVD

LOT

2021010111

2022-04-06


mindray

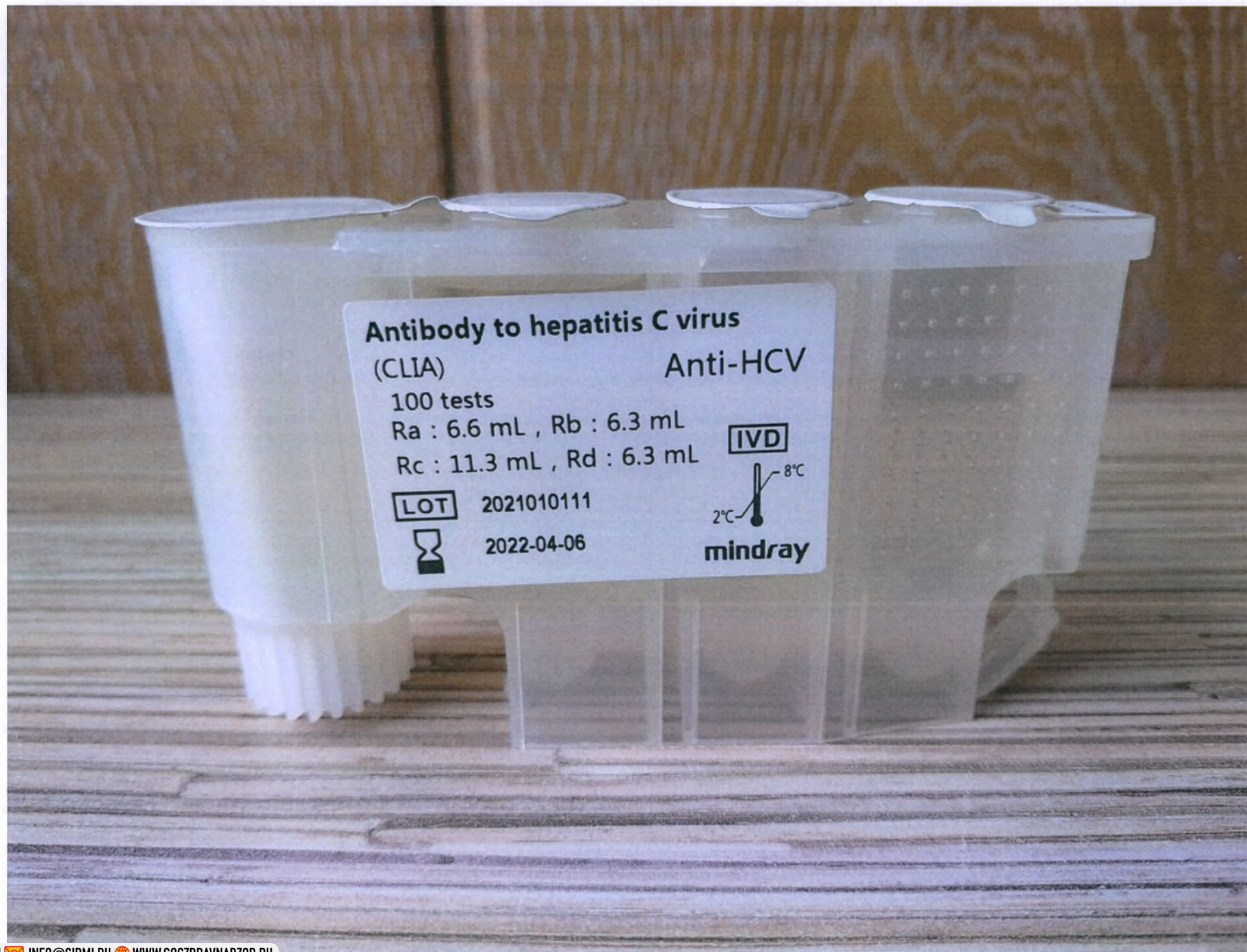mindray

Anti-HCV

Antibody to Hepatitis C Virus (CLIA)

Order Information

Catalog No.	Package Size
Anti-HCV111	2×50 tests
Anti-HCV112	2×100 tests

Intended Use

The CL-series Anti-HCV assay is a Chemiluminescent Immunoassay (CLIA) for the qualitative determination of antibody to hepatitis C virus (Anti-HCV) in human serum or plasma.

Summary

Hepatitis C virus (HCV) is an enveloped, positive-sense single-stranded RNA virus which has been classified as an own genus in the family of Flaviviridae. It is the most common cause of bloodborne¹⁻³ as well as community-acquired non-A, non-B hepatitis worldwide⁴. The HCV genome consists of ~9.5 kb encoding for a 3000 amino acid polypeptide of structural and non-structural domains⁵. Like other RNA viruses, and exhibits

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