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Mindray Building, Keji 12th Road South, High-Tech Industrial Park, Nanshan, 518057
Shenzhen, People's Republic of China

место производства: Shenzhen Mindray Bio-Medical Electronics Co., Ltd.,
1203 Nanhuan Avenue, Guangming District, 518106 Shenzhen, People's Republic of China

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CO: 1×2.0 mL, CI: 1×2.0 mL

mindray

SHENZHEN MINDRAY BIO-MEDICAL ELECTRONICS CO., LTD.
Mindray Building, Key 12th Road South, High Tech Industrial Park,
Nanshan, Shenzhen, 518057, P.R. China

IVD

2018

mindray

HIV Calibrators

Order Information

Catalog No.	Package Size
HW211	CO: 1×2.0 mL/box CI: 1×2.0 mL/box

Intended Use

The Mindray HIV Calibrators (HIV CAL) are intended to calibrate the Mindray CL-series Chemiluminescence Immunoassay Analyzer for the qualitative determination of HIV p24 antigen, and antibodies to HIV-1 and/or HIV-2 in human serum and plasma.

Summary

HIV Calibrators are prepared by adding recombinant p24 antigen, antibody into buffer solution. The Mindray CL measurement system calculates an HIV result based on the ratio of the sample RLU to the control RLU (S/CO) for each specimen and control. Control RLU = Mean RLU of CO + (Mean RLU of CI - Mean RLU of CO) × calibration factor. The calibration factor is stored in calibration card for each lot. Mindray CL measurement system is composed of Mindray CL-series Chemiluminescence Immunoassay

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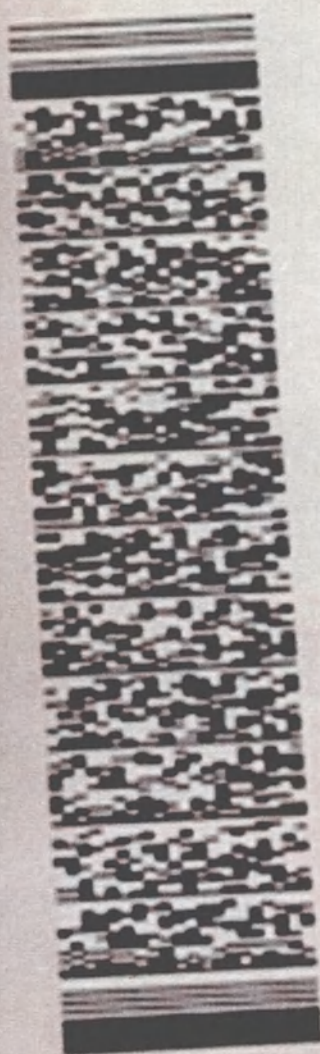
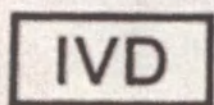
SHENZHEN MINDRAY BIO-MEDICAL ELECTRONICS CO., LTD.

Mindray Building, Keji 12th Road South, Hi-tech Industrial Park,
Nanshan, Shenzhen, 518057, P.R. China

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