Approved by Mauro Beccaro

MB

Date: 23-09-2020_____

Mauro Beccaro

(Signature)

Name of Responsible Person: Mauro Beccaro

Stamp

AL.CHI.MI.A. s.r.l.
Viale Austria, 14
35020 Ponte San Nicolò (PD)
Tel. 049.8962074 - Fax 049.8962071
Partita IVA 00063370282

Instruction for use:
Ophthalmic kits for
long-term intraocular tamponade
in vitreoretinal surgery

Manufactured by:
AL.CHI.MI.A. S.R.L., Italy

Ophthalmic kits for long-term intraocular tamponade in vitreoretinal surgery.

NAME OF THE DEVICE

ophthalmic kits for long-term intraocular tamponade in vitreoretinal surgery:

1. RSO 006-00 ophthalmic kit: RS-OIL ECS 1000 cS silicone oil, viscosity of 1000 cSt in a 10 mL syringe; 1 plunger; 1 adapter; 1 product insert, 4 labels for product traceability.
2. RSO 007-00 ophthalmic kit: RS-OIL ECS 5000 cS silicone oil, viscosity of 5000 cSt in a 10 mL syringe; 1 plunger; 1 adapter; 1 product insert, 4 labels for product traceability.
3. RSO 010-00 ophthalmic kit: RS-OIL ECS 1300 cS silicone oil, viscosity of 1300 cSt in a 10 mL syringe; 1 plunger; 1 adapter; 1 product insert, 4 labels for product traceability.
4. RSO 011-00 ophthalmic kit: RS-OIL ECS 5700 cS silicone oil, viscosity of 5700 cSt in a 10 mL syringe; 1 plunger; 1 adapter; 1 product insert, 4 labels for product traceability.
5. RSO 013-00 ophthalmic kit: RS-OIL ECS 2000 cS silicone oil, viscosity of 2000 cSt in a 10 mL syringe; 1 plunger; 1 adapter; 1 product insert, 4 labels for product traceability.

INTENDED USE OF THE MEDICAL DEVICE

Kits for long-term intraocular tamponade are used in ophthalmic surgery. Silicone oil included in the kit is used as a replacement of the vitreous body and for intraocular tamponade in cases of retinal detachment. The oil can be injected intraocularly for up to 2 months.

According to international literature, practicing eye surgeons disagree on the maximum period of eye exposure to the oil. The eye exposure period is determined by the surgeon subject to assessment of the anatomical effect achieved. According to research studies (articles), eye exposure to the silicone oil for more than 2 months is not recommendable; however, based on patient's interests, the physician may increase the tamponade period. Emulsification is an absolute indication for removal of the silicone oil.

INFORMATION ABOUT INTENDED USERS OF THE MEDICAL DEVICE

Ophthalmic surgeons performing ophthalmic surgeries.

CONDITIONS OF USE

Solutions are used in the settings of health care institutions only.

Operating conditions: temperature range from +15°C to +32°C; 20 – 65% humidity.

INDICATIONS FOR USE

The kits are used in ophthalmic surgeries for:

- intraocular hemorrhage when rebleeding can be expected.
- long-term tamponade in vitreoretinal surgeries for patients with primary or recurrent retinal detachment complicated by retinal perforation, large retinal tears or proliferative vitreoretinopathy, cytomegalovirus retinitis;
- prevention of the need to maintain face down posturing, especially in pediatric patients, patients with mental and physical disabilities;
- early rehabilitation of vision in comparison with other methods.
- enabling airborne transport or climbing after surgery for retinal detachment, since atmospheric pressure change may cause recurrent detachment, the risk of which is minimized when using the silicone oil.

OPERATING INSTRUCTIONS

Make sure the packaging is intact.

Take the syringe out of the package.

For manual injection: Remove the adapter (1) rotating it counter-clockwise; screw the plunger (2) into the rubber stopper of the syringe (3).

For injection via mechanical/automated systems: remove the plunger (2) and rotate the adapter (1) clockwise to firmly secure it onto the syringe; connect the syringe to the mechanical/automatic system.

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In relation to the instance on record made by the proprietor/legal representative/attorney of the enterprise, we certify that the said applicant is a duly authorized signatory for any documents and acts to be used in trade with foreign countries.

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PADOVA, 28/09/2020

WHEN USING PERFLUORO-N-OCTANE OR PERFLUORODECALIN DURING A SINGLE SURGICAL PROCEDURE, FLUID/AIR EXCHANGE SHOULD BE PERFORMED PRIOR TO INJECTION OF THE SILICONE OIL TO ENSURE THE COMPLETE REMOVAL OF PERFLUOROCARBONS.

Inject the silicone oil in accordance with the common medical practice. Remove any residues of the silicone oil. Remove the silicone oil from the patient's eye upon reaching the satisfactory results and dispose as the general sanitary waste.

TECHNICAL SPECIFICATIONS

Specifications of RS-OIL ECS 1000 cS silicone oil, viscosity of 1000 cSt, included into RSO 006-00 ophthalmic kit

Specifications of RS-OIL ECS 1000 cS silicone oil, viscosity of 1000 cSt, included into RSO 006-00 ophthalmic kit

Item	Standards
Composition	Silicone oil (polydimethylsiloxane)
Appearance	Clear, free of foreign matter
APHA color	< 15
Refractive index at 25°C	1,4013 – 1,4053
Viscosity mm ² /s (cSt) at 25°C	950 – 1050
Specific gravity (g/cm ³) at 25°C	0,967 – 0,975
Phenolic compounds (absorption in the UV spectrum in the range of 250 – 270 nm)	0,0 – 0,2
Volatile content %	0,10 – 0,30
Volatility, %	max. 0,1
Identification with infrared spectrophotometry	Passed
Purity, % (active ingredient content)	100%
Heavy metals ppm	< 5
Surface tension, mN/m	21,2 – 21,5 against air
Interfacial tension, mN/m	40 – 44 against water
Acidity, mL NaOH 0.01 N	0,10 – 0,15
Acid value (with 0.01 n KOH) mg KOH/g	< 0,019
Spectral transmission range, nm	300-1100

Specifications of RS-OIL ECS 5000 cS silicone oil, viscosity of 5000 cSt, included into RSO 007-00 ophthalmic kit

Item	Standards
Composition	Silicone oil (polydimethylsiloxane)
Appearance	Clear, free of foreign matter
APHA color	< 15
Refractive index at 25°C	1,4013 – 1,4053
Viscosity mm ² /s (cSt) at 25°C	4750 – 5250
Specific gravity (g/cm ³) at 25°C	0,967 – 0,975
Phenolic compounds (absorption in the UV spectrum in the range of 250 – 270 nm)	< 0,20
Volatile content %	0,10 – 0,30
Volatility, %	max. 0,1
Identification with infrared spectrophotometry	Passed
Purity, % (active ingredient content)	100%
Heavy metals ppm	< 5
Acidity, mL NaOH 0.01 N	0,12 – 0,15
Acid value (with 0.01 n KOH) mg KOH/g	< 0,019
Surface tension, mN/m	21,2 – 21,5 against air
Interfacial tension, mN/m	40 – 44 against water
Spectral transmission range, nm	300-1100

Specifications of RS-OIL ECS 1300 cS silicone oil, viscosity of 1300 cSt, included into RSO 010-00 ophthalmic kit

Item	Standards
Composition	Silicone oil (polydimethylsiloxane)
Appearance	Clear, free of foreign matter
APHA color	< 15
Refractive index at 25°C	1,4013 – 1,4053
Viscosity mm ² /s (cSt) at 25°C	1235 – 1365
Specific gravity (g/cm ³) at 25°C	0,967 – 0,975
Phenolic compounds (absorption in the UV spectrum in the range of 250 – 270 nm)	< 0,20
Volatile content %	0,10 – 0,30
Volatility, %	max. 0,1
Identification with infrared spectrophotometry	Passed
Purity, % (active ingredient content)	100%
Heavy metals ppm	< 5
Surface tension, mN/m	21,2 – 21,5 against air
Interfacial tension, mN/m	40 – 44 against water
Acidity, mL NaOH 0.01 N	0,12 – 0,15
Acid value (with 0.01 n KOH) mg KOH/g	< 0,019
Spectral transmission range, nm	300-1100

Specifications of RS-OIL ECS 5700 cS silicone oil, viscosity of 5700 included into RSO 011-00 ophthalmic kit

Item	Standards
Composition	Silicone oil (polydimethylsiloxane)
Appearance	Clear, free of foreign matter
APHA color	< 15
Refractive index at 25°C	1,4013 – 1,4053
Viscosity mm ² /s (cSt) at 25°C	5415 – 5985
Specific gravity (g/cm ³) at 25°C	0,967 – 0,975
Phenolic compounds (absorption in the UV spectrum in the range of 250 – 270 nm)	< 0,20
Volatile content %	0,10 – 0,30
Volatility %	max. 0,1
Identification with infrared spectrophotometry	Passed
Purity, % (active ingredient content)	100%
Heavy metals ppm	< 5
Surface tension, mN/m	21,2 – 21,5 against air
Interfacial tension, mN/m	40 – 44 against water
Acidity, mL NaOH 0.01 N	0,10 – 0,15
Acid value (with 0.01 n KOH) mg KOH/g	< 0,019
Spectral transmission range, nm	300-1100

Specifications of RS-OIL ECS 2000 cS silicone oil, viscosity of 2000 cSt, included into RSO 013-00 ophthalmic kit

Item	Standards
Composition	Silicone oil (polydimethylsiloxane)
Appearance	Clear, free of foreign matter
APHA color	< 15
Refractive index at 25°C	1,4013 – 1,4053
Viscosity mm ² /s (cSt) at 25°C	1900 – 2100
Specific gravity (g/cm ³) at 25°C	0,967 – 0,975
Phenolic compounds (absorption in the UV spectrum in the range of 250 – 270 nm)	< 0,2
Volatility %	max. 0,1
Identification with infrared spectrophotometry	Passed
Purity, % (active ingredient content)	100%
Heavy metals ppm	< 5
Volatile content %	0,10 – 0,30
Surface tension, mN/m	21,2 – 21,5 against air
Interfacial tension, mN/m	40 – 44 against water
Acidity, mL NaOH 0.01 N	0,12 – 0,15
Acid value (with 0.01 n KOH) mg KOH/g	< 0,019
Spectral transmission range, nm	300-1100

Sterilization

RS-OIL ECS medical devices are sterilized according to ISO 17665-1 Sterilization of health care products by moist heat -- Part 1: Requirements for the development, validation and routine control of a sterilization process for medical devices. Sterilization of the medical devices is conducted by means of steam sterilization in an autoclave. The applied method is an overkill approach which ensures that sufficient lethality is achieved during the cycle. Confirmation of the lethality of the process is ensured by thermometric measurements and inactivation of Biological Indicators located inside the sterilizer load. The sterility assurance level (SAL) is 10^{-6} or higher.

Re-sterilization of the kits is not applicable.

PACKAGING

The primary packaging for RS-OIL ECS 1000 cS, RS-OIL ECS 5000 cS, RS-OIL ECS 1300 cS, RS-OIL ECS 5700 cS and RS-OIL ECS 2000 cS silicone oil is syringes.

Syringe filling volume: $10.0 \text{ mL} \pm 0.1 \text{ mL}$

The weight of a syringe with the silicone oil RS-OIL ECS 1000 cS, RS-OIL ECS 5000 cS, RS-OIL ECS 1300 cS; RS-OIL ECS 5700 cS; RS-OIL ECS 2000 cS is $26.73 \text{ g} \pm 0.2 \text{ g}$.

Syringes are supplied clean, sterile, verified to be endotoxins-free and ready-to-use.

Syringes are supplied with certificates of conformity and are subject to incoming quality control conducted by Product Safety Specialist of AL.CHI.MI.A. S.R.L.

Syringes are sterilized by the X-ray method and the sterility period is 24 months; during this period the syringes must be filled with the silicone oil. After filling, the final product is re-sterilized by autoclaving.

Materials for syringes manufacturing

Description	Material
Luer-lock syringe with:	COC (cyclic olefin copolymer), is compliant with Ph. Eur. TOPAS 6013, manufactured by Topas Advanced Polymers GmbH, Germany
Plunger	Polypropylene Bormed HD 810 MO, manufactured by Borealis AG, Austria
Plunger stopper	Brombutyl Datwyler FM 257/2, manufactured by Datwyler Pharma Packaging Deutschland GmbH, Germany
Tip cap	Brombutyl Datwyler FM 257, manufactured by Datwyler Pharma Packaging Deutschland GmbH, Germany
Adapter with:	Polycarbonate Lexan HP 4REU, manufactured by Sabic Innovative Plastic, Saudi Arabia
Sealing ring	Viton, manufactured by DuPont, USA.
Lubricant	Silicone MED 4159, manufactured by NuSil Technology LLC, USA

The secondary packaging for silicone oil syringes is a sterilizable peel-off pouch made of **PET12/PP40 and medical grade paper Eline, manufactured by E-Line Srl, Italy**. Pouches do not have an adhesive layer as they are subject to thermal sealing. No adhesive materials are used. No adhesive defects. Width seal is $13 \pm 3 \text{ mm}$. Opening force, maximum 30 N.

Eline medical-grade paper specification

Density: $60 \pm 1.8 \text{ g/m}^2$;

Longitudal tension strength: $6.4 \pm 0.2 \text{ kN/m}$;

Transverse tension strength: $3.4 \pm 0.1 \text{ kN/m}$;

Thickness: $83.0 \pm 2.5 \text{ um}$;

Air permeability: $11.4 \pm 0.3 \text{ um}$;

Longitudal rupture strength: $600 \pm 18 \text{ mN}$

Transverse rupture strength: $650 \pm 19 \text{ mN}$

PET12/PP40 peel-off pouch specification

Thickness: $52 \pm 1.5 \text{ um}$;

Oxygen permeability: $\text{O}_2 - 125 \pm 3.7 \text{ cm}^3/\text{m}^2/24\text{h}$

Name of the kit	Dimensions of the peel-off pouch, length x width $\pm 5 \text{ mm}$
RSO 006-00 ophthalmic kit: RS-OIL ECS 1000 cS silicone oil, viscosity of 1000 cSt in a 10 mL syringe; 1 plunger; 1 adapter; 1 instruction for use, 4 labels for product traceability.	250 mm x 100 mm
RSO 007-00 ophthalmic kit: RS-OIL ECS 5000 cS silicone oil, viscosity of 5000 cSt in a 10 mL syringe; 1 plunger; 1 adapter; 1 instruction for use, 4 labels for product traceability.	250 mm x 100 mm
RSO 010-00 ophthalmic kit: RS-OIL ECS 1300 cS silicone oil, viscosity of 1300 cSt in a 10 mL syringe; 1 plunger; 1 adapter; 1 instruction for use, 4 labels for product traceability.	250 mm x 100 mm
RSO 011-00 ophthalmic kit: RS-OIL ECS 5700 cS silicone oil, viscosity of 5700 cSt in a 10 mL syringe; 1 plunger; 1 adapter; 1 instruction for use, 4 labels for product traceability.	250 mm x 100 mm
RSO 013-00 ophthalmic kit: RS-OIL ECS 2000 cS silicone oil, viscosity of 2000 cSt in a 10 mL syringe; 1 plunger; 1 adapter; 1 instruction for use, 4 labels for product traceability.	250 mm x 100 mm

The outer packaging of the kits is a cardboard box. Dimensions of the cardboard boxes are given in the table.

Name of the kit	Dimensions of the outer packaging (cardboard box), length x width x height $\pm 5 \text{ mm}$
RSO 006-00 ophthalmic kit: RS-OIL ECS 1000 cS silicone oil, viscosity of 1000 cSt in a 10 mL syringe; 1 plunger; 1 adapter; 1 instruction for use, 4 labels for product traceability.	150 mm x 100 mm x 35 mm
RSO 007-00 ophthalmic kit: RS-OIL ECS 5000 cS	

silicone oil, viscosity of 5000 cSt in a 10 mL syringe; 1 plunger; 1 adapter; 1 instruction for use, 4 labels for product traceability.	150 mm x 100 mm x 35 mm
RSO 010-00 ophthalmic kit: RS-OIL ECS 1300 cS silicone oil, viscosity of 1300 cSt in a 10 mL syringe; 1 plunger; 1 adapter; 1 instruction for use, 4 labels for product traceability.	150 mm x 100 mm x 35 mm
RSO 011-00 ophthalmic kit: RS-OIL ECS 5700 cS silicone oil, viscosity of 5700 cSt in a 10 mL syringe; 1 plunger; 1 adapter; 1 instruction for use, 4 labels for product traceability.	150 mm x 100 mm x 35 mm
RSO 013-00 ophthalmic kit: RS-OIL ECS 2000 cS silicone oil, viscosity of 2000 cSt in a 10 mL syringe; 1 plunger; 1 adapter; 1 instruction for use, 4 labels for product traceability.	150 mm x 100 mm x 35 mm

HANDLING PRECAUTIONS

- Do not use if packaging is opened or broken.
- Expiration date is indicated on the device label. Do not use after expiration date.
- Do not shake or mix with other substances.
- For single-use only. Do not re-sterilize. The use or sterilization of disposable devices may result in cross-contamination.
- Prior to injection, release approximately 0.5 ml of silicone oil and make sure that the rubber stopper slides well into the syringe.
- Before using, perform fluid/air exchange to ensure the complete removal of perfluorocarbons.
- Prior to injection of the silicone oil, make sure there are no retinal holes or tears that can absorb the silicone oil. Avoid contact between the silicone oil and the corneal endothelium.
- During the injection, ensure that the substance is not absorbed in the areas of retinal tears.
- The device should be used only by qualified personnel in the settings of operating room of the health care institution. Any residues should be disposed as general sanitary waste. In case of unintentional damage or spillage prior to or during the surgical procedure, the spilled liquid should be cleaned up with the inert material, placed in a plastic container and disposed as prescribed by the applicable procedures.
- Empty packaging should be disposed as prescribed by the applicable regulations.

CONTRAINDICATIONS

- Silicone oil is a relatively inert substance, however it is not recommended for patients with hypersensitivity to silicone oil.
- Silicone oil is contraindicated for patient with pseudophakic eyes having silicone intraocular lenses or for a permanent replacement of the vitreous body.

SIDE EFFECTS

- Temporary elevated intraocular pressure; hypotension; glaucoma; cataract may occur after surgery.
- Penetration of silicone oil into the anterior chamber of the eye may lead to corneal swelling or opacity; emergence of Descemet's membrane folds; corneal decompensation.
Keratopathy.

LABELLING

The following symbols are used on the label:

Symbol	Definition
--------	------------

	Manufacturer
	Expiration date
	Indicates that the device complies with the requirements of the European Medical Devices Directive
	Read the instructions for use before any operation
	Indicates sterilization of the medical device by steam or dry heating
	Do not use if package is damaged
	Read the instructions for use before any operation
	This side up
	Do not re-sterilize
	Latex free
	Catalogue number
	Batch number
	For single-use only
	Environmentally friendly disposal

	Apyrogenic
--	-------------------

SUPPLY PACKAGE for RSO 006-00; RSO 007-00; RSO 010-00; RSO 011-00; RSO 013-00 ophthalmic kits:

- Primary packaging: 10.0 mL syringe containing silicone oil: RS-OIL ECS 1300 cSt, viscosity of 1300 cSt; RS-OIL ECS 1000 cSt, viscosity of 1000 cSt; RS-OIL ECS 1300 cSt, viscosity of 1300 cSt; RS-OIL ECS 5700 cSt, viscosity of 5700 cSt or RS-OIL ECS 2000 cSt, viscosity of 2000 cSt.
- Secondary packaging: a syringe with a plunger and an adapter is packed into a pouch made of non-woven fabrics.

Multiple packaging: a cardboard box containing a syringe with a plunger and an adapter in a secondary packaging. The supply package includes 4 labels for product traceability; operational documentation (instructions for use).

TRANSPORTATION

The kits should be transported at the temperature ranges from -40°C to +40°C and humidity 0-65%. Short-term deviations from the specified temperature range do not have a significant effect on the stability of the product.

The packaging integrity is maintained during normal loading and transportation procedures, preserving the sterility of the device.

STORAGE AND EXPIRATION DATE

Ophthalmic kits for preservation of the anterior chamber volume during ophthalmic surgeries are recommended to be stored at room temperature +18°C to +27°C and 20-65% humidity and kept away from the direct sunlight. Do not freeze. Fragile.

Shelf life:

RSO 006-00 ophthalmic kit: RS-OIL ECS 1000 cSt silicone oil, viscosity of 1000 cSt in a 10 mL syringe; 1 plunger; 1 adapter; 1 instruction for use, 4 labels for product traceability.	3 years
RSO 007-00 ophthalmic kit: RS-OIL ECS 5000 cSt silicone oil, viscosity of 5000 cSt in a 10 mL syringe; 1 plunger; 1 adapter; 1 instruction for use, 4 labels for product traceability.	3 years
RSO 010-00 ophthalmic kit: RS-OIL ECS 1300 cSt silicone oil, viscosity of 1300 cSt in a 10 mL syringe; 1 plunger; 1 adapter; 1 instruction for use, 4 labels for product traceability.	3 years
RSO 011-00 ophthalmic kit: RS-OIL ECS 5700 cSt silicone oil, viscosity of 5700 cSt in a 10 mL syringe; 1 plunger; 1 adapter; 1 instruction for use, 4 labels for product traceability.	3 years
RSO 013-00 ophthalmic kit: RS-OIL ECS 2000 cSt silicone oil, viscosity of 2000 cSt in a 10 mL syringe; 1 plunger; 1 adapter; 1 instruction for use, 4 labels for product traceability.	3 years

WASTE DISPOSAL

Liquid extracts discharged from patients' eyes and unused components of the medical device may be potentially biohazardous. This waste of the medical device is classified as class B waste (epidemiologically hazardous waste) - Infected and potentially infected waste, in accordance with SanPiN 2.1.7.2790-10. Materials and tools, objects contaminated with blood and/or other biological waste.

MANUFACTURER'S WARRANTY

The manufacturer warrants the quality of devices for the entire shelf life, given the consumer follows the prescribed operational, transportation and storage conditions.

COMPLAINTS

Any complaints for the quality of medical devices should be referred to: LLC "IRIS-M", 13, Marshala Vasilevskogo str., building 3, office 30, 21, 123183 Moscow.

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