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Carl-Schurz-Straße 1
41453 Neuss
Germany

Postal Address:
ESPE Platz
82229 Seefeld
Germany

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3M ESPE

☎ +49 (0)8152 700-0
☎ +49 (0)8152 700-1366
Internet: www.3MESPE.com
E-Mail: info3MESPE@3M.com
WEEE-Reg.-Nr. DE 36963167
VAT-ID: DE 120679179

To whom it may concern

February 06, 2014
Contact:
Ruediger Franke
e-mail:
ruediger.franke@mmm.com

Registration documents for Astringent Retraction Paste - IFU

We 3M Deutschland GmbH, Carl-Schurz-Str. 1, 41453 Neuss, herewith confirm that the following pages are an excerpt of the Informations for use with an additional comment about the representative of the manufacturer in Russia.

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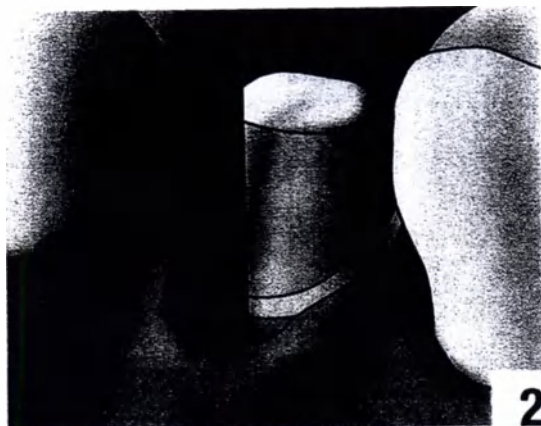
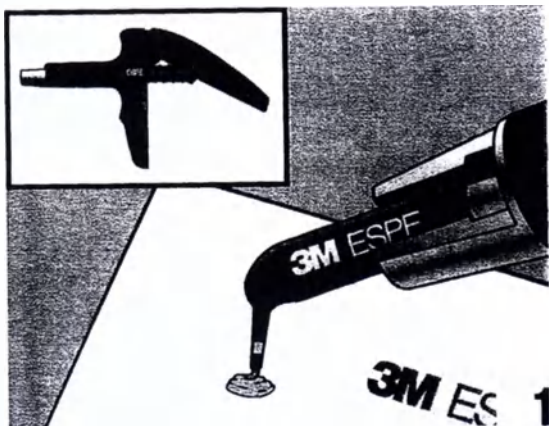
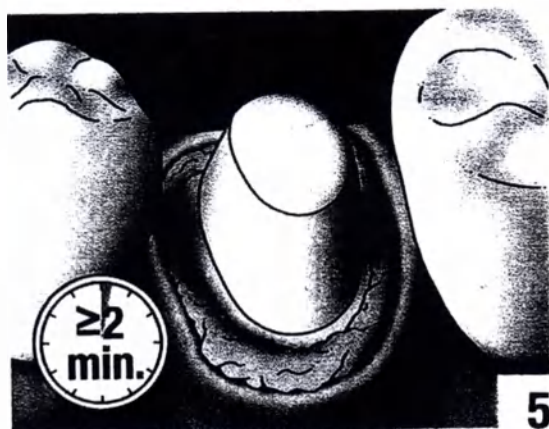
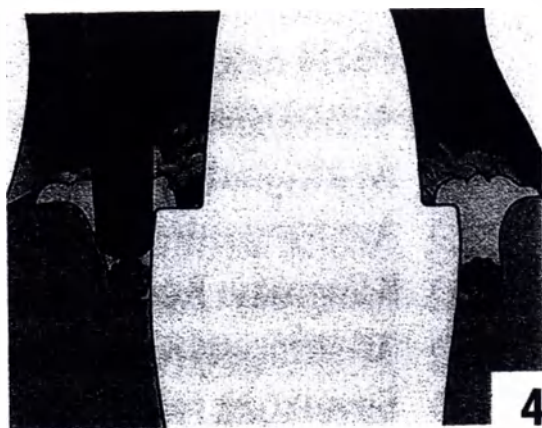
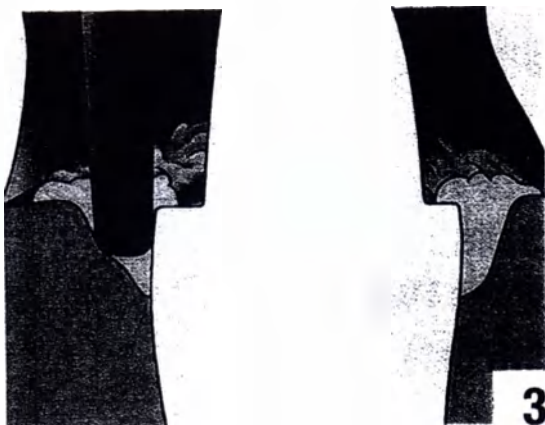
ENGLISH

Product Description

Astringent Retraction Paste is an astringent retraction paste for any situation in which a **temporary** displacement of the marginal gingiva and/or the assurance of a dry and clean **sulcus is required**. The acidic paste containing aluminum chloride is applied directly from the **retraction capsule** into the sulcus. A hemostetic effect results from the compression on the **gums by the paste** containing aluminum chloride as an astringent. This hemostatic effect **quickly closes** any small blood vessels which might be unintentionally opened. Depending on the clinical situation and working technique of the person performing the treatment, the paste can be used as an alternative to or in combination with retraction cords or other retraction methods. Any suitable, commonly available dispensers, e.g., the Capsule Applicator can be used for application of the retraction paste. A retraction capsule contains sufficient material for up to 3 teeth, but may not be used on more than one patient.

 These Instructions for Use should not be discarded for the duration of product use.

Use

- All indications for temporary displacement of the marginal gingiva and to provide for a dry sulcus when the periodontium is healthy, such as:
 - Taking impressions, either with impression material or as digital impression
 - Cementation of temporary and permanent restorations
 - Preparation of Class II and V fillings

Contraindications: the astringent retraction paste must not be used in patients suffering from a diseased periodontium, open furcations, or exposed bone.

Precautionary Measures

The user must wear protective goggles during treatment with the astringent retraction paste. If eye contact occurs, wash immediately with plenty of water and consult a physician if necessary.

3M MSDSs can be obtained from www.mmm.com, or contact your local subsidiary.

Application

- ▶ Remove a retraction capsule from the blister (immediately before use only), place it in a suitable, commonly available dispenser, and check to see that the retraction capsule is firmly seated in the dispenser.
- ▶ Remove the sealing cap from the capsule tip (immediately before application only) and discard.
 - The capsule tip is rounded and must not be shortened because this would result in sharp edges.
- ▶ Dispense a small quantity of paste onto a mixing block and discard (Fig. 1).
- ▶ Rinse the sulcus thoroughly with water and dry lightly with air before application of the paste.
- ▶ Introduce the capsule tip into the sulcus, opening the sulcus (Fig. 2).

- ▶ Move the capsule tip in the sulcus slowly and evenly around the tooth, pressing out the paste and filling the sulcus all around with enough retraction paste so that an excess appears (Fig. 3).
 - If required by the clinical situation, the retraction paste can be used in combination with a cord to open the sulcus further (Fig. 4).
- ▶ Allow the astringent retraction paste to react for at least 2 minutes in the sulcus (Fig. 5). Keep all moisture away from the site during this period.
 - An overly long residence time may cause damage to tissue.
- ▶ Remove the retraction paste completely from the sulcus after the residence time, using a mixture of air and water and with the aid of a suction device (Fig. 6).
 - The patient must not swallow the retraction paste.

Concluding the Treatment

- ▶ Carefully check the sulcus of the treated teeth and the surrounding areas. Remove any astringent retraction paste from the mouth.

Cleaning

- Contact with astringent retraction paste may cause metal parts to corrode. Remove any residual paste on the instruments immediately with water.
- Follow the corresponding instructions for use for using and disinfecting the dispenser.

Disposal

- ▶ Dispose of empty retraction capsules in the normal waste of the medical practice.

Storage and Stability

Store the product at 15–25° C/59–77° F.

Do not use after the expiration date.

Customer Information

No person is authorized to provide any information that deviates from the information provided in this instruction sheet.

Warranty

3M Deutschland GmbH warrants this product will be free from defects in material and manufacture. 3M Deutschland GmbH MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Deutschland GmbH's sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.

Limitation of Liability

Except where prohibited by law, 3M Deutschland GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Information valid as of February 2012

RUS: Authorised representative of the manufacturer in Russia: ZAO "3M Russia",
Russia 121614, Moscow, Krylatskaya str. bldg.17/3, T.+74957847474

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