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Hersteller

Symbol description

WARNING: Failure to observe may result in injury or even death.

CAUTION: Failure to observe may result in damage to or even destruction of the product.

NOTE: Special information on the operation of the instrument.

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Does not apply to USA and Canada!
US and Canadian customers, please use the separate US instructions!

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2 Intended use

The optical fibers (laser fibers) are designed for use in connection with the Calculase. Calculase II and Calculase III laser units from KARL STORZ in the wavelength spectrum of 2100 nm. They are intended for use in operating rooms and doctors' offices for human lithotripsy applications and for the cutting, ablation and vaporization of tissue.

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2.1 Indications

The use of reusable laser fibers is indicated if the following is indicated in the opinion of the attending physician.

Lithotripsy:

Stones of all types in the ureteral system (urethra, ureter, bladder and kidneys), bile duct, gall bladder, salivary duct and salivary glands.

Urology:

- Holmium laser ablation - HoLAP
- Holmium laser resection of the prostate - HoLRP
- Holmium laser enucleation of the prostate - HoLEP
- Obstruction of the bladder neck
- Prostate and bladder incisions
- Hemostasis, vaporization and excision for treatment of benign prostate hypertrophy

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