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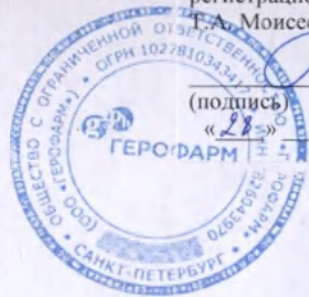
«28»

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Pen-injector for insulin injections for individual use

RinsaPen® II

with accessories

Basic device:

1. Pen-injector for insulin injections for individual use RinsaPen® II -1 pcs.
2. Instructions for use -1 pcs.

Accessories:

1. Heat insulated soft case -1 pcs.

YPSOMED

Brunnmattstrasse 6
3401 Burgdorf / Switzerland

S. Dinkel

September 24, 2020

Instruction for use

Thank you for choosing the pen-injector RinsaPen® II, developed in Switzerland.

These instructions for use contain information about how to use, store and clean your pen-injector RinsaPen® II. Please read the instructions for use carefully, even if you have already used a RinsaPen® II or other pen systems for insulin injections previously.

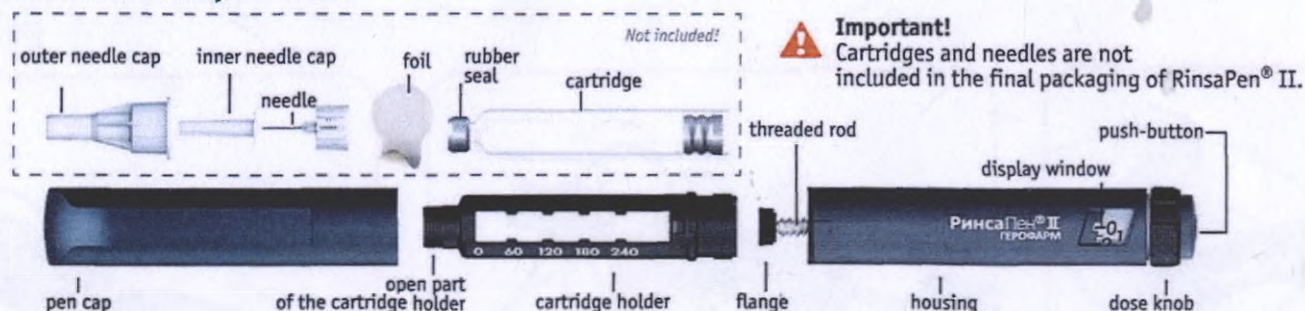
Pen-injector is easy to operate. Product users are self-injecting adult patients and children from 12 years of age. RinsaPen® II can deliver doses from 1 to 60 insulin units (I.U.) in increments of 1 unit.

Important! Pen-injector RinsaPen® II is intended for individual use only!

Intended use

Pen-injector for insulin injections for individual use RinsaPen® II with accessories is intended for injections of insulin and insulin analogs in 3 ml insulin cartridges.

Pen-injector and its components

■ Pen-injector RinsaPen® II is used only with 3 ml cartridges with insulin and insulin analogs, manufactured by OOO GEROPHARM. The company is not liable for any damage caused by the use of other cartridges.

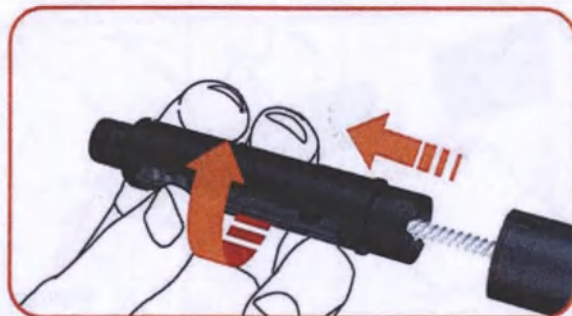
■ Pen-injector RinsaPen® II is used with standard pen needles, intended for use with pen-injectors for injections of insulin and insulin analogs, and registered in the Russian Federation. Information concerning needles' compatibility is stated on the package of pen needles.

Pen preparation Setting and changing cartridges

1 Remove the pen cap.


2 Remove the cartridge holder by twisting. Remove any empty cartridge if necessary.


3 Disinfect the rubber seal of the cartridge with the alcohol wipe and insert the cartridge in the cartridge holder with the aluminum end first.

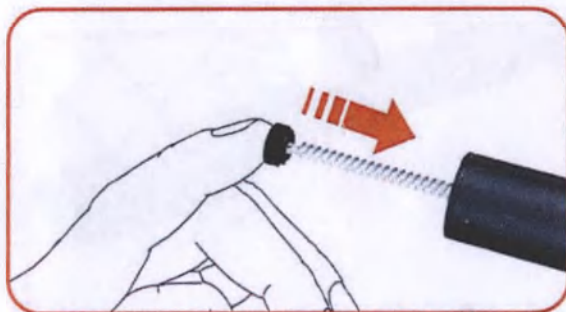- 4 With the tip of your finger, slide back the threaded rod as far as it will go. To do this you should never use the cartridge inserted in the cartridge holder.


- 5 Make sure you are using the correct insulin, prescribed for you. Check the condition and expiration date of the insulin. Do not use the cartridge if the insulin is cloudy or contains particles. Do not use if the expiration date of insulin has expired. Make sure the cartridge is not damaged. Insert the cartridge holder with the insulin cartridge in the housing by twisting, check to make sure the alignment marks coincide. Before fitting the needle disinfect the open part of the cartridge holder (use alcohol wipe).

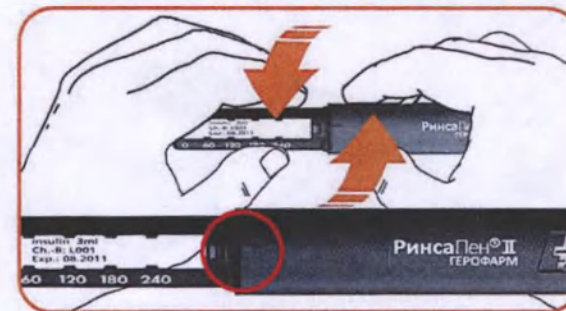
Fitting the pen needle

Follow the instruction for use for your pen needle.

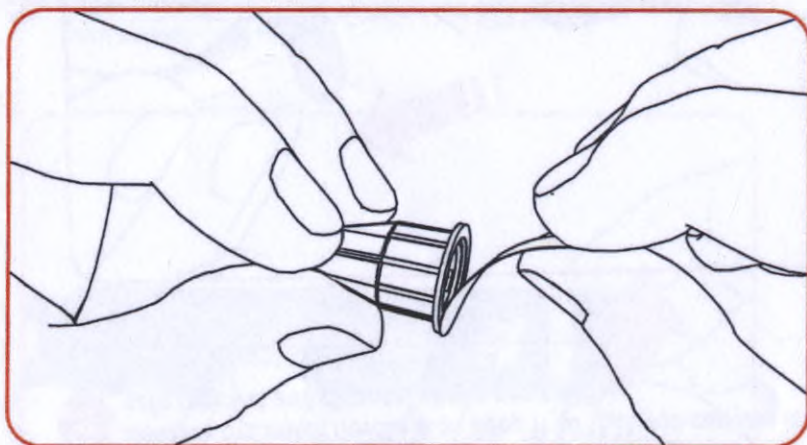
- 1 Remove the sealing foil from the outer needle cap.

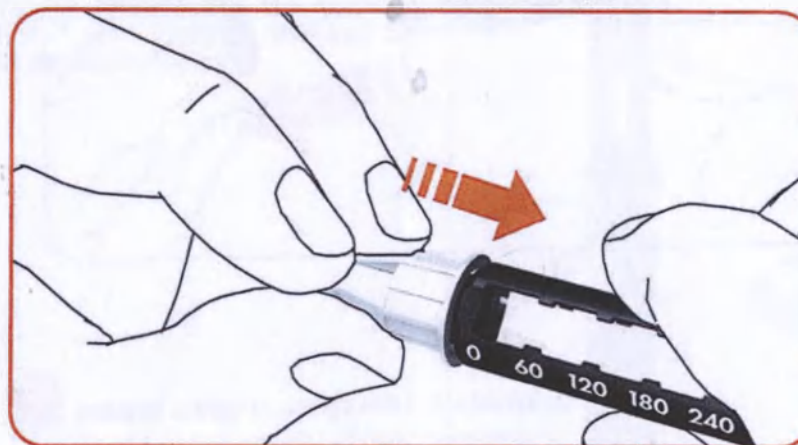
- 2 Screw the pen needle onto the cartridge holder with the help of the outer needle cap. Check to make sure the pen needle is correctly positioned.

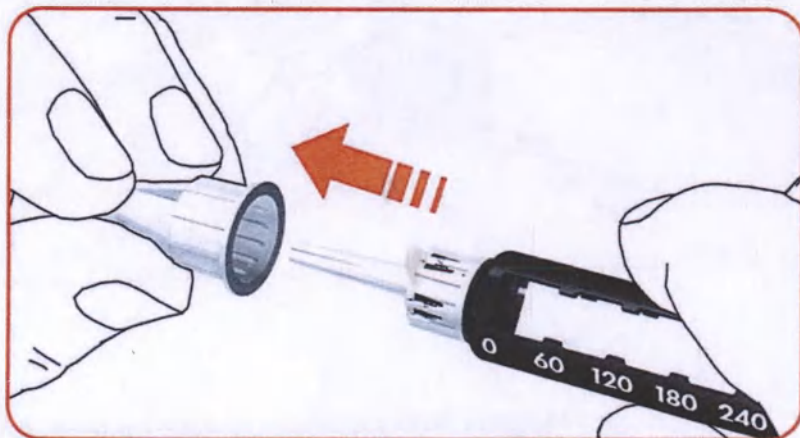- 3** Remove the outer needle cap. Keep it so that you can use it later for safe removal and disposal of the pen needle.

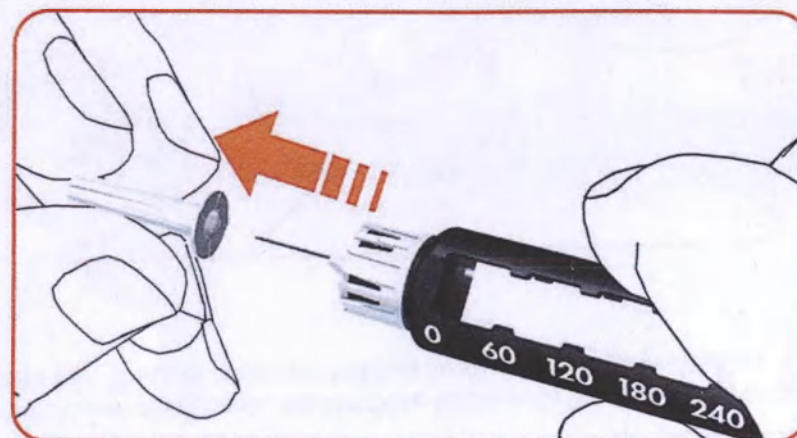- 4** Remove the inner needle cap and dispose it.

Priming/ performance test

A performance test of the pen-injector must be conducted before each injection in order to remove air bubbles from the insulin cartridge and check that the pen needle is not clogged.

- 1 Set a dose of 6 I.U. by turning the dosage knob clockwise. Check the dose set by looking at the display window vertically from above.

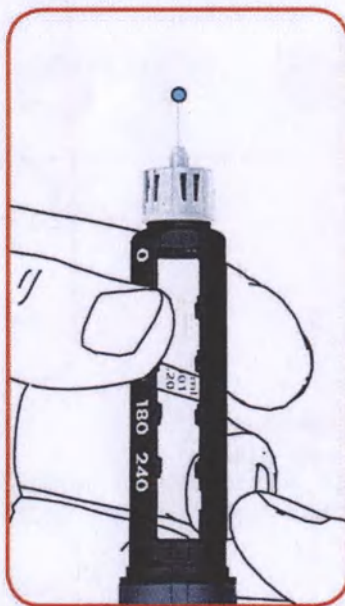
- 2 Holding the pen-injector with the pen pointing up carefully tap the cartridge holder with your finger so that any air bubbles can rise to the top.

- 3** Press the push-button right in.

- 4** A few drops of insulin should emerge from the top of the pen needle.

If no drops emerge, repeat steps 1 to 4 with a dose of 3 I.U. If after conducting performance check 3 times still no drops emerge, change the pen needle and repeat steps 1 to 4. If there is no result of the performance test or if you have any concerns, please call Customer Service (toll free number: 8 800 333 43 76).

Setting the dose

- 1 Dial the required insulin dose by turning the dosage knob clockwise. Correct the dose by turning counter clockwise. For every I.U. selected the pen generates a click.
- 2 Check the dialed dose by looking vertically from above at the display window.

Important! When using the pen-injector it is possible to set a dose that is larger than the quantity of insulin still available in the cartridge. If the amount of insulin left in the cartridge is no longer sufficient for the desired insulin dose, the pen will stop. Read the insulin dose deficit off the display window, this is the amount of insulin you have to inject from the new cartridge to complete your dose.

Please refer also to section *Additional information* (page 13).

Injection

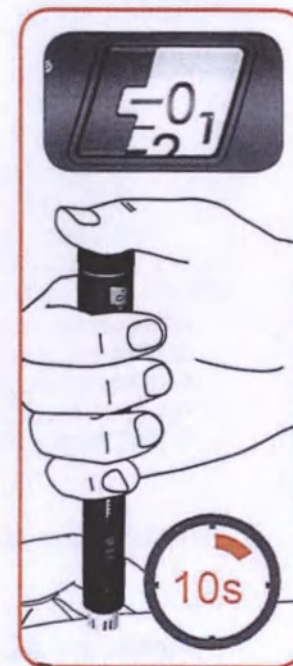
Consult your healthcare professional to choose the injection technique.

- 1 Disinfect (alcohol wipe is recommended for use) the intended place of injection, about 4-5 cm in diameter. Leave it to dry (≈ 1 minute). Insert the pen needle. Press the push-button right in and keep it fully pressed for the whole time of injection.

Important! Do not twist dosage knob during insertion.


- 2 When the entire insulin dose has been delivered, keep the push-button pressed for another 10 second to make sure the entire dose has been delivered. Check to make sure the display is '0'. Remove the pen needle slowly. Disinfect the place of injection with alcohol wipe.

Important! You can interrupt the injection by letting go of the push-button. In the display window you can read off the remaining insulin dose that has not been administered, it can be administered by pressing the push-button again.


- 3** Remove the pen needle after each injection.

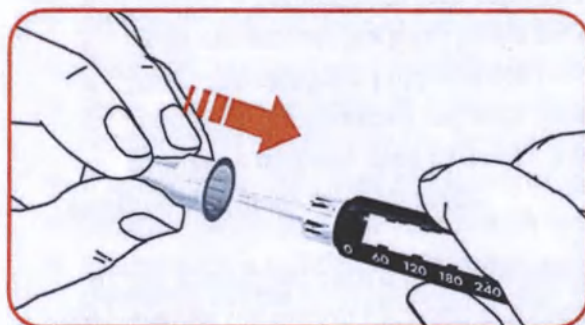Fit the outer needle cap on the needle carefully.

Unscrew the pen needle and dispose it.

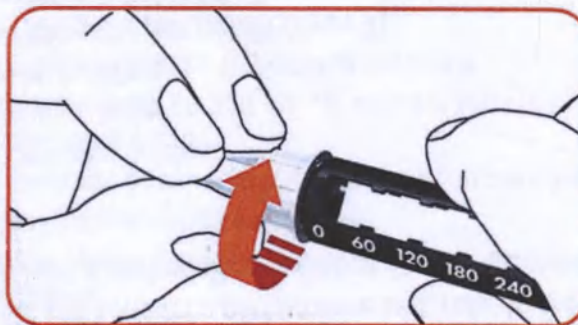- 4** Put the pen cap back on after each use.

Additional information

Audible and tactile indicators

RinsaPen® II features the following audible and tactile indications for correct dose control:

- During dose dialing and dose correction pen-injector provides an audible signal and a noticeable resistance per dialed unit.
- During injection a sound can be heard, which stops as soon as the set dose has been administered in full.

Dose deficit

- Read the insulin dose deficit off the display and remember it.
- Change the insulin cartridge and the needle and conduct a performance test, refer to the section *Priming/ Performance test* (page 8).
- Set the insulin dose deficit and inject it to get a desired dose in full.

Inserting a partially used cartridge

- Insert the partially used cartridge in the cartridge holder.
- Slide back the threaded rod with the tip of your finger.
- Insert the cartridge holder in the housing by twisting.
- Fit a new pen needle, refer to the section *Fitting the pen needle* (page 6).
- Conduct a performance test, refer to the section *Priming/ Performance test* (page 8).
- Set the insulin dose and inject it.

Symbols used

Batch code

Consult instructions for use

Manufacturing date

Keep away from heat

Storage, care and transport

- After each use put the pen cap back on.
- Store and transport your pen-injector in a pen case without a pen needle attached.
- Store your pen-injector at a temperature of between 2 °C and 40 °C, relative humidity not more than 85%. Non-compliance with storage temperatures can shorten the service life of the pen-injector.
- If a new or partially used insulin cartridge is inserted, observe the temperature information on the leaflet enclosed with the insulin cartridge.
- Generally handle your pen-injector with care. Protect it from moisture and dust.
- A damp cloth is sufficient to clean your pen-injector, please, do not use cleaning agents containing alcohol or other solvents, alcohol wipe in used only for disinfection of the cartridge holder before the injection.

Risk information

Read instructions for use of the pen-injector, pen needles and insulin cartridges carefully before using your pen-injector, use the pen-injector only after first consulting your doctor.

Special user categories

The device is designed to maximize ease of handling, however the healthcare professional has to decide case by case, if the patient is capable to handle the device and to act as self-injecting user to assure that the patient's kinetic and cognitive abilities allow a safe handling of the pen. 14

- Children may only use the pen-injector RinsaPen® II for self-administration from the age of 6 under survey of trained and experienced adults.
- Visually handicapped or blind patients must use the pen-injector only with assistance from healthcare professionals or someone who has been trained and is not visually impaired.
- Persons who are limited in their sense of hearing or touch must consult a doctor before using the pen-injector.
- Store your pen-injector, the needles and the insulin cartridges out of the reach of children and any other persons who are not acquainted with how to handle RinsaPen® II properly.
- Only use the RinsaPen® II in an environment with adequate lightning and low background noise in order to ensure perception of audible signals from the pen-injector.

Correct use

- Only use your pen-injector within the scope of treatment that has been prescribed by your doctor, following instructions concerning insulin cartridges, pen needles and dose quantities. Any changes must be made under the supervision of a doctor.
- Before every injection make sure your pen-injector contains the correct insulin type prescribed for you, check to make sure the insulin cartridge is undamaged.
- Pen needles and insulin cartridges are intended for individual use only to avoid disease transmission.
- Do not change the injection pen angle once the pen needle has been inserted into the skin. It can cause the pen needle to break off. If a broken pen needle remains stuck in the body, visit a doctor immediately.
- Do not subject your pen-injector to direct sunlight or cold temperatures.
- Do not drop the pen-injector and do not knock it against hard surfaces, do not immerse in water. Never use your pen-injector if you have any doubts about the proper device operation.
- Do not apply force to the pen-injector, it may cause damage.

- Do not use the pen-injector in case it is damaged because it may no longer operate properly.
- Do not attempt to repair a damaged pen-injector yourself. Please refer to the section *Warranty* (page 17) in case of damages or defected device.
- Replace the pen needle for every injection.

Correct administration of the medicinal product

- Conduct a performance test before every injection, refer to the section *Priming/ Performance test* (page 8) in order to rule out the risk of an incorrect dosage caused by air bubbles in the insulin cartridge or a clogged pen needle. Change the pen needle if necessary.
- After each injection check whether insulin is dripping from the pen-injector or the site of puncture. In case of insulin dripping from the site of puncture, pen needle or the pen-injector, please consult your doctor immediately, follow the instructions concerning how to handle a potential underdose.
- If an injection is interrupted by letting go of the push-button, check the display to establish the insulin dose that has not been administered.
- In case an insulin injection has been administered by mistake, please visit a doctor immediately.
- In the event of accidental needle injury involving the risk of disease being transmitted, please visit a doctor immediately.

Possible side effects

- Under- or over dosage possible, when the cartridge is empty and the remaining part of pre-set dose, indicated in the display window, has to be injected with a new cartridge.

Contraindications

The pen-injector must not be used for self-injecting purposed in the following cases:

- Children under 6 years of age
- Patients with mental disabilities
- Patients with impairments of kinetic and cognitive abilities

Disposal

Observe safety precautions for disposal of the pen-injector.

You can dispose your pen-injector in accordance with local waste regulations.

The pen injectors used as well as the pen injectors not used are considered as Class A waste and don't contain substances harmful to the environment and can be disposed of with household waste.

Remove the pen needle and insulin cartridge from the RinsaPen® II before disposal.

Dispose used pen needles and insulin cartridges according to recommendations of healthcare professionals and sanitary rules.

International standards conformity

Pen-injector conforms to the regulations of the EU directive on medical devices 93/42/EEC and ISO 11608-1.

Storage period/ service life

Storage period of the pen-injector before first use: 2,5 years

Service life of the pen-injector: 3 years.

Warranty

The warranty of RinsaPen® II covers the whole service life of 3 years after first use under conditions of proper use of the pen-injector in accordance to Instructions for use.

The warranty is limited to replacement of the pen.

Pen-injector RinsaPen® II has been developed for use with 3 ml insulin cartridges, manufactured by OOO GEROPHARM, only in such case the dose accuracy and safety are guaranteed.

Warranty does not cover damage as a result of improper or careless use, storage or cleaning, and does not apply in the event of use contrary to the Instructions for use.

Manufacturer

Ypsomed AG, Brunnmattstrasse 6, 3401 Burgdorf, Switzerland

Authorized representative of the manufacturer/ claims to be addressed to

OOO GEROPHARM

Postal address: BC "Nevskaya Ratusha", 11B Degtyarny lane, Saint Petersburg, 191144, Russian Federation

Toll free number: 8 800 333 43 76 (free throughout Russia)

www.geropharm.ru

Data of the last review: 08.09.2020

LEGALISATION

Martin Bürgi, notary public, registered in the Register of Notaries Public of the Canton of Berne, with office in Kirchberg (Switzerland), certifies:

that the Ypsomed AG, with registered office at CH-3400 Burgdorf (Switzerland), has submitted today to the notary the foregoing document (10 sheets).

Certified at the office of the notary public at Kirchberg on the twenty-eighth of September two thousand and twenty;

28.09.2020

The notary public:

A large, stylized handwritten signature in blue ink, written over the text "The notary public:".

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