Abbott GmbH & Co. KG
Max-Planck-Ring 2
65205 Wiesbaden
Postfach 1303, 65011 Wiesbaden

Tel. +49 61 22 58 0
Fax +49 61 22 58 1244

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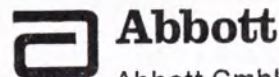
Signed: ZQ
Zaman Khan,
Associate Director Regulatory
Affairs EMEA and Pakistan,
Abbott GmbH & Co. KG.

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dated August 2016.

JAME
Alinity i HIV Ag/Ab Combo
HIV Ag/Ab Cal)

INTENDED USE
The Alinity i HIV Ag/Ab
of the Alinity i analy
qualitative detection
immunodeficiency
serum or plasma
For additional in
reagent packag

CONTENT
CAL 1 contains
saline with
Calibrator
CAL 1

a Dye:
MAT

STA
To
th
c

linity i

IV Ag/Ab Combo Calibrator

HIV Ag/Ab

REF 08P0701

G71153R01

S8P070

Created August 2016.

NAME

Alinity i HIV Ag/Ab Combo Calibrator (also referred to as HIV Ag/Ab Cal)

INTENDED USE

The Alinity i HIV Ag/Ab Combo Calibrator is for the calibration of the Alinity i analyzer when used for the simultaneous qualitative detection of HIV p24 antigen and antibodies to human immunodeficiency virus type 1 and/or type 2 (HIV-1/HIV-2) in human serum or plasma.

For additional information, refer to the Alinity i HIV Ag/Ab Combo reagent package insert and the Alinity ci-series Operations Manual.

CONTENTS

CAL 1 contains purified HIV viral lysate prepared in TRIS buffered saline with protein (bovine) stabilizer. Preservative: Sodium azide.

Calibrator	Quantity	Color
CAL 1	1 x 3.0 mL	Red ^a

^a Dye: Red D&C No. 33

MATERIALS REQUIRED BUT NOT PROVIDED

- 04R1001 Alinity ci-series Calibrator/Control Replacement Caps

STANDARDIZATION

To maintain a consistent HIV-1 p24 antigen sensitivity of < 50 pg/mL, the Alinity i HIV Ag/Ab Combo Calibrator has a HIV-1 p24 antigen concentration referenced to the international standard Agence française de sécurité sanitaire des produits de santé (AFSSAPS) panel.

PRECAUTIONS

- IVD**
- For In Vitro Diagnostic Use

Safety Precautions

- CAUTION:** This product contains human-sourced and/or potentially infectious components. Refer to the CONTENTS section of this package insert. No known test method can offer complete assurance that products derived from human sources or inactivated microorganisms will not transmit infection. Therefore, all human-sourced materials should be considered potentially infectious. It is recommended that these reagents and human specimens be handled in accordance with the OSHA Standard on Bloodborne Pathogens. Biosafety Level 2 or other appropriate biosafety practices should be used for materials that contain or are suspected of containing infectious agents.¹⁻⁴
- The calibrator is purified HIV-1 viral lysate.

The following warnings and precautions apply to: CAL 1	
Contains sodium azide.	
EUH032	Contact with acids liberates very toxic gas.
P501	Dispose of contents / container in accordance with local regulations.

Safety Data Sheets are available at www.abbottdiagnostics.com or contact your local representative.

For a detailed discussion of safety precautions during system operation, refer to the Alinity ci-series Operations Manual, Section 8.

PREPARATION FOR USE

- This product is liquid ready-to-use.
- This product may be used immediately after removal from 2 to 8°C storage.
- Prior to each use, mix by gentle inversion (5 to 10 times).

STORAGE

- Do not use past expiration date.

	Storage Temperature	Maximum Storage Time	Additional Storage Instructions
Unopened	2 to 8°C	Until expiration date	
Opened	2 to 8°C	Until expiration date	Store tightly capped with new replacement cap. Return to refrigerated storage after use.

The analyzer will track In-use Stability, which is the time the calibrator is outside of refrigerated storage while on the analyzer. The analyzer will not allow the use of the calibrator if the In-use Stability has been exceeded. Maximum In-use Stability can be found in the Assay Parameter Report. For additional information on calibrator In-use Stability, refer to the Alinity ci-series Operations Manual, Section 5.

For additional information on printing assay parameters, refer to the Alinity ci-series Operations Manual, Section 5.

INSTRUMENT PROCEDURE

- Calibrator lots may be configured using the bar code label on the calibrator carton.
- For information on configuring calibrator data, refer to the Alinity ci-series Operations Manual, Section 2.
- For instructions on ordering and loading calibrators on the instrument, refer to the Alinity ci-series Operations Manual, Section 5.

QUALITY CONTROL PROCEDURES

A single sample of each control level must be tested to evaluate the assay calibration. Ensure that assay control values are within the ranges specified in the respective control package insert.

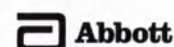
For information on ordering controls, refer to the Alinity ci-series Operations Manual, Section 5.

Once a calibration is accepted and stored, all subsequent samples may be tested without further calibration unless:

- A reagent kit with a new lot number is used.
- Daily quality control results are outside of statistically-based quality control limits used to monitor and control system performance, as described in the Quality Control Procedures section of the associated reagent package insert.
- If statistically-based quality control limits are not available then the calibration should not exceed a 30-day limit for recalibration frequency.

This assay may require recalibration after maintenance to critical parts or subsystems or after service procedures have been performed.

For additional information, refer to the assay reagent package insert and the Alinity ci-series Operations Manual.

INDICATIONS OF INSTABILITY OR DETERIORATION

Instability or deterioration should be suspected if there are precipitates, visible signs of leakage, turbidity, if calibration does not meet the appropriate package insert and/or Alinity ci-series Operations Manual criteria, or if controls do not meet the appropriate criteria.

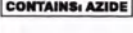
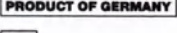
BIBLIOGRAPHY

1. US Department of Labor, Occupational Safety and Health Administration, 29 CFR Part 1910.1030, Bloodborne pathogens.
2. US Department of Health and Human Services. *Biosafety in Microbiological and Biomedical Laboratories*. 5th ed. Washington, DC: US Government Printing Office; December 2009.
3. World Health Organization. *Laboratory Biosafety Manual*. 3rd ed. Geneva: World Health Organization; 2004.
4. Clinical and Laboratory Standards Institute (CLSI). *Protection of Laboratory Workers From Occupationally Acquired Infections; Approved Guideline—Fourth Edition*. CLSI Document M29-A4. Wayne, PA: CLSI; 2014.

Note for number formatting:

- A space is used as thousands separator (example: 10 000 specimens).
- A period is used to separate the integer part from the fractional part of a number written in decimal form (example: 3.12%).

Key to Symbols

	Caution
	Consult instructions for use
	Manufacturer
	Temperature limitation
	Use by/Expiration date
	Calibrator 1
	Control Number
	Contains Sodium Azide. Contact with acids liberates very toxic gas.
	<i>In Vitro</i> Diagnostic Medical Device
	Lot Number
	Product of Germany
	List Number
	Serial Number

Alinity is a trademark of Abbott Laboratories in various jurisdictions. All other trademarks are property of their respective owners.

Abbott GmbH & Co. KG
Max-Planck-Ring 2
65205 Wiesbaden
Germany
+49-6122-580

0088

**Customer Service: Contact your local representative
or find country-specific contact information on
www.abbottdiagnostics.com**

Created August 2016.

©2016 Abbott Laboratories

PENDUM

the Instruction for Use Alinity i HIV Ag/Ab Combo Calibrator for Russia

SHIPPING AND STORAGE CONDITIONS

Recommended shipping conditions for devices are 2-8°C. Medical devices must be stored in refrigerators/refrigerator chambers to ensure regulated temperature conditions with a daily temperature record.

Medical devices must be carried out in single-use thermal containers containing coolant elements, or in reusable thermo containers with automatically maintained temperature or in refrigerated trucks using thermal indicators.

Device is stable until the expiration date indicated on the package. Device is no longer suitable for use if shipping temperature limits have been violated. Ship in an any kind of enclosed carrier vehicle in accordance with shipping rules, applicable to the respective type of transport.

WARRANTY

Manufacturer guarantees that device remains stable until the expiration date when stored and handled as intended. Manufacturer set the maximum shelf life of

Alinity i HIV Ag/Ab Combo Calibrator – 10 months from date of manufacturing.

Device should not be used if stored with deviations from manufacturer's requirements (GOST R 51088-2013).

MAINTENANCE

Not required.

REPAIRS

Not required.

DESTRUCTION AND DISPOSAL OF A MEDICAL DEVICE

Destruct and dispose of a medical device in accordance with the requirements of SanPin 2.1.7.2790-10 dated December 12, 2010 "Sanitary-epidemiological Requirements for the Management of Medical Waste". Transmit and dispose of a product and its components and packaging in accordance with local regulations.

LIST OF APPLICABLE STANDARDS

98/79/EC, 1272/2008/EC (CLP), 1907/2006/EC (REACH), 2002/364/EC, 2009/886/EC, Regulation (EC) 1069/2009, Regulation (EC) 142/2011, CLASS 2013, GHS, WHSR Part 7.1, EN ISO 9001, ISO 13485, EN ISO 14971, EN 13641, EN 13612, EN ISO 15223-1, EN 980 EN ISO 18113-1, EN ISO 18113-2, EN ISO 18113-3, EN ISO 23640*, EN ISO 20776-1, EN 13975, EN ISO 17511, EN ISO 15193, EN ISO 15194

POTENTIAL USER

The Alinity i HIV Ag/Ab Combo Calibrator is intended to be used in a variety of clinical and laboratory conditions, including, without limitation, hospitals, diagnostic centers, reference laboratories and blood transfusion facilities around the world, whose personnel have been trained in the use of the Alinity i analyzer.

INFORMATION ABOUT PRESENCE OF DRUGS IN COMPOSITION OF MEDICAL DEVICE

Does not contain any drugs

STERILIZATION

Medical device is not required to be sterile.

INFORMATION ABOUT POSSIBILITY OF MULTIPLE APPLICATION

Medical device is intended for one-off usage

MATERIALS OF ANIMAL ORIGIN

Materials of animal origin are tested and risks of using MD are indicated in SDS. Safety Data Sheets are available at www.abbottdiagnostics.com or contact your local representative.

INDICATIONS AND CONTRAINDICATIONS FOR USE

Not applicable.

TECHNICAL CHARACTERISTICS

- Calibrator 1: pH-value at 20 °C: 7.2
Density at 20 °C: 1.007 g/cm³

COMPATIBILITY RESTRICTIONS OF MEDICAL DEVICE

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