Abbott GmbH & Co. KG
Max-Planck-Ring 2
65205 Wiesbaden
Postfach 1303, 65011 Wiesbaden

Tel. +49 61 22 58 0
Fax +49 61 22 58 1244

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Signed: [Signature]
Zaman Khan,
Associate Director Regulatory
Affairs EMEA and Pakistan,
Abbott GmbH & Co. KG.

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G71281R02

C7P450

Read Highlighted Changes: Revised July 2019.

NAME

Alinity i Toxo IgG Controls (also referred to as Toxo IgG Ctrls)

INTENDED USE

The Alinity i Toxo IgG Controls are for the estimation of test precision and the detection of systematic analytical deviations of the Alinity i analyzer when used for the quantitative determination of IgG antibodies to *Toxoplasma gondii* in human serum and plasma. For additional information, refer to the Alinity i Toxo IgG reagent package insert and the Alinity ci-series Operations Manual.

CONTENTS

CONTROL - contains recalcified human plasma. Preservatives: ProClin 950 and sodium azide.

CONTROL + 1 and **CONTROL + 2** contain anti-Toxo IgG in aqueous solution with protein (bovine) stabilizer. Preservative: ProClin 300. The controls are at the following ranges and target concentrations:

Control	Quantity	Color	Anti-Toxo IgG	
			CONC (IU/mL)	RANGE (IU/mL)
CONTROL -	1 x 8.0 mL	Natural	-	≤ 0.5
CONTROL + 1	1 x 8.0 mL	Natural	6.0	3.0 - 9.0
CONTROL + 2	1 x 8.0 mL	Natural	120.0	60.0 - 180.0

NOTE: The insert ranges for the controls are not lot specific and represent the total range of values which may be generated throughout the life of the product. It is recommended that each laboratory establish its own means and acceptable ranges which should fall within the package insert ranges. Sources of variation that can be expected include:

- Calibration
- Control lot
- Reagent lot
- Calibrator lot
- Instrument

PRECAUTIONS

- **IVD**
- For *In Vitro* Diagnostic Use

Safety Precautions

-  **CAUTION:** This product contains human-sourced and/or potentially infectious components. Refer to the CONTENTS section of this package insert. No known test method can offer complete assurance that products derived from human sources or inactivated microorganisms will not transmit infection. Therefore, all human-sourced materials should be considered potentially infectious. It is recommended that these reagents and human specimens be handled in accordance with the OSHA Standard on Bloodborne Pathogens. Biosafety Level 2 or other appropriate biosafety practices should be used for materials that contain or are suspected of containing infectious agents.¹⁻⁴
- The human-sourced material used in the negative control is nonreactive for anti-Toxo IgG, HBsAg, HIV-1 RNA or HIV-1 Ag, anti-HIV-1/HIV-2, and anti-HCV.

The following warnings and precautions apply to: **CONTROL -**

WARNING	Contains methylisothiazolone and sodium azide.
H317	May cause an allergic skin reaction.
EUH032	Contact with acids liberates very toxic gas.
Prevention	
P261	Avoid breathing mist / vapors / spray.
P272	Contaminated work clothing should not be allowed out of the workplace.
P280	Wear protective gloves / protective clothing / eye protection.
Response	
P302+P352	IF ON SKIN: Wash with plenty of water.
P333+P313	If skin irritation or rash occurs: Get medical advice / attention.
P362+P364	Take off contaminated clothing and wash it before reuse.
Disposal	
P501	Dispose of contents / container in accordance with local regulations.

The following warnings and precautions apply to:

CONTROL + 1 and **CONTROL + 2**

WARNING	Contains methylisothiazolones.
H317	May cause an allergic skin reaction.
Prevention	
P261	Avoid breathing mist / vapors / spray.
P272	Contaminated work clothing should not be allowed out of the workplace.
P280	Wear protective gloves / protective clothing / eye protection.
Response	
P302+P352	IF ON SKIN: Wash with plenty of water.
P333+P313	If skin irritation or rash occurs: Get medical advice / attention.
P362+P364	Take off contaminated clothing and wash it before reuse.
Disposal	
P501	Dispose of contents / container in accordance with local regulations.

Safety Data Sheets are available at www.abbottdiagnostics.com or contact your local representative.

For a detailed discussion of safety precautions during system operation, refer to the Alinity ci-series Operations Manual, Section 8.

 Abbott

PREPARATION FOR USE

- This product is liquid ready-to-use.
- This product may be used immediately after removal from 2 to 8°C storage.
- Prior to each use, mix by gentle inversion.

STORAGE

- Do not use past expiration date.

	Storage Temperature	Maximum Storage Time	Additional Storage Instructions
Unopened	2 to 8°C	Until expiration date	
Opened	2 to 8°C	Until expiration date	Store tightly capped. Return to refrigerated storage after use.

INSTRUMENT PROCEDURE

- To obtain the recommended volume requirements for the controls, hold the bottle vertically, and dispense 4 drops of the negative control, 4 drops of positive control 1, and 4 drops of positive control 2 into each sample cup in the assigned position.
- For instructions on ordering and loading controls on the instrument, refer to the Alinity ci-series Operations Manual, Section 5.

INDICATIONS OF INSTABILITY OR DETERIORATION

Instability or deterioration should be suspected if there are precipitates, visible signs of leakage, turbidity, or if controls do not meet the appropriate package insert and/or Alinity ci-series Operations Manual criteria.

BIBLIOGRAPHY

1. US Department of Labor, Occupational Safety and Health Administration, 29 CFR Part 1910.1030, Bloodborne pathogens.
2. US Department of Health and Human Services. *Biosafety in Microbiological and Biomedical Laboratories*. 5th ed. Washington, DC: US Government Printing Office; December 2009.
3. World Health Organization. *Laboratory Biosafety Manual*. 3rd ed. Geneva: World Health Organization; 2004.
4. Clinical and Laboratory Standards Institute (CLSI). *Protection of Laboratory Workers From Occupationally Acquired Infections; Approved Guideline—Fourth Edition*. CLSI Document M29-A4. Wayne, PA: CLSI; 2014.

Note for number formatting:

- A space is used as thousands separator (example: 10 000 specimens).
- A period is used to separate the integer part from the fractional part of a number written in decimal form (example: 3.12%).

Key to Symbols

	Caution
	Consult instructions for use
	Manufacturer
	Temperature limitation
	Use by/Expiration date
CN	Control Number
CONC	Concentration
CONTAINS: AZIDE	Contains Sodium Azide. Contact with acids liberates very toxic gas.
CONTROL -	Negative Control
CONTROL + 1	Positive Control 1
CONTROL + 2	Positive Control 2
IVD	In Vitro Diagnostic Medical Device
LOT	Lot Number
PRODUCT OF GERMANY	Product of Germany
RANGE	Range
REF	List Number

Alinity is a trademark of Abbott Laboratories in various jurisdictions. All other trademarks are property of their respective owners.

Abbott GmbH & Co. KG
Max-Planck-Ring 2
65205 Wiesbaden
Germany
+49-6122-580

Customer Service: Contact your local representative or find country-specific contact information on www.abbottdiagnostics.com

Revised July 2019.

©2016, 2019 Abbott Laboratories

ADDENDUM

to the Instruction for Use Alinity i Toxo IgG Controls for Russia

SHIPPING AND STORAGE CONDITIONS

Recommended shipping conditions for devices are 2-8°C. Medical devices must be stored in refrigerators/refrigerator chambers to ensure regulated temperature conditions with a daily temperature record.

Medical devices must be carried out in single-use thermal containers containing coolant elements, or in reusable thermo containers with automatically maintained temperature or in refrigerated trucks using thermal indicators.

Device is stable until the expiration date indicated on the package. Device is no longer suitable for use if shipping temperature limits have been violated. Ship in an any kind of enclosed carrier vehicle in accordance with shipping rules, applicable to the respective type of transport.

WARRANTY

Manufacturer guarantees that device remains stable until the expiration date when stored and handled as intended. Manufacturer set the maximum shelf life of

Alinity i Toxo IgG Controls – 12 months from date of manufacturing.

MAINTENANCE

Not required.

REPAIRS

Not required.

DESTRUCTION AND DISPOSAL OF A MEDICAL DEVICE

Destruct and dispose of a medical device in accordance with the requirements of SanPin 2.1.7.2790-10 dated December 12, 2010 "Sanitary-epidemiological Requirements for the Management of Medical Waste". Transmit and dispose of a product and its components and packaging in accordance with local regulations.

LIST OF APPLICABLE STANDARDS

98/79/EC, 1272/2008/EC (CLP), 1907/2006/EC (REACH), 80/181/EEC, 2009/886/EC, 2009/3/EC, 1069/2009, Regulation (EC) 142/2011, CLASS 2013, GHS, WHSR Part 7.1, EN ISO 9001, ISO 13485, EN ISO 14971, EN 13641, EN 13612, EN ISO 15223-1, EN 980 EN ISO 18113-1, EN ISO 18113-2, EN ISO 18113-3, EN ISO 23640*, EN ISO 20776-1, EN 13975, EN ISO 17511, EN ISO 15193, EN ISO 15194, ISO 80000-1

POTENTIAL USER

The Alinity i Toxo IgG Controls is intended to be used in a variety of clinical and laboratory conditions, including, without limitation, hospitals, diagnostic centers, reference laboratories and blood transfusion facilities around the world, whose personnel have been trained in the use of the Alinity i analyzer.

INFORMATION ABOUT PRESENCE OF DRUGS IN COMPOSITION OF MEDICAL DEVICE

Does not contain any drugs

STERILIZATION

Medical device is not required to be sterile.

INFORMATION ABOUT POSSIBILITY OF MULTIPLE APPLICATION

Medical device is intended for one-off usage

MATERIALS OF ANIMAL ORIGIN

Materials of animal origin are tested and risks of using MD are indicated in SDS. Safety Data Sheets are available at www.abbottiagnostics.com or contact your local representative.

INDICATIONS AND CONTRAINDICATIONS FOR USE

Not applicable.

TECHNICAL CHARACTERISTICS

- Negative Control 1: pH-value at 20 °C: 7.5
Density at 20 °C: 1.026 g/cm³
- Positive Control 1: pH-value at 20 °C: 7.4
Density at 20 °C: 1.009 g/cm³
- Positive Control 2: pH-value at 20 °C: 7.4
Density at 20 °C: 1.009 g/cm³

COMPATIBILITY RESTRICTIONS OF MEDICAL DEVICE

No available data.

COLOR AND ODOR

Negative Control 1: Brown and odorless

Positive Control 1: Colourless and odorless

Positive Control 2: Colourless and odorless

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- US Department of Labor, Occupational Safety and Health Administration, 29 CFR Part 1910.1030, Bloodborne pathogens.
- US Department of Health and Human Services. *Biosafety in Microbiological and Biomedical Laboratories*. 5th ed. Washington, DC: US Government Printing Office; December 2009.
- World Health Organization. *Laboratory Biosafety Manual*. 3rd ed. Geneva: World Health Organization; 2004.
- Clinical and Laboratory Standards Institute (CLSI). *Protection of Laboratory Workers From Occupationally Acquired Infections; Approved Guideline—Fourth Edition*. CLSI Document M29-A4. Wayne, PA: CLSI; 2014.

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Signed: ZQ
Zaman Khan,
Associate Director Regulatory
Affairs EMEA and Pakistan,
Abbott GmbH & Co. KG.

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CONTROL + 1 and **CONTROL + 2** contain anti-Toxo IgG in aqueous solution with protein (bovine) stabilizer. Preservative: ProClin 300.

The controls are at the following ranges and target concentrations:

Control	Quantity	Color	Anti-Toxo IgG	
			CONC (IU/mL)	RANGE (IU/mL)
CONTROL -	1 x 8.0 mL	Natural	-	≤ 0.5
CONTROL + 1	1 x 8.0 mL	Natural	6.0	3.0 - 9.0
CONTROL + 2	1 x 8.0 mL	Natural	120.0	60.0 - 180.0

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- **IVD**
- For In Vitro Diagnostic Use

Safety Precautions

- **CAUTION:** This product contains human-sourced and/or potentially infectious components. Refer to the CONTENTS section of this package insert. No known test method can offer complete assurance that products derived from human sources or inactivated microorganisms will not transmit infection. Therefore, all human-sourced materials should be considered potentially infectious. It is recommended that these reagents and human specimens be handled in accordance with the OSHA Standard on Bloodborne Pathogens. Biosafety Level 2 or other appropriate biosafety practices should be used for materials that contain or are suspected of containing infectious agents.¹⁻⁴
- The human-sourced material used in the negative control is nonreactive for anti-Toxo IgG, HBsAg, HIV-1 RNA or HIV-1 Ag, anti-HIV-1/HIV-2, and anti-HCV.

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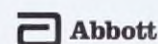
Key to Symbols

	Caution
	Consult instructions for use
	Manufacturer
	Temperature limitation
	Use by/Expiration date
	Control Number
	Concentration
	Contains Sodium Azide. Contact with acids liberates very toxic gas.
	Negative Control
	Positive Control 1
	Positive Control 2
	In Vitro Diagnostic Medical Device
	Lot Number
	Product of Germany
	Range
	List Number

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Device is stable until the expiration date indicated on the package. Device is no longer suitable for use if shipping temperature limits have been violated. Ship in any kind of enclosed carrier vehicle in accordance with shipping rules, applicable to the respective type of transport.

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No available data.

COLOR AND ODOR

Negative Control 1: Brown and odorless
Positive Control 1: Colourless and odorless
Positive Control 2: Colourless and odorless

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1. US Department of Labor, Occupational Safety and Health Administration, 29 CFR Part 1910.1030, Bloodborne pathogens.
2. US Department of Health and Human Services, *Biosafety in Microbiological and Biomedical Laboratories*. 5th ed. Washington, DC: US Government Printing Office; December 2009.
3. World Health Organization, *Laboratory Biosafety Manual*. 3rd ed. Geneva: World Health Organization; 2004.
4. Clinical and Laboratory Standards Institute (CLSI). *Protection of Laboratory Workers From Occupationally Acquired Infections; Approved Guideline—Fourth Edition*. CLSI Document M29-A4. Wayne, PA: CLSI; 2014.

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