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en

Notes for users

PROPHYpearls are intended for use in the KaVo PROPHYflex 2 and KaVo PROPHYflex 3 and only for dental treatment in the area of dentistry. Use for any other purpose is not advisable and can lead to danger.

PROPHYpearls powder is a medical product according to the relevant, national statutory provisions.

According to these provisions, PROPHYflex 2 and PROPHYflex 3 in combination with PROPHYpearls powder are to be used by a competent user only for the application described, taking into account the applicable work safety provisions, the applicable accident prevention measures and these operating instructions.

PROPHYpearls are intended for cleaning tooth surfaces

According to these provisions, it is the duty of the user

- to use only fault-free operating materials
- to ensure use for the correct purpose
- to protect himself, the patient and third parties from danger

PROPHYpearls do not dissolve in water and will, therefore, amass in the amalgam separator, necessitating more frequent emptying.

Commissioning

The PROPHYflex powder reservoir is unscrewed by rotating it counterclockwise, in the opposite direction to the arrow.

Shake the PROPHYpearls bottle or stick well before filling the powder container.

Half fill the powder reservoir (see picture). Hold the powder reservoir upright and screwing it back onto the handpiece in a clockwise direction. Should any PROPHYpearls remain in the powder reservoir, then seal the reservoir with the rubber cap.

KaVo PROPHYpearls should be stored in a cool and dry place. Protect from heat ($>60^{\circ}\text{C}$).

Procedure

The cannula tip should be held approximately 3-5 mm away from the tooth surface. It is important to position the cannula at an angle between 10° and 60° to the tooth surface to be treated, in order to ensure the rolling effect and thus to optimise the adsorption power of the spherical particles. This is in contrast to the procedure when using standard sodiumbicarbonate powders. The powder spray can be fed from the occlusal margin to the cervical boundary of the tooth without danger of injury to the periodontal pocket and the bands of the dental alveolus. When removing stained "dental plaque" biofilm, e.g. with erythrosine solution - a very efficient, visual aid, which limits the exposure time precisely to that required, is the disappearance of the red colour. Rub Vaseline onto the patient's lips to prevent the corner of the mouth from drying out or cracking. We recommend working with aspiration. The cleaning power can be varied on units in which an adjustment of the drive air pressure via the foot control is possible.

Instructing the patient

⚠ Inform your patient that he or she must not smoke or consume any tea, coffee, fruit or vegetable juices or other colour-imparting food during the first 2 to 3 hours after the treatment.

Further information on use and safety information

For safety reasons, safety goggles must be worn by user and patient. We suggest that mouth protection is worn. PROPHYpearls must not be used on patients who suffer from diseases of the respiratory tract. Patients with contact lenses should wear additional eye protection. In extreme isolated cases, particularly with the presence of pathological periodontal pockets (> 6 mm), mucosal lesions and extremely long direct exposure of the soft tissue, emphysema may occur. Root cement, demineralized enamel, fillings and filling margins should not be directly blasted. The cleaning should be performed here only indirectly via the rolling effect. We recommend using mouth protection during the work. All parts and products with sensitive surfaces should be removed from the treatment area. Metallic surfaces may appear matt after treatment. This may be rectified by using tooth polish.

Main constituent: Calcium carbonate, CAS no. 471 - 34 - 1

The additional information in the operating instructions for KaVo PROPHYflex 2 and KaVo PROPHYflex 3 must be followed.

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Kavo Dental GmbH D-89000 Ulm, Germany