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Tel : (6674) 552654-6 FAX : (6674) 552652

E-Mail : info@hycare-int.com

INSTRUCTION FOR USE FOR MEDICAL DEVICE:

Medical diagnostic (examination) disposable non-sterile gloves "BENOVY"

Managing director

(signature)

HYCARE
INTERNATIONAL CO., LTD.

/Mr. Werner Gostner/

No. 002665

14 JAN 2020

SONGKHLA

Certified Genuine Signature(s) of

WERNER GOSTNER

2019

(MRS. KAMOLWAN CHANDAKUL)

General Service Officer

Experienced Level

Ministry of Foreign Affairs of Thailand

(The Ministry of Foreign Affairs assumes no
responsibility for the contents of the documents)
3180990

TABLE OF CONTENTS

1.1 Name of the medical device	3
1.2 Manufacturer (producer) of the medical device	4
1.3 Purpose of the medical device	4
1.4 Scope of the medical device	4
1.5 Intended users of the medical device	4
1.6 Application conditions of the medical device	4
1.7 Indications for use of the medical device	5
1.8 Contradictions for use of the medical device	5
1.9 Operating principle of the medical device	5
1.10 Instruction for use of the medical device	6
1.11 Warning and precautions	7
1.12 Shelf life of the medical device	7
2 Transportation, storage and disposal conditions of the medical device	8
2.1 Transportation conditions of the medical device	8
2.2 Storage conditions of the medical device	8
2.3 Disposal conditions of the medical device	8
3 Cleaning and disinfection methods of the medical device	8
4 Specification of the medical device	9
4.1 Description of the medical device	9
4.2 Physical specification of the medical device	11
5 Adverse reactions	12
6 Circumstances, in which the consumer must consult healthcare professional	12
7 Manufacturer's warranties	12
8 Data on Authorized representative of the manufacturer (producer)	12

1 Data on the medical device

1.1 Name of the medical device

Medical diagnostic (examination) disposable non-sterile gloves "BENOVY", in hardware versions:

1. Medical diagnostic (examination) disposable non-sterile gloves "BENOVY" nitrile finger textured powder free chlorinated (beaded cuff), color: white, black, blue, violet blue, green, pink, size: XS, S, M, L, XL. Series: 35-1C. Type 185830.
2. Medical diagnostic (examination) disposable non-sterile gloves "BENOVY" nitrile finger textured powder free chlorinated (beaded cuff), color: white, black, blue, size: XS, S, M, L, XL. Series: 4-1C. Type 185830.
3. Medical diagnostic (examination) disposable gloves "BENOVY" non-sterile nitrile finger textured powder free chlorinated (beaded cuff), color: white, black, blue, size: XS, S, M, L, XL. Series: 5-1C. Type 185830.
4. Medical diagnostic (examination) disposable non-sterile gloves "BENOVY" nitrile finger textured powder free chlorinated (extended beaded cuff), color: white, blue, size: XS, S, M, L, XL. Series: 6-1C. Type 185830.
5. Medical diagnostic (examination) disposable non-sterile gloves "BENOVY" nitrile with full-surface prominent diamond texture for better grip powder free chlorinated (beaded cuff), color: orange, black, light green, blue, size: XS, S, M, L, XL. Series: 65-1C. Type 185830.
6. Medical diagnostic (examination) disposable non-sterile gloves "BENOVY" nitrile with full-surface prominent diamond texture for better grip High Density powder free chlorinated (beaded cuff), color: orange, black, light green, blue, size: XS, S, M, L, XL. Series: 83-1C. Type 185830.

1.2 Manufacturer (producer) of the medical device

Manufacturer (producer) of the medical device:

Hycare International Co., Ltd.

1197, Moo 3, Asia Highway, Khuanlang, Hat Yai Songkhla, 90110, Thailand

Address of the medical device manufacture:

1197, Moo 3, Asia Highway, Khuanlang, Hat Yai Songkhla, 90110, Thailand

1.3 Purpose of the medical device

The gloves are intended for single use as protective barrier on healthcare professional's hands during examination/treatment of patient or for other medical goals.

1.4 Scope of the medical device

Gloves are used during medical researches, diagnostic, therapeutic, dental and treatment procedures, sanitation and laboratory researches, as well as handling contaminated medical materials/devices/surfaces, providing hand hygiene, as also protection of patient and healthcare professional from cross-contamination.

1.5 Intended users of the medical device

Medical personnel, who carry out medical researches, diagnostic, therapeutic, dental and treatment procedures, sanitation and laboratory researches, as well as handling contaminated medical materials/devices/surfaces and other activities, which require compliance with the requirements for protection of patient/healthcare professional from cross-infection.

1.6 Application conditions of the medical device

Medical examination disposable gloves are used in medical or healthcare facilities indoors in a dry place, away from direct sunlight, at temperature from 0°C to 25°C and relative humidity from 65% up to 85% in accordance with the instruction for use on medical device.

Gloves for external use only (the device is non-invasive).

1.7 Indications for use of the medical device

The indications for use of the medical device are the necessity of conduction of medical researches, diagnostic, therapeutic, dental and treatment procedures, sanitation and laboratory researches, as well as handling contaminated medical materials/devices/surfaces.

1.8 Contradictions for use of the medical device

The presence of an allergic reaction to residual process chemicals in the medical device in a patient or a healthcare professional.

1.9 Operating principle of the medical device

The basis of nitrile examination disposable gloves limits movement of infectious substances, increases the efficacy of antimicrobial protection in medical facility, protecting the healthcare professional against possible infection during examination/treatment of the patient and other sanitation activities, as well as contributes to reliable capture of medical instruments and increases safety of the work.

1.10 Instruction for use of the medical device

Preliminary hygienic treatment of your hands is required before use of the gloves (treat hands with skin antiseptic or wash with soap and water). Gloves are put on dry hands only.

How to put the gloves on:

- 1) Take a glove from the box. It is recommended to touch the bounded surface of the glove corresponding to the wrist (upper edge of the cuff).
- 2) Pull on the first glove.
- 3) Take the second glove with your bare hand, touching the bounded surface of the glove corresponding to the wrist.
- 4) To avoid touching the skin of the forearm with a gloved hand, rotate the outer surface of the glove to be put on with folded fingers in the glove so that you could pull the glove over the other hand.
- 5) Having put the gloves on, it is required to touch only those objects provided for by the indications for use and conditions of the use of the gloves.
- 6) To avoid contamination, it is required to change gloves before proceeding with the new procedure.

The used gloves should be put off carefully, avoiding splashing from the glove's surfaces where microbial contamination of hands and environment is possible.

How to take the gloves off:

- 1) To take a glove off, pick it up at the level of the wrist without touching the skin of the forearm and pull it off your hand, turning it inside out.
- 2) Holding the removed glove with the gloved hand, it is necessary to place the fingers of your bare hand between the glove and wrist. Remove the second glove by rolling it down the hand and put into the first glove.
- 3) Discard the gloves and wash your hands thoroughly.

1.11 Warning and precautions

- The device is intended for single use;
- For individual use only;
- Every time it is necessary to perform hygienic treatment of your hands before putting on or changing gloves, as well as after removing the gloves;
- It is necessary to choose the gloves of the right size – they should be easily put on and should not impede movement or squeeze the hands. Do not use in case of size mismatch;
- It is necessary to choose the gloves depending on the type of the performed manipulations;
- Touching is permissible only to the bounded surface of the glove corresponding to the wrist (upper edge of the cuff);
- It is necessary to change the gloves in case of any defect, puncture, damage, contamination, fluid contact with skin under the glove, as well as stickiness. If one glove is damaged, change both ones;
- Depending on the type of the performed manipulation, to keep an optimal level of protection, it is recommended to change gloves every 60-120 minutes;
- Do not wet and keep away from moisture;
- It is necessary to put the gloves on dry hands only;
- Avoid exposure to direct sunlight;
- Gloves contain residual process chemicals that may cause an allergic reaction;
- Gloves do not guarantee 100% hand protection against punctures, cuts and damage caused by exposure to chemicals.

1.12 Shelf life of the medical device

The established shelf life of the examination disposable non-sterile nitrile gloves is 5 years.

2 Transportation, storage and disposal conditions of the medical device

2.1 Transportation conditions of the medical device

The transportation of the device may be made by all means of transport (sea, land, railway or air transport) according to the rules of transportation, which are applied to such means of transport. During transportation, the storage conditions should not be violated.

2.2 Storage conditions of the medical device

Devices should be stored in a dry place protected from direct sunlight, heat, ignition and ultraviolet sources, as well as ozone, at temperature from 0°C to 25°C and relative humidity – from 65% up to 85%.

Products should be handled with caution when they are stored at temperatures below 15°C, as during careless handling, they can be stiff and sensitive to deformation. After storage at a low temperature, gloves should be warmed up to 30°C before use throughout all product volume.

2.3 Disposal conditions of the medical device

Disposal of the medical device "Medical diagnostic (examination) disposable non-sterile gloves BENOVIY" in the Russian Federation is carried out by the medical organization as class B waste in accordance with SanPiN 2.1.7.2790-10 "Sanitary-epidemiological requirements for medical waste management" and approved guideline of the organization.

3 Cleaning and disinfection methods of the medical device

Examination gloves are disposable medical device. Reuse of the medical device is a gross violation of application methods.

4 Specification of the medical device

4.1 Description of the medical device

Form of the medical device. Medical examination gloves have straight fingers and are ambidextrous (non-anatomic). The glove's cuff may be extended or has a standard length. Rolled cuff.

Surface of the medical device. Medical gloves have a powder free surface. Powder free gloves are treated with chlorine – chlorinated.

Texture of the outer surface of the medical device. Gloves may have textured surface on fingers or full-surface prominent diamond texture.

Sizes of the medical device. Depending on the size, table 1 gives the values of width, length and thickness of the medical device.

Table 1 – Sizes of the medical device:

Value name	Size				
	XS	S	M	L	XL
Width W (figure 1), mm	≤ 80	80±10	95±10	110±10	≥110
Length l (figure 1), mm, min	220		230		
Thickness (in points, shown on figure 2), mm, min: - smooth surface - textured surface	0.08 0.11				
Thickness of rolled cuff, mm, max	2.50				

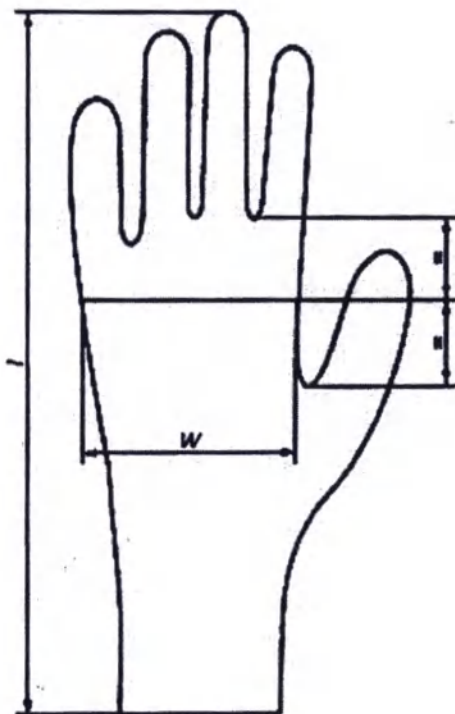
Figure 1 – Points for width and length measurement of the glove

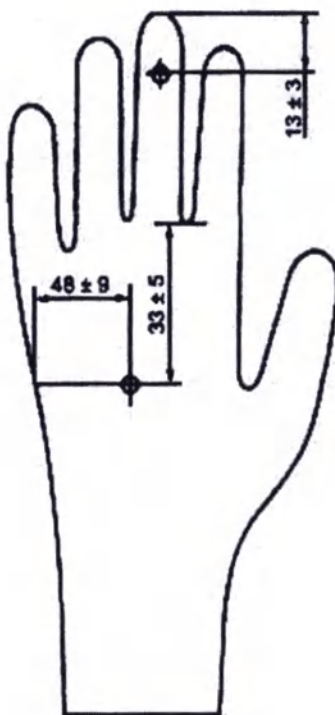
Figure 2 – Points for wall thickness measurement of the glove

4.2 Physical specification of the medical device

4.2.1 Physical and mechanical performance of the medical device

Physical and mechanical parameters of the medical device are represented in tables 2 and 3.

Table 2 – Values of physical and mechanical parameters of the medical device before accelerated aging:

Value of physical and mechanical parameters before accelerated aging		
Elongation at break, %, min	Force at break, N, min	Nominal strength, MPa, min
500	7.0	14

Table 3 – Values of physical and mechanical parameters of the medical device after accelerated aging:

Value of physical and mechanical parameters after accelerated aging		
Elongation at break, %, min	Force at break, N, min	Nominal strength, MPa, min
400	6.0	14

4.2.2. Inspection levels and acceptable quality level of the medical device

For the purposes of quality control medical gloves are selected and examined in accordance with standard ISO 2859-1.

Hermeticity. Medical gloves are hermetic. Acceptable quality level (AQL) is ≤ 1.5 .

Powder. For medical examination nitrile powder free gloves, the content of residual powder is determined in accordance with the requirements of ASTM D 6124. The value obtained does not exceed 2 mg per glove.

5 Adverse reactions

Manifestation of dermal hypersensitivity reactions, when using the device is possible.

6 Circumstances, in which the consumer must consult healthcare professional

The patient is recommended to consult healthcare professional in the event of dermal hypersensitivity reactions, when using the device.

7 Manufacturer's warranties

The manufacturer (producer) guarantees preservation of all the parameters and characteristics of the medical device taking into account acceptable quality level (AQL) ≤ 1.5 , as well as the quality of the medical gloves within all shelf life (5 years) from the date of manufacture upon the condition of package integrity, observance of transportation, storage and operation conditions in accordance with the instruction.

8 Data on Authorized representative of the manufacturer (producer)

Please contact the Authorized representative of the manufacturer for any and all purposes, related to circulation of the medical device on the territory of the Russian Federation:

"BENOVY" Company Limited ("BENOVY" Co., Ltd.)

Russia, 194292, St. Petersburg, 1, 5-y Verkhniy pereulok, litera J, office 205

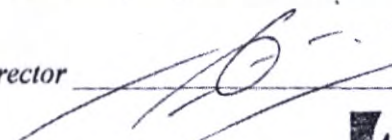
Tel.: +7 (812) 3091632

www.benovy-company.ru

e-mail: benovy@benovy-company.ru

Initial issue of operational documentation (instruction for use for medical device "Medical diagnostic (examination) disposable non-sterile gloves "BENOVY")": December 2019

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