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14th of January, 2021
"Approved"

WORLD WORK

S.R.L.

Via del Progresso, 42/48 (signature)

36054 MONTEBELLUNA VICENTINO (VI)

Partita IVA 0167940364

Tel. +39 (0)444/574297 - Fax +39 (0)444/370543

CEO

WORLD WORK S.r.l.

**INSTRUCTION FOR USE
FOR THE MEDICAL DEVICE**

CLEAN+SAFE® Dental Aspiration Devices

Revision 1

January 2021

Contents

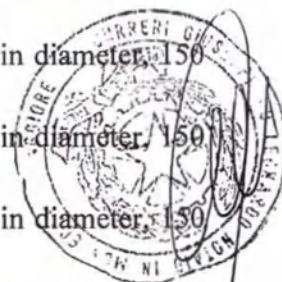
1. Name of the device	1. N: CLE
2. Information about manufacturer, developer and authorized representative of manufacturer.....	Ver
3. Intended use and potential user of the device	1.
4. Description of the device	2.
5. Indications, contraindications, and possible side effects	
6. Safety precautions	3
7. Usage procedure.....	4
8. Packaging.....	
9. Description of symbols used on package labeling	
10. Conditions of storage, transportation and operation	
11. Maintenance and repair	8
12. Disinfection	8
13. Disposal	8
14. Shelf life.....	9
15. List of international standards to which the medical device complies.....	9
16. Manufacturer's warranty	9

.. Name of the device

CLEAN+SAFE® Dental Aspiration Devices.

Versions:

1. CLEAN+SAFE® Dental Aspiration Devices: cannula with fixed hood, 6.5 mm in diameter, 130 mm in length, green.
2. CLEAN+SAFE® Dental Aspiration Devices: cannula with fixed hood, 6.5 mm in diameter, 130 mm in length, white.
3. CLEAN+SAFE® Dental Aspiration Devices: cannula with fixed hood, 6.5 mm in diameter, 130 mm in length, lilac.
4. CLEAN+SAFE® Dental Aspiration Devices: cannula with fixed hood, 6.5 mm in diameter, 130 mm in length, orange.
5. CLEAN+SAFE® Dental Aspiration Devices: cannula with fixed hood, 6.5 mm in diameter, 130 mm in length, lime.
6. CLEAN+SAFE® Dental Aspiration Devices: cannula with fixed hood, 6.5 mm in diameter, 130 mm in length, clear.
7. CLEAN+SAFE® Dental Aspiration Devices: cannula with fixed hood, 6.5 mm in diameter, 130 mm in length, blue.
8. CLEAN+SAFE® Dental Aspiration Devices: cannula with fixed hood, 6.5 mm in diameter, 130 mm in length, yellow.
9. CLEAN+SAFE® Dental Aspiration Devices: cannula with fixed hood, 6.5 mm in diameter, 130 mm in length, fuchsia.
10. CLEAN+SAFE® Dental Aspiration Devices: cannula with fixed hood, 6.5 mm in diameter, 130 mm in length, light blue.
11. CLEAN+SAFE® Dental Aspiration Devices: cannula with fixed hood, 6.5 mm in diameter, 130 mm in length, red.
12. CLEAN+SAFE® Dental Aspiration Devices: cannula with fixed hood, 6.5 mm in diameter, 130 mm in length, pink.
13. CLEAN+SAFE® Dental Aspiration Devices: cannula with fixed hood, 6.5 mm in diameter, 150 mm in length, green.
14. CLEAN+SAFE® Dental Aspiration Devices: cannula with fixed hood, 6.5 mm in diameter, 150 mm in length, white.
15. CLEAN+SAFE® Dental Aspiration Devices: cannula with fixed hood, 6.5 mm in diameter, 150 mm in length, lilac.
16. CLEAN+SAFE® Dental Aspiration Devices: cannula with fixed hood, 6.5 mm in diameter, 150 mm in length, orange.
17. CLEAN+SAFE® Dental Aspiration Devices: cannula with fixed hood, 6.5 mm in diameter, 150 mm in length, lime.
18. CLEAN+SAFE® Dental Aspiration Devices: cannula with fixed hood, 6.5 mm in diameter, 150 mm in length, clear.
19. CLEAN+SAFE® Dental Aspiration Devices: cannula with fixed hood, 6.5 mm in diameter, 150 mm in length, blue.
20. CLEAN+SAFE® Dental Aspiration Devices: cannula with fixed hood, 6.5 mm in diameter, 150 mm in length, yellow.

WORLDWORK
S.R.L.
Via del Progresso, 47/49
36054 MONTEBELLO VICENTINO (VI)
Partita IVA 01678460244
Tel. +39 (0)444/574287

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Instruction for use for the medical device:

21. CLEAN+SAFE® Dental Aspiration Devices: cannula with fixed hood, 6.5 mm in diameter, 150 mm in length, fuchsia.
22. CLEAN+SAFE® Dental Aspiration Devices: cannula with fixed hood, 6.5 mm in diameter, 150 mm in length, light blue.
23. CLEAN+SAFE® Dental Aspiration Devices: cannula with fixed hood, 6.5 mm in diameter, 150 mm in length, red.
24. CLEAN+SAFE® Dental Aspiration Devices: cannula with fixed hood, 6.5 mm in diameter, 150 mm in length, pink.
25. CLEAN+SAFE® Dental Aspiration Devices: cannula with removable hood, 6.5 mm in diameter, 130 mm in length, green.
26. CLEAN+SAFE® Dental Aspiration Devices: cannula with removable hood, 6.5 mm in diameter, 130 mm in length, white.
27. CLEAN+SAFE® Dental Aspiration Devices: cannula with removable hood, 6.5 mm in diameter, 130 mm in length, lilac.
28. CLEAN+SAFE® Dental Aspiration Devices: cannula with removable hood, 6.5 mm in diameter, 130 mm in length, orange.
29. CLEAN+SAFE® Dental Aspiration Devices: cannula with removable hood, 6.5 mm in diameter, 130 mm in length, lime.
30. CLEAN+SAFE® Dental Aspiration Devices: cannula with removable hood, 6.5 mm in diameter, 130 mm in length, clear.
31. CLEAN+SAFE® Dental Aspiration Devices: cannula with removable hood, 6.5 mm in diameter, 130 mm in length, blue.
32. CLEAN+SAFE® Dental Aspiration Devices: cannula with removable hood, 6.5 mm in diameter, 130 mm in length, yellow.
33. CLEAN+SAFE® Dental Aspiration Devices: cannula with removable hood, 6.5 mm in diameter, 130 mm in length, fuchsia.
34. CLEAN+SAFE® Dental Aspiration Devices: cannula with removable hood, 6.5 mm in diameter, 130 mm in length, light blue.
35. CLEAN+SAFE® Dental Aspiration Devices: cannula with removable hood, 6.5 mm in diameter, 130 mm in length, red.
36. CLEAN+SAFE® Dental Aspiration Devices: cannula with removable hood, 6.5 mm in diameter, 130 mm in length, pink.
37. CLEAN+SAFE® Dental Aspiration Devices: cannula with removable hood, 6.5 mm in diameter, 150 mm in length, green.
38. CLEAN+SAFE® Dental Aspiration Devices: cannula with removable hood, 6.5 mm in diameter, 150 mm in length, white.
39. CLEAN+SAFE® Dental Aspiration Devices: cannula with removable hood, 6.5 mm in diameter, 150 mm in length, lilac.
40. CLEAN+SAFE® Dental Aspiration Devices: cannula with removable hood, 6.5 mm in diameter, 150 mm in length, orange.
41. CLEAN+SAFE® Dental Aspiration Devices: cannula with removable hood, 6.5 mm in diameter, 150 mm in length, lime.
42. CLEAN+SAFE® Dental Aspiration Devices: cannula with removable hood, 6.5 mm in diameter, 150 mm in length, clear.
43. CLEAN+SAFE® Dental Aspiration Devices: cannula with removable hood, 6.5 mm in diameter, 150 mm in length, blue.
44. CLEAN+SAFE® Dental Aspiration Devices: cannula with removable hood, 6.5 mm in diameter, 150 mm in length, yellow.

45. CLEAN+SAFE® Dental Aspiration Devices: cannula with removable hood, 6.5 mm in diameter, 150 mm in length, fuchsia.
46. CLEAN+SAFE® Dental Aspiration Devices: cannula with removable hood, 6.5 mm in diameter, 150 mm in length, light blue.
47. CLEAN+SAFE® Dental Aspiration Devices: cannula with removable hood, 6.5 mm in diameter, 150 mm in length, red.
48. CLEAN+SAFE® Dental Aspiration Devices: cannula with removable hood, 6.5 mm in diameter, 150 mm in length, pink.

Hereinafter referred to as cannula, medical device, device, product.

2. Information about manufacturer, developer and authorized representative of manufacturer

Manufacturer:

WORLD WORK S.r.l.

Via del Progresso, 47-36054 Montebello Vicentino (VI), Italy

Tel.: +39 0444-574297

Fax: +39 0444-370543

E-mail: info@worldwork.it

Website: www.worldwork.it

Developer:

WORLD WORK S.r.l.

Via del Progresso, 47-36054 Montebello Vicentino (VI), Italy

Tel.: +39 0444-574297

Fax: +39 0444-370543

E-mail: info@worldwork.it

Website: www.worldwork.it

Authorized representative of the manufacturer in the Russian Federation

PROTECO Joint Stock Company (PROTECO JSC)

192102, St. Petersburg, Salova str. 57, build. 1, lit. I.

Tel./fax: +7 (812) 635-88-90

E-mail: info@protecodent.ru

Website: www.protecodent.ru

3. Intended use and potential user of the device

Intended use: Intended for removal (Aspiration) of fluids from the patient's oral cavity during dental procedures.

Scope of application: Dental offices in various medical institutions, private dental offices and dental clinics (including private ones).

Potential users: Intended for use by qualified healthcare professionals during various dental procedures as intended by the manufacturer.

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4. Description of the device

Package

The device is a tube reinforced with metal wire to change and retain the desired shape. The unit has a built-in tip and a streamlined removable tip with flexible slits (for filtration of fluids inflowing into the oral cavity) that exclude the possibility of irritation of the oral mucosa, prevent injury to the oral mucosa, and facilitate the comfortable removal of the saliva. This ensures successful removal of the risk of (Aspiration) of fluids from any areas of the patient's oral cavity including those that are particularly difficult to access.

A cannula can be attached to the Aspiration equipment by inserting it into a tapered connection of the aspiration tubing. A cannula can be attached to the tubing of various Aspiration systems duly authorized for use in accordance with the operating instructions for a specific system.

5. Indications, contraindications, and possible side effects

Indications: The device is used exclusively for removal of fluids from the patient's oral cavity during various dental procedures.

Contraindications for use: Individual incompatibility with the raw material components of the device. If irritation occurs, stop using the device and seek medical advice.

Adverse reactions: Irritation or allergic reaction due to the consumer's individual intolerance.

6. Safety precautions

- The contraindication for use of the device is the individual incompatibility with the device materials;
- Read the Instructions before use;
- Use a cannula only as intended (see "Scope of application");
- Do not use a defective cannula if its package was damaged before the initial opening.
- Store a new or unused cannula in a dry, clean, and secure place;
- After opening, close the package in a way to prevent contamination of the product;
- A cannula is for single use only! Do not reuse. Reuse can result in an infection or contamination with biological material transferred from one patient to another.
- Do not use cannula with expired shelf life.

7. Usage procedure

1. Before use, gently pull the cannula tip to make sure that it is tightly secured on the cannula.
2. Give the cannula the desired shape by bending the tube at the required angle to adapt it to the patient's oral cavity.
3. Attach the cannula end opposite to the tip to the aspiration system of a dental unit duly registered in the Russian Federation.
4. Place the end with the tip into the patient's oral cavity for its operation.
5. To attach a cannula to the aspiration system tubing 11 mm in diameter, it is necessary to additionally purchase an adapter for Aspiration cannulas with $d = 11$ mm that is duly registered in the Russian Federation by any manufacturer.

Packaging

Cannulas (100 pcs.) are packed into a multiple consumer package (polyethylene film bag). The uniform sealing along the two sides of the bag ensures the packaging integrity during storage and transportation. The material and design of the packaging do not have any negative effects on the contents, ensure the device integrity if stored in dry, clean, and well-ventilated places, minimize the risk of contamination after opening, and protect the contents if transported and stored under normal conditions.

9. Description of symbols used on package labeling

Table 1. Symbols and definitions

Symbol	Description
	Date of manufacture of a medical device
	Manufacturer of a medical device
	Use-by date
	Catalog number of a manufacturer intended for identification of a device
	LOT number of a manufacturer intended for identification of a certain device
	CE mark indicating that a device complies with the requirements of applicable regulations of European Union
	Do not use if package is damaged
	Do not reuse
	Keep away from sunlight
	Low density (high pressure) polyethylene
	Non-toxic material
	Keep dry
	Storage temperature limits
	Non-sterile
	Consult instructions for use

WORK

Via del Progresso, 47/49 S.R.L.

36054 MONTEBELLO VICENTINO (VI)

Partita IVA 01572560247

Tel. +39 (0)444/574297 - Fax +39 (0)444/370543

Instruction for use for the medical device:

10. Conditions of storage, transportation and operation

Devices packed in shipping box should be transported by all types of covered vehicles according to applicable regulations for the carriage of goods by this type of transport, observing precautions indicated on the shipping box.

Transportation conditions:

- Ambient temperature: 5 to +50 °C;
- Relative humidity: 0-100 %.

The device should be stored in a dry, well-ventilated place protected from light and sunrays. Keep away from heat, flammable substances, and/or substances (chemicals) that may affect the safety of the device.

Storage conditions:

- Ambient temperature: 5 to +50 °C;
- Relative humidity: 0-100 %.

Operation conditions:

- Ambient temperature: 5 to +50 °C;
- Relative humidity: 0-100 %.

11. Maintenance and repair

The device is disposable. Maintenance and repair are not required.

12. Disinfection

The medical devices CLEAN+SAFE® Dental Aspiration Devices do not come into contact with the wound surface, blood, or injectable preparations and do not cause damage during operation.

This device is supplied non-sterile but clean, does not require any pre-treatment, and is ready for single use. These disposable devices are subject to decontamination/deactivation (disinfection) and further disposal after being used on patients during dental procedures. Do not reuse.

After use, CLEAN+SAFE® Dental Aspiration Devices should be placed into a special container with a disinfectant solution with the concentration and for the exposure time prescribed by the manufacturer. It is necessary to use only the disinfectants and detergents that are duly registered in the Russian Federation and that are suitable for use with polyvinylchloride and polyethylene and to follow the directions concerning the concentration, temperature, and exposure time prescribed by the manufacturer of disinfectants and detergents. Following the treatment with a disinfectant, the device is disposed of in accordance with the established procedure.

13. Disposal

Medical device should be disposed of according to medical procedures established in healthcare facilities and according to local, state, or federal regulations. Selection of methods for safe disposal and destruction depends on the capacity and profile of the medical institution, availability of waste disinfection/disposal facilities, strategy of waste neutralization/disposal adopted in certain administrative area (incineration, landfilling, recycling).

The medical devices CLEAN+SAFE® Dental Aspiration Devices are considered hazardous waste of epidemiologic potential (devices contaminated with blood and/or other biological fluids of patients and devices that come into contact with infected patients) after their use as intended.

In this case, the medical organization carries out only collection, decontamination, and temporary storage of the abovementioned waste. Decontamination should be performed in accordance with the country-specific sanitary guidelines and other normative legal acts for all medical institutions regardless of their specialization. Collection is carried out using bags attached to a trolley or disposable, color-coded containers for hazardous medical waste indoors or in places for temporary storage of medical waste before transporting for destruction or disposal.

Specialized organizations licensed to handle hazardous medical waste of epidemiologic potential transport medical waste from medical institutions and recycle (dispose of) and destroy it.

Consumer packaging, defective and unused products with expired shelf life are classified as epidemiologically safe wastes (non-hazardous waste that does not come into contact with biological fluids of patients or infected patients), similar to municipal solid waste (MSW).

Non-hazardous waste is collected using reusable containers or disposable bags (excluding color-coded bags for hazardous waste). Disposable bags are placed inside reusable containers. Waste containers should be marked and placed at a special site (indoors). After emptying, reusable containers are subject to washing and disinfection. The procedure for washing and disinfection of reusable containers is determined in accordance with the organization-specific waste management flowchart.

Transportation and disposal are performed considering the requirements established for a specific territory in accordance with the country-specific sanitary laws concerning the maintenance of populated areas and handling of industrial and household waste.

14. Shelf life

The shelf life of a cannula is 5 years from the date of manufacture provided that the storage and transportation conditions are observed.

15. List of international standards to which the medical device complies

Directive 93/42/EEC, UNI IEC EN ISO 14971, UNI CEI EN ISO 15223-1, EN ISO 10993-1, UNI CEI EN 1041.

16. Manufacturer's warranty

The manufacturer guarantees the compliance of the medical device with the requirements that have impact on the quality, effectiveness, safety, and intended use provided that the user observes the operation, transportation, and storage conditions.

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