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Do not fold the needle before and after use.

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For professional use only in the practice of dentistry.

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Needles^{DHT}

Single use needles. Do not sterilize or reuse.
Store in original packaging until ready to use.
In order to benefit from all of the advantages of the DHT needles, proceed as indicated below.
For maximum performance, the use of a syringe allowing the use of a pen grip (support points) and an electronic injection is highly recommended.

Be sure to correctly orientate the needle indicator for this type of anesthesia:

ANESTHESIA OF THE ATTACHED MUCOSA:

For a pain free penetration even with children, orientate the indicator opposite the mucosa and incline the needle so that it is almost parallel to the mucosa (1). While injecting very slowly, delicately penetrate the bevel by less than a millimeter.

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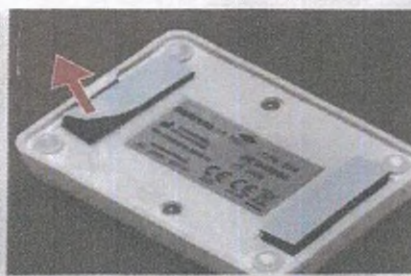
Ref. SA0226

Adhesive Strips


Adhesive strips to fix your DHT electronic control box.
Fit the adhesive strips as indicated. It is recommended to wait a minimum of 2 hours before unclipping the unit in order to prevent the adhesives from becoming unstuck. The fixation will be optimal after 12 hours. This fixation system allows you to clip/unclip the control box at will.

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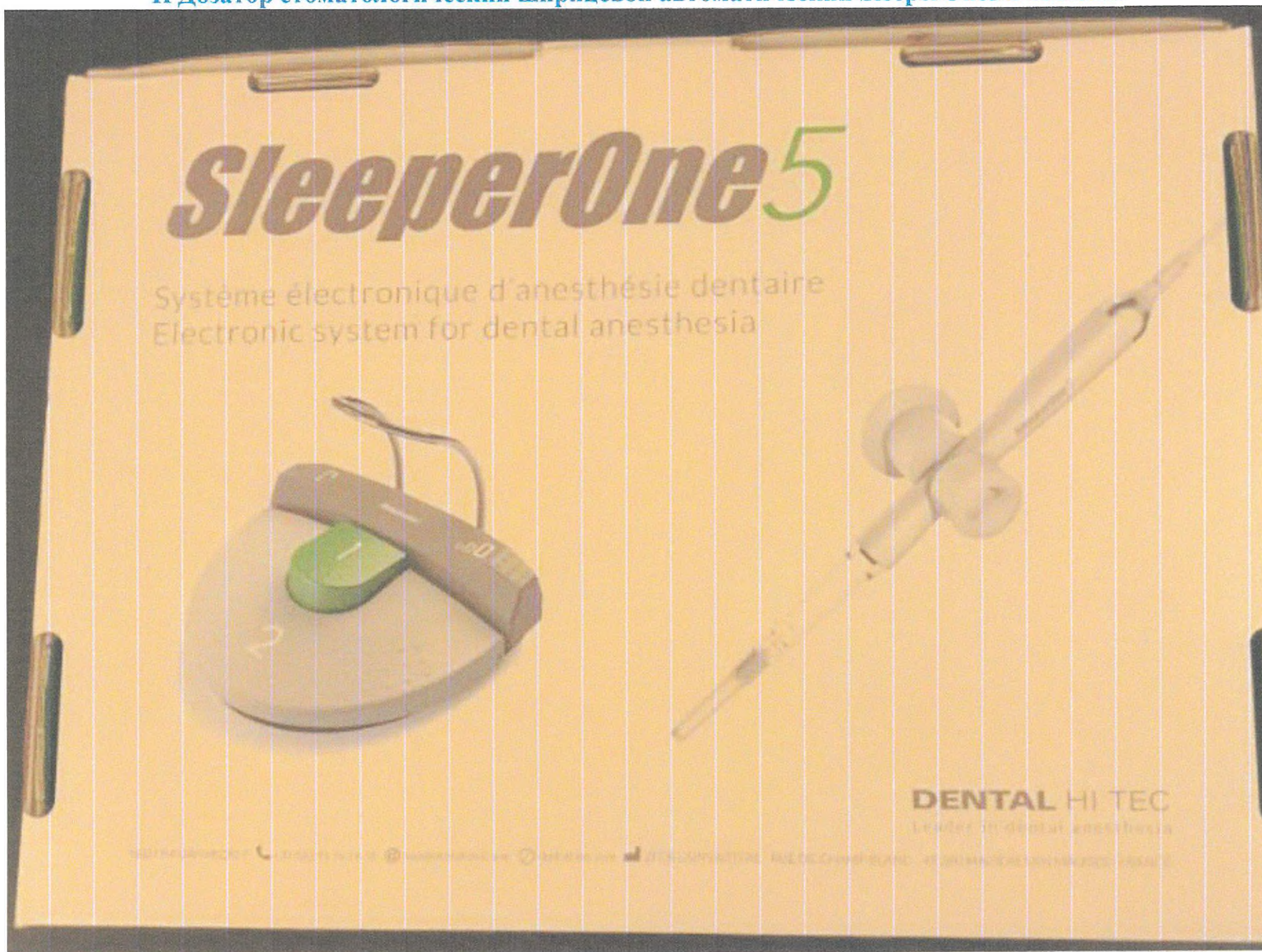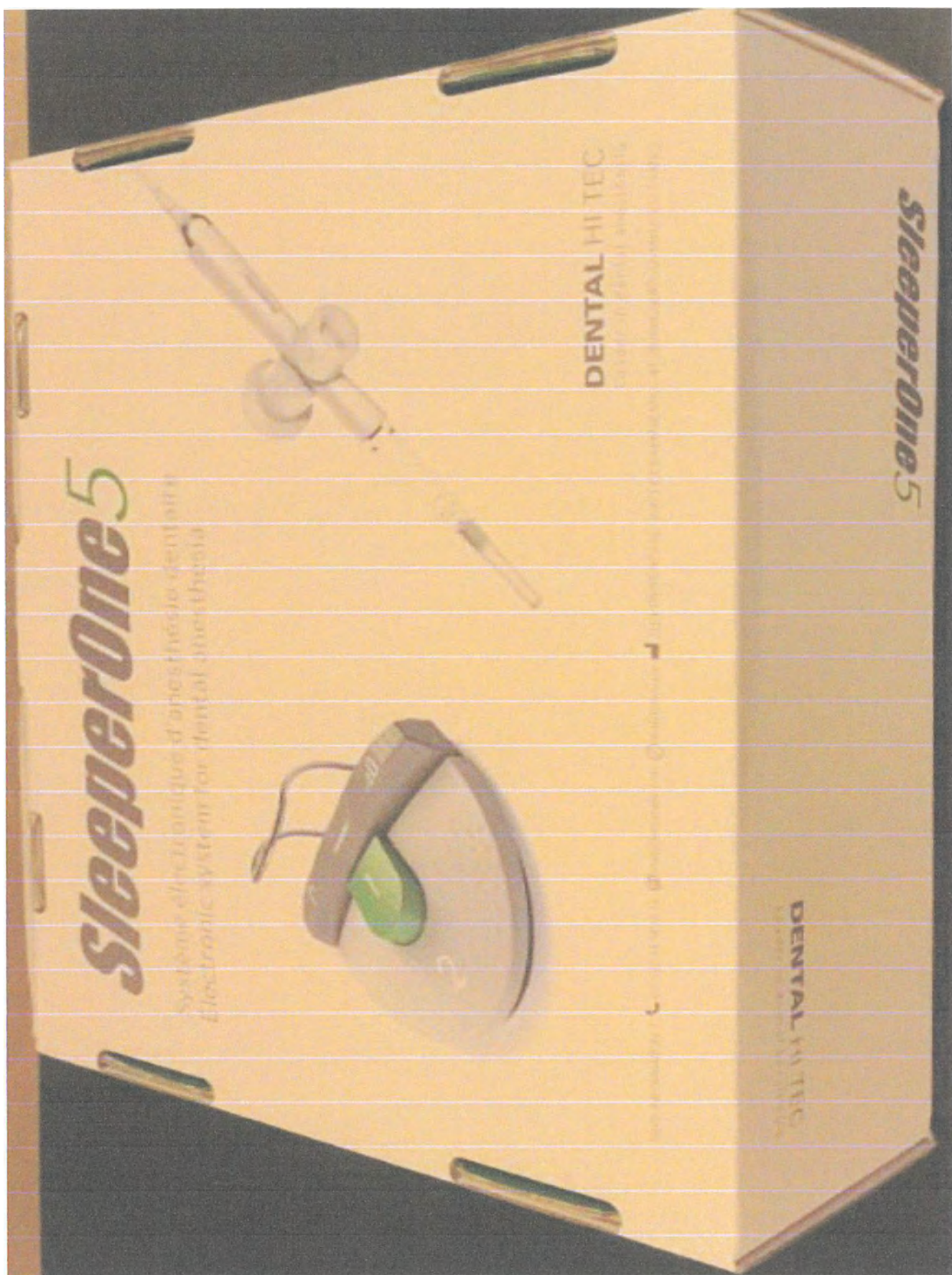
REF 55.1B SN 55B00878
2018

Altitude < 3000 m
15% 30% 40%
1000HP 2000HP 3000HP

CONDITIONS DE STOCKAGE
STORAGE CONDITIONS

10% 15% 20% 30% 40%
1000HP 2000HP 3000HP

CONDITIONS DE TRANSPORT
TRANSPORT CONDITIONS

- 1 Accompanying documentation
- 1 Set of disposables is available upon the country
- 1 Power supply cord
- 1 Power supply
- 1 Option key
- 1 Set of sterilizable accessories
- 1 Handpiece holder
- 1 Pedal hoop
- 1 Wireless foot pedal
- 1 Handpiece and control box

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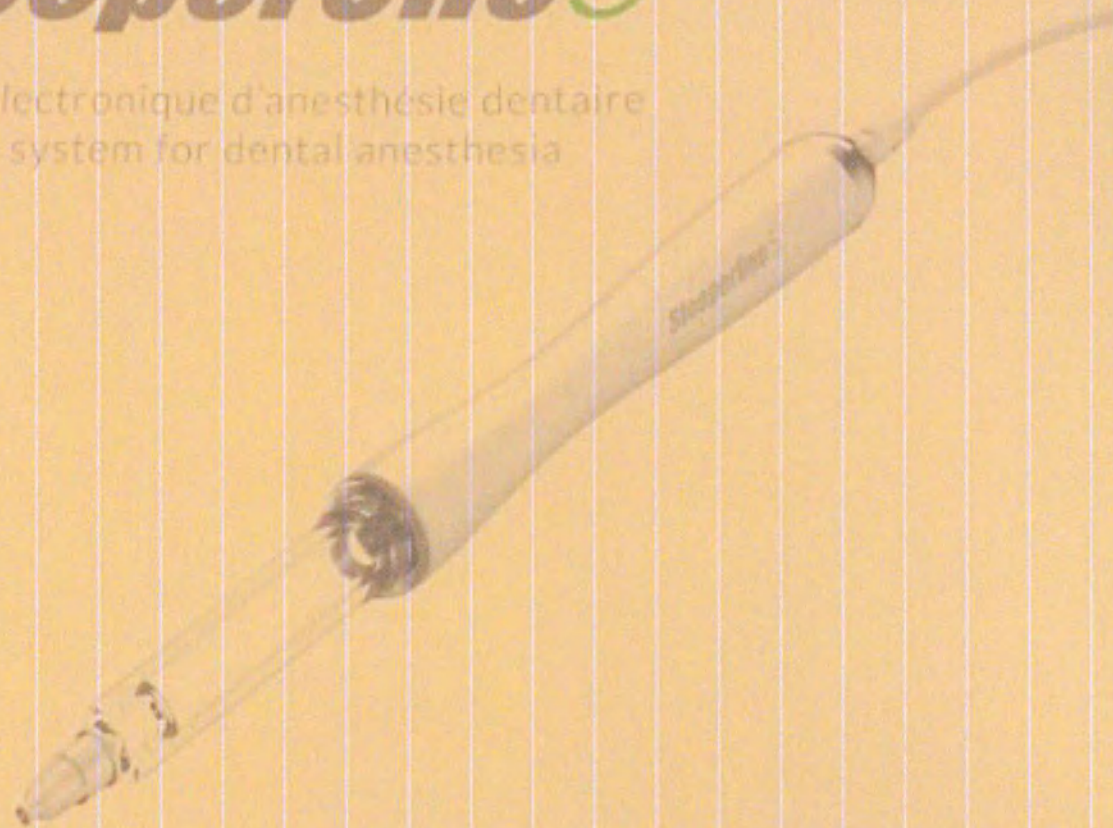
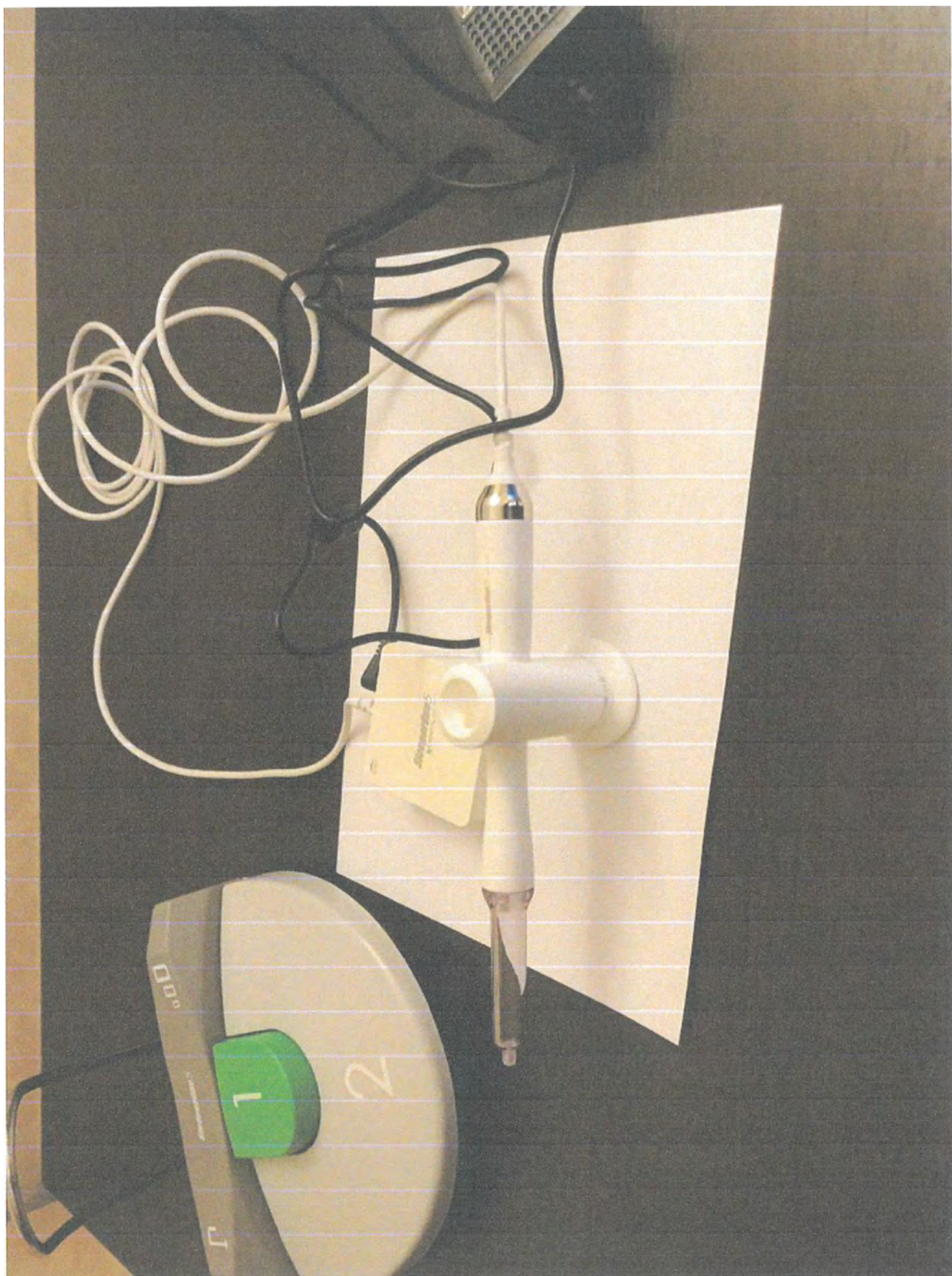
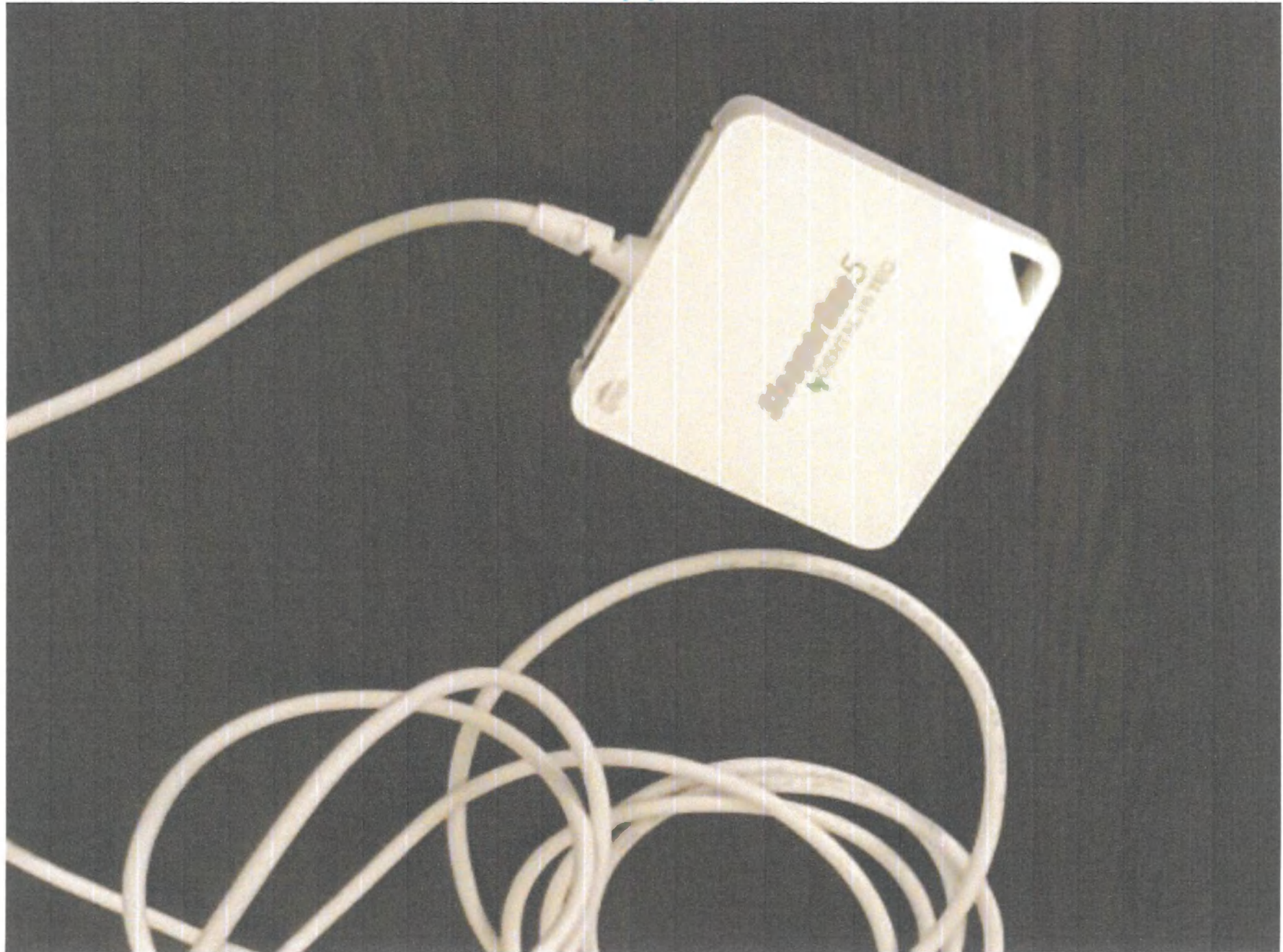
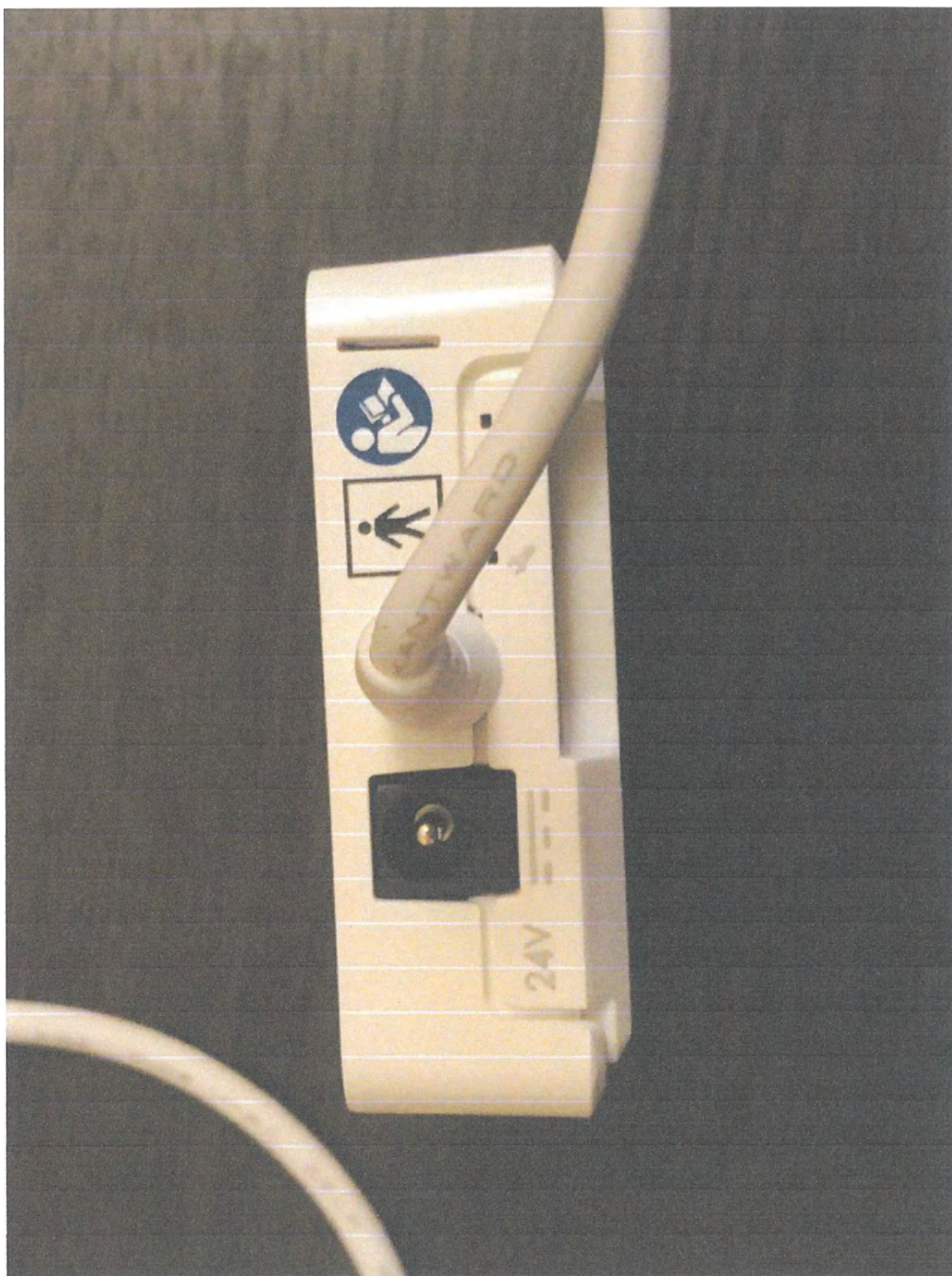
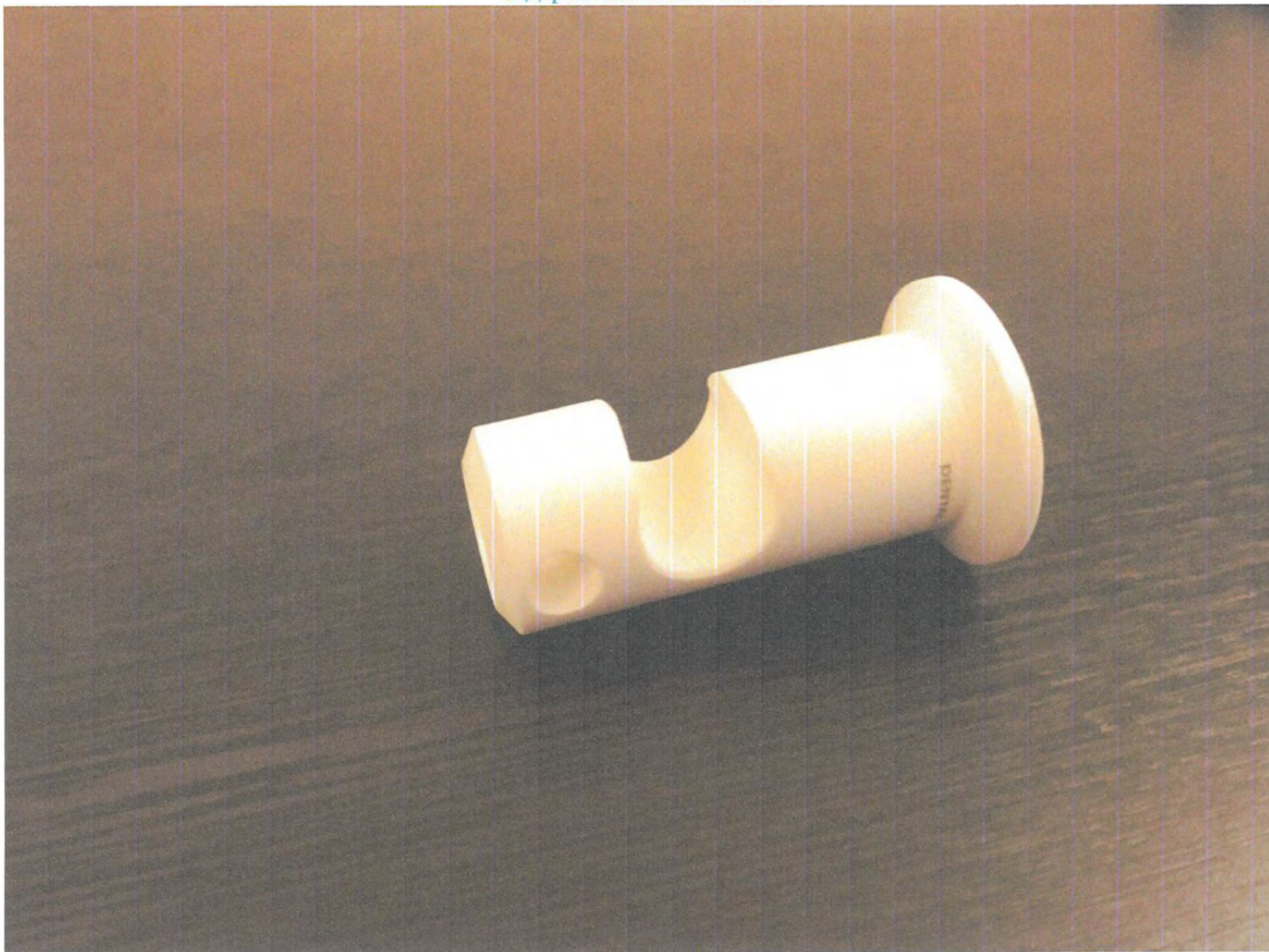
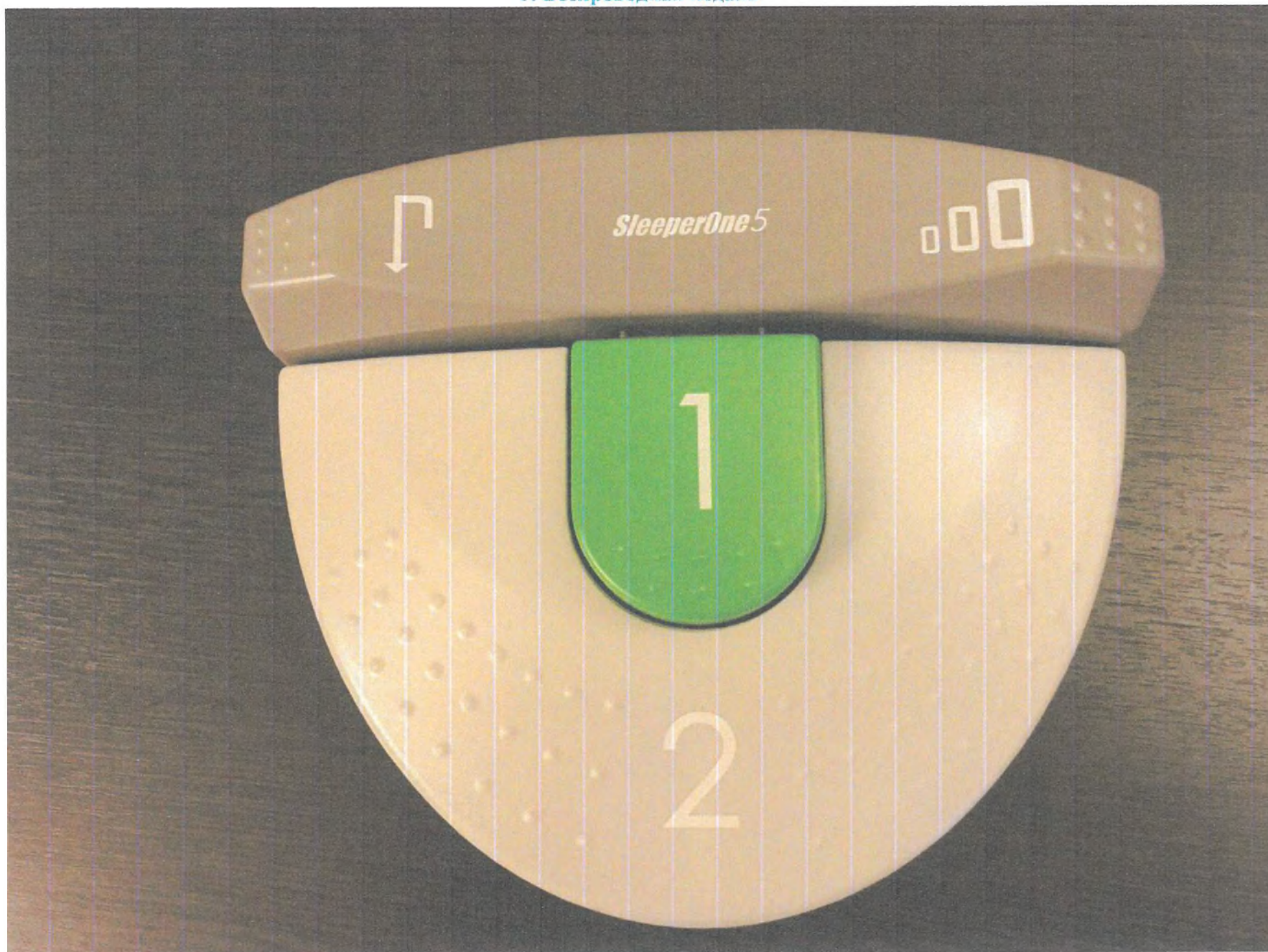
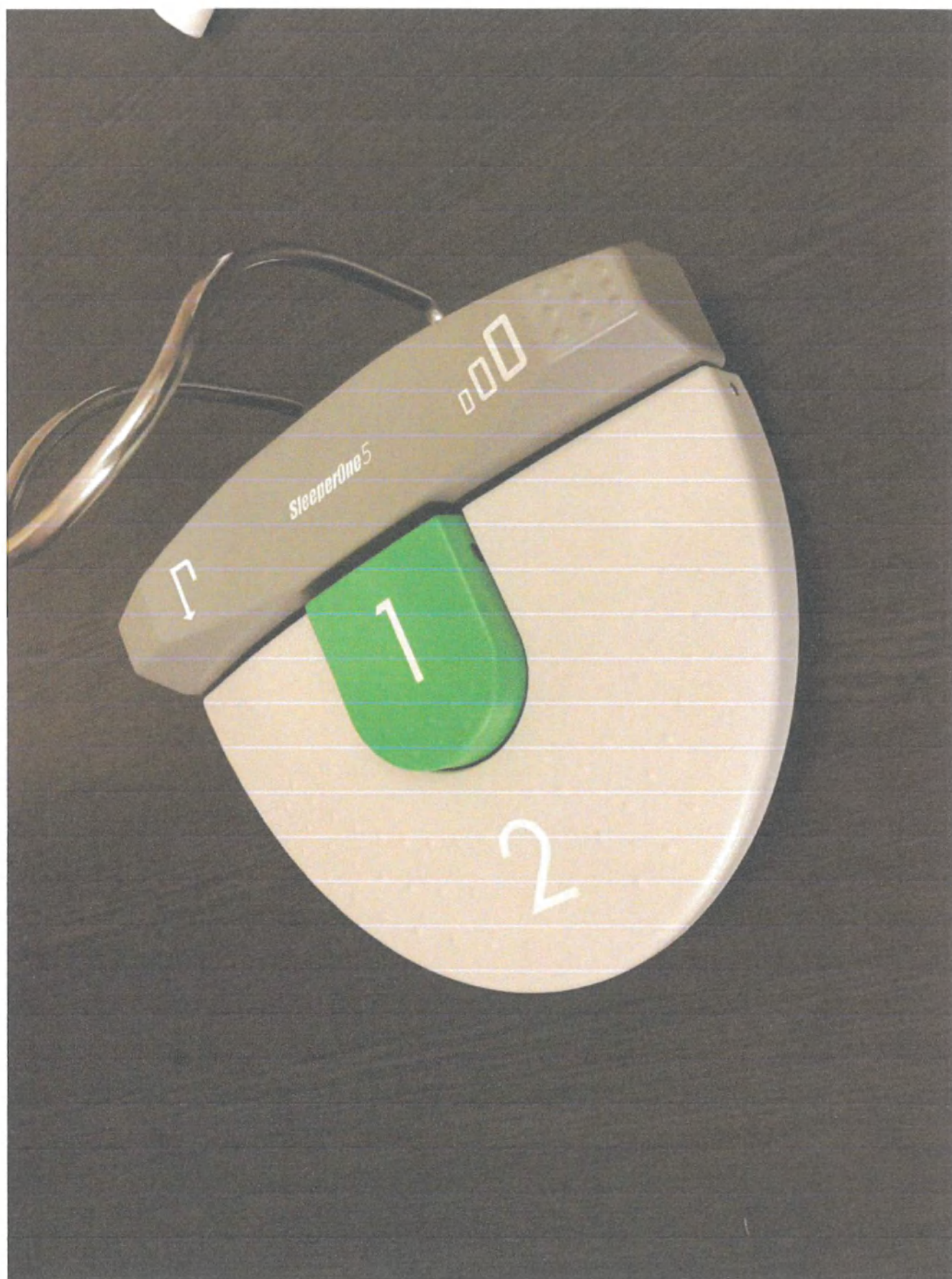
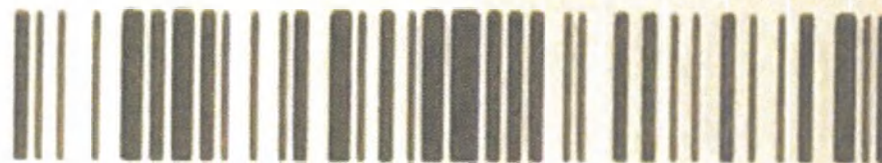
EAC

MEAN WELL ENTERPRISES CO.,LTD.
No. 28, Wuquan 3rd Rd., Wugu Dist.,
New Taipei City, 24891, Taiwan

Manual:
[www.
meanwell.
com/
manual.html](http://www.meanwell.com/manual.html)

This device complies with
Part 15 of the FCC Rules.
Operation is subject to the
following two conditions:
(1) this device may not cause
harmful interference, and
(2) this device must accept
any interference received,
including interference that
may cause undesired
operation.

OYB8306409

Order No.:
GSM36B24-P6L

AC / DC MEDICAL ADAPTOR

MODEL NO. : GSM36B24

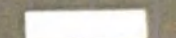
INPUT : 100-240VAC, 50/60Hz, 0.9-0.45A

OUTPUT : 24V $\equiv$ 1.5A, 36W MAX.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

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Needles^{DHT}

Single use needles. Do not resterilize or reuse.
Store in original packaging until ready to use.
In order to benefit from all of the advantages of the DHT needles, proceed as indicated below.
For maximum performance, the use of a syringe allowing the use of a pen grip (support points) and an electronic injection is highly recommended.

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More information : www.dentalhitec.com
(videos with needles DHT...)

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Needles

Needles

Single use needles. Do not reuse or reuse.
Store in original packaging and ready to use.
To use, insert the needle into the syringe barrel.
(1) Reutilization. Utilization of a single needle for
two consecutive procedures. Utilization of a single needle for
two consecutive procedures. Utilization of a single needle for
two consecutive procedures.

Be sure to correctly orientate the needle indicator for this type
of anesthesia.

ANESTHESIA OF THE ATTACHED MUCOSA

For a single use attached mucosa anesthesia, use the
needle to inject the attached mucosa, without any contact
with the mucosa. (2) Do not use the needle for
injection of attached mucosa anesthesia in attached
mucosa anesthesia.

INTRALIGAMENTARY ANESTHESIA

For a single use intraligamentary anesthesia, use the
needle to inject the intraligamentary anesthesia, without
any contact with the mucosa. (2) Do not use the needle
for injection of intraligamentary anesthesia in
intraligamentary anesthesia.

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- © 3M United Kingdom PLC, 3M Centre, Cain Road, Bracknell, Berks., RG12 8HT,