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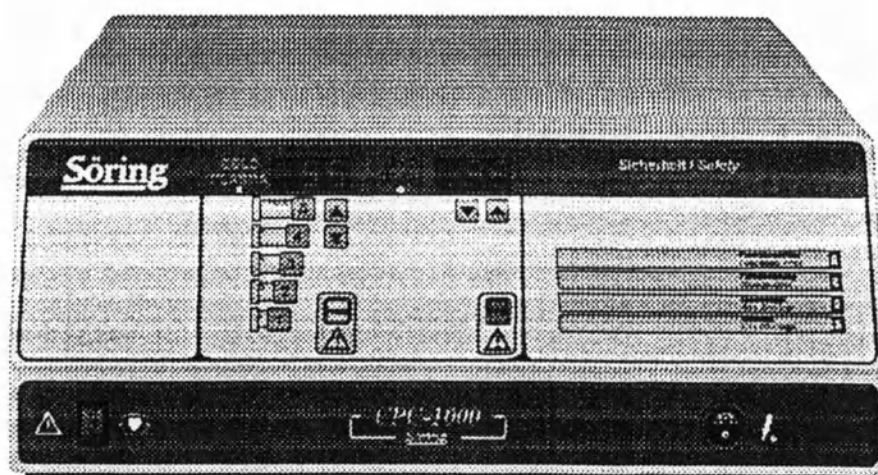
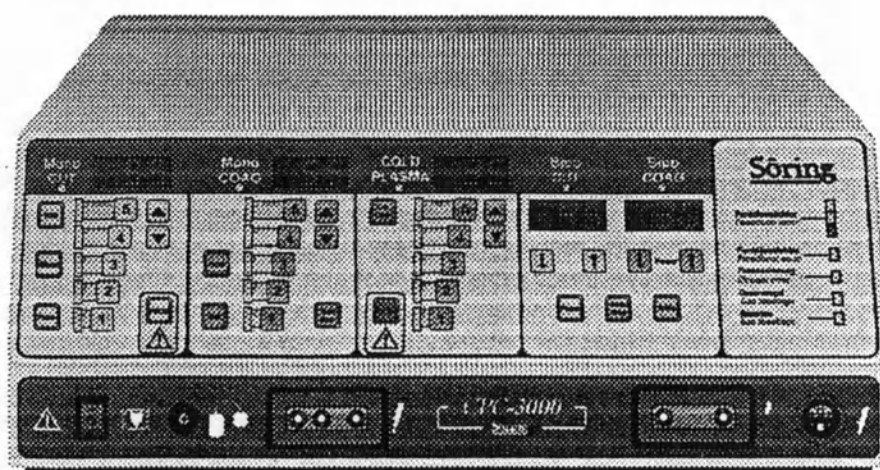
1. Introduction

This operating-manual describes the function and handling of the Radiofrequency-Surgery-Equipment CPC. The manual serves as an instruction-reference and should be read thoroughly before operating the equipment. Only then the correct handling of the equipment is ensured.

In case of incorrect handling no liability will be taken on by the manufacturer.

The CPC unit is a RF-surgery equipment conforming to DIN IEC 601 part 1, respectively EN60601-1 utilized only for rooms for medical use. Every important functions are controlled and supervised by a microprocessor.

The CPC unit is only meant for the use in general surgery. Operations on the heart, and the eyes are not admissible!!!

This equipment has not been developed for continual operation (constant power-supply for a longer period). The period for use fulfils the demands for operation of the EN60601-2-2 (10s load, 30s break).

This RF-Surgery unit meets the requirements of the (EMC) guideline 93/42/EWG of the European Community and is therefore labeled with the  -sign!

Before starting operation the supplies of the appropriate handpieces must always be checked for external damages.

The Söring company only takes the liability if accessories from the Accessories-Order-List were used!!!

At the end of the normal life cycle of the unit the unit has to be brought either to a legitimated person for waste disposal of electronics or the Söring company.

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Attention!, see also covering documents

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