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2021

Contents

1.	INTRODUCTION.....	3
2.	INFORMATION ABOUT THE MEDICAL DEVICE MANUFACTURER.....	3
3.	INFORMATION ABOUT THE AUTHORIZED REPRESENTATIVE OF THE MANUFACTURER.....	3
4.	NAME OF THE MEDICAL DEVICE.....	3
5.	INTENDED USE OF THE MEDICAL DEVICE.....	3
6.	INDICATIONS	3
7.	CONTRAINDICATIONS	3
8.	SIDE EFFECTS	4
9.	WARNINGS	4
10.	PRECAUTIONS.....	4
11.	SCOPE OF APPLICATION OF THE MEDICAL DEVICE	5
12.	TECHNICAL DESCRIPTION AND CHARACTERISTIC.....	5
13.	COMPOSITION	27
14.	SUPPLY PACKAGE	29
15.	HYGIENE.....	32
16.	GENERAL DESCRIPTION AND METHOD OF APPLICATION...	33
17.	CLEANING AND DISINFECTION.....	41
18.	MAINTENANCE	41
19.	OPERATING, STORAGE AND TRANSPORTATION CONDITIONS.....	41
20.	EXPIRATION DATE OF THE MEDICAL DEVICE.....	41
21.	LABELING.....	43
22.	PACKAGING.....	47
23.	ENVIRONMENTAL REQUIREMENTS	47
24.	DISPOSAL.....	47
25.	MANUFACTURER WARRANTY.....	47
26.	RECLAMATION.....	47

Annex 1 i-Shade label, Illustrated technical guide, Order form for NEO Spectra ST
Shade Guide, The information leaflet

1. INTRODUCTION

WARNING! For safe usage of NEO Spectra ST Kits for anterior and posterior teeth restoration in design variants, as indicated, all users should carefully read these instructions for use before using the medical device. The instruction for use is written to familiarize the consumer with the design of the medical device, its functions, as well as the operation conditions and rules. The Instruction for Use is intended for medical professionals.

- Always follow these Instruction for Use.
- The Instruction for use should be kept in the immediate vicinity of the device.

2. INFORMATION ABOUT THE MEDICAL DEVICE MANUFACTURER AND MANUFACTURING SITE

Dentsply De Trey GmbH

Address: De-Trey-Strasse 1, 78467 Konstanz, GERMANY

E-mail: service-konstanz@dentsplysirona.com

Manufacturing site: Dentsply De Trey GmbH, De-Trey-Strasse 1, 78467 Konstanz, GERMANY

3. INFORMATION ABOUT THE AUTHORIZED REPRESENTATIVE OF THE MANUFACTURER

Dentsply Sirona Limited Liability Company

Address: 18 Andropov Prospect, Building 6, Offices 1, 39, 40, Moscow 115432, Russia

Tel.: +7 (495) 725-10-87

E-mail: MoscowRus-Reception@dentsplysirona.com

4. NAME OF THE MEDICAL DEVICE

NEO Spectra ST Kits for anterior and posterior teeth restoration in design variants.

5. INTENDED USE OF THE MEDICAL DEVICE

This medical device is intended for all types of direct restorations of the cavities of the anterior and posterior teeth, and for the indirect fabrication of inlays, onlays and veneers.

6. INDICATIONS

- Direct restorations of cavities of all classes of the anterior and posterior teeth.
- For the indirect fabrication of inlays, onlays and veneers.

7. CONTRAINDICATIONS

The medical device is contraindicated for patients who have previously had allergic reactions to methacrylate resins or to any other component of the composite material.

8. SIDE EFFECTS

- Eye contact: Irritation, possible damage to the cornea.
- Skin contact: Irritation, possible allergic response. A red rash may appear on the skin.
- Contact with mucous membranes: inflammation (see section "Warnings")

9. WARNINGS

The material contains methacrylates and polymerizable monomers which may be irritating to skin, eyes and oral mucosa and may cause sensitization by skin contact and allergic contact dermatitis in susceptible persons.

- Avoid contact with eyes to prevent irritation and possible damage to the cornea. In case of contact with eyes, rinse with plenty of water and seek for medical attention.
- Avoid skin contact to prevent irritation and possible allergic response. In case of contact, reddish rashes may appear on the skin. If skin contact occurs, remove material with cotton wool and alcohol and wash thoroughly with soap and water. In case of skin sensitization or rash, discontinue use and seek medical attention.
- Avoid contact with oral soft tissues/mucosa to prevent inflammation. If accidental contact occurs, remove material from the tissues, Flush mucosa with plenty of water and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.

10. PRECAUTIONS

This product is intended to be used only as specifically outlined in these Directions for Use. Any use of this product inconsistent with these Directions for Use is at the discretion and sole responsibility of the dental practitioner.

- If material is refrigerated, allow sufficient time for material to warm to room temperature before use.
- Contact with saliva, blood and sulcus fluid during application may result in a failed restoration. Use adequate isolation such as rubber dam.
- Use appropriate protective measures for dental personnel and patients, such as glasses and rubber dam in accordance with local best practice.
- Devices marked with "single use" on the label are intended for single use only. Discard after use. Do not reuse in other patients in order to prevent cross-contamination.
- The syringes cannot be reprocessed. To prevent syringes from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the syringes are handled with clean/disinfected gloves. Discard syringes if contaminated.
- As an additional precaution, the syringe can be protected from gross debris, but not all possible contamination, by using a protective barrier.
- It is recommended to use Compules® Tips with the original Compules® Tips Gun from Dentsply Sirona.
- Compules® Tips Gun is only intended for Compules® or Compula® Tips from Dentsply Sirona.
- Read the Compules® Tips Gun Instructions for Use for complete information.
- The shade guide and individual shade guide tabs cannot be reprocessed. To prevent the shade guide and the individual shade guide tabs from exposure to spatter or spray of body fluids

or contaminated hands it is mandatory that the shade guide and the individual shade guide tabs are handled with clean/disinfected gloves.

- The material should extrude easily.

DO NOT APPLY EXCESSIVE FORCE.

Excessive pressure may cause the Compules® Tip to rupture or to eject from the Compules® Tips Gun.

- Close syringes tightly immediately after use.

• Interaction:

- Do not use materials containing eugenol or hydrogen peroxide in combination with this device as they may interfere with the hardening of the product.

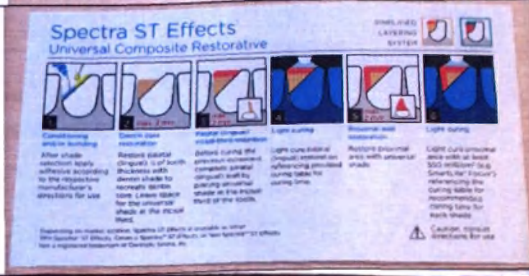
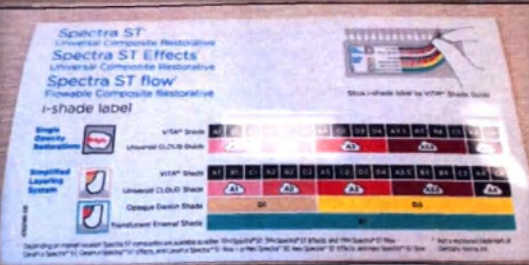
11. SCOPE OF APPLICATION OF THE MEDICAL DEVICE

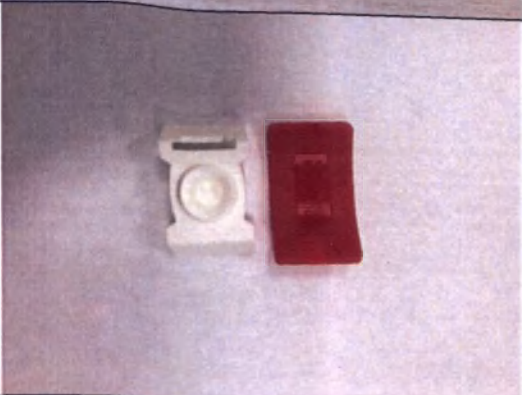
The scope of application is dentistry.

The product is intended for use by dentists and is used in dental clinics and dental laboratories.

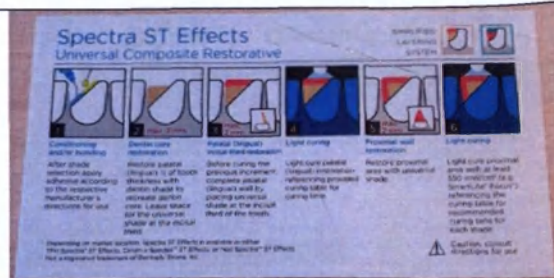
12. TECHNICAL DESCRIPTION AND CHARACTERISTIC

Device, description	Image
1. NEO Spectra ST LV Intro Kit in syringes, consisting of:	
1. Light-cured restoration composite material NEO Spectra ST LV in syringes, 3 g each, shades: A1 (1 pce), A2 (1 pce), A3 (1 pce), A3,5 (1 pce), A4 (1 pce) - 5 pcs. Syringe length, mm: $155 \pm 5\%$ Outer diameter, mm: $9.50 \pm 1\%$ Inner diameter, mm: $7.80 \pm 1\%$ Weight, g: $15.0 \pm 5\%$	
2) Prime&Bond® universal adhesive in bottle, 2.5 ml – 1 pce Provides easy adhesive application for direct and indirect restorations and adheres to enamel, dentin, composites, zirconia and metals. Dimensions of the bottle: Height, mm: $39.6 \pm 5\%$ Diameter, mm: $20.7 \pm 1\%$ Volume, ml: $2.5 \pm 1\%$ Weight, g: $2.3 \pm 1\%$	

3) Applicators - 50 pcs Length, mm: 84 ± 5 % Diameter, mm: 3.0 ± 1 % Weight, g: 0.28 ± 1 %	
4) CliXdish plastic mixing well – 1 pce Mixing well / adhesive protection. Length, mm: 41.3 ± 5 % Width, mm: 26.0 ± 5 % Weight, g: 7.2 ± 1 %	
5) Illustrated technical guide - 1 pce	
6) Order form for NEO Spectra ST Shade Guide - 1 pce Supplied if necessary.	
7) i-Shade label - 1 pce	
8) Instruction for use on paper and/or in electronic form - 2 pcs	

9) Information leaflet - 1 pce Supplied if necessary.	 Try the complete Neo Spectra™ ST portfolio for efficient esthetics. Learn more ►
2. NEO Spectra ST LV Eco Kit in syringes, consisting of:	
1. Light-cured restoration composite material NEO Spectra ST LV in syringes, 3 g each, shades: A2 (3 pcs), A3 (3 pcs) – 6 pcs Syringe length, mm: $155 \pm 5\%$ Outer diameter, mm: $9.50 \pm 1\%$ Inner diameter, mm: $7.80 \pm 1\%$ Weight, g: $15.0 \pm 5\%$	
2) Prime&Bond® universal adhesive in bottle, 2.5 ml – 1 pce Provides easy adhesive application for direct and indirect restorations and adheres to enamel, dentin, composites, zirconia and metals. Dimensions of the bottle: Height, mm: $39.6 \pm 5\%$ Diameter, mm: $20.7 \pm 1\%$ Volume, ml: $2.5 \pm 1\%$ Weight, g: $2.3 \pm 1\%$	
3) Applicators - 50 pcs Length, mm: $84 \pm 5\%$ Diameter, mm: $3.0 \pm 1\%$ Weight, g: $0.28 \pm 1\%$	
4) CliXdish plastic mixing well – 1 pce Mixing well / adhesive protection. Length, mm: $41.3 \pm 5\%$ Width, mm: $26.0 \pm 5\%$ Weight, g: $7.2 \pm 1\%$	

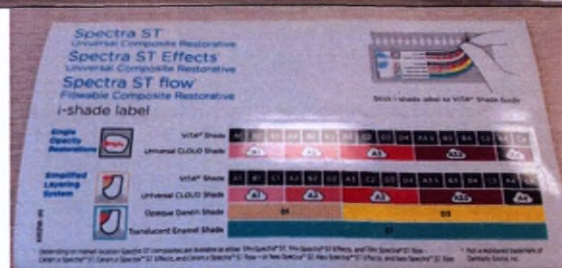
5) Illustrated technical guide - 1 pce

6) Order form for NEO Spectra ST Shade Guide - 1 pce
Supplied if necessary.

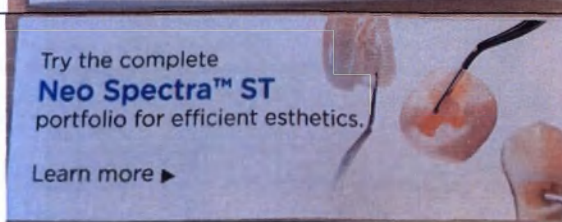
7) i-Shade label - 1 pce

8) Instruction for use on paper and/or in electronic form - 2 pcs

9) Information leaflet - 1 pce
Supplied if necessary.

3. NEO Spectra ST HV Intro Kit in syringes, consisting of:

1. Light-cured restoration composite material NEO Spectra ST HV in syringes, 3 g each, shades: A1 (1 pce), A2 (1 pce), A3 (1 pce), A3,5 (1 pce), A4 (1 pce) - 5 pcs.

Syringe length, mm: $155 \pm 5\%$

Outer diameter, mm: $9.50 \pm 1\%$

Inner diameter, mm: $7.80 \pm 1\%$

Weight, g: $15.0 \pm 5\%$

2) Prime&Bond® universal adhesive in bottle, 2.5 ml – 1 pce

Provides easy adhesive application for direct and indirect restorations and adheres to enamel, dentin, composites, zirconia and metals.

Dimensions of the bottle:

Height, mm: $39.6 \pm 5\%$

Diameter, mm: $20.7 \pm 1\%$

Volume, ml: $2.5 \pm 1\%$

Weight, g: $2.3 \pm 1\%$

3) Applicators - 50 pcs

Length, mm: $84 \pm 5\%$

Diameter, mm: $3.0 \pm 1\%$

Weight, g: $0.28 \pm 1\%$

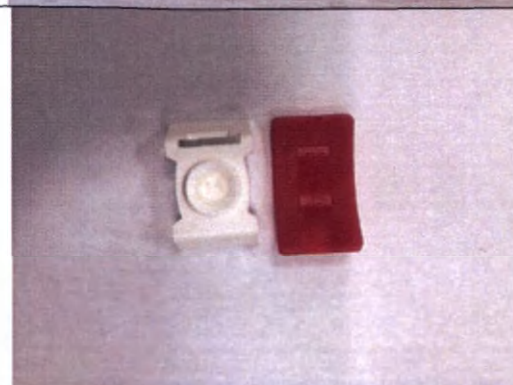
4) CliXdish plastic mixing well – 1 pce

Mixing well / adhesive protection.

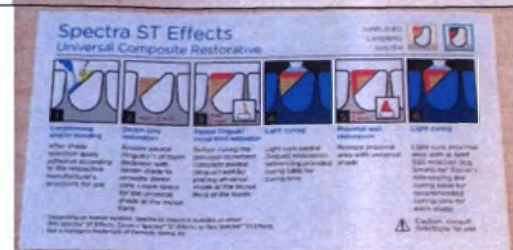
Length, mm: $41.3 \pm 5\%$

Width, mm: $26.0 \pm 5\%$

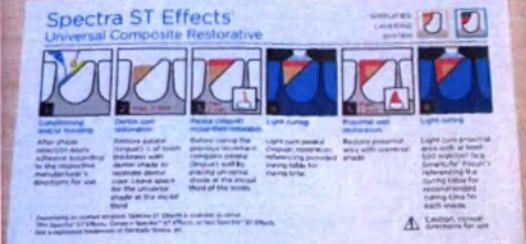
Weight, g: $7.2 \pm 1\%$

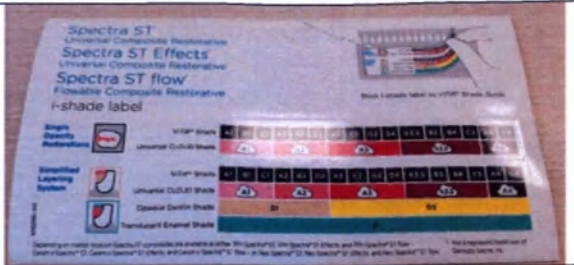
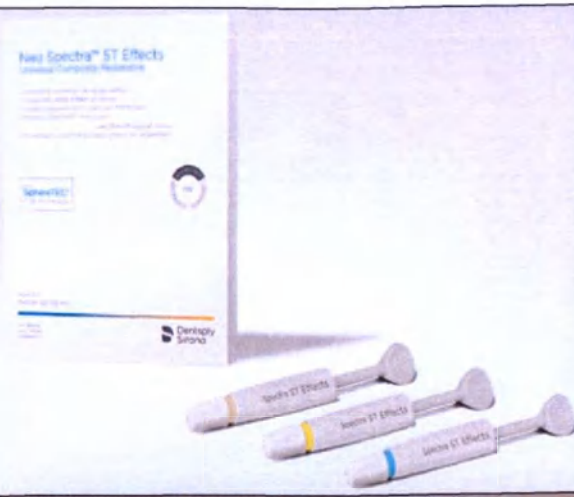
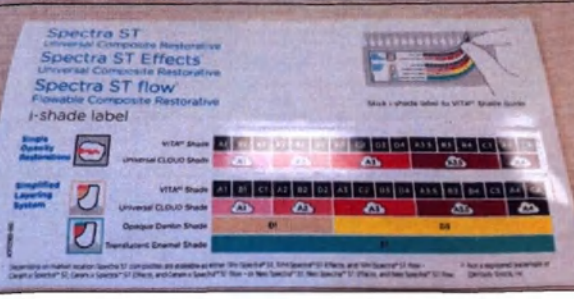
5) Illustrated technical guide - 1 pce

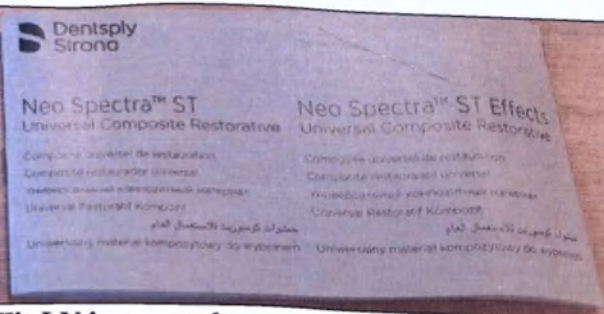
6) Order form for NEO Spectra ST Shade Guide - 1 pce Supplied if necessary.	 Neo Spectra ST Composites Order your free shade guide
7) i-Shade label - 1 pce	 Spectra ST Universal Composite Restorative Spectra ST Effects Universal Composite Restorative Spectra ST flow Flowable Composite Restorative i-shade label
8) Instruction for use on paper and/or in electronic form - 2 pcs	 Dentsply Sirona Neo Spectra™ ST Universal Composite Restorative Neo Spectra™ ST Effects Universal Composite Restorative
9) Information leaflet - 1 pce Supplied if necessary.	 Try the complete Neo Spectra™ ST portfolio for efficient esthetics. Learn more ►
4. NEO Spectra ST HV Eco Kit in syringes, consisting of:	
1. Light-cured restoration composite material NEO Spectra ST HV in syringes, 3 g each, shades: A2 (3 pcs), A3 (3 pcs) – 6 pcs Syringe length, mm: $155 \pm 5\%$ Outer diameter, mm: $9.50 \pm 1\%$ Inner diameter, mm: $7.80 \pm 1\%$ Weight, g: $15.0 \pm 5\%$	

2) Prime&Bond® universal adhesive in bottle, 2.5 ml – 1 pce Provides easy adhesive application for direct and indirect restorations and adheres to enamel, dentin, composites, zirconia and metals. Dimensions of the bottle: Height, mm: $39.6 \pm 5\%$ Diameter, mm: $20.7 \pm 1\%$ Volume, ml: $2.5 \pm 1\%$ Weight, g: $2.3 \pm 1\%$	
3) Applicators - 50 pcs Length, mm: $84 \pm 5\%$ Diameter, mm: $3.0 \pm 1\%$ Weight, g: $0.28 \pm 1\%$	
4) CliXdish plastic mixing well – 1 pce Mixing well / adhesive protection. Length, mm: $41.3 \pm 5\%$ Width, mm: $26.0 \pm 5\%$ Weight, g: $7.2 \pm 1\%$	
5) Illustrated technical guide - 1 pce	
6) Order form for NEO Spectra ST Shade Guide - 1 pce Supplied if necessary.	

7) i-Shade label - 1 pce	
8) Instruction for use on paper and/or in electronic form - 2 pcs	
9) Information leaflet - 1 pce Supplied if necessary.	
5. NEO Spectra ST Effects Intro Kit in syringes, consisting of:	
1. Light-cured restoration composite material NEO Spectra ST Effects in syringes, 3 g each, shades: D1 (1 pce), D3 (1 pce), E1 (1 pce) – 3 pcs Syringe length, mm: $155 \pm 5\%$ Outer diameter, mm: $9.50 \pm 1\%$ Inner diameter, mm: $7.80 \pm 1\%$ Weight, g: $15.0 \pm 5\%$	
2) Illustrated technical guide - 1 pce	
3) i-Shade label - 1 pce	

4) Instruction for use on paper and/or in electronic form - 1 pce

6. *NEO Spectra ST Intro Kit LV in compules, consisting of:*

1) **Composite material NEO Spectra ST LV (Compules® tips) in compules, 0.25 g each, shades: A1 (4 pcs), A2 (4 pcs), A3 (8 pcs), A3.5 (4 pcs), A4 (4 pcs) - 24 pcs**

Dimensions of the compule:

Outside diameter of the needle, mm: $1.6 \pm 1\%$

Flange diameter, mm: $8.0 \pm 1\%$

Compule height, mm: $20.5 \pm 1\%$

Outside diameter of compule tips, mm:

$5.35 \pm 1\%$

Weight, g: $0.8 \pm 1\%$

2) **Prime&Bond® universal adhesive in bottle, 2.5 ml – 1 pce**

Dimensions of the bottle:

Height, mm: $39.6 \pm 5\%$

Diameter, mm: $20.7 \pm 1\%$

Volume, ml: $2.5 \pm 1\%$

Weight, g: $2.3 \pm 1\%$

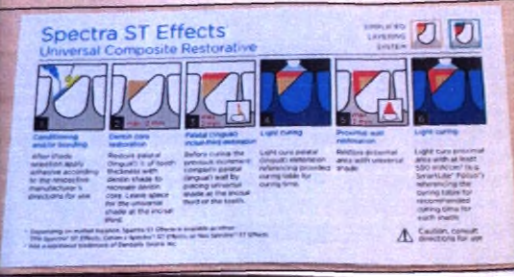
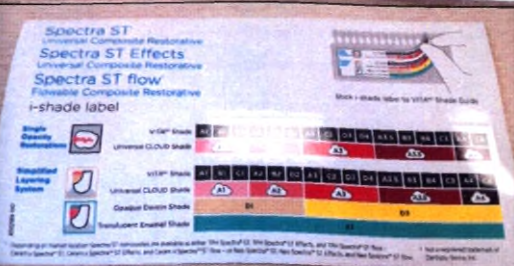
3) **Compules® Tips Gun - 1 pce**

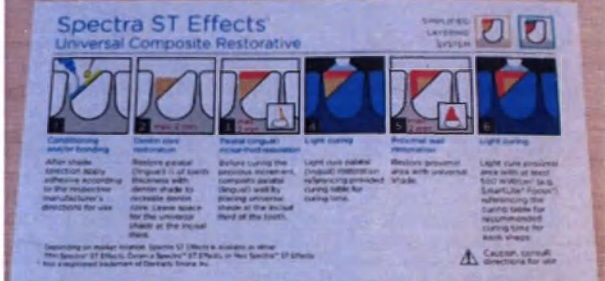
Length, mm: $99.0 \pm 5\%$

Width, mm: $157.0 \pm 5\%$

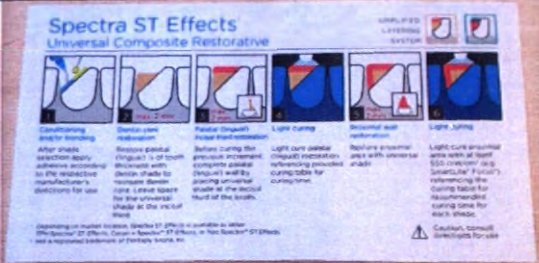
Weight, g: $48.0 \pm 1\%$

4) Applicators - 50 pcs Length, mm: $84 \pm 5 \%$ Diameter, mm: $3.0 \pm 1 \%$ Weight, g: $0.28 \pm 1 \%$	
5) CliXdish plastic mixing well – 1 pce Mixing well / adhesive protection. Length, mm: $41.3 \pm 5\%$ Width, mm: $26.0 \pm 5\%$ Weight, g: $7.2 \pm 1\%$	
6) Illustrated technical guide - 1 pce	
7) Order form for NEO Spectra ST Shade Guide - 1 pce Supplied if necessary.	
8) i-Shade label - 1 pce	
9) Instruction for use on paper and/or in electronic form - 2 pcs	

10) Information leaflet - 1 pce Supplied if necessary.	
7. NEO Spectra ST Discovery Kit LV in compules, consisting of:	
1) Composite material NEO Spectra ST LV (Compules® tips) in compules, 0.25 g each, shade: A2 - 4 pcs. Dimensions of the compule: Outside diameter of the needle, mm: $1.6 \pm 1\%$ Flange diameter, mm: $8.0 \pm 1\%$ Compule height, mm: $20.5 \pm 1\%$ Outside diameter of compule tips, mm: $5.35 \pm 1\%$ Weight, g: $0.8 \pm 1\%$	
2) Illustrated technical guide - 1 pce	
3) Instruction for use on paper and/or in electronic form - 1 pce	
8. NEO Spectra ST LV Risk Free Trial Kit in compules, consisting of:	
1) Composite material NEO Spectra ST LV (Compules® tips) in compules, 0.25 g each, shades: A1 (4 pcs), A2 (4 pcs), A3 (8 pcs), A3.5 (4 pcs), A4 (4 pcs) - 24 pcs Dimensions of the compule: Outside diameter of the needle, mm: $1.6 \pm 1\%$ Flange diameter, mm: $8.0 \pm 1\%$ Compule height, mm: $20.5 \pm 1\%$ Outside diameter of compule tips, mm: $5.35 \pm 1\%$ Weight, g: $0.8 \pm 1\%$	

2) Composite material NEO Spectra ST LV (Compules® tips) in compules, 0.25 g each, shade: A2 - 4 pcs. Dimensions of the compule: Outside diameter of the needle, mm: $1.6 \pm 1\%$ Flange diameter, mm: $8.0 \pm 1\%$ Compule height, mm: $20.5 \pm 1\%$ Outside diameter of compule tips, mm: $5.35 \pm 1\%$ Weight, g: $0.8 \pm 1\%$	
3) Prime&Bond® universal adhesive in bottle, 2.5 ml – 1 pce Dimensions of the bottle: Height, mm: $39.6 \pm 5\%$ Diameter, mm: $20.7 \pm 1\%$ Volume, ml: $2.5 \pm 1\%$ Weight, g: $2.3 \pm 1\%$	
4) Compules® Tips Gun - 1 pce Length, mm: $99.0 \pm 5\%$ Width, mm: $157.0 \pm 5\%$ Weight, g: $48.0 \pm 1\%$	
5) Applicators - 50 pcs Length, mm: $84 \pm 5\%$ Diameter, mm: $3.0 \pm 1\%$ Weight, g: $0.28 \pm 1\%$	

6) CliXdish plastic mixing well – 1 pce Mixing well / adhesive protection. Length, mm: $41.3 \pm 5\%$ Width, mm: $26.0 \pm 5\%$ Weight, g: $7.2 \pm 1\%$	
7) Illustrated technical guide – 2 pce	
8) i-Shade label - 1 pce	
9) Order form for NEO Spectra ST Shade Guide - 1 pce Supplied if necessary.	
10) Instruction for use on paper and/or in electronic form - 3 pcs	
11) Information leaflet - 1 pce Supplied if necessary.	

9. NEO Spectra ST Intro Kit HV in compules, consisting of

1. Composite material NEO Spectra ST HV (Compules® tips) in compules, 0.25 g each, shades: A1 (4 pcs), A2 (4 pcs), A3 (8 pcs), A3.5 (4 pcs), A4 (4 pcs) – 24 pcs

Dimensions of the compule:

Outside diameter of the needle, mm: $1.6 \pm 1\%$

Flange diameter, mm: $8.0 \pm 1\%$

Compule height, mm: $20.5 \pm 1\%$

Outside diameter of compule tips, mm: $5.35 \pm 1\%$

Weight, g: $0.8 \pm 1\%$

2) Prime&Bond® universal adhesive in bottle, 2.5 ml – 1 pce

Dimensions of the bottle:

Height, mm: $39.6 \pm 5\%$

Diameter, mm: $20.7 \pm 1\%$

Volume, ml: $2.5 \pm 1\%$

Weight, g: $2.3 \pm 1\%$

3) Compules® Tips Gun - 1 pce

Length, mm: $99.0 \pm 5\%$

Width, mm: $157.0 \pm 5\%$

Weight, g: $48.0 \pm 1\%$

4) Applicators - 50 pcs

Length, mm: $84 \pm 5\%$

Diameter, mm: $3.0 \pm 1\%$

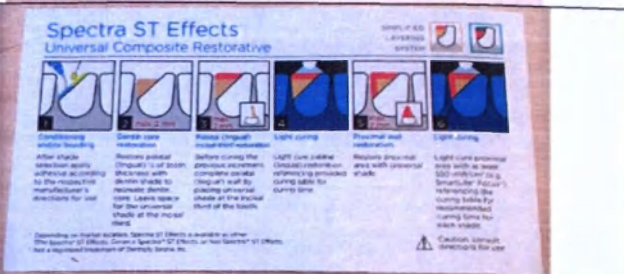
Weight, g: $0.28 \pm 1\%$

5) CliXdish plastic mixing well – 1 pce
 Mixing well / adhesive protection.
 Length, mm: $41.3 \pm 5\%$
 Width, mm: $26.0 \pm 5\%$
 Weight, g: $7.2 \pm 1\%$

6) Illustrated technical guide - 1 pce

7) Order form for NEO Spectra ST Shade Guide - 1 pce
 Supplied if necessary.

8) i-Shade label - 1 pce

9) Instruction for use on paper and/or in electronic form - 2 pcs

10) Information leaflet - 1 pce
 Supplied if necessary.

10. NEO Spectra ST Discovery Kit HV in compules, consisting of:

1) Composite material NEO Spectra ST HV (Compules® tips) in compules, 0.25 g each, shade: A2 - 4 pcs.

Dimensions of the compule:

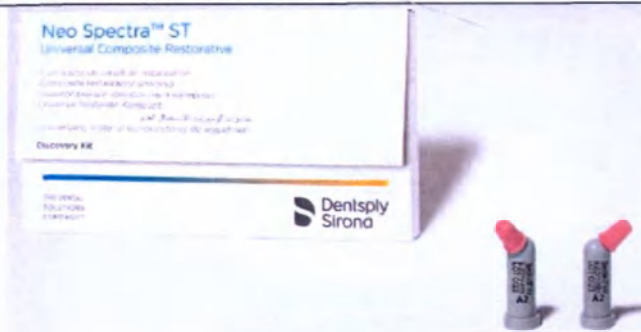
Outside diameter of the needle, mm: $1.6 \pm 1\%$

Flange diameter, mm: $8.0 \pm 1\%$

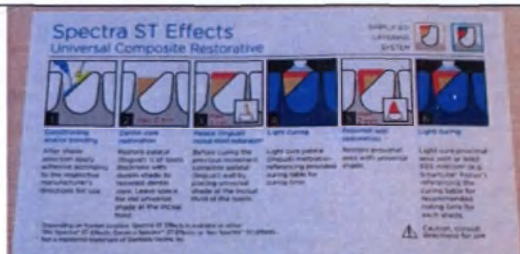
Compule height, mm: $20.5 \pm 1\%$

Outside diameter of compule tips, mm: $5.35 \pm 1\%$

Weight, g: $0.8 \pm 1\%$

2) Illustrated technical guides - 1 pce

3) Instruction for use on paper and/or in electronic form - 1 pce

11. NEO Spectra ST HV Risk Free Trial Kit in compules, consisting of:

1) Composite material NEO Spectra ST HV (Compules® tips) in compules, 0.25 g each, shades: A1 (4 pcs), A2 (4 pcs), A3 (8 pcs), A3.5 (4 pcs), A4 (4 pcs) - 24 pcs

Dimensions of the compule:

Outside diameter of the needle, mm: $1.6 \pm 1\%$

Flange diameter, mm: $8.0 \pm 1\%$

Compule height, mm: $20.5 \pm 1\%$

Outside diameter of compule tips, mm: $5.35 \pm 1\%$

Weight, g: $0.8 \pm 1\%$

2) Composite material NEO Spectra ST HV (Compules® tips) in compules, 0.25 g each, shade: A2 - 4 pcs.

Dimensions of the compule:

Outside diameter of the needle, mm: $1.6 \pm 1\%$

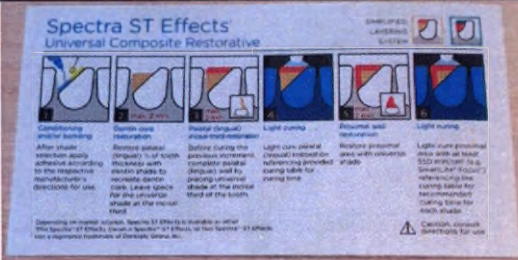
Flange diameter, mm: $8.0 \pm 1\%$

Compule height, mm: $20.5 \pm 1\%$

Outside diameter of compule tips, mm: $5.35 \pm 1\%$

Weight, g: $0.8 \pm 1\%$

3) Prime&Bond® universal adhesive in bottle, 2.5 ml – 1 pce Dimensions of the bottle: Height, mm: $39.6 \pm 5\%$ Diameter, mm: $20.7 \pm 1\%$ Volume, ml: $2.5 \pm 1\%$ Weight, g: $2.3 \pm 1\%$	
4) Compules® Tips Gun - 1 pce Length, mm: $99.0 \pm 5\%$ Width, mm: $157.0 \pm 5\%$ Weight, g: $48.0 \pm 1\%$	
5) Applicators - 50 pcs Length, mm: $84 \pm 5\%$ Diameter, mm: $3.0 \pm 1\%$ Weight, g: $0.28 \pm 1\%$	
6) CliXdish plastic mixing well – 1 pce Mixing well / adhesive protection. Length, mm: $41.3 \pm 5\%$ Width, mm: $26.0 \pm 5\%$ Weight, g: $7.2 \pm 1\%$	
7) Illustrated technical guide - 2 pce	

8) i-Shade label - 1 pce

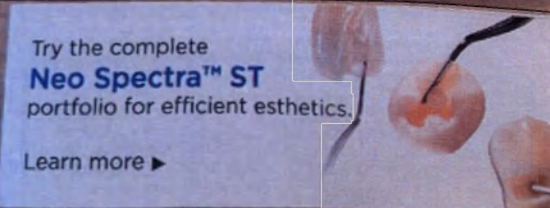
9) Order form for NEO Spectra ST Shade Guide - 1 pce
Supplied if necessary.

10) Instruction for use on paper and/or in electronic form - 3 pcs

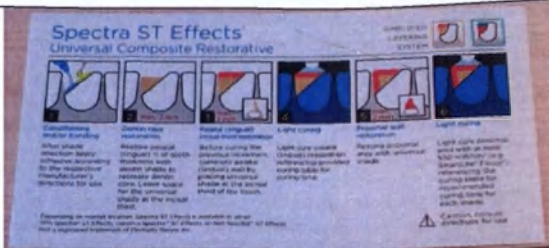
11) Information leaflet - 1 pce
Supplied if necessary.

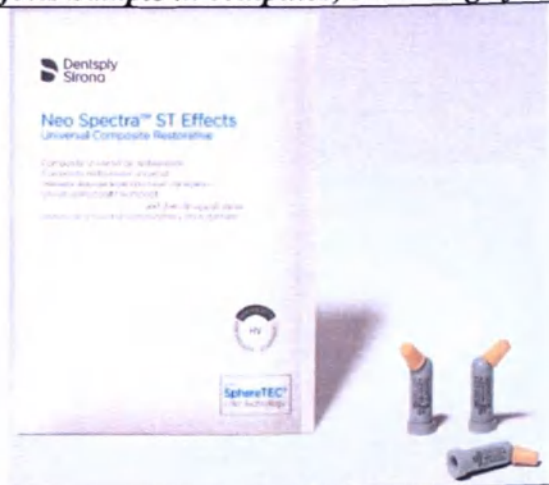
12. NEO Spectra ST Effects Intro Kit in compules, consisting of:

1) Composite material NEO Spectra ST Effects (Compules® tips) in compules, 0.25 g each, shades: D1 (12 pcs), D3 (8 pcs), E1 (4 pcs) - 24 pcs
Dimensions of the compule:
Outside diameter of the needle, mm: $1.6 \pm 1\%$
Flange diameter, mm: $8.0 \pm 1\%$
Compule height, mm: $20.5 \pm 1\%$
Outside diameter of compule tips, mm: $5.35 \pm 1\%$
Weight, g: $0.8 \pm 1\%$

2) Illustrated technical guides - 1 pce.	
3) i-Shade label - 1 pce.	
4) Instruction for use on paper and/or in electronic form - 1 pce	
5) Information leaflet - 1 pce Supplied if necessary	

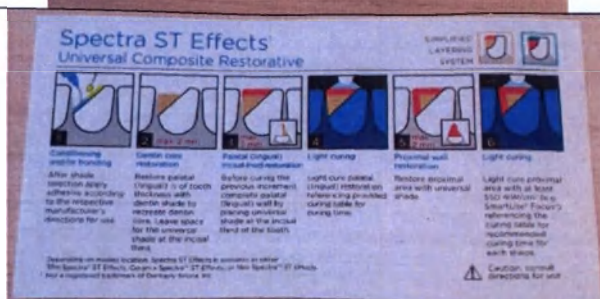
13. NEO Spectra ST, NEO Spectra ST Effects Sample in compules, consisting of:

1) Composite material NEO Spectra ST Effects (Compules® tips) in compules, 0.25 g each, shades: D1 (4 pcs), E1 (4 pcs) - 8 pcs Dimensions of the compule: Outside diameter of the needle, mm: $1.6 \pm 1\%$ Flange diameter, mm: $8.0 \pm 1\%$ Compule height, mm: $20.5 \pm 1\%$ Outside diameter of compule tips, mm: $5.35 \pm 1\%$ Weight, g: $0.8 \pm 1\%$	
---	--

2) **Composite material NEO Spectra ST HV (Compules® tips) in compules, 0.25 g each, shade A2 - 4 pcs**
Dimensions of the compule:
 Outside diameter of the needle, mm: $1.6 \pm 1\%$
 Flange diameter, mm: $8.0 \pm 1\%$
 Compule height, mm: $20.5 \pm 1\%$
 Outside diameter of compule tips, mm: $5.35 \pm 1\%$
 Weight, g: $0.8 \pm 1\%$

3) **Illustrated technical guide - 1 pce**

4) **Instruction for use on paper and/or in electronic form - 1 pce**

Characteristics	Value	Value	Value	Value	Value	Value	Value	Value	Value	Value	Value	Value	Value
	NEO Spectra ST LV, shade: A1	NEO Spectra ST LV, shade: A2	NEO Spectra ST LV, shade: A3	NEO Spectra ST LV, shade: A3.5	NEO Spectra ST LV, shade: A4	NEO Spectra ST HV, shade: A1	NEO Spectra ST HV, shade: A2	NEO Spectra ST HV, shade: A3	NEO Spectra ST HV, shade: A3.5	NEO Spectra ST HV, shade: A4	NEO Spectra ST Effects, shade: D1	NEO Spectra ST Effects, shade: D3	NEO Spectra ST Effects, shade: E1
Physical and mechanical properties													
Compressive strength	no less than 400	no less than 400	no less than 400	no less than 400	no less than 400	no less than 400	no less than 400	no less than 400	no less than 400	no less than 400	no less than 400	no less than 400	no less than 400
Flexural strength, MPa	no less than 122	no less than 122	no less than 122	no less than 122	no less than 122	no less than 115	no less than 115	no less than 115	no less than 115	no less than 115	no less than 115	no less than 115	no less than 115
N value characterizing the elasticity in bending, MPa	8,5 ±5%	8,5±5%	8,5±5%	8,5±5%	8,5±5%	8,5±5%	8,5±5%	8,5±5%	8,5±5%	8,5±5%	8,5±5%;	8,5±5%	8,5±5%
Flow, mm	24 – 29	24 – 29	24 – 29	24 – 29	24 – 29	17 - 22	17 - 22	17 - 22	17 - 22	17 - 22	17 - 22	17 - 22	17 - 22
Cure time, sec	from 10 to 40 sec (depending on the lamp power)	from 10 to 40 sec (depending on the lamp power)	from 10 to 40 sec (depending on the lamp power)	from 10 to 40 sec (depending on the lamp power)	from 10 to 40 sec (depending on the lamp power)	from 10 to 40 sec (depending on the lamp power)	from 10 to 40 sec (depending on the lamp power)	from 10 to 40 sec (depending on the lamp power)	from 10 to 40 sec (depending on the lamp power)	from 10 to 40 sec (depending on the lamp power)	from 10 to 40 sec (depending on the lamp power)	from 10 to 40 sec (depending on the lamp power)	from 10 to 40 sec (depending on the lamp power)
Sensitivity to ambient light, s	300	310	170	160	260	220	330	200	210	270	170	160	150

Depth of cure for 10 s, 800 mW/cm ² , mm	no less than 2,15	no less than 1,94	no less than 2,04	no less than 1,97	no less than 1,77	no less than 2,14	no less than 2,04	no less than 1,97	no less than 1,85	no less than 1,77	no less than 2,12	no less than 1,94	no less than 2,66
Depth of cure for 20 s, 500 mW/cm ² , mm	no less than 2,25	no less than 2,05	no less than 2,13	no less than 2,05	no less than 2,06	no less than 2,57	no less than 2,19	no less than 2,18	no less than 2,05	no less than 1,98	no less than 2,28	no less than 2,11	no less than 2,54
Radiopacity, %	3,1 mm equiv. Al	3,1 mm equiv. Al	3,1 mm equiv. Al	3,1 mm equiv. Al	3,1 mm equiv. Al	3,0 mm equiv. Al	3,0 mm equiv. Al	3,0 mm equiv. Al	3,0 mm equiv. Al	3,0 mm equiv. Al	3,0 mm equiv. Al	3,0 mm equiv. Al	3,0 mm equiv. Al
Aesthetic properties													
Transparency coefficient, %	41 ± 2	41 ± 2	41 ± 2	41 ± 2	41 ± 2	41 ± 2	41 ± 2	41 ± 2	41 ± 2	41 ± 2	63 ± 2	55 ± 2	38 ± 2

13. COMPOSITION

Neo Spectra ST HV, Neo Spectra ST, Effects Neo Spectra ST LV	
Urethane resin EBPADMA	
Polysiloxane modified with methacrylates (organically modified ceramics)	
BHT (butyl hydroxytoluene stabilizer)	
Lumilux (fluorescent substance)	
2-hydroxy-4-methoxybenzophenone	
Camphorquinone	
Ethyl 4 (dimethylamino) benzoate (DMABE)	
Me2-DPI (bis (4-methylphenyl) iodonium hexafluorophosphate)	
Barium-aluminum-borosilicate glass containing fluoride with added silane	
Barium-aluminum-borosilicate glass containing fluoride with the addition of silane (SphereTEC® filler)	
Ytterbium fluoride	
Titanium dioxide	
Black iron oxide pigment	
Yellow iron oxide pigment	
Red iron oxide pigment	

Syringe materials	
Components	Materials
Barrel	Polypropylene
Luer lock	Polybutylene terephthalate
Plunger piston	Polypropylene
Colored syringe ring	Polyethylene PE-HD
	Colour A1: Masterbatch UN Lightpink
	Colour A2: Masterbatch PE ROSA
	Colour A3: Masterbatch PE Rot
	Colour A3.5: Masterbatch UN Rot
	Colour A4: Masterbatch PP Violet
	Colour D1: Masterbatch UN IVORY
	Colour D3: Masterbatch PE YELLOW
	Colour E1: Masterbatch PE TURQUOISE
	Polypropylene
Cap	Polypropylene
Sleeve	Polypropylene

Compule materials	
Component	Material
Body frame	Polybutylene terephthalate reinforced with glass fiber
Tip	Polyethylene PE-HD
	Colour A1: Masterbatch UN Lightpink
	Colour A2: Masterbatch PE ROSA
	Colour A3: Masterbatch PE Rot
	Colour A3.5: Masterbatch UN Rot
	Colour A4: Masterbatch PP Violet
	Colour D1: Masterbatch UN IVORY
	Colour D3: Masterbatch PE YELLOW
	Colour E1: Masterbatch PE TURQUOISE
Compule tube	Stainless steel 1.4301

Applicator materials	
Component	Material
Handle material	PP homopolymer
	Stabilizers
Nylon flock	Titanium dioxide
	Polyamide nylon-6.6
	Absorbed water

Adhesive, palette, gun materials	
Component	Materials are presented in Registration Certificate
Prime & Bond® universal adhesive in bottle (2.5 ml)	Registration Certificate No. RZN (Roszdravnadzor) 2018/7377 dated July 18, 2018
CliXdish plastic mixing well	Registration Certificate No. RZN (Roszdravnadzor) 2018/7377 dated July 18, 2018
Compules® Tips Gun	Registration Certificate No. RZN (Roszdravnadzor) 2017/5954 dated July 10, 2017

Package materials	
Secondary packaging	Materials
Folded boxes	Chromo board
Plastic sphere for compulsive introductory kits	Polypropylene
Compules Tips – inner bags	Polyethylene terephthalate (PET) film laminated with aluminum foil and polyethylene (PE)
Compules Tips liners – outer bags	Polyethylene terephthalate (PET) film laminated with polyethylene (PE)
Syringe Refills – outer pouches	Polyethylene terephthalate (PET) film laminated with aluminum foil and polyethylene (PE)

14. SUPPLY PACKAGE

NEO Spectra ST Kits for anterior and posterior teeth restoration in design variants:

1. NEO Spectra ST LV Intro Kit in syringes, consisting of:
 - 1) Light-cured dental restorative material NEO Spectra ST LV in syringes, 3 g each, shades: A1 (1 pce), A2 (1 pce), A3 (1 pce), A3.5 (1 pce), A4 (1 pce) - 5 pcs;
 - 2) Prime & Bond® universal adhesive in bottle, 2.5 ml - 1 pce;
 - 3) Applicators - 50 pcs;
 - 4) CliXdish plastic mixing well - 1 pce;
 - 5) Illustrated technical guide - 1 pcs;
 - 6) Order form for NEO Spectra ST Shade Guide - 1 pce (if necessary);
 - 7) i-Shade label - 1 pce;
 - 8) Instruction for use on paper and/or in electronic form - 2 pcs;
 - 9) Information leaflet - 1 pce (if necessary).
2. NEO Spectra ST LV Eco Kit in syringes, consisting of:
 - 1) Light-cured dental restorative material NEO Spectra ST LV in syringes, 3 g each, shades: A2 (3 pcs), A3 (3 pcs) – 6 pcs;
 - 2) Prime & Bond® universal adhesive in bottle, 2.5 ml - 1 pce;
 - 3) Applicators - 50 pcs;
 - 4) CliXdish plastic mixing well - 1 pce;
 - 5) Illustrated technical guide - 1 pce;
 - 6) Order form for NEO Spectra ST Shade Guide - 1 pce (if necessary);
 - 7) i-Shade label- 1 pce;
 - 8) Instruction for use on paper and/or in electronic form - 2 pcs;
 - 9) Information leaflet - 1 pce (при необходимости).
3. NEO Spectra ST HV Intro Kit in syringes, consisting of:
 - 1) Light-cured dental restorative material NEO Spectra ST HV in syringes, 3 g each, shades: A1 (1 pce), A2 (1 pce), A3 (1 pce), A3.5 (1 pce), A4 (1 pce) - 5 pcs;
 - 2) Prime & Bond® universal adhesive in bottle, 2.5 ml - 1 pcs;
 - 3) Applicators - 50 pcs;
 - 4) CliXdish plastic mixing well - 1 pce;
 - 5) Illustrated technical guide - 1 pce;
 - 6) Order form for NEO Spectra ST Shade Guide - 1 pce (if necessary);
 - 7) i-Shade label - 1 pce;
 - 8) Instruction for use on paper and/or in electronic form - 2 pcs;
 - 9) Information leaflet - 1 pce (if necessary).
4. NEO Spectra ST HV Eco Kit in syringes, consisting of:
 - 1) Light-cured dental restorative material NEO Spectra ST HV in syringes, 3 g each, shades: A2 (3 pcs), A3 (3 pcs) – 6 pcs
 - 2) Prime & Bond® universal adhesive in bottles, 2.5 ml - 1 pce;
 - 3) Applicators - 50 pcs;
 - 4) CliXdish plastic mixing well - 1 pce;
 - 5) Illustrated technical guide - 1 pce;
 - 6) Order form for NEO Spectra ST Shade Guide - 1 pce (if necessary);
 - 7) i-Shade label - 1 pce;
 - 8) Instruction for use on paper and/or in electronic form - 2 pcs;
 - 9) Information leaflet - 1 pce (if necessary).

5. NEO Spectra ST Effects Intro Kit in syringes, consisting of:
 - 1) Light-cured dental restorative material NEO Spectra ST Effects in syringes, 3 g each, shades: D1 (1 pce), D3 (1 pce), E1 (1 pce) - 3 pcs;
 - 2) Illustrated technical guide - 1 pce;
 - 3) i-Shade label - 1 pce;
 - 4) Instruction for use on paper and/or in electronic form - 1 pce

6. NEO Spectra ST Intro Kit LV in compules, consisting of:
 - 1) Composite material NEO Spectra ST LV (Compules® tips) in compules, 0.25 g each, shades: A1 (4 pcs), A2 (4 pcs), A3 (8 pcs), A3.5 (4 pcs), A4 (4 pcs) - 24 pcs;
 - 2) Prime & Bond® universal adhesive in bottles, 2.5 ml - 1 pce;
 - 3) Compules® Tips Gun - 1 pce;
 - 4) Applicators - 50 pcs;
 - 5) CliXdish plastic mixing well - 1 pce;
 - 6) Illustrated technical guide - 1 pce;
 - 7) i-Shade label- 1 pce;
 - 8) Order form for NEO Spectra ST Shade Guide - 1 pce (if necessary);
 - 9) Instruction for use on paper and/or in electronic form - 2 pcs;
 - 10) Information leaflet - 1 pce (if necessary).

7. NEO Spectra ST Discovery Kit LV in compules, consisting of:
 - 1) Composite material NEO Spectra ST LV (Compules® tips) in compules, 0.25 g each, shade A2 - 4 pcs
 - 2) Illustrated technical guide - 1 pce;
 - 3) Instruction for use on paper and/or in electronic form - 1 pce.

8. NEO Spectra ST LV Risk Free Trial Kit in compules, consisting of:
 - 1) Composite material NEO Spectra ST LV (Compules® tips) in compules, 0.25 g each, shades: A1 (4 pcs), A2 (4 pcs), A3 (8 pcs), A3.5 (4 pcs), A4 (4 pcs) - 24 pcs;
 - 2) Composite material NEO Spectra ST LV (Compules® tips) in compules, 0.25 g each, shade A2 - 4 pcs;
 - 3) Prime & Bond® universal adhesive in bottles, 2.5 ml each - 1 pce;
 - 4) Compules® Tips Gun - 1 pce;
 - 5) Applicators - 50 pcs;
 - 6) CliXdish plastic mixing well - 1 pce;
 - 7) Illustrated technical guide - 2 pcs;
 - 8) i-Shade label - 1 pce;
 - 9) Order form for NEO Spectra ST Shade Guide - 1 pce (if necessary);
 - 10) Instruction for use on paper and/or in electronic form - 3 pcs;
 - 11) Information leaflet - 1 pce (if necessary).

9. NEO Spectra ST Intro Kit HV in compules, consisting of:
 - 1) Composite material NEO Spectra ST HV (Compules® tips) in compules, 0.25 g each, shades: A1 (4 pcs), A2 (4 pcs), A3 (8 pcs), A3.5 (4 pcs), A4 (4 pcs) - 24 pcs;
 - 2) Prime & Bond® universal adhesive in bottle, 2.5 ml - 1 pce;
 - 3) Compules® Tips Gun - 1 pce;
 - 4) Applicators - 50 pcs;
 - 5) CliXdish plastic mixing well - 1 pce;

- 6) Illustrated technical guide - 1 pce;
 - 7) i-Shade label - 1 pce;
 - 8) Order form for NEO Spectra ST Shade Guide - 1 pce (if necessary);
 - 9) Instruction for use on paper and/or in electronic form - 2 pcs;
 - 10) Information leaflet - 1 pce (if necessary).
10. NEO Spectra ST Discovery Kit HV in compules, consisting of:
- 1) Composite material NEO Spectra ST HV (Compules® tips) in compules, 0.25 g each, shade A2 - 4 pcs;
 - 2) Illustrated technical guides - 1 pce;
 - 3) Instruction for use on paper and/or in electronic form - 1 pce.
11. NEO Spectra ST HV Risk Free Trial Kit in compules, consisting of:
- 1) Composite material NEO Spectra ST HV (Compules® tips) in compules, 0.25 g each, shades: A1 (4 pcs), A2 (4 pcs), A3 (8 pcs), A3.5 (4 pcs), A4 (4 pcs) - 24 pcs;
 - 2) Composite material NEO Spectra ST HV (Compules® tips) in compules, 0.25 g each, shade A2 - 4 pcs;
 - 3) Prime & Bond® universal adhesive in bottle, 2.5 ml - 1 pce;
 - 4) Compules® Tips Gun - 1 pce;
 - 5) Applicators - 50 pcs;
 - 6) CliXdish plastic mixing well - 1 pce;
 - 7) Illustrated technical guides - 2 pcs;
 - 8) i-Shade label - 1 pce;
 - 9) Order form for NEO Spectra ST Shade Guide - 1 pce (if necessary);
 - 10) Instruction for use on paper and/or in electronic form - 3 pcs;
 - 11) Information leaflet - 1 pce (if necessary).
12. NEO Spectra ST Effects Intro Kit in compules, consisting of:
- 1) Composite material NEO Spectra ST Effects (Compules® tips) in compules, 0.25 g each, shades: D1 (12 pcs), D3 (8 pcs), E1 (4 pcs) - 24 pcs;
 - 2) Illustrated technical guides - 1 pcs;
 - 3) i-Shade label - 1 pce;
 - 4) Instruction for use on paper and/or in electronic form - 1 pce;
 - 5) Information leaflet - 1 pce (if necessary).
13. NEO Spectra ST, NEO Spectra ST Effects Sample in compules, consisting of:
- 1) Composite material NEO Spectra ST Effects (Compules® tips) in compules, 0.25 g each, shades: D1 (4 pcs), E1 (4 pcs) - 8 pcs;
 - 2) Composite material NEO Spectra ST HV (Compules® tips) in compules, 0.25 g each, shade A2 - 4 pcs;
 - 3) Illustrated technical guide - 1 pce;
 - 4) Instruction for use on paper and/or in electronic form - 1 pce.

15. HYGIENE

Cross contamination.

Infection.

- Do not reuse single use products. They should be disposed of in accordance with local regulations.
- Syringes cannot be reprocessed. Dispose of contaminated syringes in accordance with local regulations.
- Reprocess reusable products according to the instructions.

a. Compules® Tips Gun

Refer to the Compules® Tips Gun Instructions for Use at www.dentsplysirona.com and www.dentsply.eu/IFU for reprocessing instructions. Upon request, we can send you a free printed manual in the language you need within 7 days. You can find the order form at www.dentsply.eu/IFU.

b. Compules® Tips – cross contamination

Cross contamination.

Infection.

- Do not reuse Compules® Tips. Dispose of Compules® Tips in accordance with local regulations.

c. Syringes – cross contamination

Cross contamination.

Infection.

- Syringes cannot be reprocessed.
- To prevent contamination of syringes with splashes or droplets of body fluids or material from untreated hands, it is mandatory that the syringe is handled offside the dental unit with clean/disinfected gloves. Do not reuse syringes if contaminated.
- Dispose contaminated syringes according to local regulations.

Replace the original syringe cap before storage. It is recommended to use a protective barrier to prevent splashing of body fluids or contaminated hands or oral tissues on the syringes. The use of protective barriers is an additional precaution against gross debris, but not against all contamination.

Incidental contact of the syringe with water, soap, or a water-based hospital-level disinfectant solution will not damage the syringe body. Do not allow the solution to come into contact with the material contained. Discard composite material that has been in contact with any fluid or non-sterile instrument.

Dry the syringe with a lint-free disposable cloth. Wipe the syringe gently without pressure.

d. Shade guide and Individual Shade Guide Tabs - Cross Contamination

	Cross contamination. Infection. • The shade guide and individual shade guide tabs cannot be reprocessed. • Use clean/sterile gloves to prevent contamination from splashes, body fluids or contaminated hands. Do not reuse contaminated shade guide and individual shade guide tabs. • Dispose of contaminated shade guide and individual shade guide tabs in accordance with local regulations.
---	---

16. GENERAL DESCRIPTION AND METHOD OF APPLICATION

Neo Spectra ST/Neo Spectra ST Effects is nano-ceramic, light-cured, radiopaque universal composite based on the innovative SphereTEC® filler technology for both direct and indirect restorations.

SphereTEC® technology (granular, spherical filler) drives filler granules like balls in a bearing (ball-bearing effect), which provides new levels of handling properties to Neo Spectra ST/Neo Spectra ST Effects:

- **Neo Spectra ST/Neo Spectra ST Effects easily adapt to cavity surfaces but does not stick to hand instruments.**
- **Neo Spectra ST/Neo Spectra ST Effects are easy to sculpt precisely and have a high slump resistance.**
- **Neo Spectra ST/Neo Spectra ST Effects provide a firm consistency but are at the same time easy to extrude from Compules® Tips.**

Neo Spectra ST includes five universal shades of intermediate translucency, from  to , each of them named "CLOUD shade".

Each "CLOUD shade" is designed to match VITA® shade guide. Overall, the five universal shades cover all 16 classical VITA® shades and provide an aesthetic clinical result. Neo Spectra ST Effects includes 2 dentin shades (D1 and D3), and translucent enamel shade (E1), which can be used in combination with the Neo Spectra ST universal to create restorations for additional opacity and/or translucency for highly aesthetic results.

Spherical granulated fillers, in turn, consisting of submicron primary glass fillers, provide a quick and easy polishing procedure resulting in an outstanding gloss.

- Prime & Bond® universal is a combination of Etch & Rinse, Selective Etch and Self Etch adhesive. It provides easy adhesive application for direct and indirect restorations and is capable of adhering to enamel, dentin, composites, zirconia and metals. Prime & Bond® universal is compatible with conventional methacrylate light-cured composite restorations and cements.

Registration Certificate No. RZN 2018/7377 dated July 18, 2018.

- CliXdish plastic mixing well.

Registration Certificate No. RZN 2018/7377 dated July 18, 2018.

- Compules® Tips Gun.

Registration Certificate No. RZN 2017/5954 dated July 10, 2017.

Illustrated technical guide of NEO Spectra ST dental material in pictures with explanations. The text of the document is contained in Annex 1 to the instruction for use for the medical device.

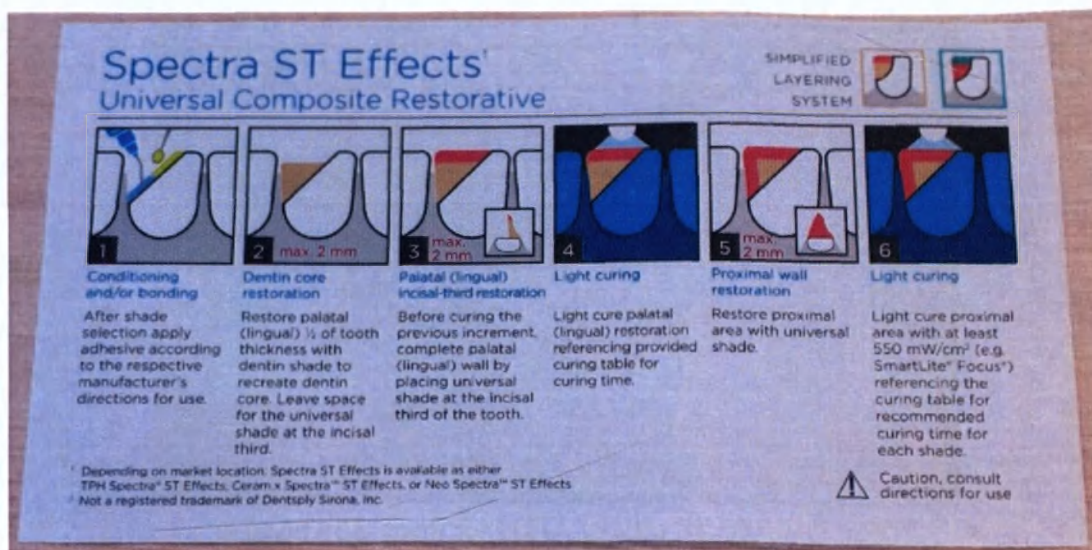
Fig.1 Illustrated technical guide

If the Neo Spectra ST Shade Guide is not available, we recommend sticking the i-Shade label to the back of the VITA[®] Classic Scale. Essential for determining the shade of the tooth and using the correct shade of NEO Spectra ST Dental Material.

The text of the document is contained in Annex 1 to the instruction for use technical file for the medical device.

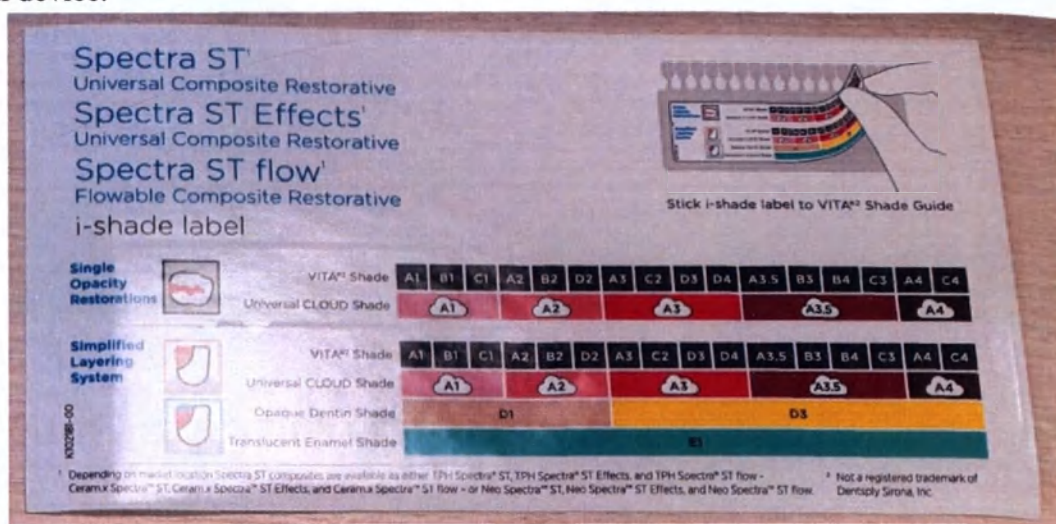
Fig.2 i-Shade label

Order form for NEO Spectra ST Shade Guide is required to order the Neo Spectra ST Shade Guide. Supplied if necessary.

The text of the document is contained in Annex 1 to the instruction for use technical file for the medical device.

Fig.3 Order form for NEO Spectra ST Shade Guide

The information leaflet contains general information about the NEO Spectra ST dental material and the available design variants.

The text of the document is contained in Annex 1 to the instruction for use technical file for the medical device.

Fig.4 The information leaflet

METHOD OF APPLICATION OF THE MEDICAL DEVICE

Color matching

1. Before choosing a shade, clean the surface of the tooth with a prophylaxis paste to remove plaque or stains on the surface of the tooth.

2. Match the shade when the surface of the tooth is hydrated.

The restorative material Neo Spectra ST was developed to allow the tooth structure enamel and dentin to blend together with the composite with a life-like translucent result. The 5 A shades of Neo Spectra ST reproduce the traditional 16 shades of the VITA® shade guide.

To achieve the desired highly aesthetic result, Neo Spectra ST Effects offers 2 dentin shades and 1 enamel shade, that work with the Neo Spectra™ ST universal CLOUD shades in a simplified layering system to provide added opacity and/or translucency as needed.

There are two tools for choosing a shade that can be used:

- Neo Spectra ST Shade Guide. The Neo Spectra ST Shade Guide consists of shaded inlays made of Neo Spectra ST composite material. To match the shade, remove the individual tabs from the holder. To determine the material shade required for the restoration using the Neo Spectra ST shade, select the shade that best matches the surface to be restored on the shaded insert.

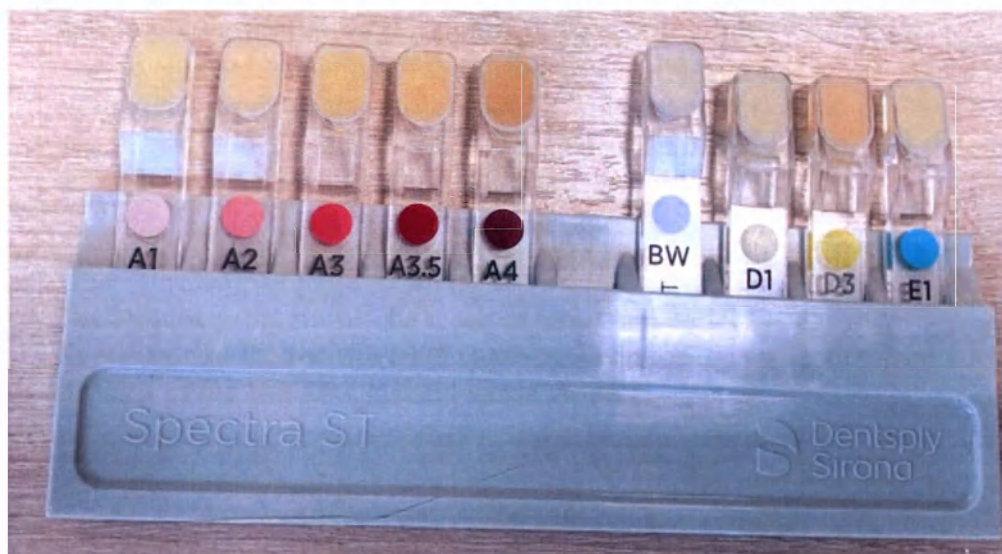
Fig.5 Neo Spectra ST Shade Guide

It is a scale for the selection of the shade of a human tooth for the selection of a suitable shade for restoration.

- VITA® classical shade guide in combination with i-shade label. The i-shade label simplifies shade matching of the restoration and demonstrates the correspondence of each shade of Neo Spectra ST to several shades of the classic VITA® scale.

Fig. 6 Correspondence of shades of Neo Spectra ST to VITA® shades

* VITA® is not a registered trademark of Dentsply Sirona.

The VITA® Shade Guide (RC No. FSZ 2011/09519, No. FSZ 2010/07252) is designed to precisely match the shade of the restored tooth. The VITA® Shade Guide is essential for a high-quality and correct selection of shades for filling or prosthetics. The shade guide helps to choose the color of the tooth / prosthesis so that it is in harmony with the shade of the adjacent teeth, the skin tone of the patient's face.

*Not a product of Dentsply De Trey GmbH.

16.1.1 Selecting a shade using the Neo Spectra ST/Neo Spectra ST Effects shade guide

Use the shade tabs of the Neo Spectra ST/Neo Spectra ST Effects shade guide to select the shade that is closest in color to the area of the dentition to be restored.

Avoid contamination of the shade guide (color guide).

To prevent contamination of the shade guide with splashes or drops of body fluids or material from contaminated hands, it is mandatory that the shade guide is handled offside the dental unit with clean/disinfected gloves. To determine the tooth shade, remove individual tabs from the holder.

16.1.2 Shade selection using VITA® classical shade guide and i-shade label

The i-shade label should be close at hand.

To ensure that the i-shade label is available chairside, we recommend sticking it to the back of the VITA® Classical shade guide.

1. Select the corresponding tooth on the VITA® classical shade guide, the central part of which best matches the area of the tooth to be restored.

2. Use the i-shade label (see table below) to determine the appropriate shade of Neo Spectra ST universal CLOUD shade and Neo Spectra ST Effects shade(s) if necessary.

VITA® Classical shade guide	A 1	B 1	C 1	A 2	B 2	D 2	A 3	C 2	D 3	D 4	A3, 5	B 3	B 4	C 3	A 4	C 4
Recommend d shades of Neo Spectra ST	A1			A2			A3				A3,5			A		
Recommend d shades of Neo Spectra ST Effects	D1						D3									
	E1															

16.2 Cavity preparation

1. Clean the surface of the tooth with a fluoride-free prophylaxis paste (eg. Nupro[®] prophylaxis paste).
2. Prepare the cavity (unless not necessary, e.g., cervical lesion).
3. Use adequate isolation such as rubber dam.
4. Rinse the surface with a water spray and carefully remove rinsing water. Do not desiccate the tooth structure.

16.2.1 Matrix placement and wedging (Class II restorations)

Place a matrix (e.g. Palodent[®] Matrix System) and a wedge. Burnishing of the matrix band will improve the contact and contour. Pre-wedging or matrix ring placement is recommended.

16.3 Pulp protection, tooth conditioning/dentin pretreatment, adhesive application

Refer to the adhesive manufacturer's recommendations for pulp protection, tooth conditioning, and / or adhesive application. Once the surfaces have been properly treated, they should not be contaminated. Immediately proceed with material application.

16.4 Application

Apply the composite in increments immediately after applying the adhesive. The composite material is cured in increments up to 2 mm thickness.

Compules[®] Tips application

1. Insert a Compules[®] Tip into the notched opening of the Compules[®] Tips Gun. Make sure that the collar on the e Compules[®] Tip is inserted first.

2. Remove the colored cap from the compule. The tip of the compule can be rotated 360° for easy introduction of the material into the cavity or to the mixing pad. DO NOT USE EXCESSIVE FORCE.

Indirect application: Dispense the required amount of material onto a mixing pad and protect from light. Apply material into the cavity with a plastic instrument.

- or -

Direct intraoral application: Dispense the material directly into the cavity using a slow, steady pressure.

3. To remove the used Compules[®] Tips, make sure the plunger of the Compules[®] Tips Gun is pulled back completely by allowing the handle to open to its widest position. Apply a downward motion to the front end of the Compules[®] Tip and remove.

Excessive force.

Injury.

1. Apply slow and steady pressure to the Compules[®] Tips Gun.
2. Do not use excessive force. Compules[®] Tips rupture or ejection from Compules[®] Tips Gun may result.

Syringe application

Avoid syringe contamination.

To prevent contamination of the syringe with splashes or drops of body fluids or contaminated hands, it is mandatory that the syringe is handled offside the dental unit with clean/disinfected gloves.

1. Remove the cap.
2. Slowly turn the syringe handle clockwise and dispense the required amount of material onto a mixing pad.
3. Point the front tip of the syringe upwards, turn the handle counterclockwise to prevent oozing of the material.
4. Immediately close the syringe with the cap.
5. Protect the composite on the mixing pad from light.

16.4.1 Applying of the composite material

1. Fill the whole cavity incrementally. Each layer is no more than 2 mm.
2. Cure each layer according to the table below.

Technique Tips:

- Use a single opacity for most common cases: Because the Neo Spectra™ ST universal CLOUD shades are designed to replace both dentin and enamel, a single universal CLOUD shade is sufficient for most restorations. If esthetic blending of more than one universal CLOUD shade is desired, shade layers may be individually cured to form a foundation for subsequent shade placement.

- Use multiple opacities for esthetically demanding cases: For esthetically demanding cases such as a large Class IV or a through-and-through Class III, Neo Spectra™ ST Effects dentin and enamel shades can be used with Neo Spectra™ ST universal CLOUD shades in a simplified layering system to provide added opacity and/or translucency.

- Restore the palatal half of the tooth with dentin shade². Leave space for the universal CLOUD shade along the incisal edge.

- Place the universal CLOUD shade along the incisal edge and polymerize the finished palatal layer.

- Restore the facial half of the tooth with the universal CLOUD shade². If increased translucency along the incisal edge is desired, place the enamel shade, then polymerize the completed facial layer.

² Light curing of individual increments required when layer thickness reaches 2 mm.

16.5 Light curing

1. Cure each increment of the composite with a suitable curing light³. Cure shades A, E1 for 10 seconds, and shades D1, D3 for 30 seconds when using a curing light that has a minimum irradiance of 800 mW/cm².

When minimum irradiance is between 550 mW/cm² and 800 mW/cm², shades A are cured for 20 seconds, E1 for 10 seconds, and D1 and D3 for 40 seconds.

Consult the curing light manufacturer's recommendations for compatibility and curing recommendations.

Neo Spectra ST and Neo Spectra ST Effects curing recommendations			
Shades	Power mW/cm ²	Time of cure	Layer thickness
A1-A4	≥ 550	20 seconds	2 mm
	≥ 800	10 seconds	
D1, D3	≥ 550	40 seconds	2 mm
	≥ 800	30 seconds	
E1	≥ 550	10 seconds	2 mm

Time of cure.

The curing time table above also appears on all outer packages.

Insufficient curing.

Inadequate polymerization.

1. Check the compatibility of the curing light.
2. Check the length of the cure cycle.
3. Check the minimum irradiance.
4. Cure each area of each increment for the recommended curing time.
5. Check the distance to the surface to be cured.

16.6 Finishing and polishing

1. Contour the restoration using finishing or diamond burs.
2. Use the Enhance[®] system for additional finishing.
3. Before the polishing stage, make sure the surface is smooth and free from defects.
4. For achieving a very high luster of the restoration, we recommend Enhance[®] and Enhance[®] PoGo[®] finishing and polishing systems and the use of Prisma[®] Gloss[™] and Extra fine polishing pastes.

Follow the manufacturer's recommendations for contouring, finishing and/or polishing.

³ Curing light designed to cure materials containing camphorquinone (CQ) initiator. Peak of spectrum in the range of 440-480 nm.

17. CLEANING AND DISINFECTION

The medical device is not subject to cleaning or disinfection. The medical device is not sterile or sterilizable.

Refer to the Compules® Tips Gun Instructions for Use at www.dentsplysirona.com and www.dentsply.eu/IFU for cleaning or disinfection instructions. Upon request, we can send you a free printed manual in the language you need within 7 days. You can find the order form at www.dentsply.eu/IFU.

18. MAINTENANCE

The medical device is not subject to maintenance and repair.

19. OPERATING, STORAGE AND TRANSPORTATION CONDITIONS

Operating conditions

Parameter	Values
Temperature	from +5°C to +42 °C
Humidity	up to 98% relative humidity at temperature 25°C
Atmosphere pressure	from 70 kPa to 106 kPa

Products are regularly operated inside the oral cavity, if there are biological fluids in it at nominal temperatures: from + 32 ° C to + 42 ° C, for accessories not used in the oral cavity: from + 10 ° C to + 35 ° C.

Storage conditions.

Parameter	Values
Temperature	from +2°C to +28 °C
Humidity	no more than 70%
Atmosphere pressure	from 70 kPa to 106 kPa

Transport conditions

Parameter	Values
Temperature	from +5°C to +30 °C
Humidity	up to 70%
Atmosphere pressure	from 70 kPa to 106 kPa

20. EXPIRATION DATE OF THE MEDICAL DEVICE

Expiration date of kits:

1. NEO Spectra ST LV Intro Kit in syringes – 2 years.
2. NEO Spectra ST LV Eco Kit in syringes – 2 years.
3. NEO Spectra ST HV Intro Kit in syringes – 2 years.
4. NEO Spectra ST HV Eco Kit in syringes – 2 years.

5. NEO Spectra ST Effects Intro Kit in syringes – 3 years.
 6. NEO Spectra ST Intro Kit LV in compules – 2 years.
 7. NEO Spectra ST Discovery Kit LV in compules – 3 years.
 8. NEO Spectra ST Intro Kit LV in compules completed with NEO Spectra ST Discovery Kit LV in compules – 2 years.
 9. NEO Spectra ST Intro Kit HV in compules – 2 years.
 10. NEO Spectra ST Discovery Kit HV in compules – 3 years.
 11. NEO Spectra ST Intro Kit HV in compules completed with NEO Spectra ST Discovery Kit HV in compules – 2 years.
 12. NEO Spectra ST Effects Intro Kit in compules – 3 years.
 13. NEO Spectra ST, NEO Spectra ST Effects Sample in compules – 3 years.
- Expiration of Prime&Bond® universal adhesive in bottle - 2 years.
- Expiration for Compules® Tips Gun – 5 years.
- Expiration of CliXdish plastic mixing well – 5 years.
- Expiration of applicators – 5 years.

21. LABELING

Individual packaging labeling of compules:

Labeling is applied to individual packaging in accordance with GOST R ISO 15223-1-2014.

- the name of the medical device;
- batch code;
- expiration date;

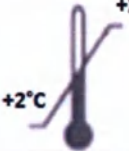
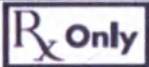
Symbol	Meaning
	Batch code
	Use by....
	Refer to Instructions for use
	Compliance with EU directives
Shade	Shade

Individual packaging labeling of syringes:

Labeling is applied to consumer packaging in accordance with GOST R ISO 15223-1-2014.

- the name and address of the manufacturer;
- the name of the medical device;
- manufacturing date;
- number of the medical device according to the manufacturer's catalog;
- batch code;
- expiration date;
- for other information and warning labels and symbols, see below

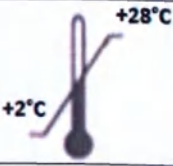
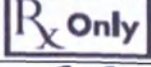
Symbol	Meaning
	Manufacturer
	Number according to the manufacturer's catalog
	Batch code
	Use by....
	Refer to Instructions for use/ Instruction in electronic form
	Caution! Refer to Instructions for use
Shade	Shade

	Do not reuse
	Avoid exposure to sunlight
	Temperature Range
	Keep away from moisture
	Symbol of the trade mark "Green dot"
	Symbol "Attention! This product may only be sold to a physician, or supplied on the order of a physician. "
	Compliance with EU directives

Consumer packaging labeling

Labeling is applied to consumer packaging in accordance with GOST R ISO 15223-1-2014.

- the name and address of the manufacturer;
- the name of the medical device;
- manufacturing date;
- number of the medical device according to the manufacturer's catalog;
- batch code;
- expiration date;
- for other information and warning labels and symbols, see table below

Symbol	Meaning
	Manufacturer
	Number according to the manufacturer's catalog
	Batch code
	Use by....
	Refer to Instructions for use
	Caution! Refer to Instructions for use
Shade	Shade
	Do not reuse
	Avoid exposure to sunlight
	Temperature Range
	Keep away from moisture
	Symbol of the trade mark "Green dot"
	Symbol "Attention! This product may only be sold to a physician, or supplied on the order of a physician. "
	Compliance with EU directives

Transport packaging labeling

Labeling is applied to the transport packaging in accordance with GOST R ISO 15223-1-2014.

- the name and address of the manufacturer;
- the name of the medical device;
- manufacturing date;
- number of the medical device according to the manufacturer's catalog;
- batch code;
- catalog number
- expiration date;
- for other information and warning labels and symbols, see table below

Symbol	Meaning
	Manufacturer
	Manufacturing date
	batch code
	number according to the manufacturer's catalog
	Temperature Range
	Humidity range
	Caution! Refer to Instructions for use
	Use by

22. PACKAGING

For transportation, NEO Spectra ST kits are packed in a corrugated cardboard box in accordance with GOST R 52901, pasted over with polyethylene tape with a sticky layer in accordance with GOST 20477 so as to exclude the possibility of opening without violating the integrity of the package.

23. ENVIRONMENTAL REQUIREMENTS

This medical device does not have a negative impact on humans and the environment during use, transportation and storage

24. DISPOSAL

Dispose the product in accordance with SanPiN 2.1.3684-21 for class B waste (epidemiologically hazardous waste). After hardware disinfection methods using physical methods and changing the appearance of the waste, excluding the possibility of their reuse, class B waste can accumulate, temporarily stored, transported, destroyed and disposed of together with class A waste (epidemiologically safe waste, in composition close to solid household waste). The packaging of disinfected medical waste of class B must have a label indicating that the waste has been disinfected.

25. MANUFACTURER WARRANTY

The manufacturer guarantees the safety of the material, as well as all their parameters and characteristics, during the entire shelf life - subject to the integrity of the individual packaging and observing the conditions of storage, transportation and use in accordance with the instructions for use.

26. RECLAMATION

Organization authorized by the manufacturer to accept claims related to product quality on the territory of the Russian Federation:

Dentsply Sirona Limited Liability Company (Dentsply Sirona, LLC)

Address: 18 Andropov Prospect, Building 6, Offices 1, 39, 40, Moscow 115432, Russia

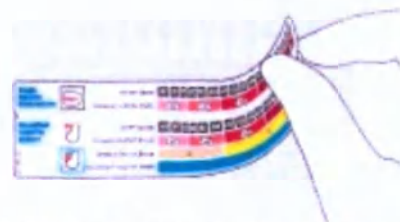
Tel.: +7 (495) 725-10-87

E-mail: MoscowRus-Reception@dentsplysirona.com

ANNEX 1 i-Shade label, Illustrated technical guide, Order form for NEO Spectra ST Shade Guide, The information leaflet

i-Shade label

Spectra ST¹
Universal Composite Restorative
Spectra ST Effects¹
Universal Composite Restorative
Spectra ST flow¹
Flowable Composite Restorative
i-shade label

Stick i-shade label to VITA^{®2} Shade Guide

Single
Opacity
Restorations

VITA^{®2} Shade
Universal CLOUD Shade

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Simplified
Layering
System

VITA^{®2} Shade
Universal CLOUD Shade

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K102181-00

Opaque Dentin Shade

Translucent Enamel Shade

¹ Depending on market location Spectra ST composites are available as either TPH Spectra[®] ST, TPH Spectra[®] ST Effects, and TPH Spectra[®] ST flow – Ceramix Spectra[™] ST, Ceramix Spectra[™] ST Effects, and Ceramix Spectra[™] ST flow – or Neo Spectra[™] ST, Neo Spectra[™] ST Effects, and Neo Spectra[™] ST flow.

² Not a registered trademark of Dentsply Sirona, Inc.

Order form for NEO Spectra ST Shade Guide

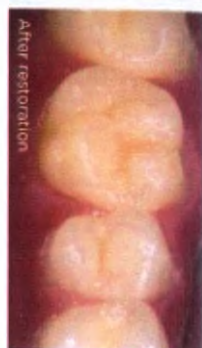
Order your free shade guide

Neo Spectra[™] ST
Composites

Neo Spectra[™] ST
Composites

Easy and reliable shade match
The Neo Spectra[™] ST shade guide is made of original material and enables direct shade selection without the need of translation.

K102286-00 © Dentsply Sirona Q2/2019

Order your free **Neo Spectra™ ST** shade guide

online www.dentsplysirona.com/freespectrastshadeuide
 per fax +49 (0)7531 583-660116
 per mail xplore@dentsplysirona.com

☐ Yes, I want to order
X a free shade guide

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Universal Composite Restorative

SIMPLIFIED
LAYERING
SYSTEM
 

1	2 max. 2 mm	3 max. 2 mm	4	5 max. 2 mm	6
Conditioning and/or bonding	Dentin core restoration	Palatal (lingual) incisal-third restoration	Light curing	Proximal wall restoration	Light curing
After shade selection apply adhesive according to the respective manufacturer's directions for use.	Restore palatal (lingual) 1/3 of tooth thickness with dentin shade to recreate dentin core. Leave space for the universal shade at the incisal third.	Before curing the previous increment, complete palatal (lingual) wall by placing universal shade at the incisal third of the tooth.	Light cure palatal (lingual) restoration referencing provided curing table for curing time.	Restore proximal area with universal shade.	Light cure proximal area with at least 550 mW/cm ² (e.g. SmartLite [®] Focus [®]) referencing the curing table for recommended curing time for each shade.

¹ Depending on market location, Spectra ST Effects is available as either TPH Spectra[®] ST Effects, Ceram.x Spectra[™] ST Effects, or Neo Spectra[™] ST Effects.

² Not a registered trademark of Dentsply Sirona, Inc.

 **Caution, consult directions for use**

The information leaflet

Try the complete
Neo Spectra[™] ST
 portfolio for efficient esthetics.

Learn more ►

**PREFERRED HANDLING
 AND SIMPLIFIED
 ESTHETICS FOR EVERY
 COMPOSITE CASE.**

From common Class II restorations to esthetically demanding anterior cases, there's a Neo Spectra[™] ST composite solution.

Our portfolio offers a complete range of handling options and esthetic capabilities for every direct restoration. They all include our unique SphereTEC[™] technology, so achieving efficient esthetics has never been simpler. For every restoration you face, there's a Neo Spectra[™] ST composite that has you covered.

The key advantages offered by the Neo Spectra™ ST composite portfolio are made possible by the unique design of our proprietary SphereTEC® filler technology.

Optimized handling for easy placement.

Enhanced chameleon blending ability for simplified shading.

Excellent durability, polish, and stain resistance for lasting esthetics.

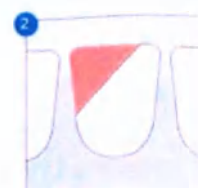
Neo Spectra™ ST Effects

Universal Composite Restorative

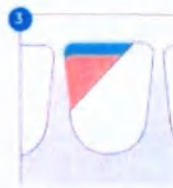
COMPLEX ANTERIOR CASES, SIMPLIFIED.

Restore lingual 1/2 of tooth thickness with composite (C1, D2, D3, D4).

Restore facial 1/2 of tooth thickness with composite (A1, B1, C1, D1).

For a highly translucent incisal edge, optionally use the incisal shade for the facial incisal edge increment.

Neo Spectra™ ST flow

Flowable Composite Restorative

ESTHETIC FLOWABLE VERSATILITY, MADE EFFICIENT.

- Chameleon blending enables universal C1/D2/D3/D4 shades A1-A4 to cover the full VITA® range.
- Shade BW, D1, D3, E1 also available for special cases.
- High-strength, polish, and stain resistance.

Flow-on-demand handling provides optimum versatility.

Neo Spectra™ ST Effects

Universal Composite Restorative

Kit	Compules® Tips	Syringe
A1, A2	607.01.942	607.01.952
A3, A3.5, A4	607.01.943	607.01.953
A1, A2; A3; A3.5; A4	607.01.941	607.01.951

607.01.940 COMPULES® TIPS INTRO KIT
 41 x HV (Shag) Compules® Tips (0.25 g each)
 (12 x D1, 8 x D3, 4 x E1)
 1 Vitrak® Technique Guide
 1 Shade label

607.01.950 SYRINGE INTRO KIT - HV
 8 x HV Syringes (D1, D3, E1) (1.5 g each)
 1 Vitrak® Technique Guide
 1 Shade label

607.01.869 SHADE GUIDE
 1 x shade guide folder
 9 shade guide tabs (A1, A2, A3, A3.5, A4, B1, B2, B3, B4)

Neo Spectra™ ST flow

Flowable Composite Restorative

607.01.910 COMPULA® TIPS INTRO KIT
 24 x Compula® Tips (0.25 g each)
 (4 x A1, 4 x A2, 8 x A3, 4 x A3.5, 4 x A4)
 1 Bottle Prime&Bond Universal™ Adhesive (2.5 mL)
 Accessories

607.01.920 SYRINGE INTRO KIT
 5 x Syringes (A1, A2, A3, A3.5, A4) (1.8 g each)
 1 Bottle Prime&Bond Universal™ Adhesive (2.5 mL)
 Accessories

607.01.932 SYRINGE TIPS REFILL
 60 Syringe Tips

Refill Shade	VITA® Shades
A1	A1, B1, C1
A2	A2, B2, D2
A3	A3, C2, D3, D4
A3.5	A3.5, B3, B4, C3
A4	A4, C4
BW	(Black, White)
D1	(Opaque Dentin)
D3	(Opaque Dentin)
E1	(The deepest Enamel)

TO LEARN MORE ABOUT THE
 NEO SPECTRA™ ST PORTFOLIO, VISIT
www.dentsplysirona.com/NeoSpectraST-composites

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