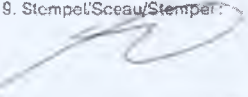
B 00313252

B 00313252

APOSTILLE

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«day» month (numerals)

Exploitation documentation

DIRECTIONS FOR USE OF THE MEDICAL PRODUCT

**Medical absorbing products iD Slip Expert, iD Slip Basic, iD Slip, iD
Innofit, iD Pants, iD Pants For her
(see Paragraph 1 on page 2)**

I, the undersigned, Henri Van den Bossche, notary holding residence at Buggenhout, confirm that the signature on this document is the signature of Mr. Charles Bouaziz and is well known by me.

This confirmation of the signature only concerns its authenticity and does not imply a certification by the notary of the capacity or the powers of the aforementioned signatory.

Done on the 12th of February 2019 at Buggenhout.

**«Ontex BVBA», Genthof 5, 9255 Buggenhout, Belgium
2019**

1. Name of the medical product (hereinafter "MP")

Medical absorbing products iD Slip Expert, iD Slip Basic, iD Slip, iD Innofit, iD Pants, iD Pants For her in the following dimensions (hereinafter "Products"):

1. Adult diapers iD Slip Expert, size: S, M, L, XL.
2. Adult diapers iD Slip Basic, size: S, M, L, XL.
3. Adult diapers iD Slip, size: S, M, L, XL.
4. Adult diapers iD Innofit, size: M, L.
5. Adult pants iD Pants, size: S, M, L, XL.
6. Adult pants iD Pants For her, size: M, L.

2. Information on the medical product

Name of the operating company: Ontex BVBA

Registered address of the company: «Ontex BVBA», Genthof 5, 9255 Buggenhout, Belgium.

Name of the manufacturing companies:

1. «Ontex BVBA», Belgium.
2. «Ontex BVBA Eeklo», Belgium.
3. «Ontex Tüketim Ürünleri Sanayi Ve Ticaret Anonim Sirketi», Turkey.
4. «Ontex Sante France SAS», France.
5. «Ontex CZ s.r.o.», Czech Republic.
6. LLC «Ontex RU», Russia.
7. «Ontex Manufacturing Italy S. R.L.», Italy.

Registered address of the company:

1. Genthof 5, 9255 Buggenhout, Belgium.
2. Korte Moeie 53, 9900 Eeklo, Belgium.
3. Selimpasa Mah. 6093 Sokak No. 78 Silivri, Istanbul, Turkey.
4. Quai du Rivage, 62119 Dourges, France.
5. Turnov, Vesecko cp. 491, PSC 51101, Czech Republic.
6. LLC «Ontex RU», 142400, Moscow Region, Noginsk district, 56km+900m of Highway M-7 „Volga“, 1, bld. 3, Russia.
7. Localita Cucullo, 66026 Ortona, Italy.

Company profile of the manufacturer:

Production, sale and distribution of hygienic and medical products both under own brands and retail brands.

Authorised representative of the manufacturer: LLC «Ontex RU», 105064, Moscow, Zemlyanoy val St, 9, floor 8, pom.II, room 79, www.ontex.ru, Russia.

3. Description of the product

Medical absorbing products iD Slip Expert, iD Slip Basic, iD Slip, iD Innofit, iD Pants, iD Pants For her (see Paragraph 1 on page 2) – single use.

3.1 Description of the product structure

Photo of the diaper is minimized.



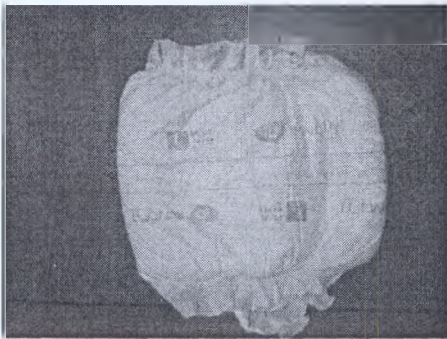
Photo of the diaper in the unfolded state.


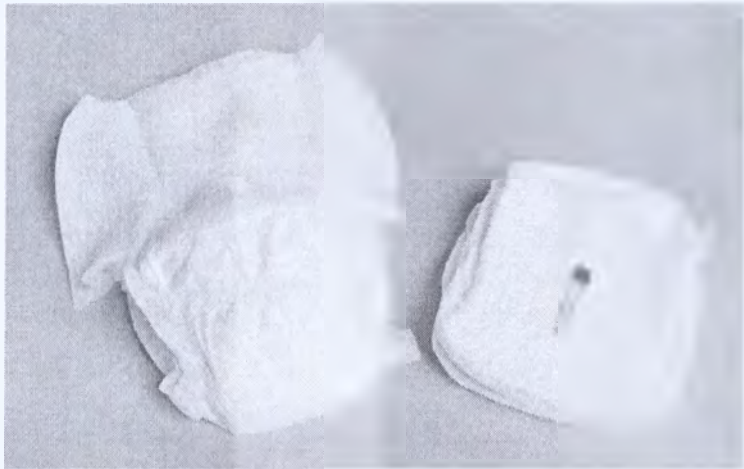
Photo of a diaper pant in the expanded and collapsed views.

Schematic representation of a diaper pant for adults

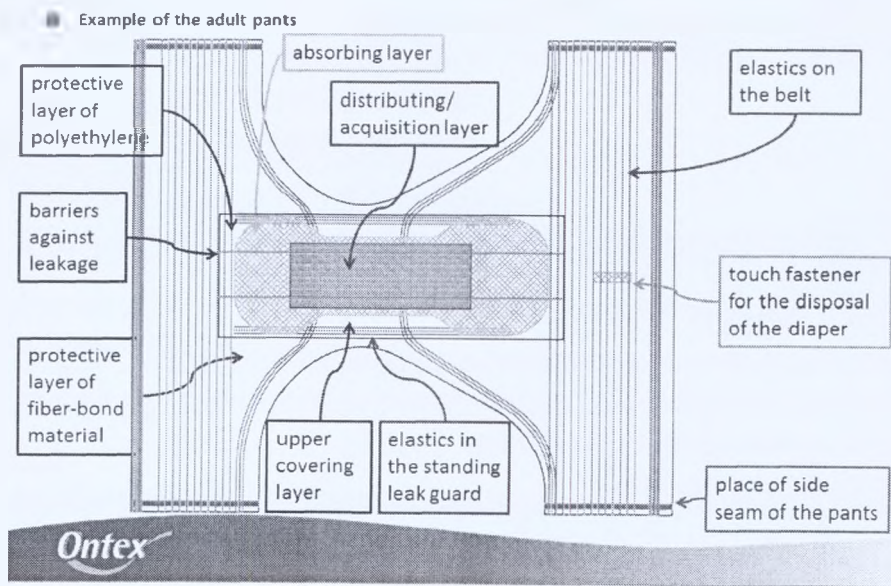Schematic representation of adult diaper

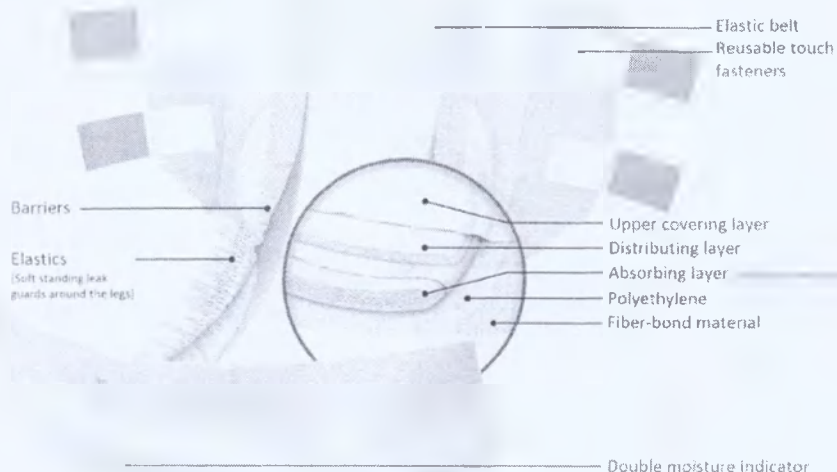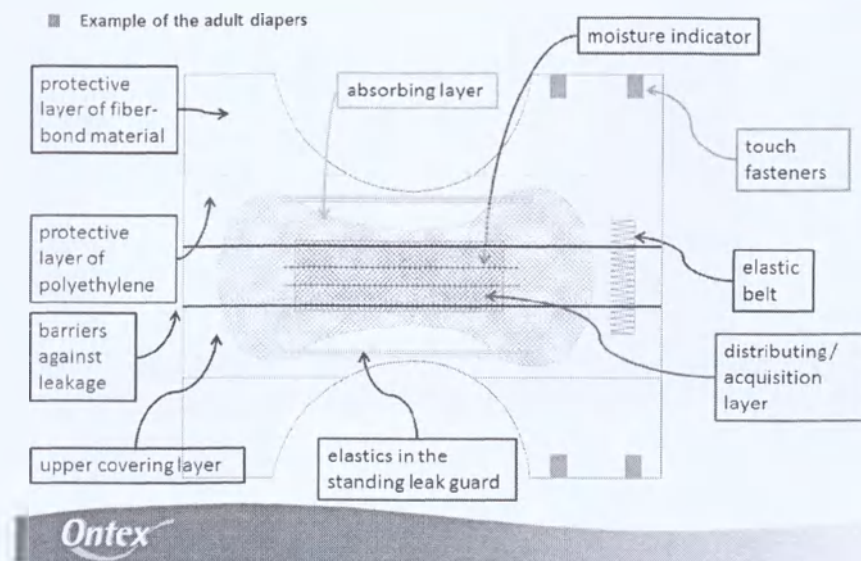3.2 Designation of the product given by the producer

The products are designed for patients suffering from incontinence moderate or severe. Products protect the clothing and pants from soaking and contamination by bio effluents. The products are designed for use in conditions of therapeutic and preventive medical institutions and at home.

3.3 Indication for use of the products

Products are used in cases of urinary incontinence at medium or severe incontinence.

3.4 Contra indications for use of the products:

- dermatopathy of externalia;
- idiosyncrasy for the materials of the products, expressed hypersensitivity reactions.

3.5 Side effects of the products

Side effects of the product were not identified by the manufacturer.

3.6 Scope of application of the product

Products are used at general healthcare institutions as well as domiciliary (in everyday life).

3.7 Use instructions of the product

Before application of the product one should:

- wash hands before and after the change of the product;
- change the product every 3-4 hours or more often, if the product soaks quicker;
- touch fasteners should not disturb the belly and legs, that is normal blood circulation should not be disturbed;
- do not tear the used product, don't wash it, don't throw to the sewage line;
- do not cut or change the form of the product because it may influence its effectiveness;
- wash genitals or clean the intimate zone with the help of special napkins for intimate hygiene at each change of the product.

3.8 Methods of product application:

- Unwrap the product;
- Turn the lying patient on the side, put the narrower part of the diaper between the patient's legs. Then lay the patient on the back in such way that the rare wider part of the diaper was under the clunus.
- Fix the product with the help of touch fasteners from the both sides of the diaper or wear like the underwear (as pants);
- Only intended for single use;
- Wrap the used product into the paper napkin and throw to the dustbin.

3.9 Completeness

Products are supplied completed in accordance with the engineering and technical documents and order requirements. Completeness of the product delivery should comply with the information specified in the Table No 1.

Table No 1. Completeness of the delivery

No	Name	Number (pcs.)
1.	Medical absorbing products iD Slip Expert, iD Slip Basic, iD Slip, iD Innofit, iD Pants, iD Pants For her (see Paragraph 1 on page 2)	from 1 to 30
2.	Instruction for use	

4. Technical characteristics of the medical product

4.1 Size, weight and functional performance of the products

Size, weight and functional performance (diapers and pants) of the products must match the data specified in the table Table No 2 - 7.

Table No 2. Sizes of diapers for adults iD Slip Expert

Наименование размера	Sizes in accordance with the identifying signs			
	Small (S)	Medium (M)	Large (L)	Extra Large (XL)
Waist line*, cm	50-90	70-120	100-150	120-170
Length/Width of the diaper*, mm	685x490	795x660	970x790	960x820
Length of the absorbing layer*, mm	550	610	720	720

Width of the absorbing layer in the front part*, mm	300	280	280	305
Width of the absorbing layer in the rare part*, mm	315	300	300	325
Width of the absorbing layer in the middle part*, mm	165	160	160	190
Total width of the touch fasteners*, mm	60	60	50	60
Distance between the upper and lower touch fastener*, mm	110	75	95	95
Width of the inner frills*, mm	35	35	35	40
Length*2/width of the elastic belt*, cm	19x4	19x4	19x4	19x4
Total weight, not less than, g	79,5	86,0	100	132
Full absorption, g, not less than	1000	1800	2000	2800
Weight of the absorbent, g, not less than	5,0	15,5	16,0	16,0
Rewet, no more, g	4,4	4,4	4,4	4,4
Absorptive capacity after centrifuge, not less than, g	120	440	500	560
Absorption time, not less than, cm3 /s	2,3	2,3	2,3	2,3
Double moisture indicator	yes	yes	yes	yes

*acceptable variations on target indicators specified in the table are of +/-10%

Table No 3. Sizes of diapers for adults iD Slip Basic

Наименование размера	Sizes in accordance with the identifying signs			
	Small (S)	Medium (M)	Large (L)	Extra Large (XL)
Waist line*, cm	50-90	70-130	100-160	120-170
Length/Width of the diaper*, mm	685x490	800x660	970x790	950x805
Length of the absorbing layer*, mm	550	615	720	725
Width of the absorbing layer in the front part*, mm	300	290	280	305
Width of the absorbing layer in the rare part*, mm	315	300	280	325
Width of the absorbing layer in the middle part*, mm	165	160	170	190
Total width of the touch fasteners*, mm	60	60	56	65
Distance between the upper and lower touch fastener*, mm	70	78	97	100
Width of the inner frills*, mm	40	35	40	45
Length*2/width of the elastic belt*, cm	19x4	19x4	19x4	19x4
Total weight, not less than, g	80,0	86,5	90,5	130,0
Full absorption, g, not less than	1000	1300	1450	1450
Weight of the absorbent, g, not less than	5	9	10	12
Rewet, no more, g	4,4	4,4	4,4	4,4
Absorptive capacity after centrifuge, not less than, g	120	220	330	330
Absorption time, not less than, cm3 /s	2,3	2,3	2,3	2,3
Double moisture indicator	yes	yes	yes	yes

*acceptable variations on target indicators specified in the table are of +/-10%

Table No 4. Sizes of diapers for adults iD Slip

Наименование размера	Sizes in accordance with the identifying signs			
	Small (S)	Medium (M)	Large (L)	Extra Large (XL)
Waist line*, cm	50-90	70-130	100-160	120-170
Length/Width of the diaper*, mm	685x490	805x660	970x790	950x805
Length of the absorbing layer*, mm	550	605	720	720
Width of the absorbing layer in the front part*, mm	300	290	280	305
Width of the absorbing layer in the rare part*, mm	315	290	280	325
Width of the absorbing layer in the middle part*, mm	165	160	170	190
Total width of the touch fasteners*, mm	60	60	60	70
Distance between the upper and lower touch fastener*, mm	130	118	130	148
Width of the inner frills*, mm	40	35	40	45
Length*2/width of the elastic belt*, cm	19x4	19x4	19x4	19x4
Total weight, not less than, g	79,5	86,0	96,0	134,0
Full absorption, g, not less than	1000	1300	2000	2800
Weight of the absorbent, g, not less than	5	10	8	6
Rewet, no more, g	4,4	4,4	4,4	4,4
Absorptive capacity after centrifuge, not less than, g	120	220	500	560
Absorption time, not less than, cm ³ /s	2,3	2,3	2,3	2,3
Double moisture indicator	yes	yes	yes	yes

*acceptable variations on target indicators specified in the table are of +-10%

Table No 5. Sizes of diapers for adults iD Innofit

Наименование размера	Sizes in accordance with the identifying signs	
	Medium (M)	Large (L)
Waist line*, cm	75-110	100-135
Length/Width of the diaper*, mm	800x710	870x830
Length of the absorbing layer*, mm	600	620
Width of the absorbing layer in the front part*, mm	210	230
Width of the absorbing layer in the rare part*, mm	220	260
Width of the absorbing layer in the middle part*, mm	215	245
Total width of the touch fasteners*, mm	56	56
Distance between the upper and lower touch fastener*, mm	40	45
Width of the inner frills*, mm	40	40
Length*2/width of the elastic belt*, cm	223x142	253x145
Total weight, not less than, g	79	90
Full absorption, g, not less than	1300	1450
Weight of the absorbent, g, not less than	10	11
Rewet, no more, g	4,4	4,4
Absorptive capacity after centrifuge, not less than, g	220	330
Absorption time, not less than, cm ³ /s	2,3	2,3

*acceptable variations on target indicators specified in the table are of +-10%

Table No 6. Sizes of diapers for adults iD Pants

Наименование размера	Sizes in accordance with the identifying signs			
	Small (S)	Medium (M)	Large (L)	Extra Large (XL)
Waist line*, cm	60-90	75-110	100-135	130-170
Length *, mm	800	830	900	900
Width *, mm	582	728	811	925
Length of the absorbing layer*, mm	480	480	480	400
Width of the absorbing layer in the front part*, mm	190	190	190	190
Width of the absorbing layer in the rare part*, mm	165	160	170	190
Length of the barriers *, mm	40	35	40	45
Width of the belt *, mm	180	190	190	195
Total weight, not less than, g	66	70,5	75	80,5
Full absorption, g, not less than	1000	1300	1450	1450
Weight of the absorbent, g, not less than	5	9	8	10
Rewet, no more, g	4,4	4,4	4,4	4,4
Absorptive capacity after centrifuge, not less than, g	120	220	330	330
Absorption time, not less than, cm ³ /s	2,3	2,3	2,3	2,3
Double moisture indicator	yes	yes	yes	yes

*acceptable variations on target indicators specified in the table are of +/-10%

Table No 7. Sizes of diapers for adults iD Pants For her

Наименование размера	Sizes in accordance with the identifying signs	
	Medium (M)	Large (L)
Waist line*, cm	75-110	100-135
Length *, mm	680	740
Width *, mm	670	770
Length of the absorbing layer*, mm	430	430
Width of the absorbing layer in the front part*, mm	110	110
Width of the absorbing layer in the rare part*, mm	85	85
Length of the barriers *, mm	490	490
Width of the waistband (unstretched) *, mm	230	260
Total weight, not less than, g	37	40
Full absorption, g, not less than	1300	1450
Weight of the absorbent, g, not less than	9	9
Rewet, no more, g	4,4	4,4
Absorptive capacity after centrifuge, not less than, g	220	330
Absorption time, not less than, cm ³ /s	2,3	2,3

*acceptable variations on target indicators specified in the table are of +/-10%

4.2 Materials of the products

Materials specified in the Table No 8 are used for manufacturing of the products.

Table No 8. Materials of the product

Layer of the products	Materials of the products
Upper covering layer	Hydrophilic fiber-bonded material of polypropylene under trademark «Regent», LLC «First Chemical Corporation», Russia
Distributing layer	Fiber-bonded material of polypropylene under trademarks «DRY WEB TDL2», «DRY WEB T28», «DRY WEB TDL3 G», «DRY

	WEB T19», «DRY WEB T12G» produced by «TWE Meulebeke BVBA», Belgium
Absorbing layer	Cellulose bleached without chlorine under trademark «GP Cellulose» produced by «GP Cellulose LLP», USA Polymer superabsorbent under trademark «LG Chem GS-3070HD» produced «LG Chem, Ltd», the Republic of Korea
Protective layer	Polyethylene film under trademark «Hyfol PP Plus», produced by «RKW SE», Germany. Hydrophobic fiber-bonded material of polypropylene under trademark «Avgol» produced by LLC «Avgol Ros», Russia.
Touch fastener	Self-adhesive plastic band (polypropylene) under trademark «AVERY DENNISON F5 2400 Wave C» produced by «AVERY DENNISON Belgie b.v.b.a», Belgium
Standing leak guard	Hydrophobic fiber-bonded material of polypropylene under trademark «Avgol» produced by LLC «Avgol Ros», Russia.
Belt	Plastic band (polystirol) under trademark «3M EBL Soft» produced by «3M Deutschland GmbH», Germany.
Elastics	Elastomer polyurethane thread under trademark "Spandex" produced by «HYOSUNG SPANDEX (JIAXING) CO. LTD», the Republic of Korea.
Double moisture indicator	Hot melt glue (moisture indicator) under trademark «Swift Melt 1525» produced by «H.B. Fuller Deutschland GmbH», Germany
Glue	Hot-melt adhesive: Hot melt glue «Bostik» on the base of styrole copolymer under trademarks «H2465-04», «H4256» produced by «Bostik Nederland B.V.», Netherlands.

Layers (seams) of the products should be fastened tightly with the help of thermal processing or hot-melt glue, or other method providing the tightness of the glued layers and seams of the diapers. Seams should be smooth.

5. Classification of the medical product

In accordance with the degree of potential risk of the application in medical purposes medical product "Medical absorbing products iD Slip Expert, iD Slip Basic, iD Slip, iD Innofit, iD Pants, iD Pants For her in the following dimensions (see Paragraph 1 on page 2)" belong to the 1 class according to Annex 2 of the Order No 4n "On approval of nomenclature classification of the medical products" as of June 06, 2012 of the Ministry of Health of the Russian Federation with subsequent amendments.

Type of contact with human body: brief contact with intact skin and mucous membranes of the human body.

6. Marking of the products

Ready-made products are marked on the package or on the labels; place of marking application is specified in the design documents.

Marking of individual packing contains:

- The name of the manufacturing company and (or) its trademark;
- The address of the manufacturing company, including the country of origin;
- The name of the products;
- The size of the product;
- The number of products in the package;

- The materials used in the manufacture of products;
- The batch number on the side of the packaging in the format: LOT Line: hour/day/year/minute/month;
- The date of manufacture (refer to batch number);
- The usage instruction;
- The date of the expiration date (see batch number: EXP month/year);
- Method of application (drawings and/or words);
- The purpose of the product;
- The contraindications to the use of the product;
- The storage conditions;
- The information on recycling products;
- The bar code of product (if available);
- The information about the registration of the product;
- The information about what the product is intended for single use;
- The name of the importing company on the territory of the Russian Federation;
- The address of the importing company on the territory of the Russian Federation;

The of signs and pictograms on the label:

- "Do not reuse"

- "Protect from moisture"

- Protect from sunlight

- Disposal: do not dispose into the sewers

- The Loop

- the level of absorbance

It is allowed to mark other data including information about consumer properties and the use of the products.

Data are printed or embossed that ensure their readability and preservation (stability) during the storage, transportation, handling and intended usage of the products.

Marking of the transportation tare with the following information:

- the name of the products;

- the article of the products;
- the number of the products in the transportation tare;
- the name and the address of the manufacturing company;
- the name and the address of the company authorized by the manufacturer, of the importer;
- the bar code of the transportation tare (if available);
- the date of manufacturing of the products (month, year);
- the expiry date;
- conditions of storage, transportation, recycling possibility;
- icons and symbols according to the legislation requirements of the consignee country.

7. Packaging

Several pieces of products are packed into the bags made of polyethylene which protects the products in transportation and storage. Seams in the bags of polyethylene should be welded.

Products of one group, structure, linear sizes, technical and decorative version are packed in one bag.

Package should provide the protection of the products from the contamination, mechanical damage, weather impact in transportation and storage as well as the most complete usage of vehicle capacity (tonnage) and easy performance of loading-unloading operations.

Boxes of corrugated cardboard and other packing materials ensuring the protection of the products may be used as transportation tare.

It is allowed to use the other packing materials including those that are manufactured by the manufacturing company which provide the necessary durability and protection of the products in transportation and storage.

8. Transportation and storage

Products are transported by all kinds of vehicles in accordance with the cargo transportation rules operating on the transport of given type at a temperature of from -50 °C to +50 °C, a relative humidity of 80%. Material is not hazardous at any kind of transportation.

Ready-made products should be stored in sheltered storage facilities. Storage conditions should exclude the impact of moisture and aggressive environment on the products.

To preserve the quality product should be preserved in original package at the room temperature from +5 °C to +25 °C and relative humidity no more than 70 % in childproof and moisture protected sites. Don't expose the products to direct sunlight.

The temperature of using from +10 °C up to +45 °C.

9. Safety requirements

Work with products does not require special safety measures.

Structure of the products does not contain materials which are hazardous for the health of the person during the operation.

Products are explosive safe, inflammable when exposed to the open fire without explosions.

In case of fire materials of the products and package should be extinguished with foam, extinguishing powder, carbon dioxide, water, sand.

10. Requirements for environment protection

Requirements for environment protection during the application of the product

Products do not release toxic substances to the environment and do not impact (at direct contact) the health of the person and the environment during the transportation, storage and operation. Products are explosive safe, inflammable when exposed to the open fire without explosions. Special environment protective measures from harmful influence are not required during the operation.

Requirements for environment protection and disposal of the products

The main possible harmful impact on the environment is the contamination of the atmosphere, soils and waters of the inhabited areas as a result of:

- unorganized waste landfilling on the territory of manufacturing company or outside its territory;
- random landfilling in unspecified places.

Products and materials used for their manufacture do not impose risk for the health of people and for the environment both during the operation and after its completion.

11. Disposal

It is allowed to dispose waste and materials during the manufacture by contract with company which has the required licence. (To dispose with the help of the method approved by the local legislation). Products are classified under the waste class A – epidemiological waste safe according SanPiN 2.1.7.2790-10. Collection of category a waste is carried out in reusable or disposable bags. The color scheme of containers is not limited, except for yellow or red. Disposable bags should be placed inside reusable tanks, which in turn are attached to the trolley. The reusable tank must have a lid tightly adjacent to the surface to ensure sealing (full or partial) of the container and must be marked. For a hazard class is set to white.

- ✓ Products can be disposed of by the following methods:
- ✓ In combined devices. I.e. the clearance of balances under the simultaneous action of radiation and water vapor.
- ✓ Thermochemical destruction: grinding, heating and further disinfection.
- ✓ Thermo - combustion.

The product can be disposed as household waste after its usage. It is not allowed to throw the used products into the toilet.

12. Maintenance and repair

Not applicable. Products are not objects of maintenance and repair.

13. Warranty liabilities

Manufacturer assures the compliance of the products with the requirements of the product specifications and guarantees the replacement of the products in case of nonconformity throughout the shelf life.

Shelf life and storage.

Shelf life and storage for the product is 5 years from the moment of manufacture/packing. Date of manufacture is specified with news ticker on the side of the package.

After the expiration date this medical product should be disposed.

14. Information about reclamations

If defects of the products appear during warranty period as well as at primary acceptance of the products, the owner of the product should send a reclamation to the address of the selling organization – request for the replacement according to the form No 1.

The address where the representative of the company carrying out warranty liabilities should arrive, phone number;		
Date	Short description of the defect	Date of the sending of the reclamation

Send the reclamations to the address:
LLC «Ontex RU», 105064, Moscow, Zemlyanoy val St, 9, Russia.
Postal address: LLC «Ontex RU», 105064, Moscow, Zemlyanoy val St, 9, floor 8, pom.II, room 79, Russia.
Tel.: 8 495 646 92 21; website: www.ontex.ru

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