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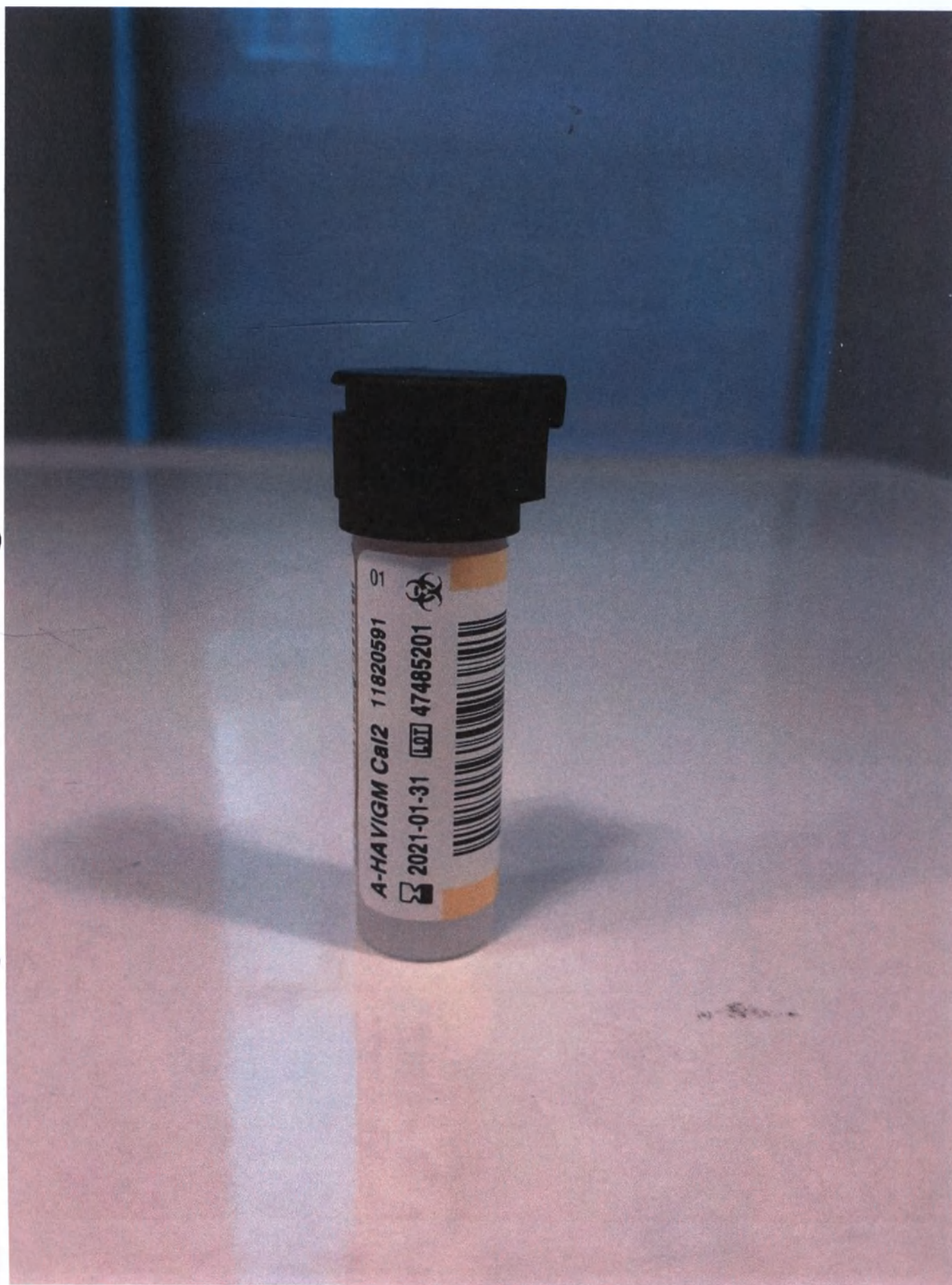
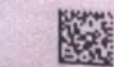
English

Precautions and warnings

For in vitro diagnostic use.
Exercise the normal precautions required for handling all laboratory reagents.
Disposal of all waste material should be in accordance with local guidelines.
Safety data sheet available for professional user on request.
All human material should be considered potentially infectious.

The calibrators (A-HAVIGM Cal1 and A-HAVIGM Cal2) have been prepared exclusively from the blood of donors tested individually and shown to be free from HBsAg and antibodies to HCV and HIV. The testing methods used assays approved by the FDA or cleared in compliance with the European Directive 98/79/EC, Annex II, List A.

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Method Sheet download

The box at the top of this document shows the product code together with the respective document version.

Please go to <http://e-labdoc. Roche.com> to download this document.

The website displays either the version listed here or, if available, any later version that is also valid for this product code. If you have no internet access, please contact your local Roche affiliate to obtain the document free of charge.

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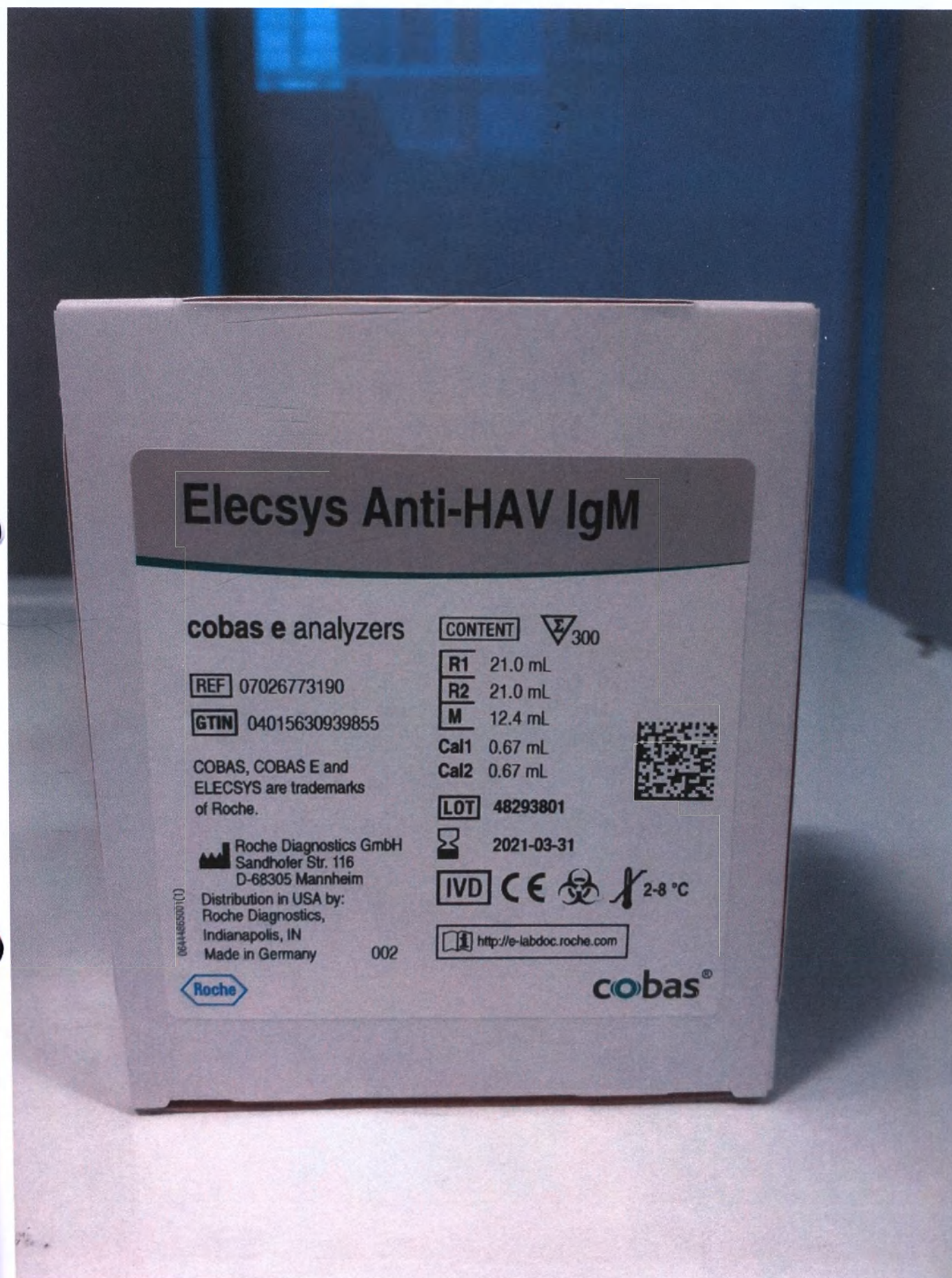
Cal1 0.67 mL

Cal2 0.67 mL

COBAS, COBAS E and
ELECSYS are trademarks
of Roche.

LOT 48293801

 2021-03-31

 Roche Diagnostics GmbH
Sandhofer Str. 116
D-68305 Mannheim

IVD    2-8 °C

Distribution in USA by:
Roche Diagnostics,
Indianapolis, IN
Made in Germany

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REF 07026773190

For USA: **CONTENT**

- R1** Monoclonal anti-HAV antibody (mouse)
labeled with ruthenium complex 0.15 µg/mL;
anti-human-Fdy antibody (sheep) 0.04 mg/mL
- R2** Biotinylated monoclonal anti-h-IgM antibody
(mouse) 0.4 µg/mL; HAV antigen (cell culture)
25 U/mL (Roche units)
- M** Streptavidin-coated microparticles 0.72 mg/mL
- Cal1** Human serum, negative for anti-HAV IgM
- Cal2** Anti-HAV IgM (human) approximately 5 U/mL
(Roche units) in human serum

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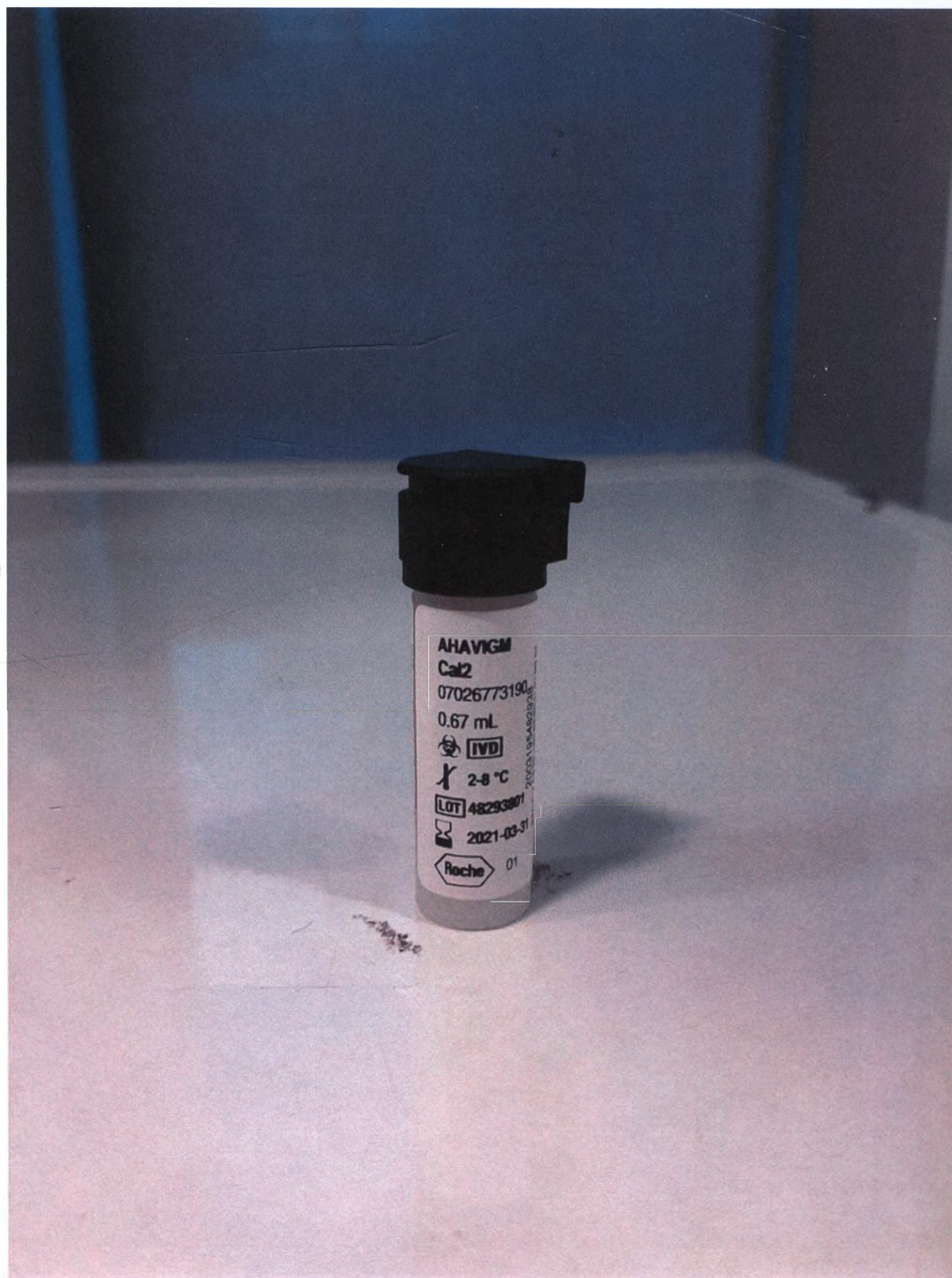
<https://usdiagnostics.roche.com>**English****Precautions and warnings**

For in vitro diagnostic use.

Exercise the normal precautions required for handling all laboratory reagents. Disposal of all waste material should be in accordance with local guidelines. Safety data sheet available for professional user on request.

For USA: Caution: Federal law restricts this device to sale by or on the order of a physician.

All human material should be considered potentially infectious.

The calibrators (AHAVIGM Cal1 and AHAVIGM Cal2) have been prepared exclusively from the blood of donors tested individually and shown to be free from HBsAg and antibodies to HCV and HIV. The testing methods used assays approved by the FDA or cleared in compliance with the European Directive 98/79/EC, Annex II, List A.

The serum containing anti-HAV IgM and the HAV antigen (cell culture) were inactivated using β -propiolactone and UV-radiation.

However, as no inactivation or testing method can rule out the potential risk of infection with absolute certainty, the material should be handled with the same level of care as a patient specimen. In the event of exposure, the directives of the responsible health authorities should be followed.

Avoid foam formation in all reagents and sample types (specimens, calibrators and controls).

Method Sheet download

The product code together with the respective document version (if applicable) is stated in the download information boxes listed above. Please go to the website listed in the relevant box to download this document. The website displays either the version listed here or, if available, any later version that is also valid for this product code. If you have no internet access, please contact your local Roche affiliate to obtain the document free of charge.

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trademarks of Roche.