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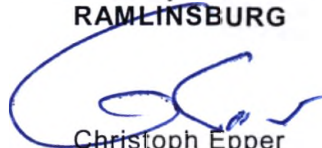
Legalization of signature

The municipal administration office of Ramlinsburg, Switzerland, certifies the authenticity of the signature of Mr. Michel Schaer, born on April 5. 1957, place of origin Huttwil BE, resident in Brännliweg 16, 4433 Ramlinsburg, Switzerland, having proved his identity by presenting his passport No. X4134502.

Ramlinsburg, June, 3., 2021

Municipal Administration Office
RAMLINSBURG

Christoph Epper
District Secretary

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DIANATAL®

obstetric gel

Facilitates birth
Protects mother & child

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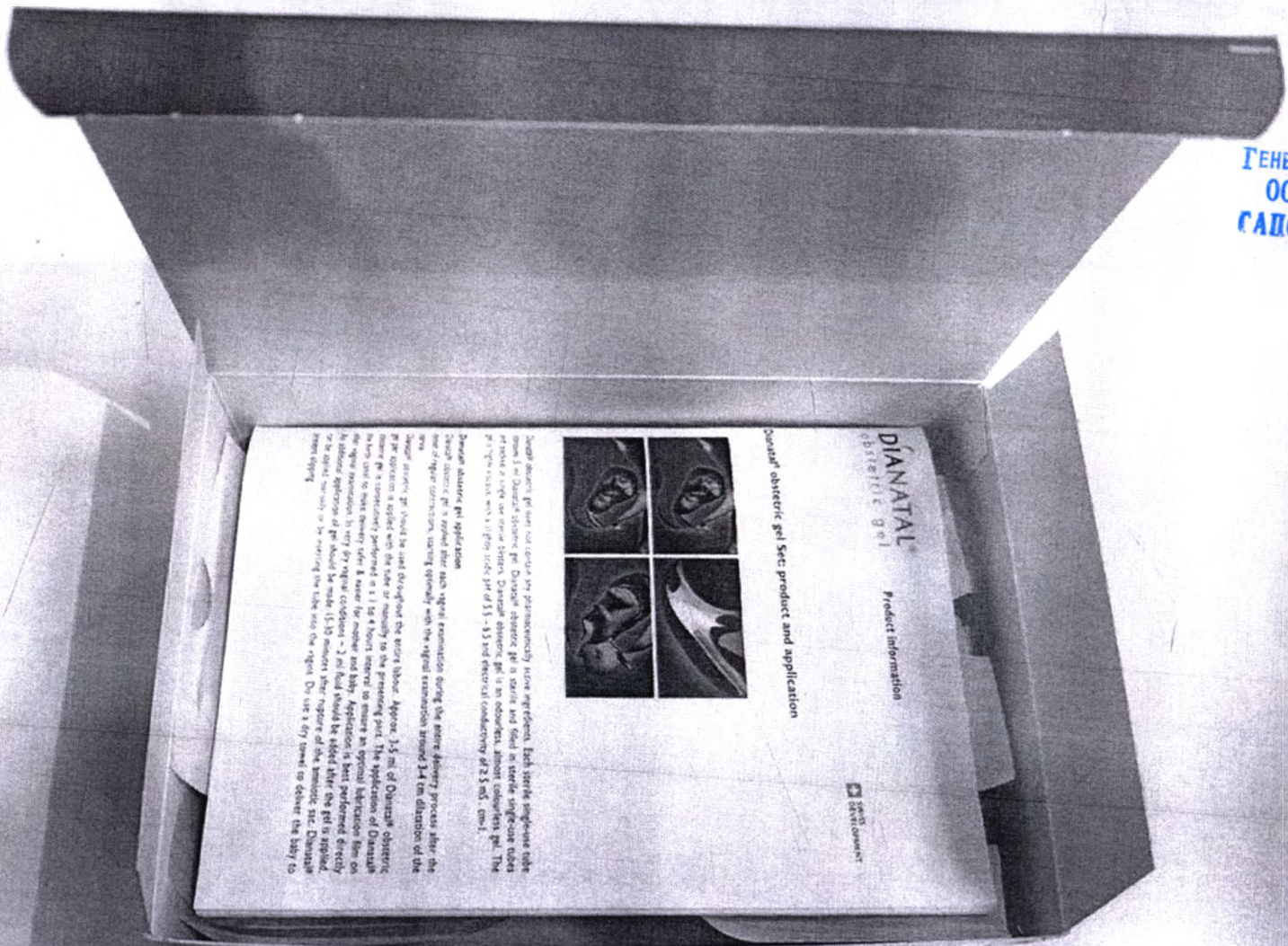
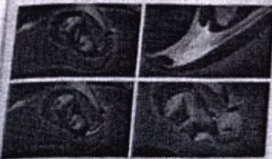
DIANATAL®
obstetric gel

Product information


Dianatal® obstetric gel Set: product and application

Dianatal® obstetric gel does not contain any pharmacologically active ingredients, each sterile ampoule tube contains 5 ml Dianatal® obstetric gel. Dianatal® obstetric gel is sterile and filled in sterile single use tubes and packed in single use sterile blisters. Dianatal® obstetric gel is an odorless, smooth colorless gel. The gel is highly viscous with a slightly acidic pH of 5.5-6.5 and electrical conductivity of 2.1 mS/cm.

Dianatal® obstetric gel application
Dianatal® obstetric gel is applied after each vaginal examination during the entire delivery process after the onset of regular contractions, starting initially with the vaginal examination around 3-4 cm dilation of the cervix.

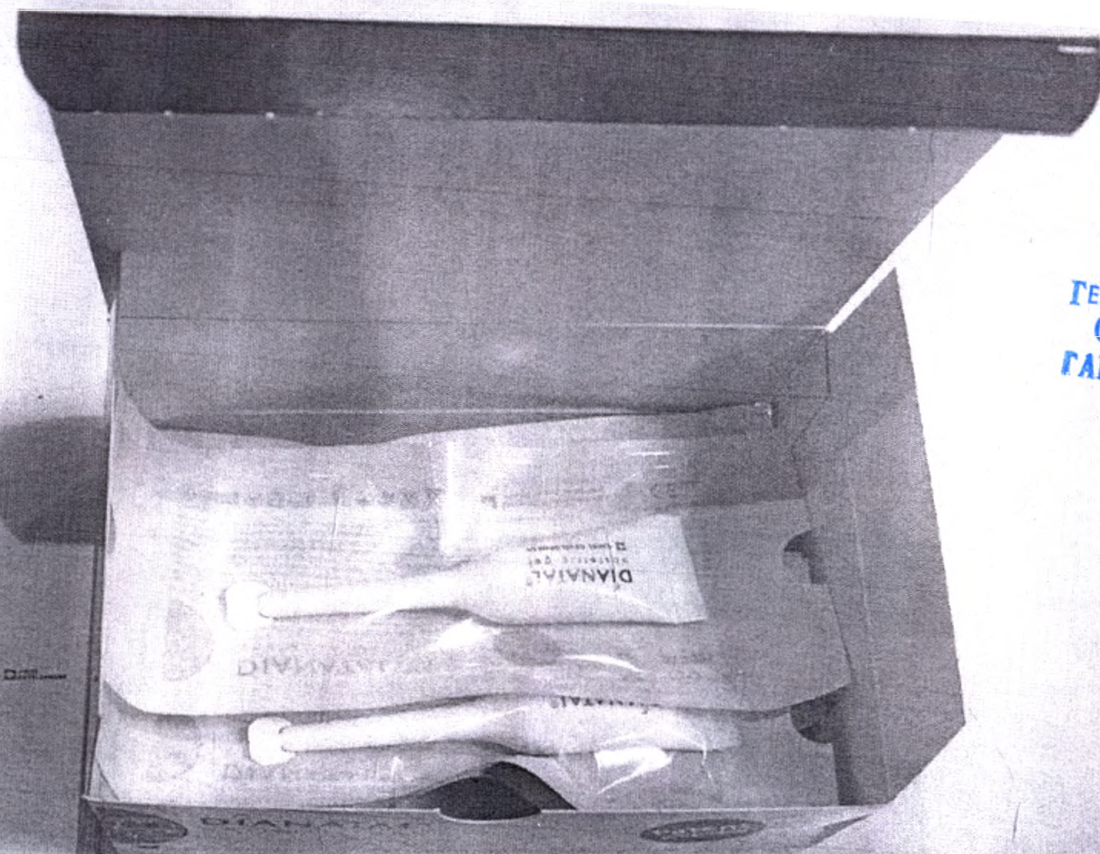Dianatal® obstetric gel should be used throughout the entire labour. Approx. 3-5 ml of Dianatal® obstetric gel per application is applied with the tube or manually to the perineal part. The application of Dianatal® obstetric gel is continuously performed in a 1 to 4 hours interval to ensure an optimal lubrication for the on the birth canal to make delivery safer & easier for mother and baby. Application is best performed directly after vaginal examination. In very dry vaginal conditions ~ 2 ml fluid should be added after the gel is applied. Additional application of gel should be made 15-30 minutes after rupture of the amniotic sac. Dianatal® can be applied manually or by inserting the tube into the vagina. Do use a dry towel to deliver the baby to prevent slipping.

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02.2021

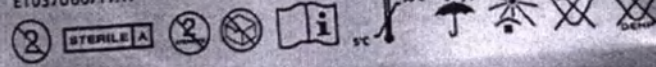
SWISS
DEVELOPMENT

Tube with 5 ml sterile Dianatal® obstetric gel


Ingredients: Aqua purificata, Propylenglycolum, Hydroxyethylcellulosum, Carbomerum

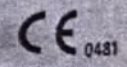For labor facilitation and perineum protection • Zur Geburtserleichterung und Dammschutz • Para la facilitación del parto y la protección del perineo • Pour faciliter la naissance et pour protéger le périnée • Para facilitação do parto e protecção do perineo • Για διευκόλυνση του τοκετού και προστασία του περινέου • Megkönnyíti a szülést és segíti a gát védelmét • Per la facilitazione del parto e la protezione del perineo • Ułatwienie porodu oraz ochrony kroczu w trakcie porodu • Om de geboorte te vergemakkelijken en het perineum te beschermen • Pro usnadnění porodu a k ochraně perinea • Для облегчения родов и защиты промежности • Na uľahčenie pôrodu a ochranu perinea • Se използва при вагинално раждане за улесняване на раждането и предпазване тазовото дъно и перинеума • Kasvatatakse vaginaalse sünnituse ajal sünnituse kergendamiseks ning vaagnapõhja ja lahkliha kaitsmiseks • Paredzēts lietošanai vaginālo dzemdību laikā, lai atvieglotu un veicinātu dzemdības un aizsargātu iegurņa pamatni un starpeni • Naudojamas gimdymo palengvinimui ir dubens dugno bei tarpvietės apsaugojimui vaginalinio gimdymo metu • Se utilizează în timpul nașterii vaginale, pentru a facilita nașterea și pentru a proteja diafragma pelviană și perineul • Se upotrebljava za vreme prirodnog poroda da bi se isti olakšao i zaštitili dno karlice i medica (perineum) • Uporablja se tijekom prirodnog poroda kako bi se olakšao porod i zaštitili dno karlice i medica (perineum) • Doğumu kolaylaştırmak ve perineyi korumak için

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Tantan AG, Dellenbodenweg 8