All-Russian Classification of  
Products code 93 9890

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# Operating Manual

BLACK EYE ENDOSCOPIC MARKER

Produced by: The Standard Co., Ltd, Korea

## 1. APPLICATION

Black Eye is a specially formulated, biocompatible agent (suspension) designed for endoscopic marking or "tagging" of lesions in the GI tract (such as colon cancer and polyps).

### 1.1. Supply package:

The syringe with a Luer lock fitting contains 5cc of the Black Eye sterile suspension for marking (5 ml):

- individual packaging: 1 syringe
- overall packaging: 10 syringes.

## 2. BASIC SPECIFICATIONS

Specifications:

Syringe volume: 5 ml.

Every syringe contains 5cc of sterile non pyrogenic suspension for marking.

Weight of syringe (with sterile suspension) is 20 gm

The syringe is pre-filled and has a Luer lock fitting.

Sterile, single use.

Duration of contact with the internal environment of the organism: permanent mark.

Washout period: the marker is not washed out. The mark is permanent.

Sterile: yes.

The suspension is sterilised by filtration.

PH- 5.5-7

The syringe is sterilised by autoclaving.

Non pyrogenic.

Non toxic.

The Black Eye suspension is sterilised by filtration and aseptically filled in a ready to use, disposable, polypropylene syringe.

## 3. INDICATIONS AND CONTRAINDICATIONS

Black Eye is a suspension containing highly purified, very fine carbon particles. The particles remain in the sub-mucosal injection site creating a permanent mark. Each steam sterilized, non-pyrogenic syringe contains 5cc of the marker. The composition of Black Eye is as follows:

- Water for Injection 78%-82%
- Glycerol 10%-20%
- Polysorbate 80 1%-2%
- Benzyl Alcohol 1%-10%
- Simethicone 0.01% – 0.05%
- High Purity Carbon Black 0,1%-0.5%

### INDICATION FOR USE

Black Eye is indicated for endoscopically marking lesions (such as colon cancer and polyps) in the GI tract when the endoscopist anticipates the lesion will require surgical removal within 30 days.

### CONTRAINDICATIONS

Black Eye is contraindicated in patients with a known hyper-sensitivity to any of the ingredients in the product.

## WARNING AND PRECAUTIONS

Some settling of the carbon pigment is normal. The syringe should be shaken vigorously prior to loading the injection needle.

Black Eye is aseptically processed single-use only product. Leakage of product may be evidence of damage. Do not use. Discard any unused product after the syringe has been opened. Although the material is non-toxic, the marker may stain clothing or porous materials.

When using a sclerotherapy needle in the colon, the needle should be inserted into the colon wall at an angle so that the beveled tip of the needle is entirely beneath the mucosa. If the needle is inserted perpendicular to the mucosa, there is a risk of penetrating the serosa and injecting the marker directly into the peritoneal cavity.

## COMPLICATIONS

- Leakage from the puncture site
- Idiopathic inflammatory bowel
- abdominal abscesses
- Inflammatory pseudotumors
- Focal peritonitis and staining
- Post operation adhesion
- Transmural injection to small bowel
- Pigmentation peritoneal adenocarcinoma(tumor inoculation)
- Infected intramural hematoma and abscess
- Rectus muscle hematoma and abscess
- Spillage of dye into peritoneal cavity

## 4. SETTING-UP PROCEDURES

The syringe is pre-filled and has a Luer lock fitting to ensure a secure connection to the 21–25G sclerotherapy needle with a needle length of 4mm or less.

**IMPORTANT:** The Black Eye syringe should be shaken vigorously for 15-20 seconds prior to loading the injection catheter. Examine the syringe to verify that the pigment is fully suspended. Then attach the syringe to the Luer fitting on the needle. With the needle retracted, insert the catheter through the biopsy channel of the endoscope.

When the needle is properly positioned, insert the tip 30°–45° tangentially into the submucosal space of the colon. **CAUTION:** Do not insert the needle perpendicular to the colon as this may lead to perforation of the serosa and injection of the marker directly into the peritoneal cavity!

## 5. OPERATIONAL PROCEDURE

Inject 0.5-0.75 ml of the marker into each injection site. This volume typically produces a mark on the colon measuring 1-2 cm in diameter. When properly placed there is an immediate black blush of the marker at the injection site. Observe for leakage of the marker into the colon lumen. If this occurs, stop the injection and cautiously insert the needle further into the submucosa.

Black Eye is a suspension containing highly purified, very fine carbon particles. The particles remain in the sub-mucosal injection site creating a permanent mark.

Black Eye is indicated for endoscopically marking lesions (such as colon cancer and polyps) in the GI tract when the endoscopist anticipates the lesion will require surgical removal within 30 days.

A minimum of four marker injections is recommended, one in each of the four quadrants of the colon around the area of the lesion. This technique assures that at least one of the marks can be observed by the operating surgeon from the serosal side of the colon and, if another endoscopy is necessary, the lesion can be observed from inside the lumen as well.

The maximum recommended dose is 8 ml per patient.

## 6. DISPOSAL INFORMATION

According to the Sanitary and Epidemiological Requirements for Disposal of Medical Waste: Sanitary and Epidemiological Regulations and Norms 2.1.7.2790-10, Class B.

## 7. PACKAGING

Packaging: a syringe in an individual plastic packaging (polypropylene). Overall packaging: plastic tube (polypropylene).

## 8. TRANSPORTATION

Black Eye can be shipped by any mode of covered transport in accordance to the requirements of the Russian National Standard 50444 and shipping rules for every means of transport.

Shipping by sea should meet the Rules for the Safe Carriage of General Cargoes by Sea. Dispatch type: containers and small shipment.

Store the product at room temperature. Store the syringes in the canister provided with the syringe tips pointed upward. Do not store with the syringe tips pointing down. Storing the syringes with tips down may allow a portion of the pigment to settle into the narrow syringe outlet and prevent the pigment from being re-suspended.

## 9. WARRANTY

The product warranty period is 2 years from the day of manufacture. It is guaranteed that the product has no defects connected with production and materials quality. The product would be replaced if there was no physical/mechanical intervention to the product, inconsistent with the requirements of the operation manual. Make sure that any malperformance appears not due to neglect or noncompliance with instructions. The storage period is 2 years.

## 10. RULES OF STORAGE AND USAGE

### Storage:

Store the product at room temperature. Store the syringes in the canister provided with the syringe tips pointed upward. Do not store with the syringe tips pointing down. Storing the syringes with tips down may allow a portion of the pigment to settle into the narrow syringe outlet and prevent the pigment from being re-suspended.

Period of validity: 2 years.

In case of temperature storage up to 30°C, the product will be damaged in 6 to 8 weeks.

Do not use if packaging has been damaged.

Do not use after the expiration time.

Single use: yes.

## WARRANTY CERTIFICATE

### Note of Sale

Product category \_\_\_\_\_

Manufacturer \_\_\_\_\_

Model \_\_\_\_\_

Serial/identification number \_\_\_\_\_

Date of sale \_\_\_\_\_

Seller organization name \_\_\_\_\_

Seller's signature \_\_\_\_\_

The product is tested, no defects found.

Operating rules were read and accepted by:

Buyer's signature \_\_\_\_\_

Warranty period – 2 years.

For warranty questions, please contact:

Endo Stars LLC, legal address: Morskoy prospect, 18, lit. A, 11-H, 197110, Saint-Petersburg, Russia; tel./fax: +7-800-555-56-57, e-mail: [endo@endo-stars.com](mailto:endo@endo-stars.com).

ATTENTION! Make sure that the Warranty Certificate is filled in correctly. The serial/identification number must correspond exactly with the one in the Certificate. All fields in the Certificate (except the fields for the warranty center) must be filled in.

In case of inaccurate or incomplete filling of the Certificate, the warranty for the product is regarded as invalid.

The product should not bear the marks of mechanical or thermal action, as well as of various chemical agents. The product use is allowed only in accordance with the Operators Manual.

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