NOTARIAL CERTIFICATE

I, **CHOUNG YOUN SIN**, a Notary Public officiating at Klang, Malaysia hereby certify that the **INSTRUCTION FOR USE FOR THE MEDICAL DEVICE** and other documents annexed hereto were presented to me and are to my personal knowledge which information I verily believe to be true belongs to that of **TOP GLOVE SDN. BHD. (COMPANY NO: 220483-T)** of Lot 4969, Jalan Teratai, Batu 6, Off Jalan Meru, 41050 Klang, Selangor Darul Ehsan, Malaysia.

IN FAITH AND TESTIMONY WHEREOF I, CHOUNG YOUN SIN have hereunto set my hand and affixed my seal of office at Klang aforesaid on 5th May 2021.


.....
CHOUNG YOUN SIN
Notary Public
Klang, Selangor

Attorney General approval reference number PN/100/5/39/97 dated 9th January 2020 and is valid until 13th February 2022 under the Notary Public Act 1959

20th of January, 2021

«Approved by»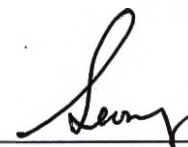

(signature)


Senior General Manager

Leong Chew Mun

Top Glove Sdn. Bhd.

Instruction for use for the medical device

Clean+Safe® Nitrile Examination Gloves

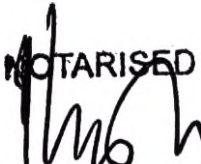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Contents

1. Medical device name and versions	3
2. Information about medical device developer and manufacturer, authorized representative of manufacturer	4
3. Intended use and principles of action	5
4. Indications, contraindications, precautions and side effects associated with intended use of medical device	5
5. Device description and design	6
6. Usage procedure	7
7. Sizes	8
8. Technical and functional characteristics	9
9. Standards used to confirm compliance with the Essential Requirements of MDD 93/42/EEC	9
10. Package	9
11. Description of symbols used on the package labeling	9
12. Requirements for maintenance and repair of a medical device	10
13. Conditions of storage, transportation and operation	10
14. Sterilization and disinfection	10
15. Information for disposal or destruction of a medical device	10
16. Warranty and shelf life	11

1. Medical device name and versions

Clean+Safe® Nitrile Examination Gloves:

Versions:

1. Clean+Safe® Non-Sterile Nitrile Medical Examination Gloves: single chlorinated, powder free, with textured fingertips, with lotion, sizes: XS, S, M, L, XL
2. Clean+Safe® Non-Sterile Nitrile Medical Examination Gloves: single chlorinated, powder free, with textured fingertips, two-colored, sizes: XS, S, M, L, XL
3. Clean+Safe® Non-Sterile Nitrile Medical Examination Gloves: single chlorinated, powder free, with textured fingertips, biodegradable, sizes: XS, S, M, L, XL
4. Clean+Safe® Non-Sterile Nitrile Medical Examination Gloves: single chlorinated, powder free, with textured fingertips, Grip Plus, sizes: XS, S, M, L, XL
5. Clean+Safe® Non-Sterile Nitrile Medical Examination Gloves: single chlorinated, powder free, with textured fingertips, long cuff LC1, sizes: XS, S, M, L, XL
6. Clean+Safe® Non-Sterile Nitrile Medical Examination Gloves: single chlorinated, powder free, with textured fingertips, long cuff LC2, sizes: XS, S, M, L, XL
7. Clean+Safe® Non-Sterile Nitrile Medical Examination Gloves: single chlorinated, powder free, with textured fingertips, high elasticity LM1, sizes: XS, S, M, L, XL
8. Clean+Safe® Non-Sterile Nitrile Medical Examination Gloves: single chlorinated, powder free, with textured fingertips, high elasticity LM2, sizes: XS, S, M, L, XL
9. Clean+Safe® Non-Sterile Nitrile Medical Examination Gloves: single chlorinated, powder free, textured surface PT1, sizes: XS, S, M, L, XL
10. Clean+Safe® Non-Sterile Nitrile Medical Examination Gloves: single chlorinated, powder free, textured surface PT2, sizes: XS, S, M, L, XL
11. Clean+Safe® Non-Sterile Nitrile Medical Examination Gloves: single chlorinated, powder free, textured surface PT3, sizes: XS, S, M, L, XL
12. Clean+Safe® Non-Sterile Nitrile Medical Examination Gloves: single chlorinated, powder free, textured surface PT4, sizes: XS, S, M, L, XL, XXL.
13. Clean+Safe® Non-Sterile Nitrile Medical Examination Gloves: single chlorinated, powder free, with textured fingertips FT1, sizes: XS, S, M, L, XL
14. Clean+Safe® Non-Sterile Nitrile Medical Examination Gloves: single chlorinated, powder free, with textured fingertips FT2, sizes: XS, S, M, L, XL.
15. Clean+Safe® Non-Sterile Nitrile Medical Examination Gloves: single chlorinated, powder free, with textured fingertips FT3, sizes: XS, S, M, L, XL.
16. Clean+Safe® Non-Sterile Nitrile Medical Examination Gloves: single chlorinated, powder free, with textured fingertips FT4, sizes: XS, S, M, L, XL.
17. Clean+Safe® Non-Sterile Nitrile Medical Examination Gloves: single chlorinated, powder free, with textured fingertips FT5, sizes: XS, S, M, L, XL.
18. Clean+Safe® Non-Sterile Nitrile Medical Examination Gloves: single chlorinated, powder free, with textured fingertips FT6, sizes: XS, S, M, L, XL.
19. Clean+Safe® Non-Sterile Nitrile Medical Examination Gloves: single chlorinated, powder free, with textured fingertips FT7, sizes: XS, S, M, L, XL.
20. Clean+Safe® Non-Sterile Nitrile Medical Examination Gloves: single chlorinated, powder free, with textured fingertips FT8, sizes: XS, S, M, L, XL.

21. Clean+Safe® Non-Sterile Nitrile Medical Examination Gloves: polymer coated, powder free, with textured fingertips PC1, sizes: XS, S, M, L, XL.
22. Clean+Safe® Non-Sterile Nitrile Medical Examination Gloves: polymer coated, powder free, with textured fingertips PC2, sizes: XS, S, M, L, XL.
23. Clean+Safe® Non-Sterile Nitrile Medical Examination Gloves: polymer coated, powder free, with textured fingertips PC3, sizes: XS, S, M, L, XL.
24. Clean+Safe® Non-Sterile Nitrile Medical Examination Gloves: polymer coated, powder free, with textured fingertips PC5, sizes: XS, S, M, L, XL.
25. Clean+Safe® Non-Sterile Nitrile Medical Examination Gloves: polymer coated, powder free, with textured fingertips PC6, sizes: XS, S, M, L, XL.

Hereinafter referred to as gloves, devices, Clean+Safe® gloves, Clean+Safe® nitrile gloves.

2. Information about medical device developer and manufacturer, authorized representative of manufacturer

Developer

TOP GLOVE SDN. BHD.

Address: Lot 4969, Jalan Teratai Batu 6, Off Jalan Meru, 41050 Klang, Selangor D. E. Malaysia

Manufacturer

TOP GLOVE SDN. BHD.

Address: Lot 4969, Jalan Teratai Batu 6, Off Jalan Meru, 41050 Klang, Selangor D. E. Malaysia

Manufacturing site of the medical device:

TOP GLOVE SDN. BHD.

Lot 4969, Jalan Teratai Batu 6, Off Jalan Meru, 41050 Klang, Selangor D. E. Malaysia

Authorized representative of the manufacturer (producer)

PROTECO JSC

Legal address: 192102, St. Petersburg, Salova st. 57, build. 1, lit. I.

Operational address: 196128, St. Petersburg, Varshavskaya st., 5, bldg. 2, office 401, Business Center "Varshavsky"

Tel.: +7 (812) 635-88-90; E-mail: info@protecodent.ru

3. Intended use and principles of action

Clean+Safe® gloves are intended to provide the two-sided barrier to protect both a patient and healthcare personnel from different types of contamination during medical research, diagnostic and therapeutic procedures, or may be used for other sanitary purposes, as well as in cosmetology.

Clean+Safe® nitrile gloves are recommended for the following users:

- healthcare personnel in cosmetology clinics and healthcare facilities, such as hospitals, maternity hospitals, clinics and doctors' offices (including private) and other healthcare facilities.
- Individuals using the device for its intended purpose specified by the manufacturer.

4. Indications, contraindications, precautions and side effects associated with intended use of medical device

Indications

Use in medical examinations, diagnostic or therapeutic procedures, or for other sanitary purposes. Use in cosmetology.

Contraindications

Contraindication for use of Clean+Safe® nitrile gloves is individual intolerance to raw materials. If irritation occurs, stop using the device and seek medical attention.

Precautions

Read instructions before use. Do not use after expiration date. Do not use if the glove is damaged!

Do not wear the same pair of single use gloves for care of more than one patient. When wearing gloves, change them during care of the same patient if moving from one body site to another one with different bacterial flora.

Hand hygiene must be performed before putting on and after removing gloves. After handwashing using a cleaning agent, let your hands dry completely before putting on gloves.

If gloves are contaminated with secretions, blood, etc. in order to avoid contamination of hands during their removal, use a swab (tissue) moistened with a solution of a disinfectant (or antiseptic) to remove visible contaminants. Remove gloves; immerse them in a disinfectant solution and dispose of them. Treat hands with an antiseptic.

Gloves should be removed immediately, if they are damaged (or there is a suspicion that their integrity is compromised) or there is indication for hand hygiene.

4.1.Side effects


Side effects that may occur during intended use of a medical device: allergic reactions due to intolerance of the device materials; non-allergic contact dermatitis due to individual intolerance to raw material components of the device including additives used in the manufacturing process.

5. Device description and design

Clean+Safe® non-anatomically shaped (ambidextrous (for use on both hands)) nitrile gloves made of synthetic rubber (nitrile butadiene rubber), have a beaded wrist cuff that snugs the wrist. The bead is rolled


inside the glove and eliminates roll-ups and accumulation of infectious agents between the bead and the external surface. There are various types of surface modifications: powder free surface with textured palms or fingertips only or surface treated by single chlorination or polymer coating (polymer internal coating). Polymer coating and chlorination prevent the sides of gloves from sticking together and make the glove easier to remove.

Figure 1. Clean+Safe® Nitrile Medical Examination Gloves (1 – length, 2 – width)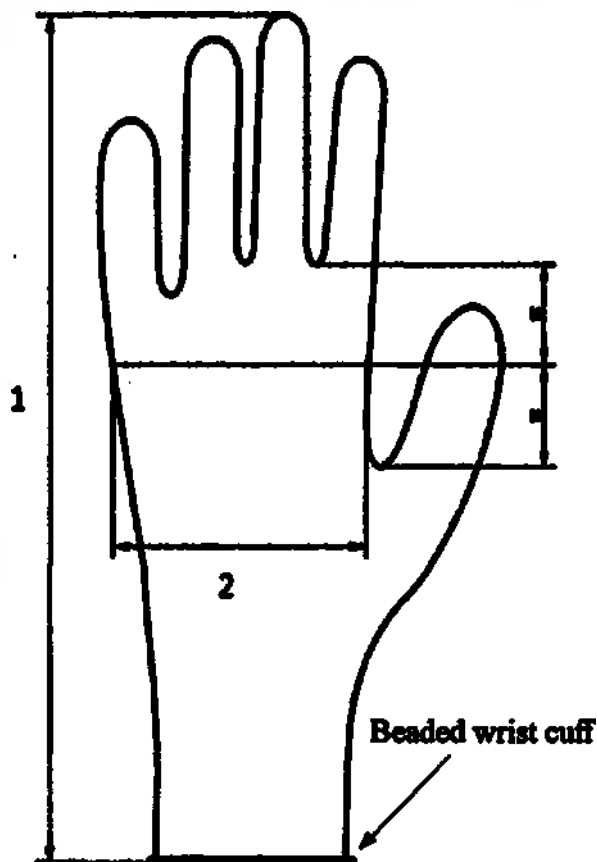


6. Usage procedure

1. Take out a glove from its original box.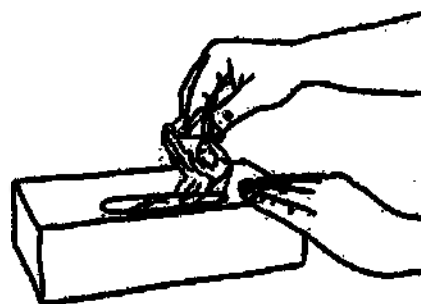


2. Touch only restricted surface of the glove corresponding to wrist (at the top of the cuff).

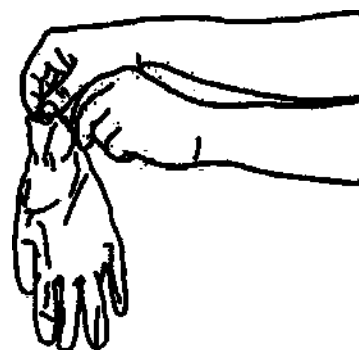
3. Don the first glove.


4. Take the second glove with bare hand and touch only restricted surface of glove corresponding to wrist.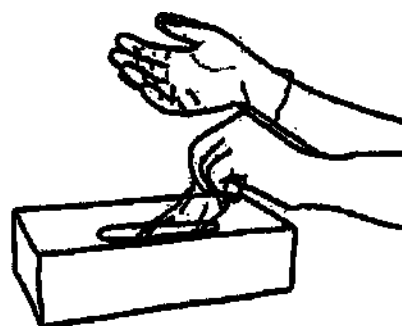


5. To avoid touching the skin of the forearm with gloved hand, turn external surface of the glove to be donned on the folded fingers of the gloved hand, thus permitting to glove the second hand.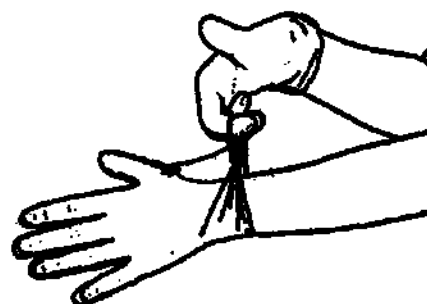


6. Once gloved, hands should not touch surrounding objects the contact with which is an indication for glove change.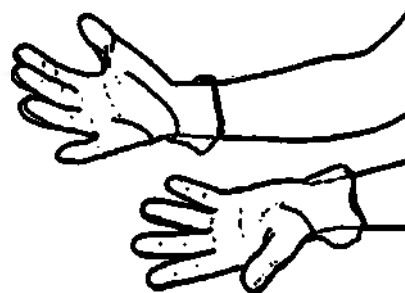

7. After use pinch one glove at the wrist level to remove it, without touching the skin of the forearm, and peel away from the hand, thus allowing the glove to turn inside out.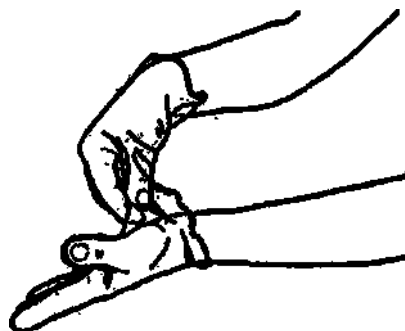

8. Hold the removed glove in the gloved hand and slide the fingers of the other hand between the glove and the wrist. Remove the second glove by rolling it down the hand and fold into the first glove.

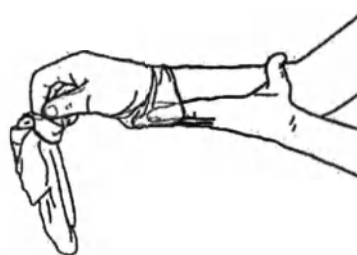9. Discard the removed gloves.

10. Then perform the hand hygiene by rubbing your hands with a hygienic antiseptic agent or by washing them with soap and water.

7. Sizes.

Size	XS	S	M	L	XL	XXL
Width, ± 3	76 mm	84 mm	94 mm	105 mm	113 mm	123 mm
Length (mm),	Not less than 240 mm 300 $\pm$ 10 mm (long cuff)					

8. Technical and functional characteristics.

- Physical properties: Tensile strength, not less than 14 MPa, Elongation at break, not less than 500 %, (before accelerated aging), Tensile strength, not less than 14 MPa, Elongation at break, not less than 400 % (after accelerated aging).

- Physical properties: Average tear resistance, before and after accelerated aging is not less than 6,0 H (except versions, produced in conformity to ASTM D 6319).
- Hermiticity, control level/AQL – G1/1.5.
- Powder residue: 2 mg/glove.

9. Standards used to confirm compliance with the Essential Requirements of MDD 93/42/EEC

ASTM D6319-10, EN 455-1, EN 455-2, EN 455-3, EN 455-4, EN 11193-1, EN ISO 14971, ISO 2859-1, ISO 10993-1, ISO 10993-5, ISO 10993-10, ISO 10993-12, ISO 15223-1, MEDDEV 2.1/1, MEDDEV 2.4/1, MEDDEV 2.5.1/Rec 1, MEDDEV 2.5/5, MEDDEV 2.5/9, MEDDEV 2.5/10, MEDDEV 2.7/1, MEDDEV 2.12/1, MEDDEV 2.12/2, MEDDEV 2.12/Rec 1.

10. Package

Medical devices “Clean+Safe® Nitrile Examination Gloves” are packed in a secondary (consumer) packaging – a cardboard box in the amount: 100 pcs. (50 pairs).

11. Description of symbols used on the package labeling:

Symbol	Description	Symbol	Description
	Manufacturer		Manufacturing date
	Lot number		Keep dry
	Non-sterile		Product must not be disposed as household waste
	Temperature limits		Do not reuse
	Keep away from sunlight		Reference number (article)
	Consult instructions for use		Product does not contain natural rubber latex

12. Requirements for maintenance and repair of a medical device

The devices are disposable and easy to use, do not require special skills and training of a user. Do not need maintenance and repair.

13. Conditions of storage, transportation and operation

Storage. Devices should be stored in well-ventilated areas away from heat, direct sunlight, direct exposure of lighting systems and electrical equipment. Packages must be protected from contamination, dust, moisture and mechanical damage.

Storage conditions:

- temperature: 0 to +37 °C;
- relative humidity: 65-80 % (non-condensing);
- atmospheric pressure: 755±20 mmHg.

Transportation. Gloves packed in shipping box should be transported by all types of covered vehicles according to applicable regulations for the carriage of goods by this type of transport, observing the precautions indicated on the shipping box.

Transportation conditions:

- temperature: 0 to +37 °C;
- relative humidity: 65-80 % (non-condensing);
- atmospheric pressure: 755±20 mmHg.

Operation. Devices are used in healthcare facilities, ambulance vehicles, in field and home conditions, to protect the hands of medical staff when using the product for its intended purpose.

Operation conditions:

- temperature: 0 to +37 °C;
- relative humidity: 65-80 % (non-condensing);
- atmospheric pressure: 755±20 mmHg.

14. Sterilization and disinfection

Clean+Safe® Nitrile Examination Gloves are single use non-sterile products.

15. Information for disposal or destruction of a medical device

The medical devices Clean+Safe® Nitrile Examination Gloves are considered hazardous waste of epidemiologic potential (devices contaminated with blood and/or other biological fluids of patients and devices that come into contact with infected patients) after their use as intended.

In this case, a medical organization carries out only collection, decontamination, and temporary storage of the abovementioned waste. Decontamination should be performed in accordance with the country-specific sanitary guidelines and other normative legal acts for all medical institutions regardless of their specialization. Collection is carried out using bags attached to a trolley or disposable, color-coded containers for hazardous medical waste indoors or in places for temporary storage of medical waste before transporting for destruction or disposal.

Specialized organizations licensed to handle hazardous medical waste of epidemiologic potential transport medical waste (used gloves) from medical institutions and recycle (dispose) and destroy it.

Consumer packaging, defective and unused products with expired shelf life are classified as epidemiologically safe wastes (non-hazardous waste that does not come into contact with biological fluids of patients or infected patients), similar to municipal solid waste (MSW).

Non-hazardous waste is collected using reusable containers or disposable bags (excluding color-coded bags for hazardous waste). Disposable bags are placed inside reusable containers. Waste containers should be marked and placed at a special site (indoors). After emptying, reusable containers are subject to washing and disinfection. The procedure for washing and disinfection of reusable containers is determined in accordance with the organization-specific waste management flowchart.

Transportation and disposal should be performed considering the requirements established for a specific territory in accordance with the country-specific sanitary laws concerning the maintenance of populated areas and handling of industrial and household waste.

16. Warranty and shelf life

Manufacturer guarantees compliance of all parameters and characteristics of gloves for entire shelf life: 5 years (3 years for versions: Clean+Safe® Non-Sterile Nitrile Medical Examination Gloves: single chlorinated, powder free, with textured fingertips, with lotion, sizes: XS, S, M, L, XL; Clean+Safe® Non-Sterile Nitrile Medical Examination Gloves: single chlorinated, powder free, with textured fingertips, two-colored, sizes: XS, S, M, L, XL; Clean+Safe® Non-Sterile Nitrile Medical Examination Gloves: single chlorinated, powder free, with textured fingertips, biodegradable, sizes: XS, S, M, L, XL; Clean+Safe® Non-Sterile Nitrile Medical Examination Gloves: single chlorinated, powder free, with textured fingertips, Grip Plus, sizes: XS, S, M, L, XL) provided that the conditions of package integrity, storage, transportation and operation are observed in accordance with the instructions.

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