State of Minnesota

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SECRETARY OF STATE

APOSTILLE

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1. Country: United States of America

This Public Document: Instructions for Use

2. has been signed by Angela Maxcine Draper

3. acting in the capacity of Notary Public, State of Minnesota

4. bears the seal/stamp of Angela Maxcine Draper, Notary Public, State of Minnesota

CERTIFIED

5. at: St. Paul, Minnesota

6. Date: January 28, 2014

7. by: Secretary of State, State of Minnesota

8. File No: 20140128026JW

9. Seal/Stamp:

10. Signature:

Mark Ritchie
Mark Ritchie
Secretary of State

3M ESPE
Dental Products

3M Center
2510 Conway Avenue
St. Paul, MN 55144-1000

Statement

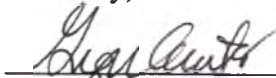
To Whom It May Concern:

**Subject: Instructions for Use for Lava™ Ultimate CAD/CAM Restorative
And Lava™ Ultimate Implant Crown Restorative**

In my capacity as Senior Regulatory Affairs Specialist, 3M ESPE Dental Products, St. Paul, Minnesota, USA, I am providing the attached Instructions for Use for the subject products below:

- **Lava™ Ultimate CAD/CAM Restorative** (this IFU applies to Lava Ultimate CAD/CAM Restorative for CEREC and also to Lava Ultimate CAD/CAM Restorative for E4D).
- **Lava™ Ultimate Implant Crown Restorative.**

Sincerely,


Ginger Cantor, RAC
Senior Regulatory Affairs Specialist
3M ESPE Dental Products

24 January 2014
Date

Attachment

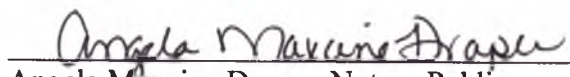Signed and sworn to before me this 24 day of January, 2014.

Angela Maxcine Draper, Notary Public
My Commission Expires: January 31, 2014

ENGLISH Instruction for Use

Lava Ultimate CAD/CAM Restorative

Product Description

The 3M ESPE Lava Ultimate CAD/CAM Restorative is a strong, wear-resistant and highly esthetic mill block that provides a fast and easy-to-use alternative to porcelain blocks for milling CAD/CAM indirect restorations. The material is specially processed to enhance its properties for use in CAD/CAM milling procedures. Lava Ultimate restorative is a Resin Nano Ceramic containing approximately 80 % (by weight) nanoceramic particles bound in the resin matrix. The ceramic particles are made up of three different ceramic fillers that reinforce a highly cross linked polymeric matrix.

The fillers are a combination of non-agglomerated/non-aggregated 20 nanometer (nm) silica filler, non-agglomerated/non-aggregated 4 to 11 nm zirconia filler, and aggregated zirconia/silica cluster filler (comprised of 20 nm silica and 4 to 11 nm zirconia particles). Lava Ultimate restorative is milled into dental restorations using a dental CAD/CAM system; the restorations should be bonded to tooth structure and/or implant abutments with adhesive resin cement.

Indications

Lava Ultimate restorative is indicated for inlays, onlays, veneers, and full crown restorations, including implant supported crowns.

Lava Ultimate restorative is not indicated for uses other than those listed.

Tooth Preparation

Prepare tooth for maximum mechanical retention.

Do

- For crowns, **prep tooth with 5-6 taper**
- All internal edges and line angles should be rounded
- Non-beveled shoulder finish lines are recommended
- **Mechanically clean** any contamination (e.g. debris, powder, temp cement) from the tooth prep and rinse with a strong stream of water
- **Slightly air dry** (surface should appear glossy)

Don't

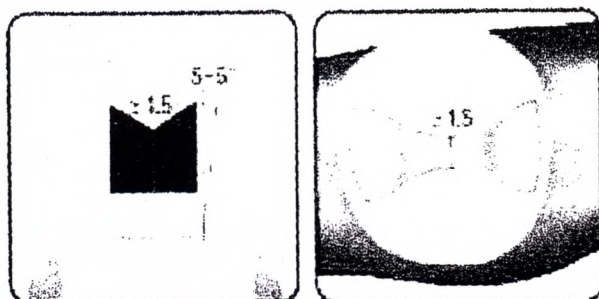
- Do not undercut
- Do not use any chemical agents after preparation, (e.g. H₂O₂, EDTA, bicarbonates, astringents, desensitizers, solvents)
- Do not use laser etching
- Do not use eugenol cement for cementation of temporary restoration

Inlay and Onlay


Traditional inlay /onlay design is recommended. Do not undercut. Taper the cavity walls 5-6 degrees to the long axis of the preparation. All internal edges and angles should be rounded.

Incisal/occlusal reduction is 1.5 to 2 mm clearance in centric occlusion and all excursions.

Inlay


Onlay

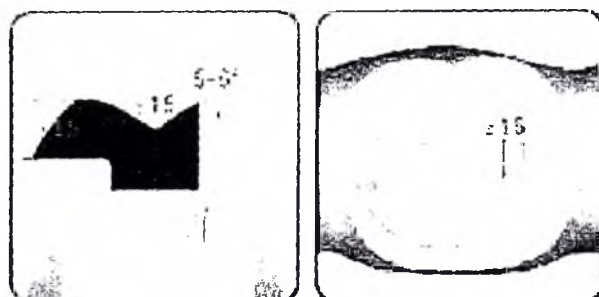Full Crown

Axial reduction is 1.2 to 1.5 mm with a 5-6 degree taper. Incisal/occlusal reduction is 1.5 to 2 mm clearance in centric occlusion and all excursions. Shoulders must extend at least 1.0 mm lingual to proximal contact area. Non-beveled shoulder finish lines are recommended. All line angles should be rounded and preparation surfaces smooth.

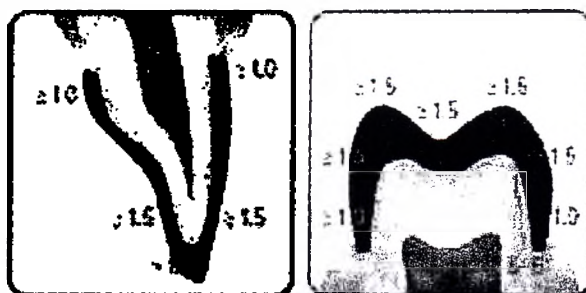Full Crown

Veneer

Standard reduction of the labial surface is 0.6 mm and 0.4 mm at the gingival portion because the enamel is thinner in this area. The reduction of the incisal, labial-lingual angle is 0.5 to 1.5 mm. Keep preparation margins in the enamel. Margins for the veneers should be above the gingival tissues. Chamfer or rounded shoulder should be used for all preparations. Proximal extensions should be far enough into the proximal region so margins do not show and have no proximal-lingual undercuts.

Veneer

Pulp Protection

If a near-pulp or pulp exposure has occurred, use a calcium hydroxide layer first, followed by an application of 3M™ ESPE™ Vitrebond™ Liner/Base or 3M™ ESPE™ Vitrebond™ Plus Liner/Base to minimize sensitivity.

Temporization

Place a provisional restoration on the tooth if protection is needed until the permanent restoration is cemented. (3M™ ESPE™ Protemp™ Temporization Materials or 3M™ ESPE™ Iso-Form™ Crowns are recommended). Non-eugenol cement must be used for cementation of the temporary restoration.

Milling and Block Sizes

Selection of the proper size of mill block for the restoration to be milled is based on completion of all design steps required for the restoration. You will be prompted by the mill to insert a block corresponding to the size of the restoration. The fabrication steps are described in the Instructions for use and user manuals of the different CAD/CAM systems. The manufacturer's instructions must be followed.

Lava Ultimate CAD/CAM block is supplied to the user in a highly cured state.

This material should not be fired under any circumstances during finishing or build up/Add on.

Extraoral Polishing Procedure

1. Clean restoration in an ultrasonic cleaner or steam cleaner; gently blow dry with an air syringe.
2. Trial fit the final restoration to the preparation. If desired, gently adjust contacts, occlusion and accentuate grooves using a fine needle diamond.
3. Grind off sprue with coarse rubber wheel or medium-to-fine bur.
4. Finish sprue area with medium rubber tool.
5. Pre-polish with fine rubber tool.
6. Using a soft bristle brush, spin brush in polishing agent to incorporate paste into brush.
7. Apply with low speed hand piece, working slowing into surface. The luster quickly appears as the restoration is polished.
8. Buff with a muslin rag wheel.

Characterization

Pit and Fissure Staining

To fully maintain the excellent polish retention and stain resistance of Lava Ultimate resin nano ceramic material, only pit and fissure staining is recommended. Applying light-cure glazes or cervical painting is not recommended.

1. Use a bur or sandblast to roughen the restoration surface. If additional grooves are desired, carve with a fine bur. Maintain minimum wall thickness as described in the sections above (e.g., Inlay and Onlay, Full Crown, Veneer).

2. Apply 3M™ ESPE™ Single Bond Universal Adhesive to the surface of the restoration and scrub it in for 20 seconds. Alternatively use a Silane followed by a bonding agent as per manufacturer's Instructions for Use.
3. Air thin with oil-free, moisture-free air until the solvent is evaporated and the adhesive no longer moves over the surface.
4. Light cure. Follow the stain manufacturer's Instructions for Use.
5. Apply Sinfony™ Magic Stains (or other methacrylate-based, light-cured stains).
6. Light cure. Follow the stain manufacturer's Instructions for Use.
7. Finish with rubber instruments and polishing paste as described in the **Extraoral Polishing Section** above.

Bonding

Lava Ultimate Restorations must be bonded with an adhesive resin cement. 3M ESPE recommends 3M™ ESPE™ RelyX™ Ultimate Clicker™ Adhesive Resin Cement with Single Bond Universal Adhesive, a primer and adhesive in one bottle. RelyX Ultimate Clicker cement with Single Bond Universal adhesive form a highly versatile bonding system that works for many materials and indications.

Pre-Treatment of Lava Ultimate Restoration

Note: Complete all try-in, adjustment, finishing and polishing steps before starting the bonding pre-treatment.

1. Clean restoration in an ultrasonic cleaner or steam cleaner. Gently blow dry with air.
2. Sandblast with aluminum oxide grain size $\leq 50 \mu m$ (e.g. Cojet™ Sand or Rocotec™ Soft) at two bars (30 psi) until entire bonding surface appears matte. Use suction to prevent dusting. **Do not etch with HF or phosphoric acid.**
3. Remove sand with alcohol and dry with oil-free, moisture-free air. Do not use other chemical agents.
4. Apply Single Bond Universal Adhesive to the bonding surface of the restoration and **scrub it in for 20 seconds.**
5. Air thin with oil-free, moisture-free air until the solvent is evaporated and the adhesive no longer moves over the surface.

Priming for adhesive resin cements other than RelyX Ultimate Clicker Adhesive Resin Cement:

6. Apply the primer, recommended by your cement. If no primer is recommended, apply Silane (e.g. 3M™ ESPE™ RelyX™ Ceramic Primer) to the bonding surface of the restoration as per silane manufacturer's instructions.

Pre-Treatment of Tooth

Note: Prepare tooth according to the instructions for use for maximum retention.

Ensure complete hemostasis and optimum gingival retraction for a dry bonding environment. Do not use any chemical agents to disinfect, clean or seal the dentin.

1. To remove debris, thoroughly clean all bonding surfaces with a slurry of pumice and water, rinse and dry.
2. Apply rubber dam to isolate the tooth.
3. Selectively etch enamel for 15 seconds or etch enamel and dentin for 15 seconds.
4. Rinse with water.
5. Lightly air dry or blot dry leaving a slightly moist surface. Do not over dry.
6. Apply Scotchbond Universal Adhesive and **scrub it in for 20 seconds.**
7. If you have pooled adhesive, use a dry microbrush to remove it.
8. Blow a gentle stream of air over the liquid for about 5 seconds until it no longer moves and the solvent has evaporated completely.
9. For maximum adhesion light cure the adhesive for 10 seconds.

Seating

Before using RelyX Ultimate Clicker Adhesive Resin Cement, please refer to the Instructions for Use provided with the product package.

Note: Use a high power light. Make sure the light guide is clean and light is delivering full **intensity**. Proper curing technique is crucial for bonding success: for each curing cycle hold light guide steady at minimum distance to the restoration surface.

1. Apply a uniform layer of RelyX Ultimate Clicker Adhesive Resin Cement to the tooth or the restoration to ensure a void-free interface.
2. Completely seat the restoration using a suitable instrument.
3. While holding the restoration firmly in place, wipe off excess using sponge pellets and floss. Cover margins with glycerin gel.

OR

4. Wait 90 seconds or tack light cure for 1 second for cement to reach gel state. While holding restoration firmly in place, remove excess cement with a sharp instrument.
5. Light cure 20 seconds per surface. Total cure time should be at least 60 seconds depending on number of surfaces.
6. For dark shades or thick restorations, allow 6 minutes of total self-cure time from the start of mixing.
7. Finish margins after cure is fully established.
8. If desired polish intraorally as described in Intraoral polishing section.

Intraoral Adjustment and Repolishing

1. Make any necessary adjustments with a diamond bur.
2. Finish with 3M™ ESPE™ Sof-Lex™ Spiral Finishing Wheel.
3. Polish with 3M™ ESPE™ Sof-Lex™ Spiral Polishing Wheel.

OR

4. Finish with diamond impregnated points/cups.
5. Apply intraoral polishing paste. Work paste slowly into surface using soft black, latch angle, intraoral prophyl brush.

Serviceability

It's easy to add on with Light-Cured Restoratives.

Lava Ultimate CAD/CAM block is supplied to the user in a highly cured state. This material should not be fired under any circumstances during finishing or build up/Add-on.

Lava Ultimate Restorative is a Resin Nano Ceramic that enables reliable characterization and re-sealing.

1. Roughen the Lava Ultimate restoration site.
2. Use Scotchbond Universal Adhesive or apply silane and adhesive according to the Instructions for Use.
3. Light cure bonding agent according to the Instructions for Use.
4. Apply 3M™ ESPE™ Filtek™ Ultimate Universal Restorative (or other methacrylatebased, light-cured restorative).
5. Light cure restorative material according to the Instructions for Use.
6. Polish the added restorative with Sof-Lex Spiral Finishing and Polishing Wheels.

Storage and Use

This product is designed to be stored and used at room temperature. Do not store Lava Ultimate Restorative in intense light or sunlight. Product should be used by the date indicated on the outer package.

Disinfection

Disinfect this product using an intermediate level disinfection process (liquid contact) as recommended by the Centers for Disease Control and endorsed by the American Dental Association. **Guidelines for Infection Control in Dental Health-Care Settings – 2003 (Vol. 52; No. RR-17), Centers for Disease Control and Prevention.**

Disposal

See the Material Safety Data Sheet (available at www.3MESPE.com or through your local subsidiary) for disposal information.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

10 Year Limited Warranty

3M ESPE Lava Ultimate CAD/CAM Restorative

Limited Warranty and Limited Remedy:

3M ESPE warrants for 10 years from date of placement that restorations made from Lava Ultimate CAD/CAM restorative will not break if fabricated using a 3M ESPE recommended milling machine in strict compliance with approved indications and instructions for use.

3M ESPE makes no other warranties including, but not limited to, any implied warranty of merchantability or fitness for a particular purpose.

3M ESPE's sole obligation and the customers' sole remedy in the event of any breakage of the restoration shall be limited to the replacement of the Lava Ultimate block used to fabricate the milled restoration, or at 3M ESPE's option, reimbursement of Lava Ultimate block purchase price. This warranty does not cover breakage resulting from accident or misuse.

Limitation of Liability: Except where prohibited by law, 3M ESPE will not be liable for any loss or damages arising from this product, whether direct, indirect, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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Lava Ultimate Implant Crown Restorative

Product Description

The 3M™ ESPE™ Lava™ Ultimate Implant Crown Restorative is a strong, wear-resistant, and highly esthetic mill block that provides a fast and easy-to-use alternative to porcelain blocks for milling Implant Crown indirect restorations. The material is specially processed to enhance its properties for use in Implant Crown milling procedures. Lava Ultimate restorative is a Resin Nano Ceramic containing approximately 80% (by weight) nanoceramic particles bound in the resin matrix. The ceramic particles are made up of three different ceramic fillers that reinforce a highly cross-linked polymeric matrix. The fillers are a combination of non-agglomerated/non-aggregated 20-nanometer (nm) silica filler, non-agglomerated/non-aggregated 4- to 11-nm zirconia filler, and aggregated zirconia/silica cluster filler (comprised of 20-nm silica and 4- to 11-nm zirconia particles).

Lava Ultimate restorative is milled into dental restorations using a Lava™ milling system (e.g., Lava™ Form, Lava™ CNC 500, Lava™ CNC 240); the restorations should be bonded to tooth structure and/or implant abutments with adhesive resin cement.

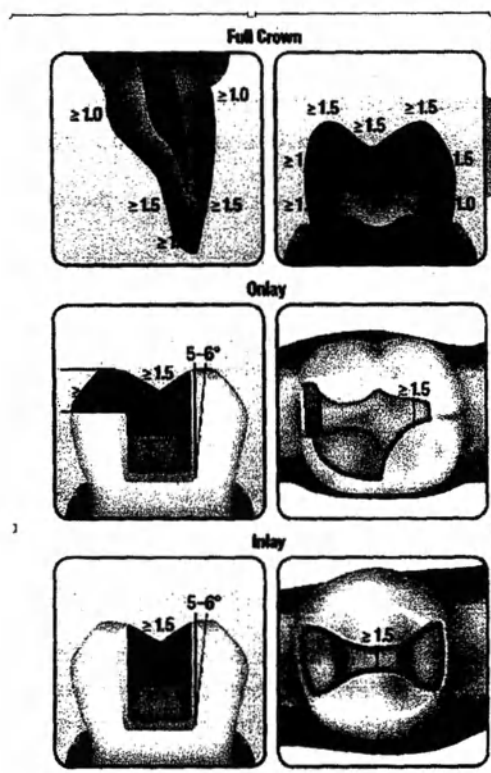
☞ These Instructions for Use should not be discarded for the duration of product use.

Indications

Lava Ultimate restorative is indicated for inlays, onlays and full crown restorations, including implant-supported crowns. Lava Ultimate restorative is not indicated for uses other than those listed.

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Do

- For crowns, prep tooth with 5-6° taper
- All internal edges and line angles should be rounded
- Non-beveled shoulder finish lines are recommended
- Mechanically clean any contamination (e.g. debris, powder, temp cement) from the tooth prep and rinse with a strong stream of water
- Slightly air dry (surface should appear glossy)

Don't

- Do not undercut
- Do not use any chemical agents after preparation, (e.g. H_2O_2 , EDTA, bicarbonates, astringents, desensitizers, solvents)
- Do not use laser etching
- Do not use eugenol cement for cementation of temporary restoration

Inlay and Onlay

We recommend preparing the inlays and onlays without a feather edge as usual for ceramic restorations. Do not undercut. Taper the cavity walls 5-6 degrees to the long axis of the preparation. All internal edges and angles should be rounded. Incisal/occlusal reduction is 1.5 to 2 mm clearance in centric occlusion and all articulations.

Full Crown

Axial reduction is 1.2 to 1.5 mm with a 5-6 degree taper. Incisal/occlusal reduction is 1.5 to 2 mm clearance in centric occlusion and all articulations. Shoulders must extend at least 1.0 mm lingual to proximal contact area. Non-beveled shoulder finish lines are recommended. All line angles should be rounded and preparation surfaces smooth.

Design

There are default settings for the cement gap parameters in the Lava Design software; they can be modified as required.

Milling and Block Sizes

Selection of the proper size of mill block for the restoration to be milled is the base for completion of all design steps required for the restoration.

There are two different sized milling blocks available (M and S).

The fabrication steps are described in the instructions for use and user manuals of the different Lava™ milling systems. The manufacturer's instructions must be followed.

Block Shades and Translucencies

- Bleach
- A1-HT, A1-LT
- A2-HT, A2-LT
- A3-HT, A3-LT
- A3.5-LT
- B1-HT, B1-LT
- C2-LT
- D2-LT

Types of Burrs

- Type 20
- Type 21

- Type 22
- Type 23

The used dental mill must be loaded with each type of burr.

Milling Machines

Lava Ultimate Implant Crown Restorative can be milled in all Lava milling machines.

To avoid contamination of Lava Ultimate Implant Crown Restorative powder on other materials, cleaning of Lava Form and Lava CNC 240 after milling Lava Ultimate restorations is mandatory.

Lava CNC 500 does not need to be cleaned after milling Lava Ultimate Implant Crown Restorative.

Lava Ultimate Implant Crown block is supplied to the user in a highly cured state. This material should not be fired under any circumstances during finishing or build-up/add-on.

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1. Use a bur or sandblast to roughen the restoration surface. If additional grooves are desired, carve with a fine bur. Maintain minimum wall thickness as described in the sections above (e.g., Inlay and Onlay, Full Crown).
2. Apply Scotchbond™ Universal Adhesive to the surface of the restoration and scrub it in for 20 seconds. Alternatively use a Silane followed by a bonding agent as per manufacturer's Instructions for Use.
3. Air thin with oil-free, moisture-free air until the solvent is evaporated and the adhesive no longer moves over the surface.
4. Light cure. Follow the stain manufacturer's Instructions for Use.
5. Apply Sinfony™ Magic Stains (or other methacrylate-based, light-cured stains).
6. Light cure. Follow the stain manufacturer's Instructions for Use.
7. Finish with rubber instruments and polishing paste as described in the Extraoral Polishing Section above.

Bonding

Lava™ Ultimate Restorations must be bonded with an adhesive resin cement. 3M ESPE recommends RelyX™ Ultimate Adhesive Resin Cement with Scotchbond™ Universal Adhesive, a primer and adhesive in one bottle. RelyX Ultimate cement with Scotchbond Universal adhesive form a highly versatile bonding system that works for many materials and indications.

Pre-Treatment of Lava™ Ultimate Restoration

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1. Clean restoration in an ultrasonic cleaner or steam cleaner. Gently blow dry with air.
2. Sandblast with aluminum oxide grain size $\leq 50 \mu\text{m}$ (e.g. Cojet™ Sand or Rocotec™ Soft) at two bars (30 psi) until entire bonding surface appears matte. Use suction to prevent dusting. Do not etch with HF or phosphoric acid.
3. Remove sand with alcohol and dry with oil-free, moisture-free air. Do not use other chemical agents.
4. Apply Scotchbond™ Universal Adhesive to the bonding surface of the restoration and scrub it in for 20 seconds.
5. Air thin with oil-free, moisture-free air until the solvent is evaporated and the adhesive no longer moves over the surface.

Priming for adhesive resin cements other than RelyX™ Ultimate Adhesive Resin Cement:

6. Apply the primer, recommended by your cement. If no primer is recommended, apply Silane (e.g. RelyX™ Ceramic Primer) to the bonding surface of the restoration as per silane manufacturer's Instructions.

Pre-Treatment of Tooth

Note: Prepare tooth according to the instructions for use for maximum retention. Ensure complete hemostasis and optimum gingival retraction for a dry bonding environment. Do not use any chemical agents to disinfect, clean or seal the dentin.

1. To remove debris, thoroughly clean all bonding surfaces with a slurry of pumice and water, rinse and dry.
2. Apply rubber dam to isolate the tooth.
3. Selectively etch enamel for 15 seconds or etch enamel and dentin for 15 seconds.
4. Rinse with water.
5. Lightly air dry or blot dry leaving a slightly moist surface. Do not over dry.
6. Apply Scotchbond™ Universal Adhesive and scrub it in for 20 seconds.
7. If you have pooled adhesive, use a dry microbrush to remove it.
8. Blow a gentle stream of air over the liquid for about 5 seconds until it no longer moves and the solvent has evaporated completely.
9. For maximum adhesion light cure the adhesive for 10 seconds.

Seating

Before using RelyX™ Ultimate Adhesive Resin Cement, please refer to the Instructions for Use provided with the product package.

Note: Use a high power light. Make sure the light guide is clean and light is delivering full intensity. Proper curing technique is crucial for bonding success: for each curing cycle hold light guide steady at minimum distance to the restoration surface.

1. Apply a uniform layer of RelyX™ Ultimate Adhesive Resin Cement to the tooth or the restoration to ensure a void-free interface.
2. Completely seat the restoration using a suitable instrument.
3. While holding the restoration firmly in place, wipe off excess using sponge pellets and floss. Cover margins with glycerin gel.
OR
4. Wait 90 seconds or tack light cure for 1 second for cement to reach gel state. While holding restoration firmly in place, remove excess cement with a sharp instrument.
5. Light cure 20 seconds per surface. Total cure time should be at least 60 seconds depending on number of surfaces.
6. For dark shades or thick restorations, allow 6 minutes of total self-cure time from the start of mixing.
7. Finish margins after cure is fully established.
8. If desired polish intraorally as described in intraoral polishing section.

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1. Make any necessary adjustments with a diamond bur.
2. Finish with Sof-Lex™ Spiral Finishing Wheel.
3. Polish with Sof-Lex™ Spiral Polishing Wheel.
OR
4. Finish with diamond impregnated points/cups.
5. Apply intraoral polishing paste. Work paste slowly into surface using soft black, latch angle, intraoral prophyl brush.

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Lava™ Ultimate Restorative is a Resin Nano Ceramic that enables reliable characterization and re-sealing.

1. Roughen the Lava™ Ultimate restoration site.
2. Use Scotchbond™ Universal Adhesive or apply silane and adhesive according to the Instructions for Use.
3. Light cure bonding agent according to the Instructions for Use.
4. Apply Filtek™ Supreme Ultra Universal Restorative (or other methacrylate-based, light-cured restorative).
5. Light cure restorative material according to the Instructions for Use.
6. Polish the added restorative with Sof-Lex™ Spiral Finishing and Polishing Wheels.

Storage and Use

This product is designed to be stored and used at room temperature. Do not store Lava Ultimate restorative in intense light or sunlight. Product should be used by the date indicated on the outer package (36 months).

Disinfection

Disinfect this product using an intermediate level disinfection process (liquid contact) as recommended by the Centers for Disease Control and endorsed by the American Dental Association. **Guidelines for Infection Control in Dental Health-Care Settings - 2003 (Vol.52; No. RR-17), Centers for Disease Control and Prevention.**

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Customer Information

No person is authorized to provide any information that deviates from the information provided in this instruction sheet.

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3M ESPE warrants for 10 years from date of placement that restorations made from Lava Ultimate Implant Crown restorative will not break if fabricated using a 3M ESPE recommended milling machine in strict compliance with approved indications and instructions for use. 3M ESPE makes no other warranties including, but not limited to, any implied warranty of merchantability or fitness for a particular purpose.

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Limitation of Liability

Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental, or consequential, regardless of the theory asserted, including warranty, contract, negligence, or strict liability.

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