Registered No. 2021 - 2431

NOTARIAL CERTIFICATE

IL-HO MOON NOTARY PUBLIC OFFICE

505, 19, Gugal-ro 60beon-gil

Giheung-gu, Yongin-si, Gyeonggi-do, Rep. of Korea

Ilho Moon

Date: 15th of April, 2021

Hyonsu Kim, CEO

Approved by

(signature)

EASTDENT Co.Ltd.

EASTDENT Co.,Ltd.

Hyon Su. Kim

H. S. Kim / President

**INSTRUCTION FOR USE
FOR THE MEDICAL DEVICE**

Dental Retraction Cord CLEAN+SAFE®

Edition 1

1 Name of the medical device

Dental Retraction Cord CLEAN+SAFE®

Versions:

1. Dental Retraction Cord CLEAN+SAFE® ProfiCord, size: 0000; 000; 00; 0; 1; 2
2. Dental Retraction Cord CLEAN+SAFE® ProfiCord X, size: 0000; 000; 00; 0; 1; 2

Hereinafter referred to as the CLEAN+SAFE® cord, ProfiCord, ProfiCord X, device.

Information about the manufacturer, developer, and authorized representative of the manufacturer

Manufacturer:

EASTDENT Co., Ltd
64, Gugal-ro, Giheung-gu, Yongin-si, Gyeonggi-do, 16972, Korea.
Tel.: + 82-31-683-5767
Fax: + 82-31-683-5768
<http://east-dent.com>

Developer

EASTDENT Co., Ltd
64, Gugal-ro, Giheung-gu, Yongin-si, Gyeonggi-do, 16972, Korea.
Tel.: + 82-31-683-5767
Fax: + 82-31-683-5768
<http://east-dent.com>

Authorized representative of the manufacturer in the Russian Federation

PROTECO JSC
Legal address: 192102, St. Petersburg, 57 Salova str., bldg. 1, lit. I.
Business address: 196128, St. Petersburg, 5 Varshavskaya str., bldg. 2, office 401, Business Center Varshavsky.
Tel./Fax: +7 (812) 635-88-90;
E-mail: info@protecodent.ru

3 Intended use and potential user of device

3.1. Intended use of device

The medical device Dental Retraction Cord CLEAN+SAFE® is intended for use in dentistry for the temporary gingival retraction to expose the neck and root of a tooth.

3.2. Potential users

CLEAN+SAFE® cord is recommended for use by dentists.

4 Description of device

The device is a knitted, 100 % cotton thread of various thickness (6 sizes), made up by four differently colored threads (see Table 1).

Due to the special knitting technique, the cord boasts high resilience, elasticity, and excellent absorbing ability, which makes the dentist's work considerably easier.

Two types of cords are manufactured: non-impregnated ProfiCord and ProfiCord X which is impregnated with aluminum chloride hexahydrate solution ($\text{pH} = 3.5 \pm 5 \%$), 0.42 ± 0.315 mg/cm, which has astringent properties.

5 Indications, contraindications, and possible side effects

Indications:

Dental Retraction Cord CLEAN+SAFE® is used for the temporary gingival retraction:

- before the impression making in fixed dentures,
- before the teeth restoration procedure utilizing adhesive systems and/or crown installation,
- in delicate procedures on the gingival margin (e.g., preparation of the subgingival margin).

Contraindications:

1. Do not use in patients allergic to aluminum chloride hexahydrate.
2. Do not use in patients with acute or necrotizing ulcerative gingivitis.

Potential adverse reactions: not detected.

6 Safety precautions

1. When using the cord at an inflamed area, the benefits and risks should be carefully considered, because the retraction of the gingival margin can result in certain damage to the tissue.
2. For professional use in dentistry only.
3. For single use. Do not re-use.
4. Keep out of reach of children.
5. Do not leave the cord in the sulcus for more than 20 minutes to avoid the irreversible damage to the gingiva.
6. Upon completion of the procedure, always remove the cord from the sulcus.
7. Close the case lid immediately after using the cord.
8. Do not use expired devices.

7 Usage procedure

Usage procedure of Dental Retraction Cord CLEAN+SAFE®

1. Check the conditions of gingival tissue (e.g., depth or thickness).

2. Select a cord with an appropriate diameter.
3. Cut the cord to the required length with a cutter and a ruler. Soak the dental cord in a astringent solution if necessary.
4. Insert the cord into the sulcus by means of a retraction cord packer.
5. Leave the cord about 5 minutes in the sulcus.
6. Remove the cord just before taking the impression.
7. Slightly moisten the cord to make its removal easier and reduce gingival hemorrhage.

8 Technical characteristics

The length of the medical device "Dental Retraction Cord CLEAN+SAFE®" is 254 (±5%)cm.

Table 1. Characteristics of CLEAN+SAFE® cord

Device model	Size	Color	Thickness, mm
Proficord	0000 (extremely fine)	Black/blue	0.5 (±5 %)
	000 (ultra-fine)	Black/gray	0.7 (±5 %)
	00 (extra-fine)	Brown/yellow	0.82 (±5 %)
	0 (fine)	Violet/light violet	0.99 (±5 %)
	1 (medium)	Blue/light blue	1.2 (±5 %)
	2 (thick)	Green/light green	1.35 (±5 %)
Proficord X	0000 (extremely fine)	Black/blue	0.5 (±5 %)
	000 (ultra-fine)	Black/gray	0.7 (±5 %)
	00 (extra-fine)	Brown/yellow	0.82 (±5 %)
	0 (fine)	Violet/light violet	0.99 (±5 %)
	1 (medium)	Blue/light blue	1.2 (±5 %)
	2 (thick)	Green/light green	1.35 (±5 %)

9. The meaning of symbols used on the secondary and shipping package labeling:

Table 2. Symbols and definitions

Symbol	Description	Symbol	Description
	Manufacturer		Date of manufacture
	Non-sterile		Do not reuse
	Reference number (article)		Lot number

Symbol	Description	Symbol	Description
	Manufacturer		Date of manufacture
	CE compliance mark		Keep away from sunlight
	Consult Instructions for Use		Temperature limits
	Keep dry		The product must not be disposed of as household waste
	Do not use if package is damaged		Use-by date
	This way up (for shipping box)		Fragile, handle with care (for shipping box)

10. Requirements for maintenance and repair of medical device

The devices do not require maintenance or repairs.

11. Sterilization, cleaning, and disinfection

The devices are intended for single use. They do not require sterilization, disinfection, or cleaning.

12. Conditions of storage, transportation, and operation

Transportation

Devices packed in shipping box should be transported by all types of covered vehicles according to applicable regulations for the carriage of goods by this type of transport, observing the precautions indicated on the shipping box.

Transportation conditions:

- temperature: +1 to +30 °C,
- relative humidity: 45–55 %,
- atmospheric pressure: 800–1060 hPa.

Storage. The packaging protects the devices from physical and environmental factors during transportation and storage. Packages must be protected from contamination, dusting, moisture, and tearing. Store the device in a dry, clean, and secure place, in a tightly closed container.

Storage conditions:

- temperature: +1 to +30 °C,
- relative humidity: 45–55 %,
- atmospheric pressure: 800–1060 hPa.

Operation. Devices are used in healthcare facilities, in conditions requiring protecting the hands of medical staff when using the product for its intended purpose.

- temperature: +1 to +30 °C,
- relative humidity: 45–55 %,
- atmospheric pressure: 800–1060 hPa.

13. Disposal

The medical device Dental Retraction Cord CLEAN+SAFE® is considered hazardous waste of epidemiologic potential (devices contaminated with blood and/or other biological fluids of patients) after their use as intended.

In this case, the medical organization carries out only collection and temporary storage of the abovementioned waste. Collection is carried out using bags or disposable, color-coded containers for hazardous medical waste indoors or in places for temporary storage of medical waste before transporting for disposal.

Specialized organizations licensed to handle hazardous medical waste of epidemiologic potential transport medical waste from medical institutions and dispose of it.

Packaging elements, defective devices, and unused expired devices are considered epidemiologically safe waste (non-hazardous waste that does not come into contact with the patient's biological fluids) that is similar to municipal solid waste (MSW).

Non-hazardous waste is collected using reusable containers or disposable bags (excluding color-coded bags for hazardous waste). Disposable bags are placed inside reusable containers. Waste containers should be marked and placed at a special site (indoors). After emptying, reusable containers are subject to washing and disinfection. The procedure for washing and disinfection of reusable containers is determined in accordance with the organization-specific waste management flowchart.

Transportation and disposal are performed considering the requirements established for a specific territory in accordance with the country-specific sanitary laws concerning the maintenance of populated areas and handling of industrial and household waste.

14. Shelf life

Shelf life of ProfiCord: not limited.

Shelf life of ProfiCord X: 3 years.

15. List of international standards with which medical device complies with.

Requirements of Directive 93/42/EEC including amendments by Directive 2007/47/EC, EN 980, EN ISO 1041, EN 1641, EN IEC 62366, EN ISO 7405, EN ISO 10993-1, EN ISO 10993-2, EN ISO 10993-5, EN ISO 10993-10, EN ISO 10993-12, EN ISO 10993-18, EN ISO 13485, EN ISO 14971, EN ISO 15223-1, MEDDEV 2.7.1, rev. 4, MEDDEV 2.12.1, rev. 8, MEDDEV 2.12.2, rev. 2.

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IL-HO MOON NOTARY PUBLIC
OFFICE

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BELONG TO SUWON DISTRICT PROSECUTORS' OFFICE

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