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PreciControl Toxo IgG

For USA: **CONTENT**

PC TOXIGG1 human serum, negative to low positive for Toxo IgG antibodies

PC TOXIGG2 human serum, positive for anti-Toxo IgG antibodies

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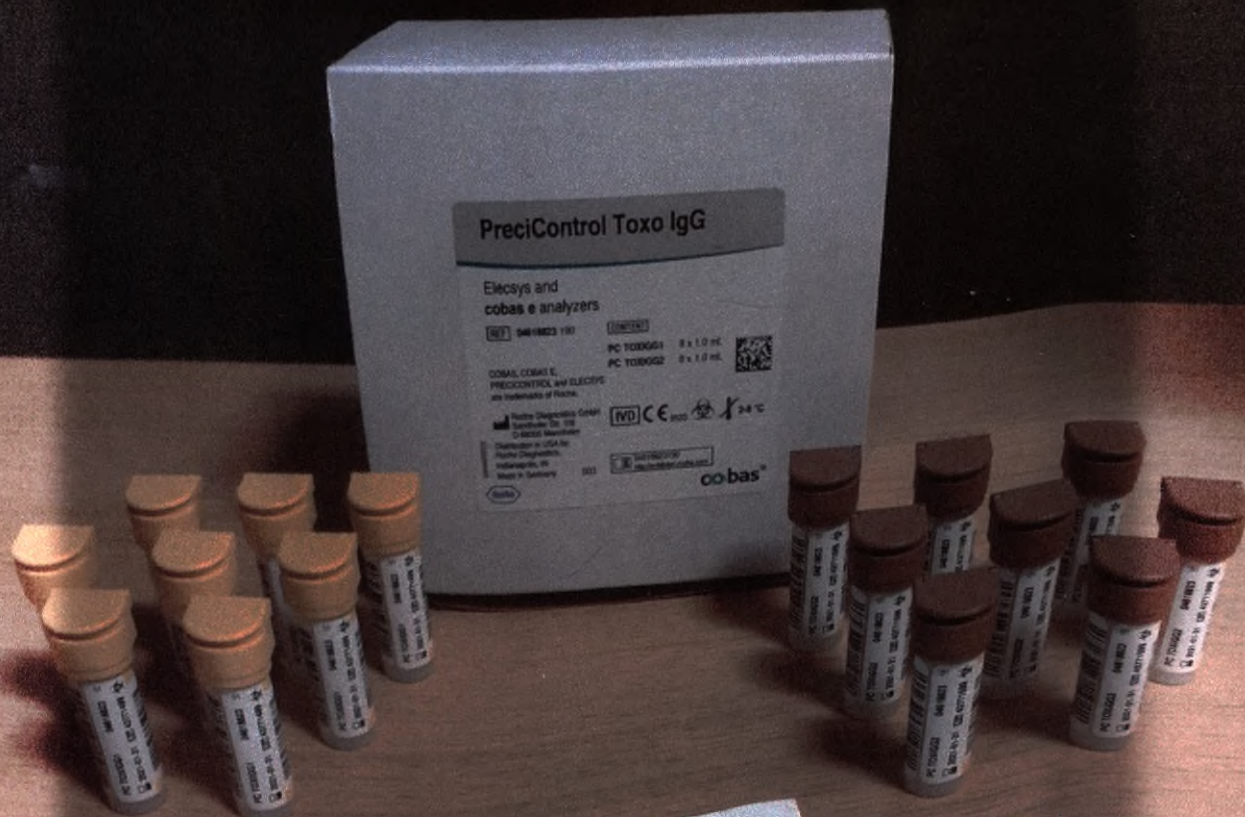
English

Precautions and warnings

For in vitro diagnostic use. Exercise the normal precautions required for the handling of human blood. Disposal of all waste material should be in accordance with the local safety data sheet available for professional use.

All human material should be considered as potentially infectious. Derived from human blood are prepared individually and shown to be free of HIV.

The sera containing anti-Toxo IgG (PC TOXIGG2) are not for intravenous use.

The testing methods applied were FTA/IT with the European Directive 98/79/EC.

However, as no testing method can provide absolute certainty, the material should be used as a patient specimen. In the event of a positive result, the responsible health authorities should be followed.

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The website displays either the version listed here or, if available, any later version that is also valid for this product code. If you have no internet access,

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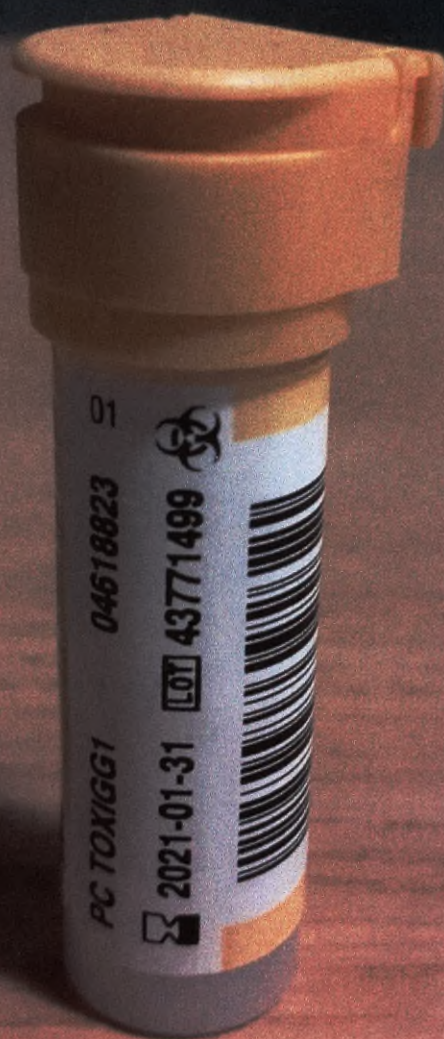
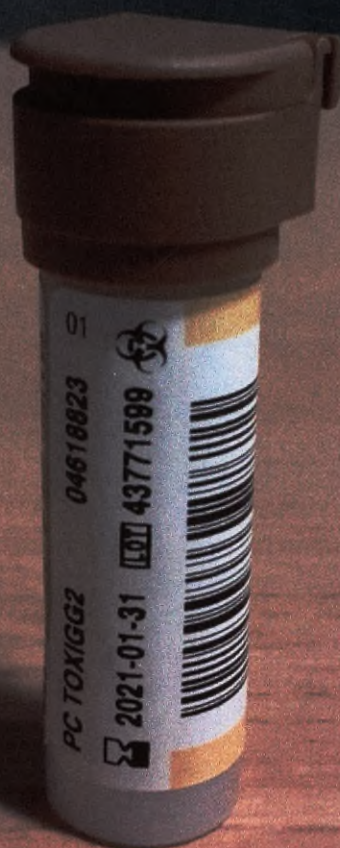
Precautions and warnings

For in vitro diagnostic use.

Exercise the normal precautions required for handling all laboratory reagents. Disposal of all waste material should be in accordance with local guidelines. Safety data sheet available for professional user on request.

All human material should be considered potentially infectious. All products derived from human blood are prepared exclusively from the blood of donors tested individually and shown to be free from HBsAg and antibodies to HCV and HIV.

The sera containing anti-Toxo IgG (PC TOXIGG1, PC TOXIGG2) were sterile filtered.

The testing methods applied were FDA-approved or cleared in compliance with the European Directive 98/79/EC, Annex II, List A.

However, as no testing method can rule out the potential risk of infection with absolute certainty, the material should be handled with the same level of care as a patient specimen. In the event of exposure, the directives of the responsible health authorities should be followed.

The controls may not be used after the expiration date.

Avoid foam formation in all reagents and sample types (specimens, calibrators and controls).

Method Sheet download

The box at the top of this document shows the product code together with the respective document version.

Please go to <http://e-labdoc.roche.com> to download this document.

The website displays either the version listed here or, if available, any later version that is also valid for this product code. If you have no internet access, please contact your local Roche affiliate to obtain the document free of charge.

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© 2010 Roche Diagnostics
Roche Diagnostics GmbH, Sandhofer Strasse 116, D-68305 Mannheim