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Bandage scissors, models: SCR-6030, SCR-6031.

2. Purpose/ indications for use

Safe cutting of thin plaster casts, bandages, clothing items, plastic and cardboard splints, coated duralumin splints, belts, etc. from various textile and polymeric materials.

3. Scope of Application

Emergency rooms of medical organizations, trauma departments of medical organizations, disaster medicine, emergency medical services, rescue services.

4. Description of design

Scissors consist of two blades/handles riveted together. The end of the upper blade is rounded, the end of the lower (longer) blade is equipped with a special tab facilitating with holding an item being cut. Furthermore, the bottom blade is serrated to avoid sliding of an item being cut.

5. Scissors material

The blades are made of austenitic-martensitic stainless steel AISI 420 (TAYYAB Surgical Forging, Pakistan). Steel chemical composition is shown in Table 1.

Table 1

Chemical element	C	Cr	Si	Mn	Fe	Ni
Content in %	Up to 0,45	Up to 14	Up to 0,8	Up to 0,8	Rest	Up to 1,0

Handles are made of PP 1040 Black polypropylene (ARAMCO Chemicals, Saudi Arabia).

6. Technical Specifications

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Name	Model SCR-6031	Model SCR-6030
Characteristic*		
Length, mm	180	145
Width, mm	90	66
Height, mm	8.7 ± 1.0	7.5 ± 1.0

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12. Safety of Use

For safety reasons, it is necessary to strictly follow all warnings in the operational documentation. Scissors have a smooth surface without potholes, pits or burrs.

13. Biological Effect Evaluation

Scissors handle material have direct non-invasive contact with the skin of the operator's hands, scissor blade materials have direct, non-invasive contact with the patient's body.

Information on the chemical composition of the steel used in the design is given in Table 1.

The medical device does not contain medicines.

The medical device does not contain animal or human cells, tissues, organs.

14. Risks of using a medical device and methods to reduce them to an acceptable level

Table 3

Hazardous situation	Factors that reduce the likelihood of a hazardous situation and causing harm
Environmental exposure to pollutants	Product disposal information in the operation manual
Presence of infectious agents on the products used	Information to a user about the need for disinfection and sterilization in the operation manual specifying the methods and parameters for product pre-sterilization cleaning, disinfection and sterilization.
Hazards associated with	Information to a user about the need to read the operation manual is given on the label.

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16. Repair and maintenance
Scissors are a non-repairable item, and do not require maintenance.

17. Disposal
Used scissors with mechanical damage, as well as scissors with an expired sterilization resource, or scissors with an expired service life should be disposed of as hazardous medical waste in accordance with the rules for the disposal of hazardous medical waste adopted in the country of use. Unused scissors with an expired shelf life should be disposed of as non-hazardous medical waste in accordance with the rules for the disposal of non-hazardous medical waste adopted in the country of use.

18. Service life
Service life is 18 months from the date of sale or 25-fold use, whichever comes first, subject to correct use and storage.

19. Shelf life
Shelf life is at least 36 months from the date of production, subject to storage conditions.

20. Packing and marking
Products are packed in bags. Bag has a label with printed information. Labeling is clear, legible and remains as such at all stages: storage, transportation and operation. Scissors bags are packed in cardboard boxes.

21. Transportation conditions

Air temperature, °C	-50 to +50
Relative air humidity	up to 90% at a temperature of 25°C

Storage of the scissors is allowed in a dry room. Products are transported packed in shipping containers by all enclosed transport types according to the good carriage regulations applicable to each transport type. Transportation process should prevent shocks, intense vibrations and moisture.

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Warranty period is 12 months from the date of sale.

23. Manufacturer

German Healthcare Industries (PVT.) LTD. (German
Healthcare Industries (PVT.) LTD.);

Address: 16/585, Shahab Pura, P.O. S.I.E., Sialkot,
Pakistan;

Tel. 0092-523-560865;

e-mail: info@germanhealthcare.net.

24. Complaints

Authorized representative of the manufacturer in the
Russian Federation: MEDPLANT LLC;

Address: 109316, Moscow, 42 Volgogradskiy
Avenue, bldg. 5, floor 2, room I, office 296-318;

Tel. +7 (495) 2236016;

email: medplant@medplant.ru.

25. Packaging labeling

Individual packaging should provide the following
information:

Product name and model: Bandage scissors,
models: SCR-6030/6031
Information on the Authorised Representative
Word "STAINLESS", «NON STERILE»
Registration certificate number and date of its
issuance;

	Manufacturer's name and address
	Manufacturing date
	Batch number
	Warning! Read the Instructions for User

A product itself is labeled as follows:

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