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REF 07026838190

GTIN 04015630940134

COBAS, COBAS E and
ELECSYS are trademarks
of Roche.

 Roche Diagnostics GmbH
Sandhofer Str. 116
D-68305 Mannheim

Distribution in USA by:
Roche Diagnostics,
Indianapolis, IN
Made in Germany 001

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REF 07026838190

For USA: **CONTENT**

- R1 HBeAg (E. coli, rDNA) > 7 ng/mL
- R2 Biotinylated monoclonal anti-HBe antibody (mouse)
> 0.8 mg/L; monoclonal anti-HBe antibody (mouse)
labeled with ruthenium complex > 0.2 mg/L
- M Streptavidin-coated microparticles 0.72 mg/mL
- Cal1 Human serum
- Cal2 Anti-HBe (human) approximately 3 IU/mL
in human serum

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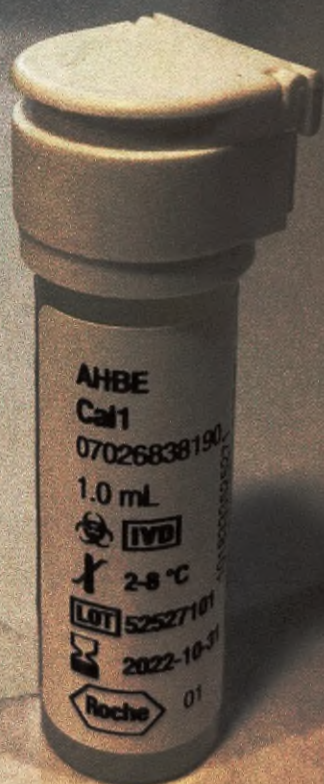
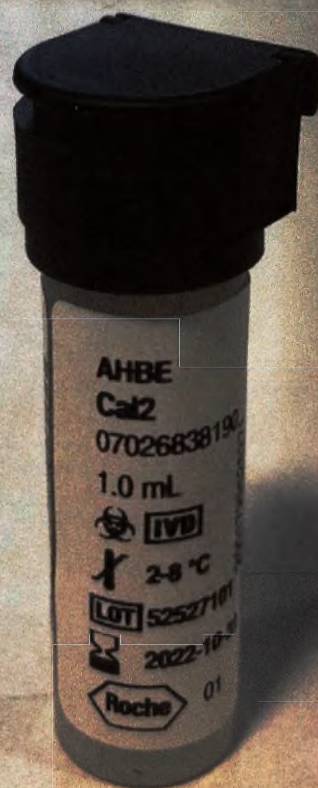
English

Precautions and warnings

For in vitro diagnostic use.
Exercise the normal precautions required for handling all laboratory reagents.
Disposal of all waste material should be in accordance with local guidelines.
Safety data sheet available for professional user on request.

This kit contains components classified as follows in accordance with the Regulation (EC) No. 1272/2008:

2-methyl-2H-isothiazol-3-one hydrochloride

EUH 208 May produce an allergic reaction.

Product safety labeling follows EU GHS guidance.

All human material should be considered potentially infectious.

The negative calibrator (AHBE Cal2) has been prepared exclusively from the blood of donors tested individually and shown to be free from HBsAg and antibodies to HCV and HIV.

The testing methods used assays approved by the FDA or cleared in compliance with the European Directive 98/79/EC, Annex II, List A.

The positive calibrator (AHBE Cal1) containing anti-HBe was tested for HIV and hepatitis C infections. The findings were negative. The serum containing anti-HBe was inactivated using β -propiolactone and UV-radiation.

However, as no inactivation or testing method can rule out the potential risk of infection with absolute certainty, the material should be handled with the same level of care as a patient specimen. In the event of exposure, the directives of the responsible health authorities should be followed.

Avoid foam formation in all reagents and sample types (specimens, calibrators and controls).

Method Sheet download

The product code together with the respective document version (if applicable) is stated in the download information boxes listed above. Please go to the website listed in the relevant box to download this document. The website displays either the version listed here or, if available, any later version that is also valid for this product code. If you have no internet access, please contact your local Roche affiliate to obtain the document free of charge.

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2-Methyl-2H-isothiazol-3-on-Hydrochlorid

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