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[제 41호 서식] SEOHYEON LAW & NOTARY PUBLIC OFFICE

Registered No. 2015 - 4048

NOTARIAL CERTIFICATE

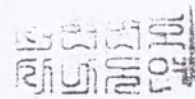
SEOHYEON LAW & NOTARY PUBLIC OFFICE

#608 BORAMCORE BUILDING 245-2 SEOHYEON-DONG

BUNDANG-GU SEONGNAM-SI GYEONGGI-DO KOREA

DECLARATION

We (applicant) : SINSIN PHARMACEUTICAL CO., LTD

do hereby solemnly and sincerely declare :

1. That the attached document(s) : OPERATIONAL DOCUMENTATION.

2.

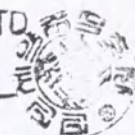
is(are) true and correct.

is(are) true translation for the text(s) originally written in the Korean language conscientiously believing the same to true and correct.

Signature

SINSIN PHARMACEUTICAL CO., LTD

PRESIDENT

Hanki Kim.

Aug. 27 ,2015

OPERATIONAL DOCUMENTATION

1. Destination

"Capsicum Medical Plaster LUXPLAST, versions: 94x124mm, 130x180mm" designed for individual use with radiculitis, neuralgia, myositis, lumbago, myalgia, arthritis, bruises, sprains, circulatory disorders and others.

2. Scope

Therapy.

3. Condition of use

Disposable medical products for personal use.

4. Basic specifications.

Patches for individual use are the non-sterile disposable products. Represent the perforated cotton base, with a mass of sticky of red color-protected peel off paper.

On the surface plasters are no mechanical damage (breaking edges).

Quality indicators "Capsicum Medical Plaster LUXPLAST, versions: 94x124mm, 130x180mm":

Indicator name	130x180 mm	94x124 mm
Length	180 mm	124 mm
Width	130 mm	94 mm

5. Contraindications

Violation of the integrity of the skin, dermatitis, weeping eczema, psoriasis, purulent skin diseases, not individual portability.

6. Side effects

Allergic reactions are possible. When the manifestation of a burning sensation in the area of application is necessary to remove the patch and lubricate the skin cream.

7. Preparation for work.

Check the expiry date on the package of the patch. The packaging must be new and must not have any damage. If packaging is broken, use another patch.

8. Mode of application

-Before using the patch degrease the skin;

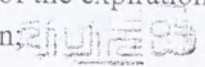
-Open the package, remove a protective film and fix the patch on the skin, gently pressing and smoothing.

-Replace as needed (you can not shoot for two days, unless there is irritation).

9. Marking.

The marking is applied directly on the packaging. Marking is applied by any means (printing, embossing, die), allowing its brightness, clarity and readability.

Marking plasters includes the following information:

- name of the producer(manufacturer), the name of the country of manufacture, address;
- importing company name, address;
- product name,trademark;
- product specificationsand quantity;
- sizes;
- manufacturing date, expiry date, batch number and date of the expiration date;
- the main raw material, which was used for the production; 
- month and year of manufacture of the product;
- the exact date or month of production;
- the shelf life of the product "Best before ..."
- disposal instructions: "Do not throw down the drain";
- rules for the use;
- productbarcode(if necessary);
- certification information (if available);
- registration number.

10. Storage conditionsand shelf life.

Shelf life 3 years from date of manufacture. Do not use beyond the expiration date. Store in a packaged format a temperature of 0⁰C to +30⁰C and relative humidity of 80% at a distance of not less than1 m from heaters, protected from direct sunlight.

11. Conditions of transportation.

Transported by any mode of transport.

During transport, use clean tools and safety precautions; do not place a heavy load on the product, Avoid contact with sharp objects, protect the product from shock; do not expose to sunlight.

Transpose at a temperature from 0⁰Cto +30⁰C and relative humidity of 80% at a distance of not less than1 m from heaters, protected from direct sunlight.

12. Warranty obligations

The manufacturer is not liable for the consequences resulting from improper use. Warranty obligations of 3 years.

13. Method of disposal

It is for bidden to lower the patch down the toilet. Used patch should be disposed of as household waste.

14. Manufacturer

SINSIN Pharmaceutical Co., Ltd., 90, Beomjigi-ro 141 Beon-gil, Danwon-gu, Ansan-si, Gyeonggi-do, Republic of Korea.

15. Authorized Representative of Russia

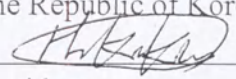
Limited Liability Company "BOLEAR MEDICA", LLC "BOLEAR MEDICA", Russia, 115201, Moscow, Kashirsky proezd, d. 13

tel./fax: +7(495) 787-92-68, e-mail: info@bolear.ru, www.bolear.ru.

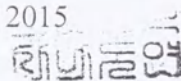
«I certify»

For SINSIN Pharmaceutical Co., Ltd.

The Republic of Korea

 Mr. Kim Han-Ki
President

“10” August 2015

“Capsicum Medical plaster LUXPLAST, versions: 94x124mm, 130x180mm”

Manufactured by SINSIN Pharmaceutical Co., Ltd.

90, Beomjigi-ro 141 Beon-gil, Danwon-gu, Ansan-si, Gyeonggi-do, Republic of Korea

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appeared before me and admitted said

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principal's subscription to the attached
DECLARATION

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Department Suwon District Prosecutors' office

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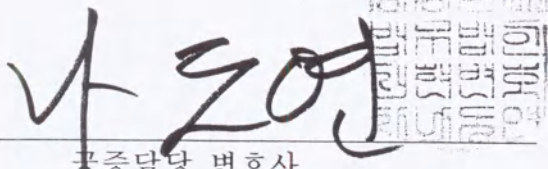
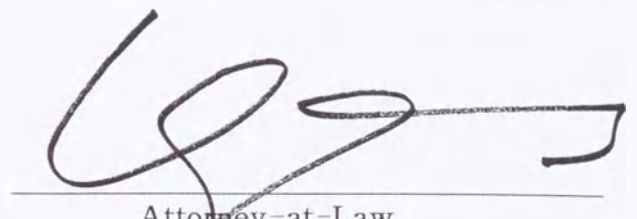
Attorney-at-Law
DO YEON NA

This office has been authorized by the Minister of Justice, the Republic of Korea, to act as Notary Public since May.15. 2003 under Law No.275

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