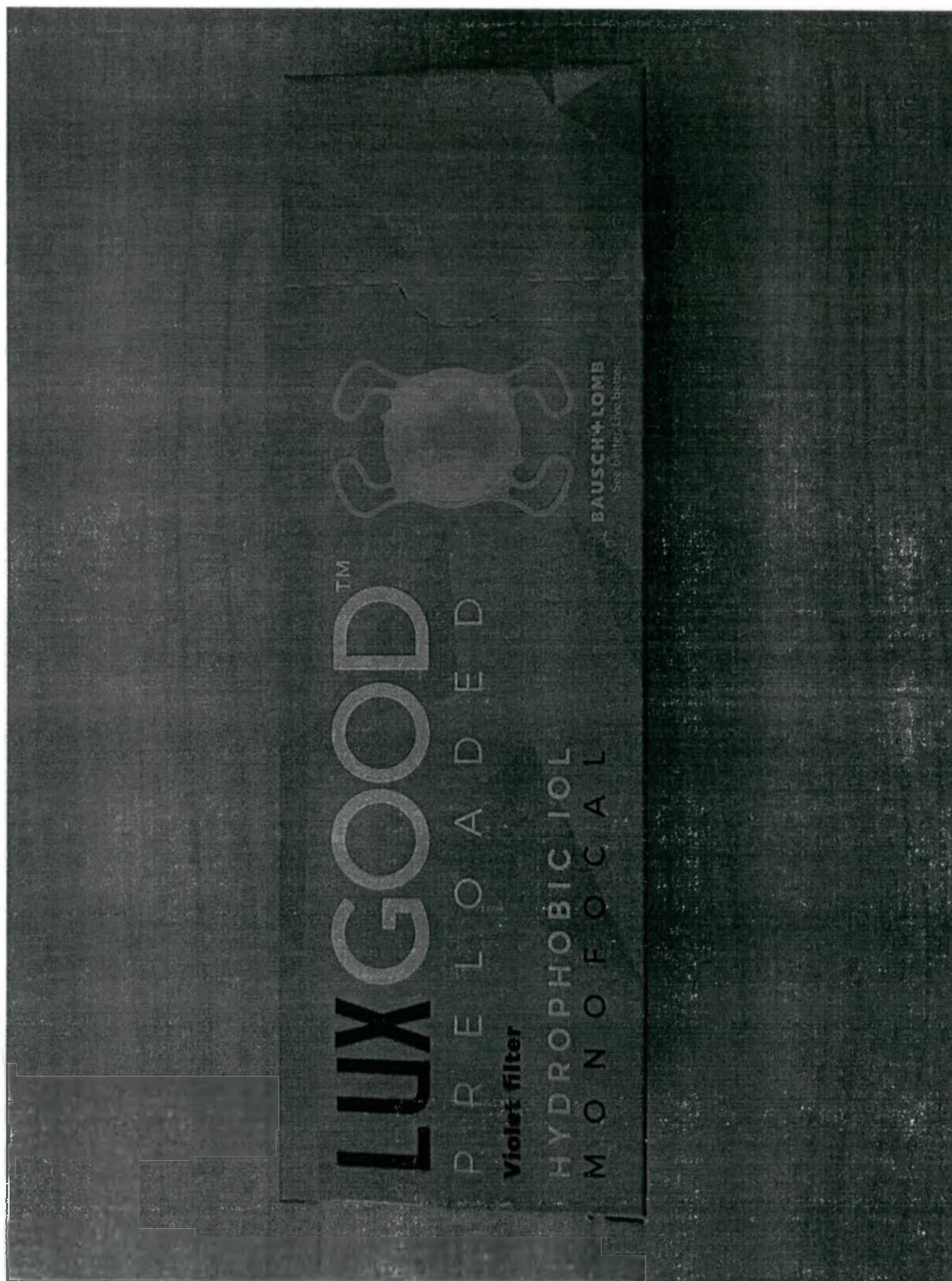
[illegible]

1) 03665493028268(17) 23 100 1/2 11 202 27

6.0 mm

11.0 mm

2020-10

LOY TE453

Do not
resist

Do not use
if package
is damaged

DE 2004 000000
DE 2004 000000

STERILE

45°C

CUTTING EDGE S.A.S
770 rue Alfred NOBEL
Immeuble le NOBEL
34000 Montpellier
FRANCE


[illegible]

YG00D + 10.00 D

SN 20247016020

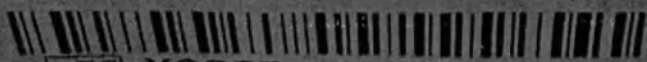🕒 2023-10-01

REF YG00D + 10.00D

Contains one preloaded sterile posterior chamber
hydrophobic acrylic intraocular lens with UV blocker.
Aberration free aspheric lens.

LUXGOOD is a trademark of Bausch & Lomb Incorporated or its affiliates.
© 2020 Bausch & Lomb Incorporated or its affiliates. CIES01

PATIENT NAME _____

DATE OF IMPLANTATION _____

IMPLANTED EYE L R

LOCATION _____

SURGEON NAME _____

BAUSCH+LOMB

1948200

BAUSCH+LOMB

Model LuxGood Crystal (LUXGOOD) foldable hydrophobic lens

Model LuxGood (LUXGOOD) yellow preloaded hydrophobic lens

EN INTRAOCULAR LENS

DESCRIPTION

The LuxGood intraocular lens (IOL) is a single-piece ultraviolet (UV) absorbing posterior chamber IOL developed to replace the natural crystalline lens in pseudophakic adult patients. The LuxGood IOL has aspheric optics. The lens is provided sterile and is designed to be folded prior to insertion in the eye and implantation in the capsular bag. The preloaded intraocular LuxGood lens is supplied with an Accospect Pro Delivery System to provide a touch-free method of delivery of the lens into the capsular bag. The LuxGood IOL is supplied in 2 different models - Model LuxGood Crystal or LuxGood. See label on the cardboard box for type of the lens.

PHYSICAL CHARACTERISTICS:

Lens / Haptic Material	Model LuxGood Crystal: Foldable hydrophobic acrylic copolymer containing a UV absorber. Model LuxGood: Foldable hydrophobic acrylic copolymer containing a UV absorber and a blue light filtering chromophore.
Material Characteristics	Index of Refraction when wet at 21°C: 1.544 Index of Refraction when in the eye at 35°C: 1.540
Spectral Transmittance	Model LuxGood Crystal: 10% transmittance at 343.5 nanometers for +20.0 diopter IOL. Model LuxGood: 10% transmittance at 343.5 nanometers for +20.0 diopter IOL. See FIG. 5, with chart 1.1, value = Wavelength (nm) and 1 value = % Transmittance
Optic Type	Model LuxGood Crystal and LuxGood: Aspheric; aberration-free
Powers	+00.00 to +34.00 Diopters
Dimensions	Body Diameter: 6.0 mm; Overall Diameter: 11 mm; Haptic Angle: 0°
Optic Orientation	The IOL is to be implanted with the anterior side of the lens facing up towards the anterior side of the eye. The orientation of the IOL can be verified by visual inspection of the lens. As illustrated in FIG. 5, when the orientation features are top right (A) and bottom left (B), you are facing the anterior side of the lens.

BAUSCH+LOMB
YG00D +10.00 D

SN 20247016020

Ø T: 11.0 mm

Ø B: 6.0 mm

Manufacturer:
CUTTING EDGE S.A.S.

CE 0197

STERILE

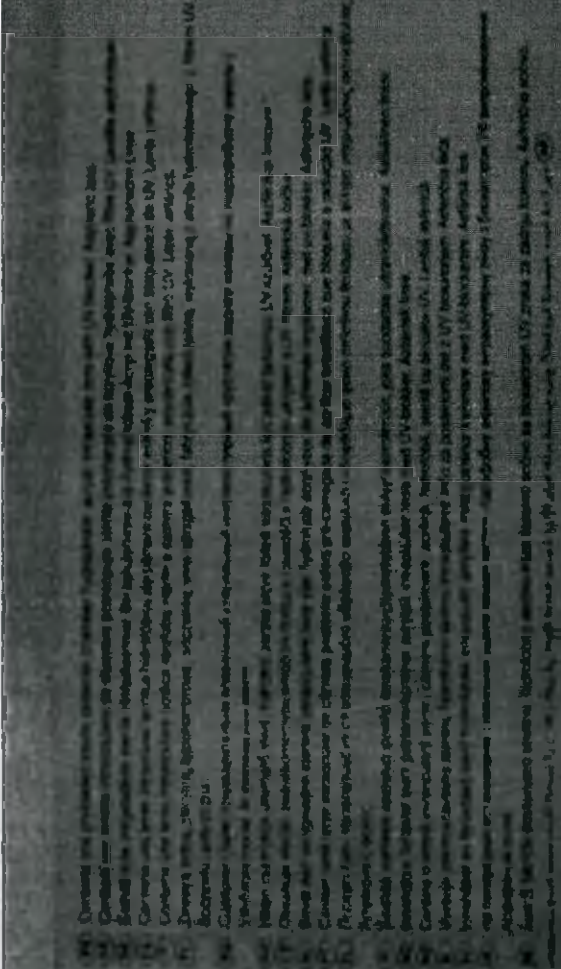
LUXSMART™

P R E L O A D E D

HYDROPHOBIC IOL
The IOL for your daily range of vision


Violet filter

BAUSCH+LOMB
See Better. Live Better.

YSMART +11.50 D

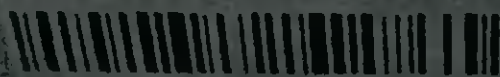
2023-06-17

SN 20149007005

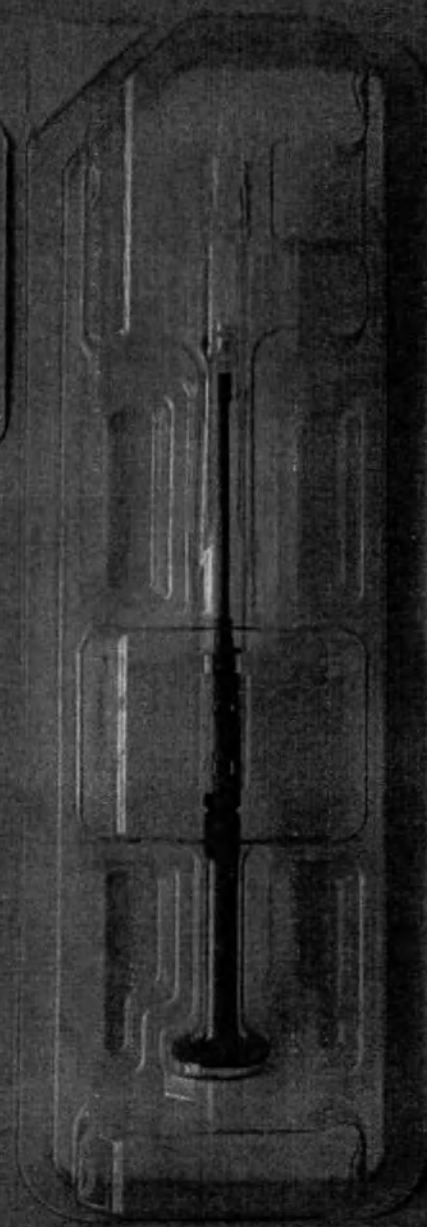YSMART + 11.50D

BAUSCH+LOMB
LUXSMART™
ASPHERIC LENS

Model LuxSmart Crystal 32/32SMART preloaded hydrophobic lens.
 Model LuxSmart CLC/SMART yellow preloaded hydrophobic lens.

EN INTRAOCULAR LENS

DESCRIPTION:
 The LuxSmart Intraocular Lens (IOL) is a single-piece ultraviolet (UV) absorbing posterior chamber IOL developed to replace the natural crystalline lens in pseudophakic adult patients. The LuxSmart IOL has aspheric optics. The lens is provided sterile and is designed to be folded prior to insertion in the eye and implantation in the capsular bag. The preloaded Intraocular LuxSmart Lens is supplied with an Accuflex Pro Delivery System to provide a touch-free method of delivery of the lens into the capsular bag. The LuxSmart IOL is supplied in 2 different models - Model LuxSmart Crystal or LuxSmart. See label on the cardboard box for type of the lens.

1945100

PHYSICAL CHARACTERISTICS:

Lens / Plastic Material	Model LuxSmart Crystal: Foldable hydrophobic acrylic copolymer containing a UV absorber. Model LuxSmart: Foldable hydrophobic acrylic copolymer containing a UV absorber and a blue light filtering chromophore.
Material Characteristics	Index of Refraction when wet at 21°C: 1.544 Index of Refraction when in the eye at 35°C: 1.540
Spectral Transmittance	Model LuxSmart Crystal: 10% transmittance at 393.5 nanometers for +20.0 diopter IOL. Model LuxSmart: 10% transmittance at 397 nanometers for +20.0 diopter IOL. See FIG. 8 with chart's X-value = Wavelength (nm) and Y-value = % Transmittance.
Optic Type	Model LuxSmart Crystal and LuxSmart: Aspheric.
Power	+00.00 to +34.00 Diopters
Dimensions	Body Diameter: 5.0 mm; Overall Diameter: 11 mm; Haptic Angle: 0°
Lens Orientation	The IOL is to be implanted with the anterior side of the lens facing up towards the anterior side of the eye. The orientation of the IOL can be verified by visual inspection of the lens. As illustrated in FIG. 9, when the orientation features are top (H), (A) and bottom left (B), you are facing the anterior side of the lens.

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