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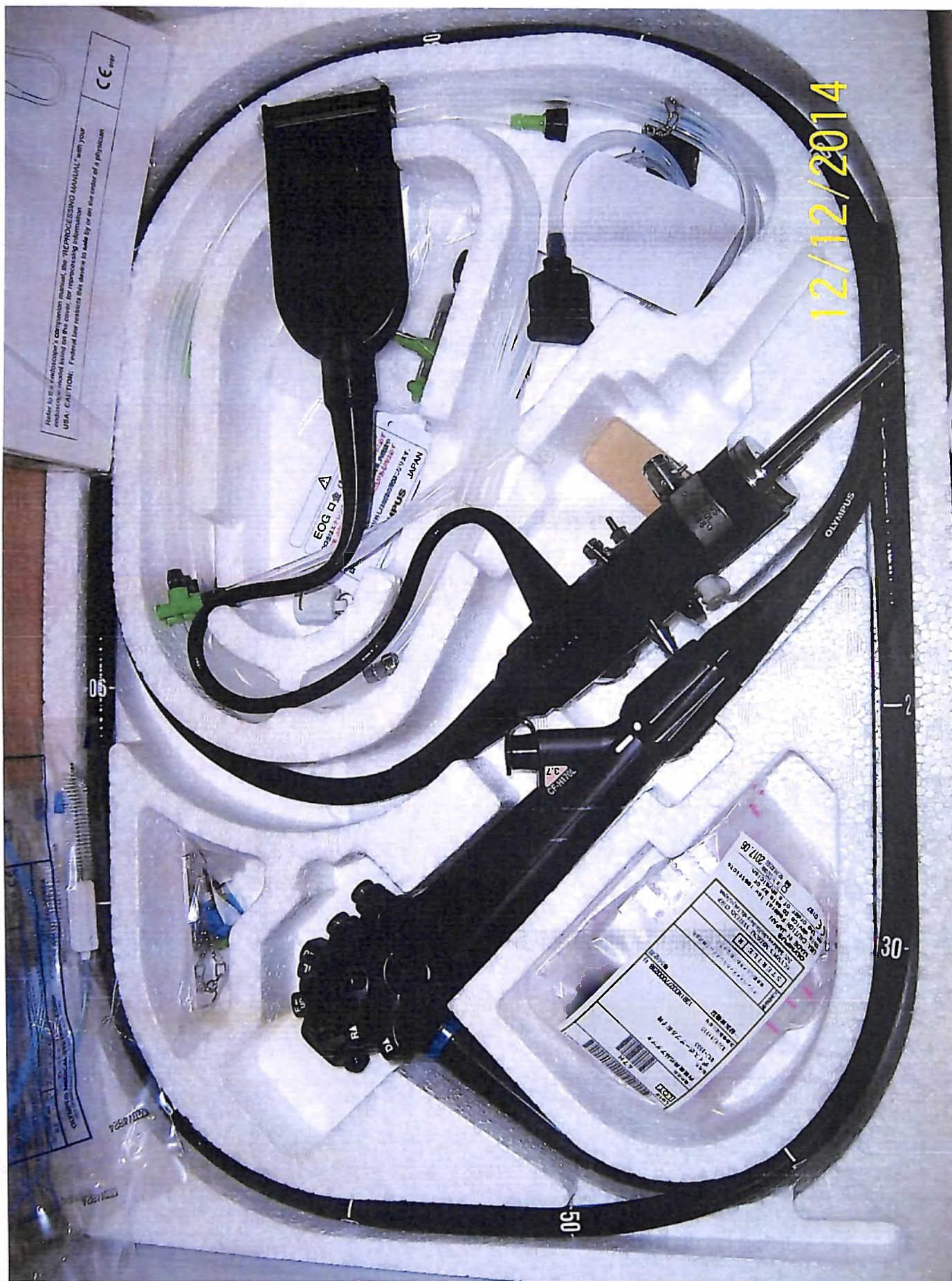
INSTRUCTIONS

OPERATION MANUAL

Symbols	1
Important Information — Please Read Before Use	2
Chapter 1. Checking the Package Contents	13
Chapter 2. Instrument Nomenclature and Specifications	17
Chapter 3. Preparation and Inspection	29
Chapter 4. Operation	63
Chapter 5. Troubleshooting	87
Appendix	99

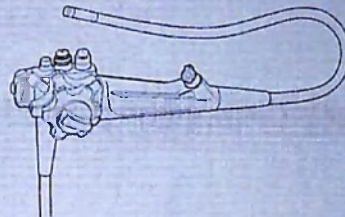
GASTROINTESTINAL VIDEOSCOPE

OLYMPUS GIF-XP170N¹⁾
OLYMPUS GIF-H170

¹⁾ This product may not be available in some areas.

COLONOVideoscope

OLYMPUS CF-H170L/I

Refer to the endoscope's companion manual, the "REPROCESSING MANUAL" with your endoscope model listed on the cover, for reprocessing information.

USA: CAUTION: Federal law restricts this device to sale by or on the order of a physician.

CE 10/12/2014

OLYMPUS

INSTRUCTIONS

GASTROINTESTINAL VIDEOSCOPE

OLYMPUS GIF-XP170N^{*1}
OLYMPUS GIF-H170

^{*1} This product may not be available in some areas.

COLONOVIDEOSCOPE

OLYMPUS CF-H170U/I


REPROCESSING MANUAL

Chapter 1	General Policy	1
Chapter 2	Function and Inspection of the Accessories for Reprocessing	9
Chapter 3	Compatible Reprocessing Methods and Chemical Agents	25
Chapter 4	Reprocessing Workflow for Endoscopes and Accessories	33
Chapter 5	Reprocessing the Endoscope (and related reprocessing accessories)	41
Chapter 6	Reprocessing the Accessories	91
Chapter 7	Reprocessing Endoscopes and Accessories Using an Automated Endoscope Reprocessor/Washer-Disinfector	103
Chapter 8	Storage and Disposal	105

Accessories:

- Biopsy valve (MB-358)
- Air/water valve (MH-436)
- Auxiliary water tube (MAJ-855)
- Suction cleaning adapter (MH-856)
- Injection tube (MH-946)
- ETO cap (MB-156)
- Channel cleaning brush (BW-201)

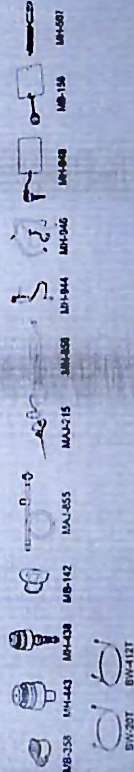- Suction valve (MH-443)
- Mouthpiece (MB-142, MA-474)
- Auxiliary water inlet cap (MAJ-215)
- Channel plug (MH-944)
- AW channel cleaning adapter (MH-948)
- Channel-opening cleaning brush (MH-507)
- Single use combination cleaning brush (BW-4127)

Refer to the endoscope's companion manual, the "OPERATION MANUAL" with your endoscope model listed on the cover, for operation information.

USA: CAUTION: Federal law restricts this device to sale by or on the order of a physician.

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