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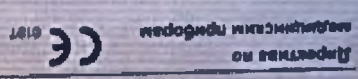
GASTROINTESTINAL VIDEOSCOPE

OLYMPUS GIF-XP170N^{*1}
OLYMPUS GIF-H170

^{*1} This product may not be available in some areas.

COLONOVIDEOSCOPE

OLYMPUS CF-H170L/I

Symbols 1

Important Information — Please Read Before Use 2

Chapter 1 Checking the Package Contents 13


Chapter 2 Instrument Nomenclature and Specifications 17

Chapter 3 Preparation and Inspection 29

Chapter 4 Operation 63

Chapter 5 Troubleshooting 87

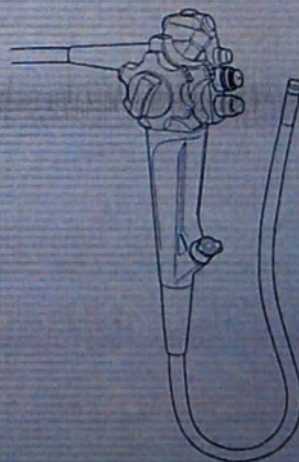Appendix 99

Refer to the endoscope's companion manual, the "REPROCESSING MANUAL" with your endoscope model listed on the cover, for reprocessing information.
USA: CAUTION: Federal law restricts this device to sale by or on the order of a physician.

CE 0187

10/12

INSTRUCTIONS

GASTROINTESTINAL VIDEOSCOPE

OLYMPUS GIF-XP170N^{*1}
OLYMPUS GIF-H170

^{*1} The product may not be available in some areas.

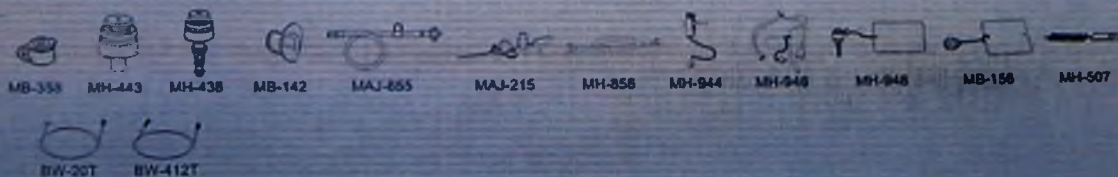
COLONOVIDEOSCOPE

OLYMPUS CF-H170L/I

Accessories:

- Biopsy valve (MB-358)
- Air/water valve (MH-438)
- Auxiliary water tube (MAJ-855)
- Suction cleaning adapter (MH-856)
- Injection tube (MH-946)
- ETO cap (MB-156)
- Channel cleaning brush (BW-20T)

- Suction valve (MH-443)
- Mouthpiece (MB-142, MA-474)
- Auxiliary water inlet cap (MAJ-215)
- Channel plug (MH-944)
- AW channel cleaning adapter (MH-948)
- Channel-opening cleaning brush (MH-507)
- Single use combination cleaning brush (BW-412T)

Refer to the endoscope's companion manual, the "OPERATION MANUAL" with your endoscope model listed on the cover, for operation information.

USA: CAUTION: Federal law restricts this device to sale by or on the order of a physician.

Chapter 1 General Policy

1

Chapter 2 Function and Inspection of the Accessories for Reprocessing

9

Chapter 3 Compatible Reprocessing Methods and Chemical Agents

25

Chapter 4 Reprocessing Workflow for Endoscopes and Accessories

33

Chapter 5 Reprocessing the Endoscope (and related reprocessing accessories)

41

Chapter 6 Reprocessing the Accessories

91

Chapter 7 Reprocessing Endoscopes and Accessories Using an Automated Endoscope Reprocessor/Washer-Disinfector

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