

Филлер-имплант инъекционный на основе гиалуроновой кислоты для внутритканевого введения в исполнении Юма Женес объем 1,0 мл



Uma Jeunesse®

For Lasting Beauty

GB Instructions for Use

Composition: Uma Jeunesse contains 24 mg/ml fully cross linked hyaluronic acid.

Contents: One sterile single use syringe with 1 ml or 2 ml Uma Jeunesse, two sterile needles 27 G 1/2", product leaflet and two labels.

Properties: Uma Jeunesse is a sterile, isotonic, apyrogenic gel made of highly purified hyaluronic acid of non-animal origin. It has not been tested on animals.

Indications: Uma Jeunesse can be used for treatment of facial wrinkles by intradermal injection and also for volume augmentation.

Mode of Action: Uma Jeunesse acts by increasing tissue volume in the extracellular matrix thus smoothing wrinkles. Always inject in mid to deep dermis.

Usage Recommendations: Pre-treat area with local anaesthetic and disinfect. Affix needle to syringe by rotating it 90° clockwise. Inject into mid to deep dermis by linear threading or serial puncture technique. Do not overcorrect or

Uma Jeunesse 1ml	UJ0513P
	2016-05
CE 1015	Cambridge Medical Aesthetics Ltd
Uma Jeunesse 1ml	UJ0513P
	2016-05
CE 1015	Cambridge Medical Aesthetics Ltd

ген. директор
2016.11.12
Международный
Медицинский
36 центр
Баилов С.Б.
МОСКВА * 90

Филлер-имплант инъекционный на основе гиалуроновой кислоты для внутритканевого введения в исполнении Юма Женес объем 2,0 мл



Uma
Jeunesse
For lasting beauty

Uma Jeunesse®
For Lasting Beauty

Instrucciones de uso

Composición: Uma Jeunesse contiene 24 mg/ml completamente reticulado de ácido hialurónico.
Contenido: Una jeringa estéril de un solo uso con 1 ml o 2 ml de Uma Jeunesse, dos agujas estériles 27G-18", el prospecto del producto y dos etiquetas.

Propiedades: Uma Jeunesse es una solución estéril, isotónica, apirógena hecha de gel de ácido hialurónico altamente purificada de origen no animal. No ha sido probado en animales.

Indicaciones: Uma Jeunesse se puede utilizar para el tratamiento de las arrugas faciales mediante inyección intradérmica y también para el aumento del volumen.

Modo de acción: Uma Jeunesse actúa aumentando el volumen de tejido en la matriz extracelular tanto, suavizar las arrugas. Inyectar a mediados de la dermis profunda.

Recomendaciones de uso: Trate previamente la zona con anestesia local y desinfectar. Coloque la aguja

Uma Jeunesse 2ml	UJT 1111A
CE 1015	2014-11
Uma Jeunesse 2ml	UJT 1111A
CE 1015	2014-11

Cartagena Medical Aesthetics Ltd

Врач-косметолог
ООО "СМЗ" ^{г. Минск}
Богачев А.Б.
«Международный Медицинский Эксперт»
Минск

