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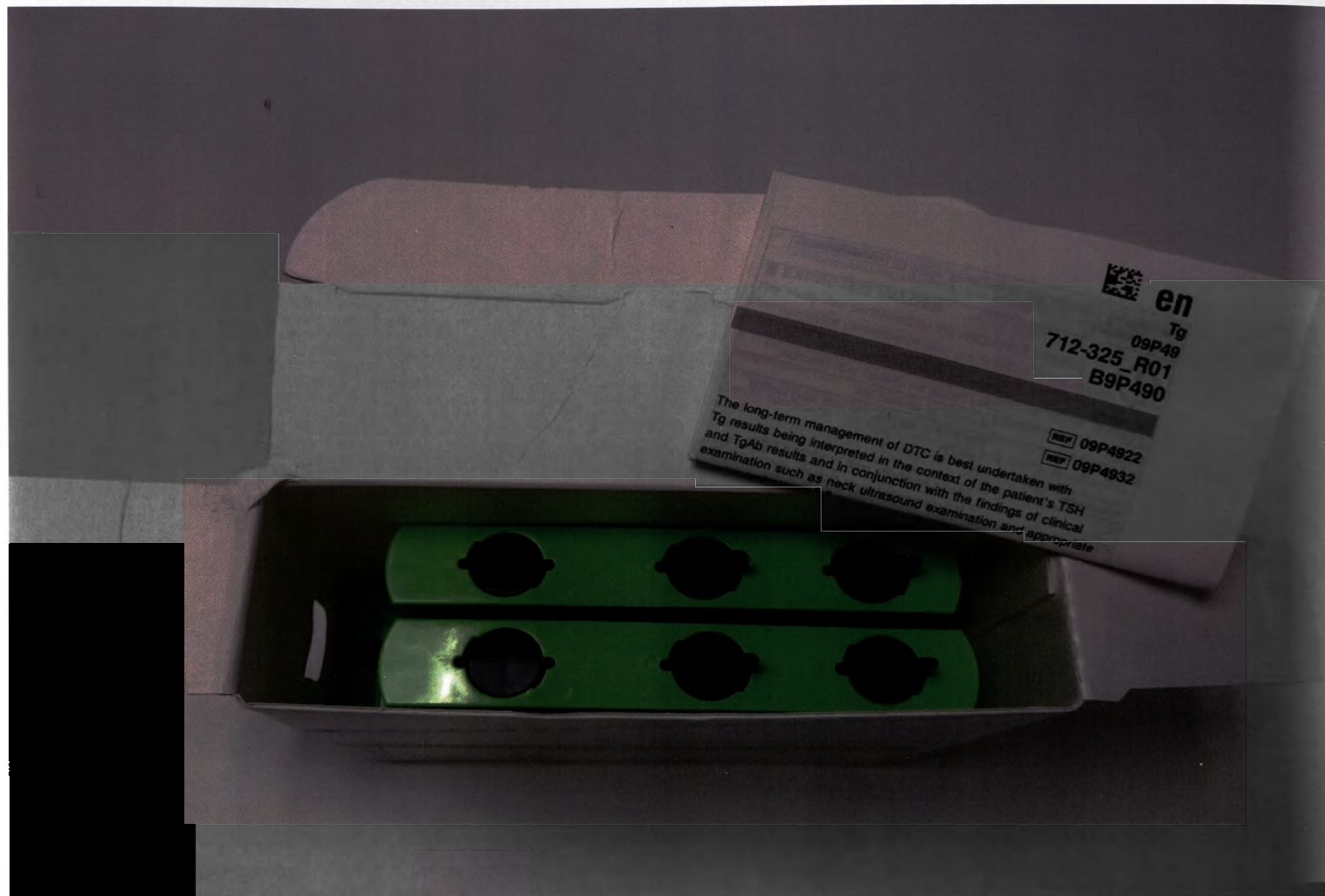
Alinity i Thyroglobulin Reagent Kit

Revised March 2020

Instructions must be carefully followed. Performance of assay results cannot be guaranteed if there are any deviations from these instructions.

NAME

Alinity i Thyroglobulin Reagent Kit (also referred to as Tg)

INTENDED USE

The Alinity i Thyroglobulin assay is a chemiluminescent microparticle immunoassay (CMIA) used for the quantitative determination of thyroglobulin in human serum and plasma on the Alinity i analyzer. Determination of thyroglobulin is used as an aid in monitoring after total thyroid ablation (thyroidectomy).

WARNING: The Alinity i Thyroglobulin assay should be used in the absence of anti-thyroglobulin antibodies. Specimens positive for anti-thyroglobulin antibodies can cause falsely decreased concentrations of thyroglobulin. Patients positive for anti-thyroglobulin antibodies should be evaluated for thyroglobulin with alternative methods.

SUMMARY AND EXPLANATION OF THE TEST

Thyroglobulin (Tg) is a homodimeric glycoprotein with a molecular weight of approximately 660 kDa.^{1, 2} It is synthesized in both normal and malignant thyroid follicular cells and is the most highly expressed

This long-term investigation of QC is based on the following: 1. Tg results being interpreted in the context of the patient's total imaging techniques.

BIOLOGICAL PRINCIPLES OF THE PROCEDURE

This assay is a delayed one-step immunoassay for the quantitative determination of Tg in human serum and plasma using chemiluminescent microparticle immunoassay (CMIA) technology. Sample and anti-Tg antibody (I15) coated paramagnetic microparticles are combined and incubated. The Tg present in the sample binds to the anti-Tg antibody (I15) coated microparticles. Anti-Tg antibody (I24) acridinium-labeled conjugate is added to create a reaction mixture. This reaction mixture is incubated. Following a wash cycle, Pre-Trigger and Trigger Solutions are added. The resulting chemiluminescent reaction is measured as relative light units (RLUs). There is a direct relationship between the amount of Tg in the sample and the RLUs detected by the system optics. For additional information on system and assay technology, refer to the Alinity ci-series Operations Manual, Section 3.

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! i

Abbott GmbH
Max-Planck-Ring 2
65205 Wiesbaden
Germany
+49-6172-540

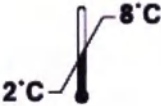
PRODUCED FOR ABBOTT BY
Euphonia Diagnostics, Inc.
251 Great Valley Parkway
Malvern, Pennsylvania, 19355, USA

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!

Abbott GmbH
Max-Planck-Ring 2
65205 Wiesbaden
Germany
+49-6172-580

PRODUCED FOR ABBOTT BY
Eugene Diagnostics, Inc.
201 Green Valley Parkway
Malvern, Pennsylvania, 19355, USA

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