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FOR USE WITH
Alinity C

Created June 2020

Instructions must be carefully followed. Reliability of assay results cannot be guaranteed if there are any deviations from these instructions.

For laboratory professional use only

NAME

Complement C3 (also referred to as C3)

INTENDED USE

The Complement C3 assay is used for the quantitation of C3 in human serum or plasma on the Alinity C system.

SUMMARY AND EXPLANATION OF THE TEST

All complement proteins are acute phase reactants and rise rapidly in concentrations during inflammatory episodes. Conversely, the complement protein catabolism may greatly increase in disease states.

Refer to the following table for a general guide to evaluation of C3 and C4 protein levels in the presence of decreased hemolysis.

Normal C4	
Normal C3	Alterations in vitro (e.g., improper specimen handling, coagulation-associated complement consumption, in vitro errors (false low C3 or C4))
Decreased C3	- Acute glomerulonephritis - Membranoproliferative glomerulonephritis - Immune complex diseases - Active SLE - Immune C3 deficiency
Decreased C4	- Immune complex diseases - Hyperimmunoglobulinemic states - Cryoglobulinemia - Hereditary angioedema - Immune C4 deficiency - Active SLE - Serum sickness - Autoimmune hemolytic anemia

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MANUFACTURED FOR
Abbott Laboratories

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