APPROVED BY

" 9/10/ 2018

EURONDA S.P.A., Italy

Via dell' Artigianato 7,  
36030 Montecchio Precalcino (VI), Italy,  
tel. +39 0445 329811, fax: +39 0445 865246  
web-site: www.euronda.com

**EURONDA S.p.a.**

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INSTRUCTION FOR USE

Disposable aspirating dental device for removal of liquids:  
Canula MONOART® EM15 Saliva Ejector with fixed tip.

## 1. Name of the device:

Disposable aspirating dental device for removal of liquids: canula MONOART® EM15 Saliva Ejector with fixed tip.

### Hardware versions:

Canula MONOART® EM15 Saliva Ejector with fixed tip, diameter 6.5 mm, length 125 mm, color yellow;

Canula MONOART® EM15 Saliva Ejector with fixed tip, diameter 6.5 mm, length 125 mm, color colorless;

Canula MONOART® EM15 Saliva Ejector with fixed tip, diameter 6.5 mm, length 125 mm, color green;

Canula MONOART® EM15 Saliva Ejector with fixed tip, diameter 6.5 mm, length 125 mm, color lilac;

Canula MONOART® EM15 Saliva Ejector with fixed tip, diameter 6.5 mm, length 125 mm, color blue;

Canula MONOART® EM15 Saliva Ejector with fixed tip, diameter 6.5 mm, length 125 mm, color orange;

Canula MONOART® EM15 Saliva Ejector with fixed tip, diameter 6.5 mm, length 125 mm, color pink;

Canula MONOART® EM15 Saliva Ejector with fixed tip, diameter 6.5 mm, length 125 mm, color lime;

Canula MONOART® EM15 Saliva Ejector with fixed tip, diameter 6.5 mm, length 125 mm, color burgundy;

Canula MONOART® EM15 Saliva Ejector with fixed tip, diameter 6.5 mm, length 125 mm, color gray;

Canula MONOART® EM15 Saliva Ejector with fixed tip, diameter 6.5 mm, length 150 mm, color yellow;

Canula MONOART® EM15 Saliva Ejector with fixed tip, diameter 6.5 mm, length 150 mm, color colorless;

Canula MONOART® EM15 Saliva Ejector with fixed tip, diameter 6.5 mm, length 150 mm, color green;

Canula MONOART® EM15 Saliva Ejector with fixed tip, diameter 6.5 mm, length 150 mm, color lilac;

Canula MONOART® EM15 Saliva Ejector with fixed tip, diameter 6.5 mm, length 150 mm, color blue;

Canula MONOART® EM15 Saliva Ejector with fixed tip, diameter 6.5 mm, length 150 mm, color orange;

Canula MONOART® EM15 Saliva Ejector with fixed tip, diameter 6.5 mm, length 150 mm, color pink;

Canula MONOART® EM15 Saliva Ejector with fixed tip, diameter 6.5 mm, length 150 mm, color lime;

Canula MONOART® EM15 Saliva Ejector with fixed tip, diameter 6.5 mm, length 150 mm, color burgundy;

Canula MONOART® EM15 Saliva Ejector with fixed tip, diameter 6.5 mm, length 150 mm, color gray.

Hereinafter referred to as “device”, “canula”.

### Instruction for Use

Disposable aspirating dental device for removal of liquids:

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tel. +39 0445 329811, fax: +39 0445 865246

web-site: [www.euronda.com](http://www.euronda.com)

Production site: Via Chizzalunga 1-36066 Sandrigo (VI) Italy

## 3. Authorized representative of the manufacturer

PROTECO JSC

Legal address: 192102, Russia, Saint Petersburg, Salova street 57, Building 1, Letter "I"

Tel. / fax: +7 (812) 635-88-90

Web-site: [www.protecodent.ru](http://www.protecodent.ru)

e-mail: [info@protecodent.ru](mailto:info@protecodent.ru)

## 4. Purpose, contraindications

Device is intended for removal (aspiration) of liquids from patient's oral cavity in the process of dental operations and procedures. A disposable, non-sterile device.

Canula MONOART® EM15 Saliva Ejector with fixed tip is recommended for the following customers:

qualified medical staff to perform various dental operations and procedures as specified by the manufacturer.

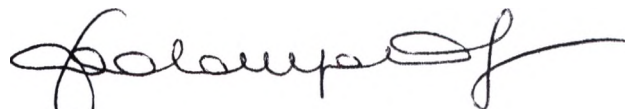
Scope of use: dental rooms in certain health care institutions, private dental rooms, dental clinics, including private ones.

Contraindications for use: the contraindication for use is the individual intolerance to the device materials.

Side effects associated with the intended purpose of the medical device: an irritant action or an allergic reaction due to the individual intolerance.

### Precautions:

- The contraindication for use is the individual intolerance to the device materials.
- Before operating, read and understand the instruction for use.
- Use the canula only in accordance with its intended purpose (refer to "Scope of use").
- Do not use a defective canula, if its package has been damaged before opening. The canula is a single-use device!
- Keep a new or an unused canula in a dry, clean and safe place.

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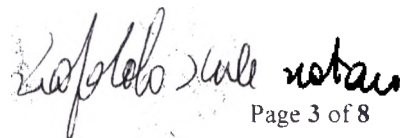
P. IVA 00595740242

Instruction for Use

Disposable aspirating dental device for removal of liquids:

Canula MONOART® EM15 Saliva Ejector with fixed tip.

Page 3 of 8

## 5. Description of the device:

The device is a tube armored with metal wire, which makes it possible to put it into a desired shape and keep it. A soft tip of a steam-line form with flexible grooves excluding the possibility of the oral mucosa irritation prevents mucosal injuries and contributes to a comfortable removal of saliva. This ensures the suction (aspiration) of liquids in any, even the hardest-to-reach areas of patient's mouth.

The connection between canula and aspiration equipment is possible by inserting the canula in the conic connector of the aspiration tube. It is possible to connect the canula to tubes of different type aspiration systems authorized according to the regulations for use in conformity with the operation manual for the system of a particular type.

The medical device "Disposable aspirating dental device for removal of liquids: Canula MONOART® EM15 Saliva Ejector with fixed tip" has been used in clinical practice for more than one decade. The device is safe for users and patients. EURONDA S.P.A, Italy has been manufacturing the medical device "Disposable aspirating dental device for removal of liquids: Canula MONOART® EM15 Saliva Ejector with fixed tip" since 1975, it has been sold in many countries. No incidents or critical complaints have been registered.

## 6. Technical specifications

Dimensions of the device:      Diameter (6.5±10%) mm length (125±10%) mm  
                                         Diameter (6.5±10%) mm, length (150±10%) mm  
                                         Inner diameter (4.5±10%) mm  
Weight: 3.5±0.3 g (at length of 125±10% mm), 4.1±0.3 g (at length of 150±10% mm)

Color: yellow, colorless, green, lilac, blue, orange, pink, lime, burgundy, gray.

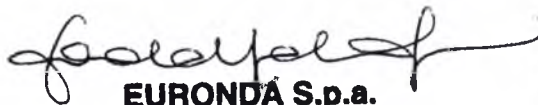
## 7. Materials

Vinyl chloride compounds, copper-plated iron wire, glue mixture of epoxy-acrylic polymers emulsified in water, cyclohexanone.

## 8. Application procedure

- 1) Bend the tube under the required angle.
- 2) Connect the canula to the aspiration system of the dental treatment unit with its side without the fixed tip.
- 3) Place the side with a fixed tip in the oral cavity of the patient to work.
- 4) At work, direct the canula onto the treated surface achieving a complete removal of liquids by adjusting the aspiration intensity depending on the capabilities of the aspiration system.
- 5) After use, the canula should be treated with a disinfectant and disposed according to the regulations.

Instruction for Use  
Disposable aspirating dental device for removal of liquids:  
Canula MONOART® EM15 Saliva Ejector with fixed tip.

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## Disinfection

The medical device "Disposable aspirating dental device for removal of liquids: Canula MONOART® EM15 Saliva Ejector with fixed tip" is not intended for the contact with an injured surface, blood or injection materials, in addition it does not cause any damage to it.

This device is non-sterile, but clean, and it does not need preliminary treatment and is intended for single use. This disposable devices are subject to disinfection/neutralization after being used in the process of dental operations, their reuse is forbidden.

After being used Canula MONOART® EM15 Saliva Ejector with fixed tip must be placed in a special container with a disinfecting solution observing the requirements for concentration and hold-up time indicated by the manufacturer. It is necessary to use only disinfecting substances and detergents suitable for application with polyvinyl chloride observing indications for concentration, temperature and operating time as instructed by the manufacturer of disinfecting substances and detergents. After being treated with a disinfecting agent the devices are to be disposed according to the established procedure.

## 9. List of standards

The medical device "Disposable aspirating dental device for removal of liquids: Canula MONOART® EM15 Saliva Ejector with fixed tip" is manufactured in accordance with ISO 13485: 2016 and meets the internal requirements and standards: ISO 9001:2015, ISO 14971:2012, EN 1041:2013, ISO 15223-1:2016, ISO 10993-1:2009, ISO 10993-5:2009, ISO 10993-10:2010.

## 10. Shelf life.

Shelf life of the canula is 5 years from the date of manufacture provided that storage and transportation conditions are observed.

## 11. Data on disposal or destruction of the medical device

Medical devices must be disposed in accordance with medical procedures adopted in healthcare institutions and organizations, and local, state or federal norms and regulations. The choice of safe disposal and destruction methods depends on capacity and field of medical organization, availability of waste decontamination/detoxification facilities, waste detoxification/disposal method used in the administrative territory (incineration, collection to waste dump, recycling).

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### Instruction for Use

Disposable aspirating dental device for removal of liquids:  
Canula MONOART® EM15 Saliva Ejector with fixed tip.

Page 5 of 8

The medical device “Disposable aspirating dental device for removal of liquids: Canula MONOART<sup>®</sup> EM15 Saliva Ejector with fixed tip” belongs to epidemiologically dangerous medical wastes: materials and instruments, items contaminated with blood and/or other body liquids.

Consumer package, defective device and device with expired shelf life refer to epidemiologically safe wastes, similar in composition to solid household waste (SHW).

## 12. Data on labeling, packaging and scope of supply

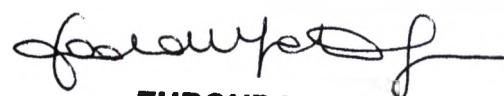
### 12.1. Labeling

The device, in quantity of 100 pcs, is packed in multiple consumer package (bag) made of polyethylene film. Even soldering along two sides of the package guarantees its integrity during transportation.

Bags are packed in a carton box for transportation.

### 12.2. Scope of supply

- The medical device “Disposable aspirating dental device for removal of liquids: Canula MONOART<sup>®</sup> EM15 Saliva Ejector with fixed tip” (The hardware version is chosen among available ones according to para. 1 in a multiple (consumer) package in quantity of 100 pcs).
- Instruction for use (in accompanying documentation).

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Instruction for Use  
Disposable aspirating dental device for removal of liquids:  
Canula MONOART<sup>®</sup> EM15 Saliva Ejector with fixed tip.

### 12.3. Description of symbols used in package labeling:

| Symbol                                                                             | Description                       | Symbol                                                                            | Description                             |
|------------------------------------------------------------------------------------|-----------------------------------|-----------------------------------------------------------------------------------|-----------------------------------------|
|   | Date of manufacture               |  | Manufacturer                            |
|   | Best before                       |  | Temperature range                       |
|   | Lot number                        |  | CE marking                              |
|   | Item number, part No.             |  | Refer to instruction for use            |
|   | Non-sterile                       |  | Protect from moisture                   |
|   | Single use                        |  | Dispose separately from household waste |
|  | Do not use if package is damaged. |                                                                                   |                                         |

### 13. Storage and transportation conditions

#### Transportation

The canula is transported by all kinds of transport in covered vehicles in accordance with shipping rules applicable to such transport.

- Air temperature: -25 °C to + 20 °C
- Relative humidity: up to 80 %

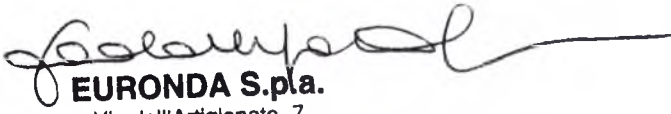
#### Storage

The canula is kept in dry and ventilated rooms.

- Air temperature: +15 °C to +25 °C
- Relative humidity: 45 to 60 %


Package must be protected from dust, moisture and risk of recontamination.

Keep a new or unused canula in a dry, clean and safe place.

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#### Instruction for Use

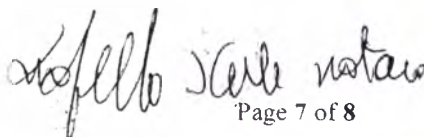Disposable aspirating dental device for removal of liquids:  
 Canula MONOART® EM15 Saliva Ejector with fixed tip.

  
 Page 7 of 8

#### 14. Manufacturer's guarantees

The manufacturer guarantees compliance of the medical device to the requirements affecting quality, effectiveness, safety and functional purpose provided that the consumer observes the operation, transportation and storage conditions.

Information on undesired events associated with use of the medical device "Disposable aspirating dental device for removal of liquids: Canula MONOART® EM15 Saliva Ejector with fixed tip" which have signs of an adverse event (incident), should be forwarded to the authorized representative of the manufacturer: "PROTECO", JSC, legal address: 192102, Russia, Saint Petersburg, Salova street 57, Building 1, Letter "I", tel.: / fax: +7 (812) 635-88-90, website: [www.protecodent.ru](http://www.protecodent.ru), e-mail: [info@protecodent.ru](mailto:info@protecodent.ru)

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