20

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CERTIFICATE

中国国际贸易促进委员会暨中国国际商会
China Council for the Promotion of International Trade is China Chamber of International Commerce

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China Council for the Promotion of International Trade
China Chamber of International Commerce

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THIS IS TO CERTIFY THAT: the seal of INTEGRAL MEDICAL
PRODUCTS CO., LTD. on the annexed INSTRUCTIONS FOR USE
NO:JCQ-T-038-IFU, V3.0 is genuine.

China Council for the Promotion
of International Trade

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Authorized
Signature:

Zhang Yan

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(Date: Oct. 31, 2022)

书查询网址 Website for verifying the certificate: <http://www.rzcepit.com/validate.html>

INSTRUCTIONS FOR USE**JCQ-T-038-IFU, V3.0**

Suction tube with Y-connector Vacon type

Approved by

Name: **Tracy Mo**Position: **General Manager**

Signature, stamp

INSTRUCTIONS FOR USE

Suction tube with Y-connector Vacon type

RU № XXX dated XX.XX.XXXX

1. Suction tube with Y-connector Vacon type, design options:

- 1.1. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 1.67 mm, length 380 mm (5F)
- 1.2. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 1.67 mm, length 430 mm (5F)
- 1.3. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 1.67 mm, length 450 mm (5F)
- 1.4. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 1.67 mm, length 500 mm (5F)
- 1.5. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 1.67 mm, length 520 mm (5F)
- 1.6. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 1.67 mm, length 530 mm (5F)
- 1.7. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 1.67 mm, length 600 mm (5F)
- 1.8. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 2.0 mm, length 380 mm (6F)
- 1.9. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 2.0 mm, length 430 mm (6F)
- 1.10. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 2.0 mm, length 450 mm (6F)
- 1.11. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 2.0 mm, length 500 mm (6F)
- 1.12. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 2.0 mm, length 520 mm (6F)
- 1.13. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 2.0 mm, length 530 mm (6F)
- 1.14. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 2.0 mm, length 600 mm (6F)
- 1.15. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 2.67 mm, length 380 mm (8F)
- 1.16. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 2.67 mm, length 430 mm (8F)
- 1.17. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 2.67 mm, length 450 mm (8F)

- Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 2.67 mm, length 500 mm (8F)
- 1.19. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 2.67 mm, length 520 mm (8F)
- 1.20. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 2.67 mm, length 530 mm (8F)
- 1.21. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 2.67 mm, length 600 mm (8F)
- 1.22. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 3.33 mm, length 380 mm (10F)
- 1.23. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 3.33 mm, length 430 mm (10F)
- 1.24. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 3.33 mm, length 450 mm (10F)
- 1.25. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 3.33 mm, length 500 mm (10F)
- 1.26. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 3.33 mm, length 520 mm (10F)
- 1.27. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 3.33 mm, length 530 mm (10F)
- 1.28. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 3.33 mm, length 600 mm (10F)
- 1.29. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 4.0 mm, length 380 mm (12F)
- 1.30. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 4.0 mm, length 430 mm (12F)
- 1.31. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 4.0 mm, length 450 mm (12F)
- 1.32. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 4.0 mm, length 500 mm (12F)
- 1.33. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 4.0 mm, length 520 mm (12F)
- 1.34. Suction tube with Y-connector, Vacon Type, without X-ray contrast strip, diameter 4.0 mm, length 530 mm (12F)
- 1.35. Suction tube with Y-connector, Vacon Type, without X-ray contrast strip, diameter 4.0 mm, length 600 mm (12F)
- 1.36. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 4.67 mm, length 380 mm (14F)
- 1.37. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 4.67 mm, length 430 mm (14F)
- 1.38. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 4.67 mm, length 450 mm (14F)
- 1.39. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 4.67 mm, length 500 mm (14F)
- 1.40. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 4.67 mm, length 520 mm (14F)
- 1.41. Suction tube with Y-connector, Vacon Type, without X-ray contrast strip, diameter 4.67 mm, length 530 mm (14F)

- .. Suction tube with Y-connector, Vacon Type, without X-ray contrast strip, diameter 4.67 mm, length 600 mm (14F)
- 1.43. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 5.33 mm, length 380 mm (16F)
- 1.44. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 5.33 mm, length 430 mm (16F)
- 1.45. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 5.33 mm, length 450 mm (16F)
- 1.46. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 5.33 mm, length 500 mm (16F)
- 1.47. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 5.33 mm, length 520 mm (16F)
- 1.48. Suction tube with Y-connector, Vacon Type, without X-ray contrast strip, diameter 5.33 mm, length 530 mm (16F)
- 1.49. Suction tube with Y-connector, Vacon Type, without X-ray contrast strip, diameter 5.33 mm, length 600 mm (16F)
- 1.50. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 6.00 mm, length 380 mm (18F)
- 1.51. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 6.0 mm, length 380 mm (18F)
- 1.52. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 6.0 mm, length 450 mm (18F)
- 1.53. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 6.0 mm, length 500 mm (18F)
- 1.54. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 6.0 mm, length 520 mm (18F)
- 1.55. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 6.0 mm, length 530 mm (18F)
- 1.56. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 6.0 mm, length 600 mm (18F)
- 1.57. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 6.67 mm, length 380 mm (20F)
- 1.58. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 6.67 mm, length 430 mm (20F)
- 1.59. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 6.67 mm, length 450 mm (20F)
- 1.60. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 6.67 mm, length 500 mm (20F)
- 1.61. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 6.67 mm, length 520 mm (20F)
- 1.62. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 6.67 mm, length 530 mm (20F)
- 1.63. Suction tube with Y-connector, Vacon type, without X-ray contrast strip, diameter 6.67 mm, length 600 mm (20F)
- 1.64. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 1.67 mm, length 380 mm (5F)

- Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 1.67 mm, length 430 mm (5F)
- 1.66. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 1.67 mm, length 450 mm (5F)
- 1.67. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 1.67 mm, length 500 mm (5F)
- 1.68. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 1.67 mm, length 520 mm (5F)
- 1.69. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 1.67 mm, length 530 mm (5F)
- 1.70. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 1.67 mm, length 600 mm (5F)
- 1.71. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 2.0 mm, length 380 mm (6F)
- 1.72. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 2.0 mm, length 430 mm (6F)
- 1.73. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 2.0 mm, length 450 mm (6F)
- 1.74. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 2.0 mm, length 500 mm (6F)
- 1.75. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 2.0 mm, length 520 mm (6F)
- 1.76. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 2.0 mm, length 530 mm (6F)
- 1.77. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 2.0 mm, length 600 mm (6F)
- 1.78. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 2.67 mm, length 380 mm (8F)
- 1.79. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 2.67 mm, length 430 mm (8F)
- 1.80. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 2.67 mm, length 450 mm (8F)
- 1.81. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 2.67 mm, length 500 mm (8F)
- 1.82. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 2.67 mm (8F) x520 mm (8F)
- 1.83. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 2.67 mm, length 530 mm (8F)
- 1.84. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 2.67 mm, length 600 mm (8F)
- 1.85. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 3.33 mm, length 380 mm (10F)
- 1.86. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 3.33 mm, length 430 mm (10F)
- 1.87. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 3.33 mm, length 450 mm (10F)
- 1.88. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 3.33 mm, length 500 mm (10F)

- Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 3.33 mm, length 520 mm (10F)
- 1.90. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 3.33 mm, length 530 mm (10F)
- 1.91. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 3.33 mm, length 600 mm (10F)
- 1.92. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 4.0 mm, length 380 mm (12F)
- 1.93. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 4.0 mm, length 430 mm (12F)
- 1.94. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 4.0 mm, length 450 mm (12F)
- 1.95. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 4.0 mm, length 500 mm (12F)
- 1.96. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 4.0 mm, length 520 mm (12F)
- 1.97. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 4.0 mm, length 530 mm (12F)
- 1.98. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 4.0 mm, length 600 mm (12F)
- 1.99. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 4.67 mm, length 380 mm (14F)
- 1.100. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 4.67 mm, length 430 mm (14F)
- 1.101. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 4.67 mm, length 450 mm (14F)
- 1.102. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 4.67 mm, length 500 mm (14F)
- 1.103. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 4.67 mm, length 520 mm (14F)
- 1.104. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 4.67 mm, length 530 mm (14F)
- 1.105. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 4.67 mm, length 600 mm (14F)
- 1.106. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 5.33 mm, length 380 mm (16F)
- 1.107. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 5.33 mm, length 430 mm (16F)
- 1.108. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 5.33 mm, length 450 mm (16F)
- 1.109. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 5.33 mm, length 500 mm (16F)
- 1.110. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 5.33 mm, length 520 mm (16F)
- 1.111. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 5.33 mm, length 530 mm (16F)
- 1.112. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 5.33 mm, length 600 mm (16F)

- J. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 6.0 mm, length 430 mm (18F)
- 1.114. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 6.0 mm, length 430 mm (18F)
- 1.115. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 6.0 mm, length 450 mm (18F)
- 1.116. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 6.0 mm, length 500 mm (18F)
- 1.117. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 6.0 mm, length 520 mm (18F)
- 1.118. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 6.0 mm, length 530 mm (18F)
- 1.119. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 6.0 mm, length 600 mm (18F)
- 1.120. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 6.67 mm, length 380 mm (20F)
- 1.121. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 6.67 mm, length 430 mm (20F)
- 1.122. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 6.67 mm, length 450 mm (20F)
- 1.123. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 6.67 mm, length 500 mm (20F)
- 1.124. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 6.67 mm, length 520 mm (20F)
- 1.125. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 6.67 mm, length 530 mm (20F)
- 1.126. Suction tube with Y-connector, Vacon Type, with X-ray contrast strip, diameter 6.67 mm, length 600 mm (20F)
2. Operation documentation
- instruction for use

Medical device overview

Vacon type suction tube with Y-connector is made of clear non-toxic medical PVC. It has open atraumatic distal end and 2 side openings. Shape and size of the tube guarantee easy passage through endotracheal and tracheotomy tubes. The tube has a Vacon type vacuum-control connector (Y-type). Connector is color coded according to the tube size by Charriere scale CH/F. Size range: 5, 6, 8, 10, 12, 14, 16, 18, 20 F.

Suction tube is specially designed for easy passage through plastic endotracheal and tracheotomy tubes with minimum irritation to patient.

The device consists of a tube (catheter) with the same inner and outer diameter along its entire length, and a connector.

The tube has an open well-rounded distal end with two elliptical side openings (eyelets) shifted from one another for easy and efficient suction. Distal end and eyelets have smooth finish to prevent sensitive trachea mucous membrane from damage. The axis of the working end is straight.

The tube is made with or without an X-ray contrast strip.

Suction tube is manufactured according to ISO 8836.

Suction tube can be used together with a vacuum control device and has a Vacon type connector. Connector is located at the proximal end of the catheter, which is connected to the source of vacuum or close to it. Outer end of the connector is installed a semi-rigid or elastomeric tube with an inner diameter of 6 mm.

vacuum control device is designed to control the flow of air and suctioned substance.
 A connector is color coded according to the nominal outer diameter conforming to ISO 8836.
 Color coding of the connector makes it possible to select suction tube of the right size, matching the size, of the endotracheal or tracheostomy tube quickly and accurately.
 Tube length – from 380 mm to 600 mm.
 The device is sterile. For single use only. Not to be re-sterilized.

Fig. 1 Suction tube with Y-connector, Vacon type

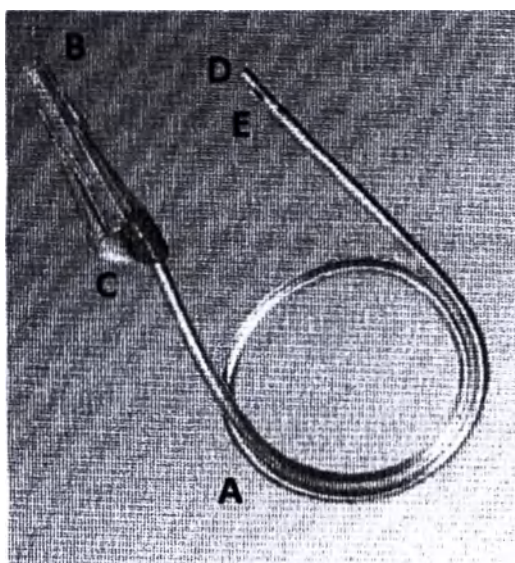
Fig. 2 Suction tube with Y-connector, Vacon type:

- A – tube main body;
- B – connector for connection to a vacuum source
- C – vacuum control device, Vacon type
- D – central opening of the tube (end opening)
- E – elliptical side openings (eyelets) at the end of the catheter.

INTENDED USE

Suction tube is designed for suction of secretions, mucus and blood with mucus from upper respiratory tract and oral cavity during anesthesia and reanimation.
 The device is intended for use in medical institutions for medical, routine and diagnostic procedures.

INDICATIONS FOR USE

Designed for suction of secretions, mucus and blood with mucus from upper respiratory tract and oral cavity during anesthesia and reanimation.

AREA OF USE.

The device is intended for use in medical institutions for medical, routine and diagnostic procedures.

INDICATIONS

Patients with spasms in one or several organs

Oxygenless patient

- Patients with cardiovascular diseases

- Patients allergic to PVC

Determined by the doctor individually for each patient, based on their condition.

POSSIBLE SIDE EFFECTS

When using a suction catheter, pain or bleeding is possible due to damage to the trachea.

PRECAUTIONS. WARNINGS

- Twisting of the tube due to patient movement or other reasons may block the fluid suction path.
- It is necessary to take into account the anthropometric parameters of the patient to avoid difficulties when inserting the aspiration catheter to the patient (resistance to insertion, sticking, twisting, blockage).
- EO sterilized. For single use only. Do not re-sterilize and do not reuse.
- Do not use, if package or product is damaged or dirty.
- Do not expose to direct sunlight.
- Use tube of appropriate size that matches the inner diameter of the endotracheal or tracheostomy tube when inserting the suction tube through the endotracheal or tracheostomy tube.
- Use immediately after opening the package.

Warnings

- Hypoacidity
- Mucous membrane of the respiratory tract is damaged
- Arrhythmia
- Obstructive atelectasis
- Airway spasm

INSTRUCTIONS FOR USE

Device preparation

- choose a tube of a suitable size according to the current regulations, that matches the inner diameter of the endotracheal or tracheostomy tube when inserting the suction tube through the endotracheal or tracheostomy tube.
- check package integrity. Do not use if package is damaged.
- remove the tube from the sterile package. In good light conditions make sure there are no visible defects.

Operating procedure

- Insert the tube into the upper airway, connect the connector to aspiration (suction) equipment. Control the degree of aspiration by adjusting the suction unit, for catheters with vacuum control - using a window for vacuum control, by closing its lumen with a finger from time to time. Do not allow the tube to suck on the tissues, as this can lead to injury and possible complications.
- Choose a tube of a suitable size according to the current regulations.
- Check package integrity. Do not use if package is damaged.
- Remove the tube from the sterile package. In good light conditions make sure there are no visible defects.

Insert the tube into the upper airway, connect the connector to aspiration (suction) equipment. Control the degree of aspiration by adjusting the suction unit, for catheters with vacuum control - using a window for vacuum control, by closing its lumen with a finger from time to time.

Do not allow the tube to suck on the tissues, as this can lead to injury and possible complications.

TOOLS USED FOR MARKING

	Lot number
	Best before date
	Production date
	Do not reuse
	Do not re-sterilize
	EO sterilized
	Do not use if package is damaged
	Refer to instructions for use
	Caution! See instructions for use
	Does not contain natural latex
NON TOXIC	Not toxic
	CE Marking

PARAMETERS, CHARACTERISTICS (PROPERTIES) OF A MEDICAL DEVICE GUARANTEED BY THE MANUFACTURER

Main parameters, characteristics (properties) of the medical device are specified in the tables below.
Where applicable, deviation from stated values $\pm 10\%$ is allowed, unless otherwise specified.

Name	Color	Nominal Outer diameter			Length, mm permissible deviation $\pm 5\%$	X-ray contrast strip	Connector type
		F	mm	Permissible tolerance of the outer diameter from the nominal value, mm			
Suction tube 5F length 380 mm, 400 mm, 430 mm, 450 mm, 500 mm, 520 mm, 530 mm, 600 mm	Grey	5	1.67	± 0.1	380 - 600	With or without X- ray contrast strip	Vacon
Suction tube 6F length 380 mm, 400 mm, 430 mm, 450 mm, 500 mm, 520 mm, 530 mm, 600 mm	Light- green	6	2.0	± 0.1	380 - 600	With or without X- ray contrast strip	Vacon
Suction tube 8F length 380 mm, 400 mm, 430 mm, 450 mm, 500 mm, 520 mm, 530 mm, 600 mm	Dark-blue	8	2.67	± 0.1	380 - 600	With or without X- ray contrast strip	Vacon
Suction tube 10F length 380 mm, 400 mm, 430 mm, 450 mm, 500 mm, 520 mm, 530 mm, 600 mm	Black	10	3.33	± 0.15	380 - 600	With or without X- ray contrast strip	Vacon
Suction tube 12F length 380 mm, 400 mm, 430 mm, 450 mm, 500 mm, 520 mm, 530 mm, 600 mm	White	12	4.0	± 0.15	380 - 600	With or without X- ray contrast strip	Vacon
Suction tube 14F length 380 mm, 400 mm, 430 mm, 450 mm, 500 mm, 520 mm, 530 mm, 600 mm	Dark	14	4.67	± 0.2	380 - 600	With or without X- ray contrast strip	Vacon
Suction tube 16F length 380 mm, 400 mm, 430 mm, 450 mm, 500 mm, 520 mm, 530 mm, 600 mm	Orange	16	5.33	± 0.2	380 - 600	With or without X- ray contrast strip	Vacon
Suction tube 18F length 380 mm, 400 mm, 430 mm, 450 mm, 500 mm, 520 mm, 530 mm, 600 mm		18	6.0	± 0.2	380 - 600	With or without X- ray contrast strip	Vacon
Suction tube 20F length 380 mm, 400 mm, 430 mm, 450 mm, 500 mm, 520 mm, 530 mm, 600 mm	Yellow	20	6.67	± 0.2	380 - 600	With or without X- ray contrast strip	Vacon

suction tube with Y-connector, Vacon type, with/without X-ray contrast strip

Catalog number breakdown

SE 12 1 Y XXX X

SE – suction tube – name

12 – 12 F – size according to Charriere scale F

1 – PVC – material

Y – adjustable connector (type: Vacon)

XXX – length, mm

X – with X-ray contrast strip (if there is no "X" in catalog number, then the tube doesn't have X-ray contrast strip)

TRANSPORT, STORAGE AND APPLICATION CONDITIONS. SHELF-LIFE

Store in a place protected from direct sunlight.

Storage

Temperature	-10°C ~ +40°C.
Atmospheric pressure	700 - 1060 hPa
Relative humidity	No more than 75%

Use

Temperature	15-30° C
Atmospheric pressure	700 - 1060 hPa
Relative humidity	35- 75%

Medical device can be transported by all means of transport in covered vehicles according to requirements applicable to each means of transport.

It is recommended to transport products in original factory packages made of thick cardboard, in a vehicle protected from direct sunlight and high humidity.

Transport conditions:

Temperature	-25 до +40 °C,
Atmospheric pressure	700 - 1060 hPa
Relative humidity	No more than 80%

Shelf life of the product is 3 years from the date of sterilization. It is prohibited to use expired products.

COMBINATION WITH OTHER DEVICES:

Combination with other devices¹:

Name	Vacuum extractor "Vakus" according to TU 9444-005-74487176-2010	Suction medical vacuum ATMOS C261 Aspirator DDS with accessories	Bronchial suction with accessories
Model	Vakus 7209	C261	-
Manufacturer	Dixion LLC (RF)	ATMOS MedizinTechnik GmbH & Co (Germany)	Dragerwerk AG & Co.(Germany)

¹ The table gives several examples of medical devices used in conjunction with a registered device.

EN 1618-1997	Catheters other than intravascular catheters - Test methods for common properties
--------------	--

WARRANTY

Tube is a sterile medical device for individual use. The manufacturer is not responsible for complications that may arise as a result of improper use of a medical device.

Do not use the device if the sterile barrier of the package is compromised, or the package is damaged in any way.

The device must be used according to indications and instructions for use. All questions related to the operation of the device can be addressed to the Authorized Representative of the manufacturer in the Russian Federation.

The manufacturer guarantees sterility of the products during its entire shelf life, when stored in prescribed storage conditions.

In case of product quality issues, please contact the Authorized Representative of the manufacturer in the Russian Federation.

INTMED LLC

Address: 111524, Moscow, Perovskaya St., 1 bldg. 12, room I, office 27

Tel.: +7 (495) 234-57-31

E-mail: andrey@intmed.ru

PROSPECTIVE CONSUMERS

Catheters are intended for use by a doctor or nurse who has undergone special training or has special skills for performing medical procedures.

Suction tube is intended for adults, children and babies.

STERILITY

Products are sterile. Sterilization method – EO sterilization.

PACKAGE. CONTENTS OF DELIVERY.

Catheters are packed in an individual paper-plastic blister package.

Material and structure of the primary package make it possible to visually identify the color of the connector.

Packaging materials do not have any harmful effect on the contents. Packaging material and design guarantee:

- a) maintaining sterility when stored in dry, clean and well-ventilated rooms;
- b) minimal risk of contamination of the contents when opening the package;
- c) protection of contents under normal conditions of shipping and storage;
- d) impossibility of re-sealing the package, if its integrity has been violated, the fact of opening the package is obvious.

Individual packages with the product are placed in the amount of 50 pcs. in a group package of corrugated cardboard along with instructions for use - 1 pc.

Transport carton contains 10 cardboard boxes. Transport packaging is corrugated cardboard boxes.

REPAIR AND MAINTENANCE

not applicable.

INFORMATION ON DISPOSAL OF MEDICAL DEVICE

Medical device is for single use only.

Follow national regulations when handling and disposing of the product, its components, and package.

Medical devices in the Russian Federation must be disposed of in local public / private organizations in accordance with SanPIN 2.1.3684-21 "Sanitary and epidemiological requirements for the maintenance of the territories of urban and rural settlements, for water bodies, drinking water and drinking water supply, atmospheric air, soils, residential premises, operation of industrial, public premises, organization and implementation of sanitary and anti-epidemic (preventive) measures.

Used catheters that have been in contact with the patient's secretions are class B waste.

Properly used catheters that have not been in contact with the patient's secretions are class A waste.

Separation, recycling or disposal of packaging material must be carried out in accordance with the requirements of current legislation regarding packaging and packaging waste.

Manufacturer

Name INTEGRAL MEDICAL PRODUCTS CO.,LTD

Address No.2 Dongze Road, High-Tech Industrial Development Zone, 312000 Shaoxing, Zhejiang, PEOPLE'S REPUBLIC OF CHINA

Tel.: 0086-575-88677000

Fax: 0086-575-88678000

E-mail: guests@integral-medical.com.cn

AUTHORIZED REPRESENTATIVE OF THE MANUFACTURER IN RUSSIAN FEDERATION INTMED LLC

Address: 111524, Moscow, Perovskaya St., 1 bldg. 12, room 1, office 27

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