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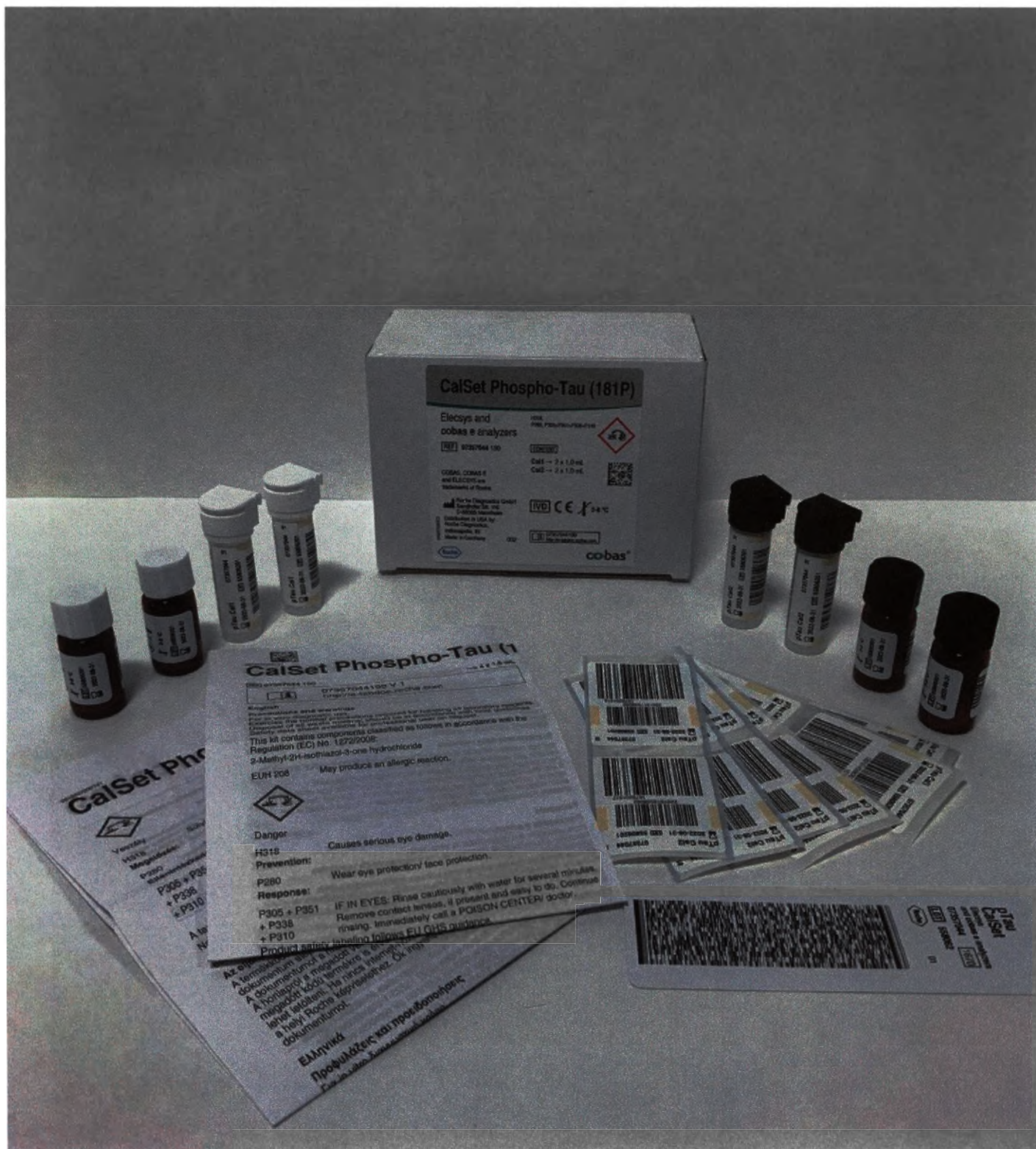
English

Intended use

CalSet Phospho-Tau (181P) is used for calibrating the quantitative Elecsys Phospho-Tau (181P) CSF assay on the Elecsys and cobas e immunoassay analyzers.

Summary

CalSet Phospho-Tau (181P) is a lyophilized buffered matrix with added synthetic peptide in two concentration ranges.

The CalSet can be used with all reagent lots.

Reagents - working solutions

- pTau Cal1: 2 bottles, each for 1.0 mL of calibrator 1
- pTau Cal2: 2 bottles, each for 1.0 mL of calibrator 2

pTau in two concentration ranges (approximately 10 pg/mL and approximately 70 pg/mL) in a buffered matrix, preservative.

The exact lot-specific calibrator values are encoded in the barcode as well as printed on the enclosed (or electronically available) calibrator barcode sheet.

Calibrator values

Traceability: The Elecsys Phospho-Tau (181P) CSF assay has been standardized against internal pTau material which is traceable to NIST amino acid reference calibrators.

Precautions and warnings

For in vitro diagnostic use for health care professionals. Exercise the normal precautions required for handling all laboratory reagents.

Infectious or microbial waste:

Warning: handle waste as potentially biohazardous material. Dispose of waste according to accepted laboratory instructions and procedures.

Environmental hazards:

Apply all relevant local disposal regulations to determine the safe disposal.

Safety data sheet available for professional user on request.

This kit contains components classified as follows in accordance with the Regulation (EC) No. 1272/2008:

Danger

H317 May cause an allergic skin reaction.

H318 Causes serious eye damage.

H411 Toxic to aquatic life with long lasting effects.

Prevention:

P261 Avoid breathing dust/fume/gas/mist/vapours/spray.

P273 Avoid release to the environment.

P280 Wear protective gloves/ eye protection/ face protection.

Response:

P305 + P351 + P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do.
+ P310 Continue rinsing. Immediately call a POISON CENTER/ doctor.

P333 + P313 If skin irritation or rash occurs: Get medical advice/attention.

P391 Collect spillage.

Product safety labeling follows EU GHS guidance.

Contact phone: all countries: +49-621-7590

Avoid foam formation in all reagents and sample types (specimens, calibrators and controls).

Handling

Carefully dissolve the contents of one bottle by adding exactly 1.0 mL of distilled or deionized water and allow to stand closed for 15 minutes to reconstitute. Mix carefully, avoiding foam formation.

Transfer the reconstituted calibrators into the supplied empty labeled snap-cap bottles.

Due to possible evaporation effects, not more than 5 calibration procedures per bottle set should be performed.

If necessary, freeze in aliquots; see section on MODULAR ANALYTICS E170, cobas e 601, cobas e 602 and cobas e 411 analyzer.

MODULAR ANALYTICS E170, cobas e 601, cobas e 602 and cobas e 411 analyzers: Unless the entire volume is necessary for calibration on the analyzers, transfer aliquots of the reconstituted calibrators into empty snap-cap bottles (CalSet Vials). Attach the supplied labels to these additional bottles. Store the aliquots at -20 °C for later use. Perform only one calibration procedure per aliquot.

Please note: Both the vial labels, and the additional labels (if available) contain 2 different barcodes. The barcode between the yellow markers is for cobas 8000 systems only. If using a cobas 8000 system, please turn the vial cap 180° into the correct position so the barcode can be read by the system. Place the vial on the instrument as usual.

Storage and stability

Store at 2-8 °C.

The lyophilized calibrators are stable up to the stated expiration date.

Stability of the reconstituted calibrators:

either at -20 °C	14 days
or at 2-8 °C	24 hours
on MODULAR ANALYTICS E170, cobas e 601, cobas e 602 and cobas e 411 analyzers at 20-25 °C	up to 2 hours

Store calibrators upright in order to prevent the calibrator solution from adhering to the snap-cap.

Materials provided

- CalSet Phospho-Tau (181P), barcode card, calibrator barcode sheet, 4 empty labeled snap-cap bottles, 2 x 6 bottle labels

Materials required (but not provided)

- [REF] 11776576322, CalSet Vials, 2 x 56 empty snap-cap bottles
 - MODULAR ANALYTICS E170 or cobas e immunoassay analyzers and Elecsys Phospho-Tau (181P) CSF assay reagents
 - Distilled or deionized water
- See the assay Method Sheet and the operator's manual for additionally required materials.

Assay

Place the reconstituted calibrators (in the system-compatible bottles with barcoded labels) in the sample zone.

Read in all the information necessary for calibrating the assay.

Ensure the calibrators are at 20-25 °C prior to measurement.

For further information, please refer to the appropriate operator's manual for the analyzer concerned, the respective application sheets, the product information and the Method Sheets of all necessary components (if available in your country).

A point (period/stop) is always used in this Method Sheet as the decimal separator to mark the border between the integral and the fractional parts of a decimal numeral. Separators for thousands are not used.

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

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