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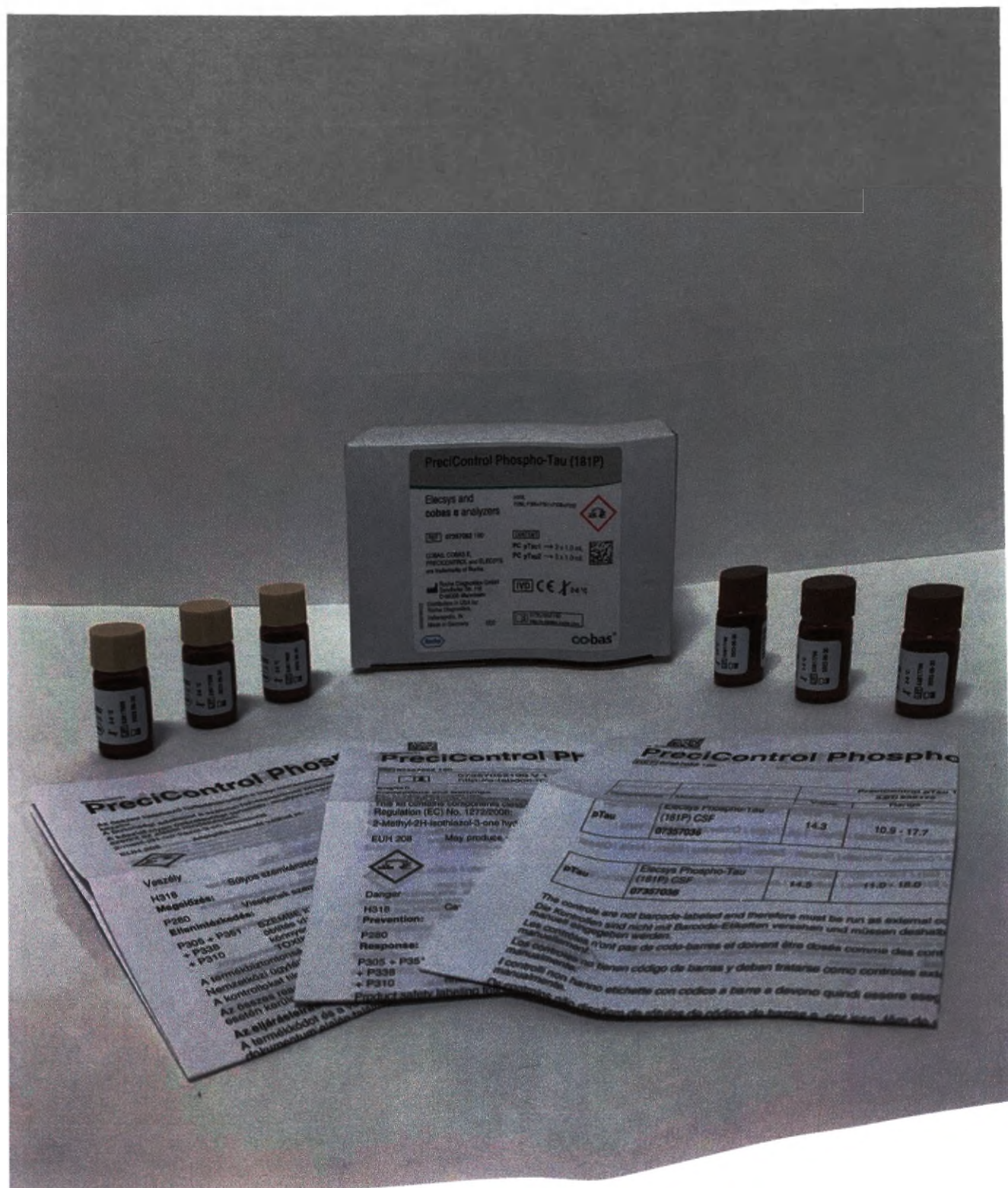
English

Intended use

PreciControl Phospho-Tau (181P) is used for quality control of the Elecsys Phospho-Tau (181P) CSF immunoassay on the Elecsys and cobas e immunoassay analyzers.

Summary

PreciControl Phospho-Tau (181P) is a lyophilized control based on buffered matrix. The controls are used for monitoring the accuracy of the Elecsys Phospho-Tau (181P) CSF immunoassay.

Reagents - working solutions

- PC pTau 1: 3 bottles, each for 1.0 mL of control based on buffered matrix with added synthetic peptide; preservative
Target value approximately 15 pg/mL
- PC pTau 2: 3 bottles, each for 1.0 mL of control CSF based on artificial CSF; preservative
Target value approximately 50 pg/mL

Note: The controls are not barcode-labeled and therefore have to be run like external controls. All values and ranges have to be entered manually. Please refer to the section "QC" in the operator's manual or to the online help of the instrument software.

Non-barcode labeled controls: Only one target value and range for each control level can be entered in the analyzer. The reagent lot-specific target values have to be re-entered each time a specific reagent lot with different control target values and ranges is used. Two reagent lots with different control target values and ranges cannot be used in parallel in the same run. The exact lot-specific target values and ranges are printed on the enclosed (or electronically available) value sheet in the reagent kit or PreciControl kit. Please make sure that the correct values are used.

Target values and ranges

The target values and ranges were determined and evaluated by Roche. They were obtained using the Elecsys Phospho-Tau (181P) CSF assay reagents and analyzers available at the time of testing.

If the target values and control ranges are updated, this information is conveyed in an additional value sheet included in the reagent kit. This value sheet lists all control lots to which the new values apply. If some of the values remain unchanged, the original values conveyed in the value sheet included in the control kit (or provided electronically), remain valid.

Results must be within the specified ranges. In the event that increasing or decreasing trends, or any other suddenly occurring deviations beyond the range limits are observed, all test steps must be checked.

When necessary, measurement of the patient sample tested should be repeated.

Traceability information is given in the Method Sheet of the relevant Elecsys assay.

Each laboratory should establish corrective measures to be taken if values fall outside the defined limits.

Precautions and warnings

For in vitro diagnostic use for health care professionals. Exercise the normal precautions required for handling all laboratory reagents.

Infectious or microbial waste:

Warning: handle waste as potentially biohazardous material. Dispose of waste according to accepted laboratory instructions and procedures.

Environmental hazards:

Apply all relevant local disposal regulations to determine the safe disposal.

Safety data sheet available for professional user on request.

This kit contains components classified as follows in accordance with the Regulation (EC) No. 1272/2008:

Danger

H217 May cause an allergic skin reaction.

H318 Causes serious eye damage.

H411 Toxic to aquatic life with long lasting effects.

Prevention:

P251 Avoid breathing dust/fume/gas/mist/vapours/spray.

P273 Avoid release to the environment.

P280 Wear protective gloves/ eye protection/ face protection.

Response:

P305 + P351 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER/ doctor.

P333 + P313 If skin irritation or rash occurs: Get medical advice/attention.

P501 Collect spillage.

Product safety labeling follows EU GHS guidance.

Contact phone: all countries: +49-621-7590

The controls may not be used after the expiration date.

Avoid foam formation in all reagents and sample types (specimens, calibrators and controls).

Handling

Carefully dissolve the contents of one bottle by adding exactly 1.0 mL of distilled or deionized water and allow to stand closed for 15 minutes to reconstitute. Mix carefully, avoiding foam formation.

Transfer the reconstituted controls into additional snap-cap bottles (ControlSet Vials). Aliquots intended for storage at -20 °C should be frozen immediately.

Perform only one control procedure per aliquot.

When measuring non-barcode controls, use only recommended sample tubes, "cup on tube" or "cup on rack".

Storage and stability

Store at 2-8 °C.

The lyophilized control is stable up to the stated expiration date.

Stability of the components in reconstituted control:	
either at -20 °C	31 days (freeze only once)
or at 2-8 °C	72 hours
on the analyzers at 20-25 °C	up to 5 hours

Store controls upright in order to prevent the control solution from adhering to the snap-cap.

Materials provided

- PreciControl Phospho-Tau (181P)

Materials required (but not provided)

- REF 06142949122, ControlSet Vials, 2 x 56 empty snap-cap bottles
- MODULAR ANALYTICS E170 or cobas e immunoassay analyzers and assay reagents

- Distilled or deionized water

See the assay Method Sheet and the operator's manual for additionally required materials.

Assay

Treat the reconstituted control in the system-compatible labeled bottles for analysis in the same way as patient samples.

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The controls are not barcode-labeled and therefore must be run as external controls. All values and ranges must be entered manually.

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