UrkR.Nr. 904 / 2023

«I CERTIFY»

GENERAL MANAGER

PRESIDENT DENTAL GMBH, GERMANY

VEDAT GÖKTAS

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OPERATION MANUAL OF MEDICAL DEVICE
« CEMENT LUTING GLASS IONOMIC «EXTRACEM» »

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UrkR Nr. 904 vom 21. 08. 2023
File No [see above] dating from 21st of August 2023

Country of Bavaria (Germany)

City of München [Munich]

I herewith certify, that

Given Names:

Mr

Family Name:

Vedat

Date of Birth [Day, Month, Year]:

Göktas

01.11.1970

[firm address]:

Kesselbodenstr. 5, 85391 Allershausen

identified by his German residence permit, has in front of me subscribed and sworn (affirmed) today the foregoing document

After having checked the commercial register at the local court "Amtsgericht München" on 14th of August, 2023 I confirm

- that the firm President Dental GmbH, with its registered rooms in Allershausen, Germany is entered in the commercial register under number HRB 209018 and
- that the said Mr. Göktas is authorized to represent the said company solely in his capacity as Managing Director.

I explanatorily annotate that the notarial certification is limited to the authenticity of the signature of the signing person and not to the content of the text which I often cannot understand and need not understand. For purposes of professional supervision, I contest that I cannot see any aspects which suggest that the before standing declaration concerns any illegal or legally problematic necessities which would prevent me from my notarization.

Munich, this 21st day of August, 2023

Dr. Heinrich Kreuzer, Notary

Perusastr. 7 [street and house number],

D-80333 München (Munich) [postal code and city]

☎ +49 (89) 290 916-0, Fax -66; notar@kreuzer-muenchen.de

of the state in which this document shall be used it may be necessary that my - gets is noted. Competent therefore is usually the President of the court „Landgericht in Munich, Palast der Justiz, Prielmayerstr. 7, D-80335 München.

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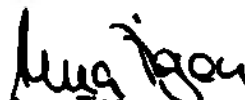
GB

EXTRACEM®

Complies with ISO 9917-1:2007 Glass Polyalkenoate Class 4.2a

**Glass Ionomer Luting Cement
Powder / Liquid
Directions For Use**

FEATURES: A radiopaque glass ionomer luting cement with prolonged working time followed by snap-set reaction. Combines excellent handling, low film thickness, high strength and contains fluoride. Excellent adhesion of porcelain, ceramic or metal to tooth structure.

RECOMMENDED INDICATIONS:

- Cementing of crowns, bridges, inlays or orthodontic brackets.

CONTRA-INDICATIONS:

- Pulp capping.

APPLICATION:

Powder: Liquid Ratio	1 Scoop : 2 Drops 2.00g : 1.00g
Mixing Time	30 seconds
Working Time (at 23 ± 1°C)	3 minutes 30 seconds Minimum
Setting Time (at 37 ± 1°C)	4 minutes 0 seconds Maximum

- 1- Clean the preparation and restoration with oil-free prophypaste, water and air dry. Use calcium hydroxide if remaining dentin is thin and pulp protection is required.
- 2- Place 1 level scoop of powder and 2 bubble-free drops of liquid on glass block or mixing pad and mix thoroughly with flat bladed spatula to creamy consistency.
- 3- Spread a thin layer of cement over all inside surfaces of restoration and seat on preparation. Have patient bite down sufficiently hard on a cotton ball to remove air and trim excess cement squeezed out. The set cement is tooth coloured (U).

STORAGE:

Store in a cool dry place (5-25°C). Close caps immediately after use.

WARNING:

Avoid contact of liquid and powder with eyes. In the event of contact, rinse with copious quantities of water and seek medical assistance.

BATCH CODE:

The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.

The expiry date is shown in year, month format. Do not use the product after this date.

SPECIALLY FORMULATED FOR USE IN DENTISTRY

PRD.100.EXC

35g

17ml

President Dental GmbH operate a policy of continuous monitoring and improvement of our products. If you have any comments about this product, please contact us at the below address stating the batch number shown on the packaging.

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