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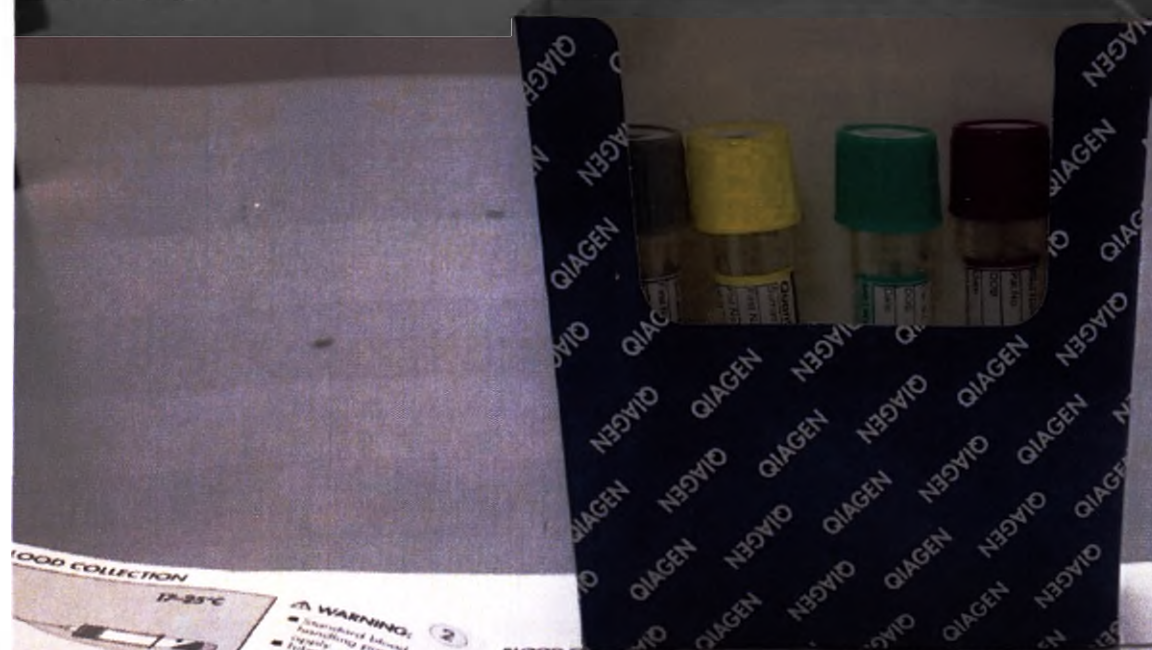
BLOOD COLLECTION

1. Collect blood by venipuncture. A black mark is acceptable. Do not allow the needle to be slow. Do not close to mark.

WARNING:
Standard blood handling procedures. Tubes must be stored at room temperature (17-25°C).

BLOOD INCUBATION

Immediately after filling tubes, shake them ten (10) times, just firmly enough to ensure the entire inner surface of the tube is coated with blood, to dissolve antigens on tube walls. Tubes should be between 17-25°C at the time of blood filling. Over-energetic shaking may cause gel disruption and could lead to aberrant results.

Label tubes appropriately.

OPTION 2 (continued)

6

CENTRIFUGATION

Centrifuge tubes prior to sampling. Spin at 2000-3000 g for 15 minutes.

Test Plasma with either QFT-Plus ELISA or LIAISON® QuantiFERON-TB Gold Plus System (REF: 311010)

Plasma in centrifuged tubes is stable for 28 days when stored at 2-8°C.

For up-to-date licensing information and product-specific disclaimers, see the QFT-Plus ELISA or the LIAISON QuantiFERON-TB Gold Plus System package inserts.

OPTION 1 (continued)

Ship tubes to laboratory at 17-27°C. (Shipped must be incubated at 37°C as soon as possible, and within 16 hours of collection). Laboratory staff must remix the tubes by inverting 10 times immediately prior to 37°C incubation.

Technical Tip

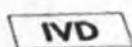
Label tubes as "Not Incubated".

CONTACT DETAILS

QIAGEN
Asia-Pacific: techservice-ap@qiagen.com
Europe: techserviceQFT-eu@qiagen.com
Middle East/Africa: techserviceQFT-ma@qiagen.com
Latin America (not including Brazil or Mexico): techservice-latam@qiagen.com
Mexico: techservice-MX@qiagen.com
Brazil: techsebr@qiagen.com

QIAGEN GmbH
QIAGEN Straße 1
40724 Hilden
Germany
+49-2103-29-0

4°C 25°C

Trademarks: QIAGEN®, Sample to Insight® QFT®, QuantiFERON® (QIAGEN Group), LIAISON® (DiaSorin).

© 2016-2018 QIAGEN, all rights reserved.

— Sample to Insight —

April 2013

8

QIAGEN is a leader in the development of innovative solutions for the life sciences industry. Our products and services are used by researchers, clinicians, and regulatory agencies worldwide. We are committed to providing high-quality products and services that meet the needs of our customers. For more information, please visit our website at www.qiagen.com.

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