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002202108280082

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XX E

WYF2108687

210808 001 00024 1/4

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CERTIFICATE

中国国际贸易促进委员会暨中国国际商会
China Council for the Promotion of International Trade & China Chamber of International Commerce

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China Council for the Promotion of International Trade
China Chamber of International Commerce

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THIS IS TO CERTIFY THAT: the seal of Jiangsu Nanfang Medical Co., Ltd. on the annexed DOCUMENT is genuine. The attached Russian text of the said DOCUMENT is true to the English text.

China Council for the Promotion
of International Trade

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Authorized
Signature:

Lyu Cuilian

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(Date: Jul. 02, 2021)

证明 网址: Website for verifying the certificate: <http://www.rzccpit.com/validate.html>

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Name and description of the medical product

Reneplast® adhesive plaster fixing (hereinafter - adhesive plaster) is a roll of viscose fabric tape wound on a plastic spool. The edges of the adhesive plaster are shaped.

Designs

Reneplast® Pepper plaster in various designs:

- Reneplast® fixing adhesive plaster, 1.25x500 cm;
- Reneplast® fixing adhesive plaster, 2.5x500 cm;
- Reneplast® fixing adhesive plaster, 5x500 cm;

Formulation

Component	Formulation
Fabric base	Viscose fabric - 100%
Adhesive layer	Adhesive mass, which includes: zinc oxide - 33.1% rubber - 26.5% rosin - 23.1% lanolin - 9.3% vaseline - 5% antioxidants - 3%

Purpose and mode of action of medical product

Specially treated fabric base of the adhesive plaster allows the skin to breathe, in creating favorable conditions for the natural healing of the damaged skin area. Using an adhesive layer, the adhesive plaster adheres tightly to skin and is securely fixed on the body surface.

Indications

Used for fixing a bandaging material, as well as tubes, probes, catheters and other medical products.

Contradictions

Hypersensitivity to the components of the product (predisposition to allergy).

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Method of use

Rinse and dry the skin in the damage area, having previously treated it with an antiseptic solution.

Before using the adhesive plaster, cut off a strip of the required size. Fix a bandage or a medical product, with tightly pressing and smoothing the edges of the adhesive plaster. The adhesive plaster can be applied in multiple layers.

If necessary, remove the adhesive plaster by gently moving in the direction of hair growth, pulling at an edge.

Conditions of use and potential consumer

Adhesive plaster can be used at home, as well as in inpatient and outpatient medical institutions.

No special knowledge is required to use Reneplast® adhesive plaster.

Potential side effects

- Skin redness in the application area;
- skin itching.

In case of side effects, immediately stop using the adhesive plaster and consult your doctor.

Precautions

- Adhesive plaster is for external use only;
- do not reuse;
- avoid contact of the adhesive base with open wounds, burns of any severity;
- do not use adhesive plaster with a damaged protective coating;
- when performing hygienic procedures, additionally protect the bandages from moisture;
- If the use of adhesive plaster brings discomfort, stop using the product and consult your doctor.

Pack size

Adhesive plaster, wound on a plastic spool, 1 pc

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each, 1.25x500 cm or 2.5x500 cm in size, is packed in a cylindrical transparent plastic package fixed on a cardboard base (consumer packaging).

Adhesive plaster, wound on a plastic spool, 1 pc each, 1.25x500 cm or 2.5x500 cm or 5x500 cm in size, is packed in a cylindrical transparent plastic package fixed on a cardboard base (consumer packaging).

Adhesive plaster, wound on a plastic spool, 1 pc each, 1.25x500 cm or 2.5x500 cm or 5x500 cm in size, is packed in a cylindrical transparent plastic package fixed on a cardboard base (consumer packaging).

Storage conditions

Store in the original packaging at a temperature of + 5 °C to + 25 °C and a relative humidity of not more than 80 %.

Keep out of the reach of children.

Shelf life

5 years.

Do not use after expiration date.

Environmental requirements

When using, transporting, storing and disposing of the plaster according to the terms of the operational documentation, negative effects on humans and the environment are excluded.

Disposal and destruction procedure

Dispose of used adhesive plaster in accordance with SanPiN 2.1.3684-21 as class A waste (epidemiologically safe waste, similar in composition to solid municipal waste).

Information about disinfection and cleaning methods

The plaster is individual and single-use.

Re-use is not provided, so no cleaning or disinfection is required.

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Sterilization

The plasters are supplied non-sterile and are not to be sterilized.

Maintenance and repair requirements

Since the patch is individual and single-use product, re-use is not provided, and it cannot be maintained or repaired.

Warranty

The manufacturing company Jiangsu Nanfang Medical Co., Ltd., guarantees the Reneplast® Pepper Adhesive compliance with the requirements of this document, subject to the conditions of use, transportation and storage.

The manufacturing company Jiangsu Nanfang Medical Co., Ltd. is not responsible for damage or accidents caused by improper use of the product, due to a fault of customers or any external unforeseen situations.

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