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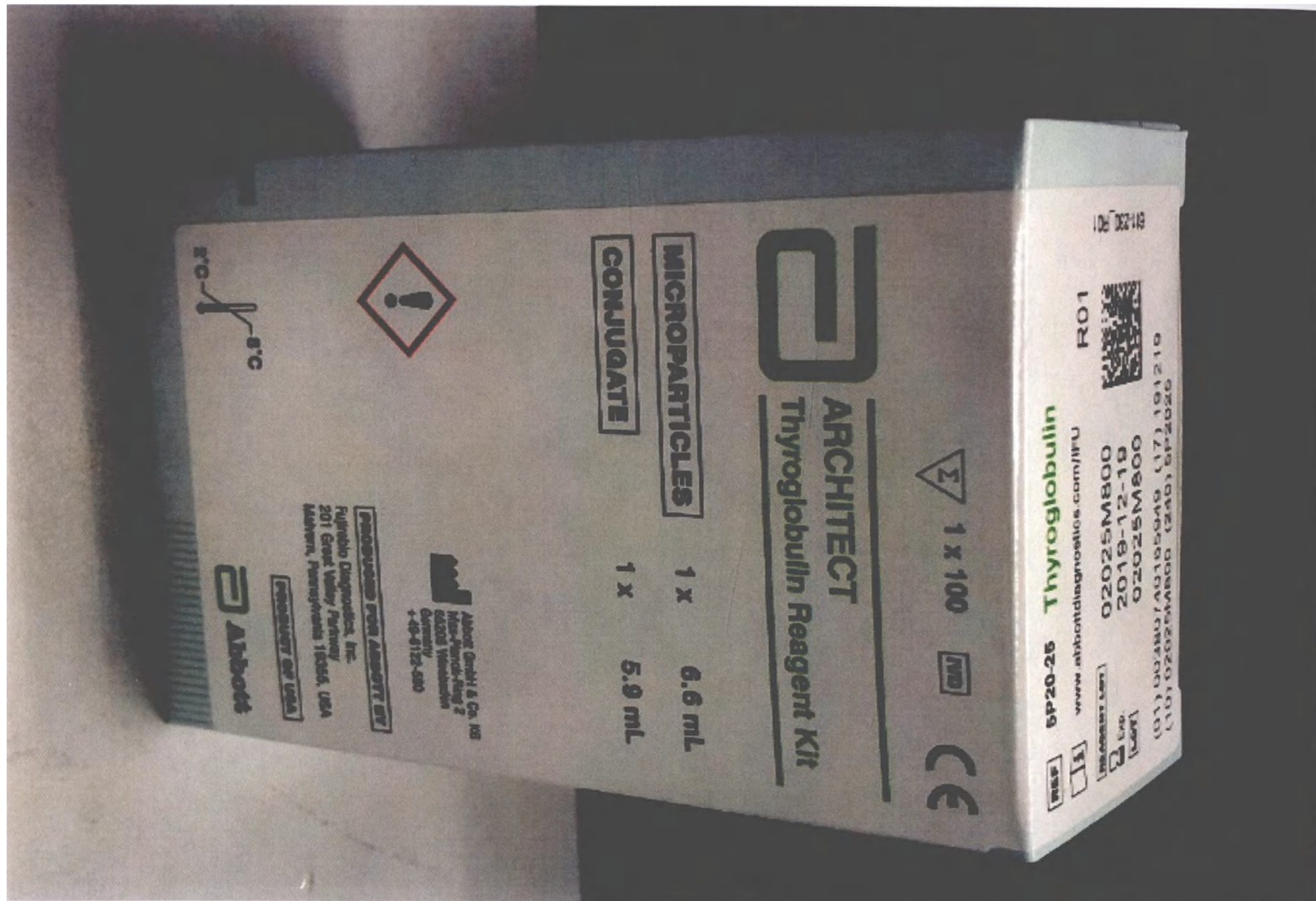
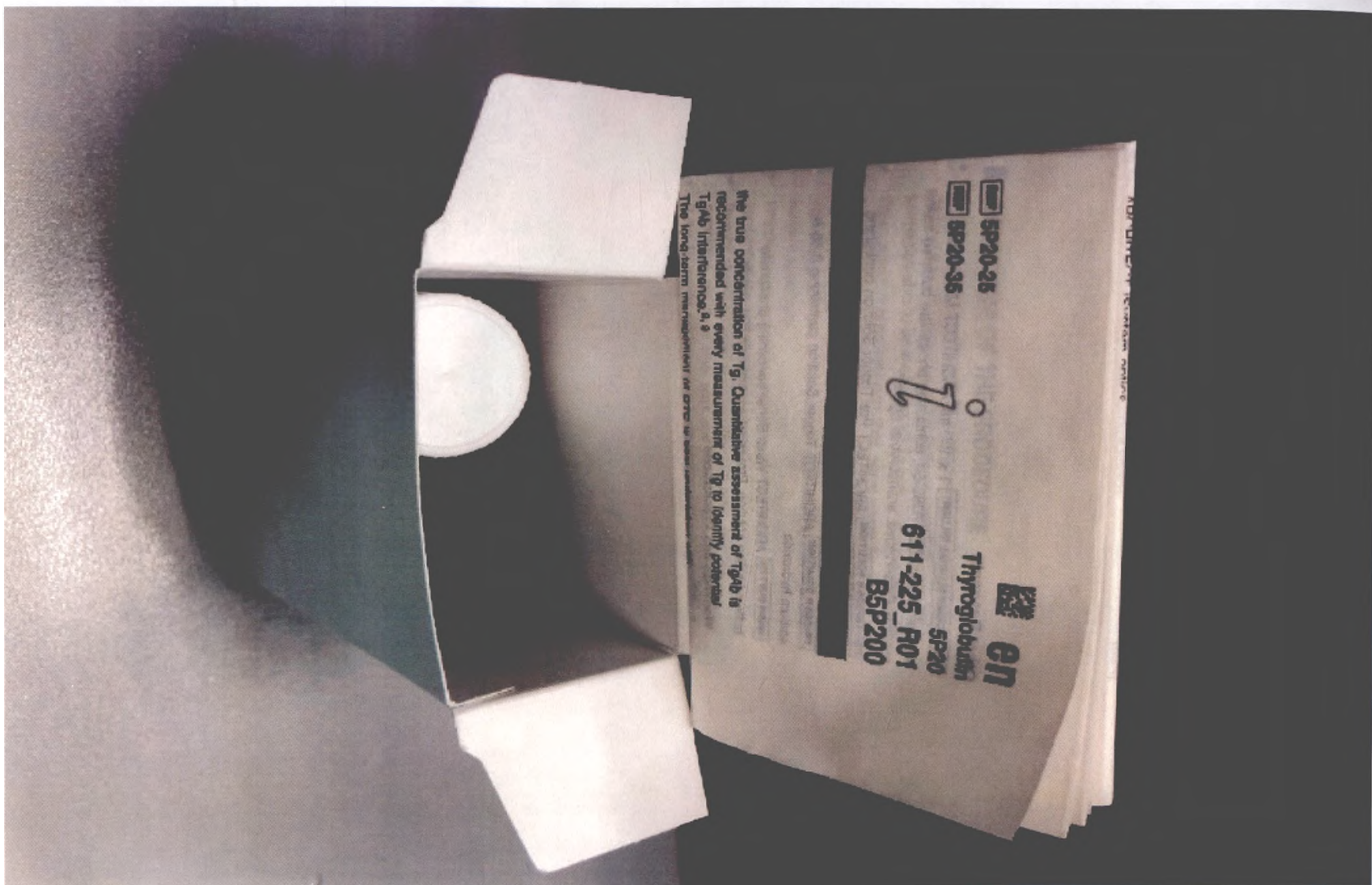
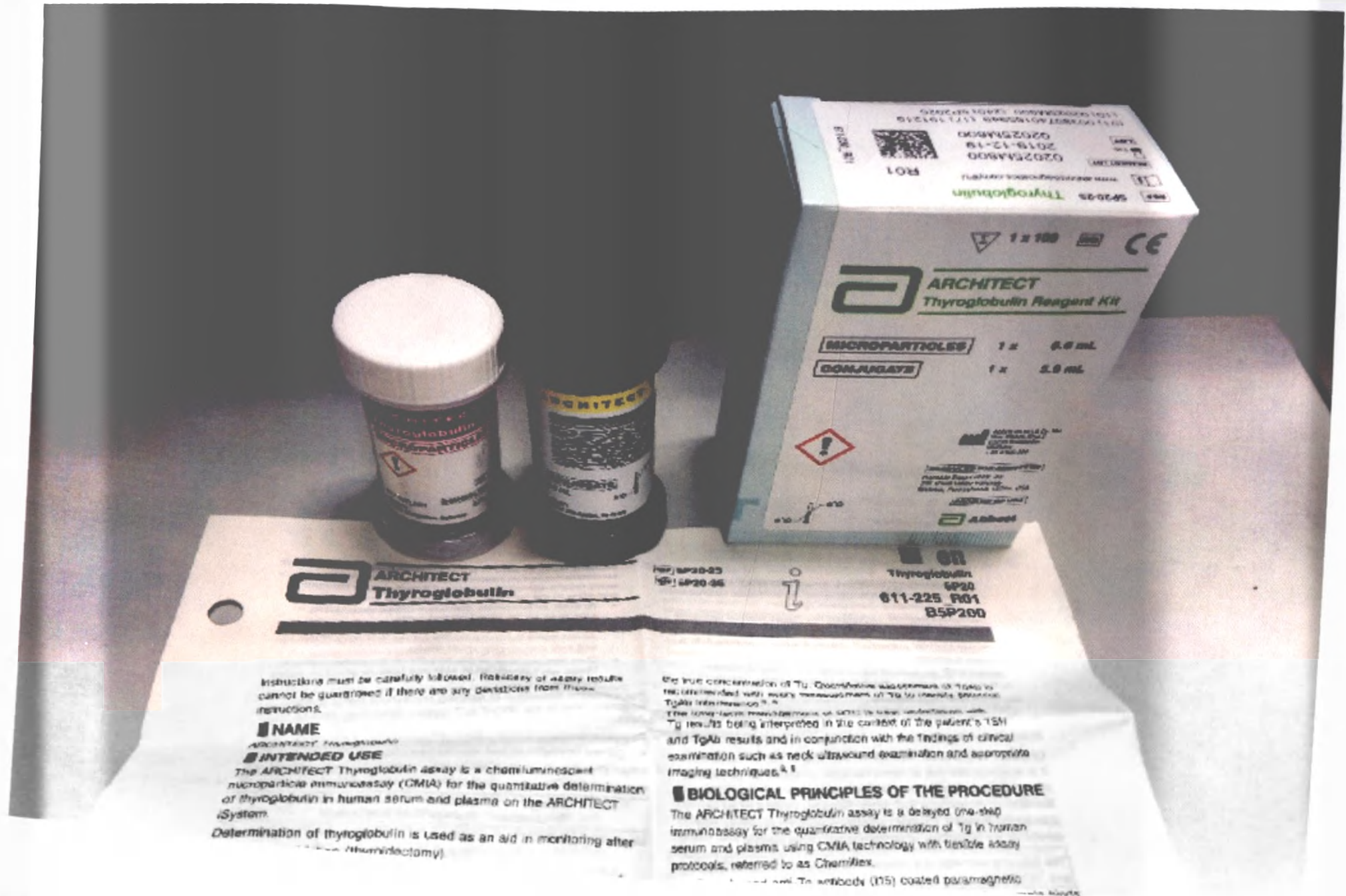
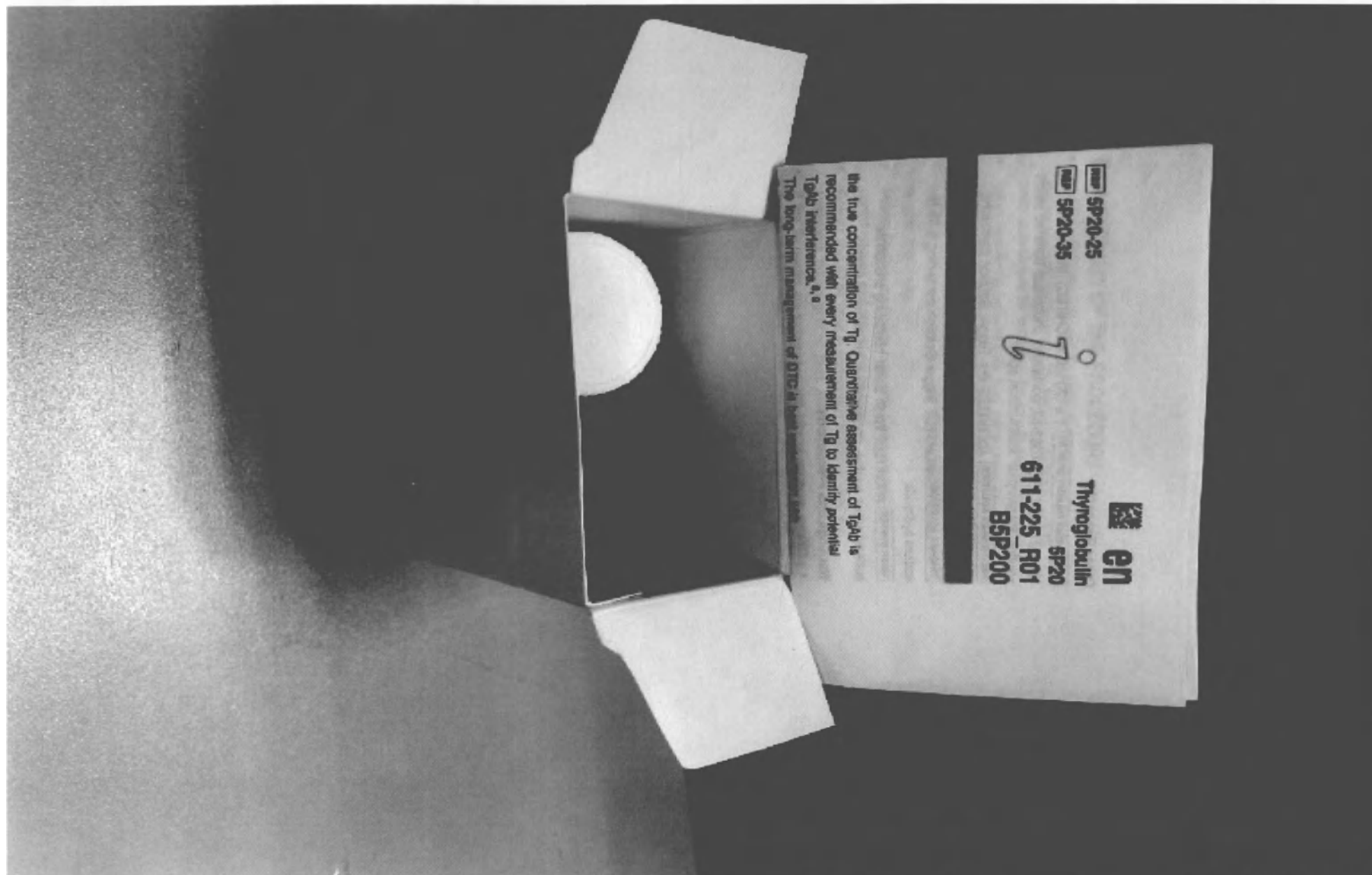
SUMMARY AND EXPLANATION OF THE TEST

Determination of thyroglobulin is used as an aid in monitoring after total thyroid ablation (thyroidectomy).

WARNING: The ARCHTECT Thyroglobulin assay should be used in the absence of anti-thyroglobulin antibodies. Specimens positive for anti-thyroglobulin antibodies can cause falsely decreased concentrations of thyroglobulin. Patients positive for anti-thyroglobulin antibodies should be evaluated for thyroglobulin with alternative methods.

INTENDED USE

The ARCHITECT Thyroglobulin assay is a chemiluminescent microparticle immunoassay (CMIA) for the quantitative determination of thyroglobulin in human serum and plasma on the ARCHITECT System.

NAME _____

Instructions must be carefully followed. Reliability of assay results cannot be guaranteed if there are any deviations from these instructions.

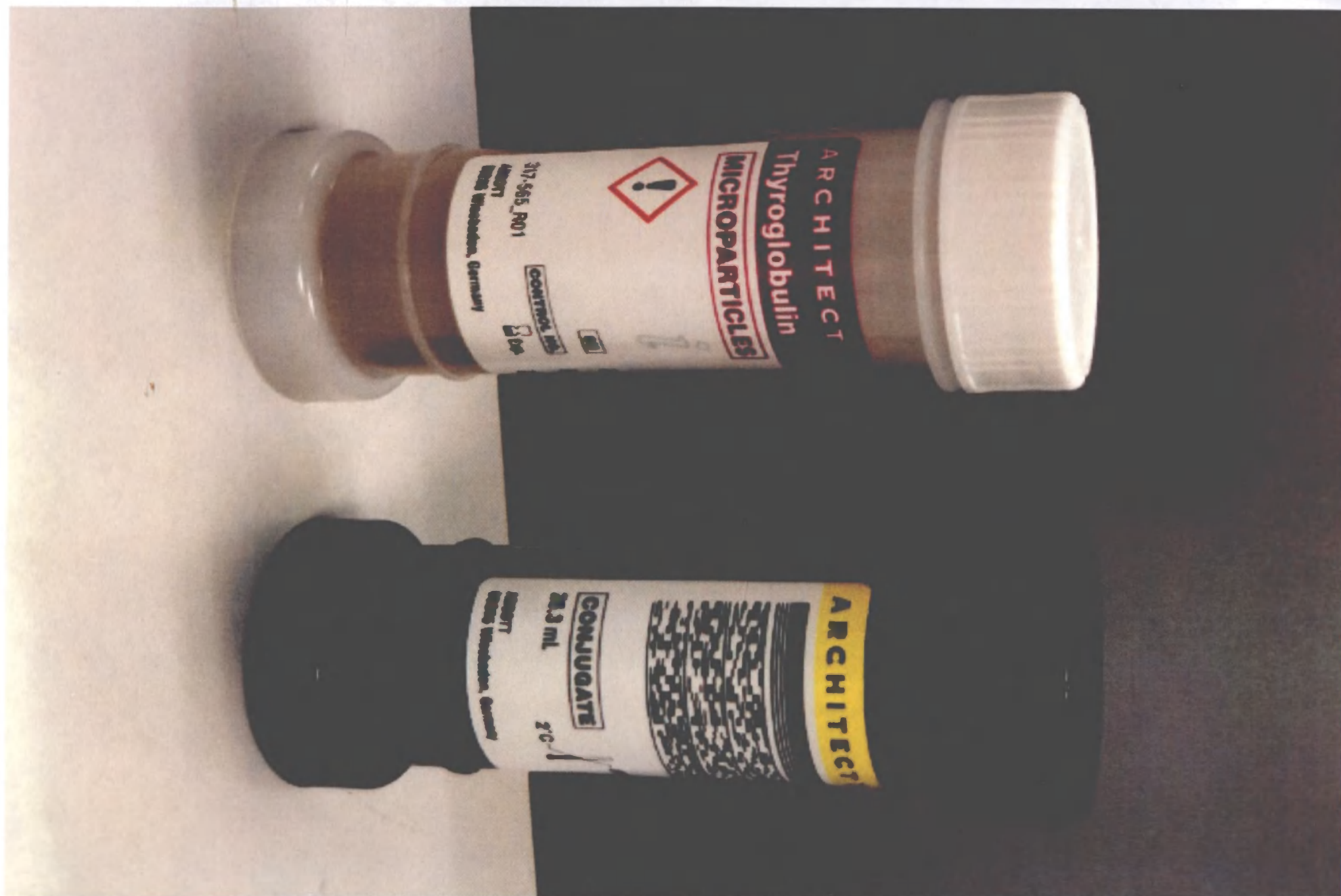
■ BIOLOGICAL PRINCIPLES OF THE PROCEDURE

The time concentration of 19. Quantitative assessments of 19a,b were associated with a significant concentration of 1a, 1b globally (p=0.0004). There was a strong concentration of 19a,b in some subgroups with 19 results being interpreted in the context of the patient's 19a and 19b results and in conjunction with the findings of clinical examination such as neck ultrasound examination and appropriate imaging techniques.^{5,6}

Solutions are added to the reaction mixture.

1. Sample and anti-Tg antibody (115) coated paramagnetic microparticles are combined. The Tg present in the sample binds to the anti-Tg antibody (115) coated microparticles.
2. After incubation, anti-Tg antibody (124) acridinium-labeled conjugate is added to the reaction mixture.
3. After further incubation and washing, Pre-Tigger and Tigger Solutions are added to the reaction mixture.
4. The resulting chemiluminescent reaction is measured as 1 unit (PULS). There is a direct relationship between the amount of Tg and the PULS detected by the reader.

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Fujirebio Diagnostics, Inc.