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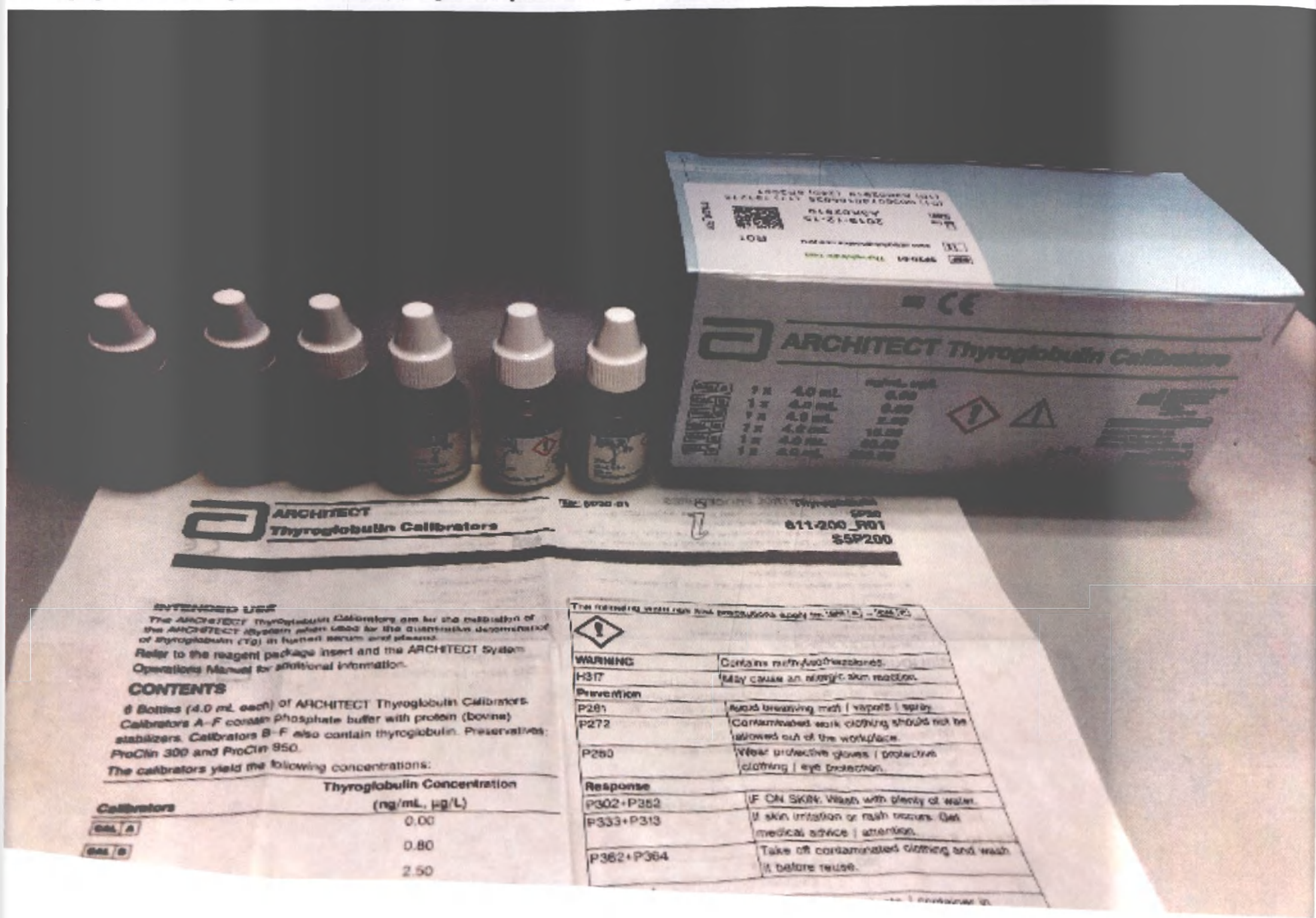
ARCHITECT
Thyroglobulin Calibrators

INTENDED USE
The ARCHITECT Thyroglobulin Calibrators are for the calibration of the ARCHITECT system when used for the quantitative determination of thyroglobulin (TG) in human serum and plasma.
Refer to the reagent package insert and the ARCHITECT system Operations Manual for additional information.

CONTENTS
6 Bottles (4.0 mL each) of ARCHITECT Thyroglobulin Calibrators. Calibrators A-F contain phosphate buffer with protein (bovine) stabilizers. Calibrators B-F also contain thyroglobulin. Preservatives: ProClin 300 and ProClin 950.
The calibrators yield the following concentrations:

Calibrators	Thyroglobulin Concentration (ng/mL, µg/L)
Cal A	0.00
Cal B	0.80
	2.50

The following warnings and precautions apply to ARCHITECT.

	WARNING	Contains methoxyethanones.
	H317	May cause an allergic skin reaction.
	Prevention	
	P261	Avoid breathing dust / vapors / spray.
	P272	Contaminated work clothing should not be allowed out of the workplace.
	P280	Wear protective gloves / protective clothing / eye protection.
	Response	
	P302+P352	IF ON SKIN: Wash with plenty of water.
	P333+P313	If skin irritation or rash occurs: Get medical advice / attention.
	P362+P364	Take off contaminated clothing and wash it before reuse.

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Limited liability company