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CERTIFICATE

中国国际贸易促进委员会暨中国国际商会
China Council for the Promotion of International Trade is China Chamber of International Commerce

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China Council for the Promotion of International Trade
China Chamber of International Commerce

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THIS IS TO CERTIFY THAT: the seal of Changzhou Jinlong Medical Plastic Appliance Co., Ltd. on the annexed DOCUMENT is genuine.

China Council for the Promotion
of International Trade

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Authorized

Signature:

HAN LI

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(Date: Nov. 14, 2022)

证明书查询 Website for verifying the certificate: <http://www.rzccpit.com/validate.html>

Changzhou Jinlong Medical Plastic Appliance Co., Ltd

APPROVED BY
Changzhou Jinlong Medical Plastic Appliance Co., Ltd

Full name: Cai Liping
Title: General Director
10 «October» 2022

Instruction for use
Disposable injection syringes with and without needles, volume: 1 ml, 2 ml, 3 ml, 5 ml,
10 ml, 20 ml, 50 ml
Versions

1. General information

Product name:

«Disposable injection syringes with and without needles, volume: 1 ml, 2 ml, 3 ml, 5 ml, 10 ml, 20 ml, 50 ml».

Versions

- Disposable injection syringe with needle 0,4 x 12 mm (27G), volume: 1 ml;
- Disposable injection syringe with needle 0,6 x 30 mm (23G), volume: 2 ml;
- Disposable injection syringe with needle 0,63 x 32 mm (23G), volume: 2 ml;
- Disposable injection syringe with needle 0,63 x 45 mm (23G), volume: 2 ml;
- Disposable injection syringe with needle 0,6 x 30 mm (23G), volume: 3 ml;
- Disposable injection syringe with needle 0,63 x 32 mm (23G), volume: 3 ml;
- Disposable injection syringe with needle 0,63 x 45 mm (23G), volume: 3 ml;
- Disposable injection syringe with needle 0,7x40 mm / 0,7 x 38 mm (22G), volume: 3 ml;
- Disposable injection syringe with needle 0,7x40 mm / 0,7 x 38 mm (22G), volume: 5 ml;
- Disposable injection syringe with needle 0,8 x 50 mm (21G), volume: 5 ml;
- Disposable injection syringe with needle 0,7x40 mm / 0,7 x 38 mm (22G), volume: 10 ml;
- Disposable injection syringe with needle 0,8 x 40 mm / 0,8 x 38 mm (21G), volume: 10 ml;
- Disposable injection syringe with needle 0,8 x 50 mm (21G), volume: 10 ml;
- Disposable injection syringe with needle 0,8 x 40 mm / 0,8 x 38 mm (21G), volume: 20 ml;

Disposable injection syringe with needle 0,9 x 50 mm (20G), volume: 20 ml;

Disposable injection syringe with needle 1,2 x 40 mm / 1,2 x 38 mm (18G), volume: 50 ml;

Disposable injection syringe with two needles: 0,45 x 12 mm (26G), 0,5 x 16 mm (25G),
volume: 1 ml; •

Disposable injection syringe with two needles: 0,63 x 32 mm (23G), 0,7 x 38 mm (22G),
volume: 2 ml;

Disposable injection syringe with two needles: 0,63 x 32 mm (23G), 0,63 x 45 mm (23G),
volume: 2 ml;

Disposable injection syringe with two needles: 0,6 x 38 mm (23G), 0,7 x 38 mm (22G), volume:
5 ml

Disposable injection syringe with two needles: 0,7 x 40 mm (22G), 0,8 x 50 mm (21G), volume:
5 ml;

Disposable injection syringe with two needles: 0,7 x 38 mm (22G), 0,8 x 38 mm (21G),
volume: 10 ml;

Disposable injection syringe with two needles: 0,7 x 40 mm (22G), 0,8 x 50 mm (21G),
volume: 10 ml;

Disposable injection syringe with two needles: 0,8 x 40 mm (21G), 0,8 x 50 mm (21G), volume:
20 ml;

Disposable injection syringe with two needles: 0,8 x 40 mm (21G), 0,9 x 50 mm (20G), volume:
20 ml;

Disposable injection syringe without needle, volume: 1 ml;

Disposable injection syringe without needle, volume: 2 ml;

Disposable injection syringe without needle, volume: 3 ml;

Disposable injection syringe without needle, volume: 5 ml;

Disposable injection syringe without needle, volume: 10 ml;

Disposable injection syringe without needle, volume: 20 ml;

Disposable injection syringe without needle, volume: 50 ml;

Manufacturer:

«Changzhou Jinlong Medical Plastic Appliance Co. Ltd»

Address: Zhenglu Town Wujin, Jiangsu 213111 China

E-mail: zly@jinlongmed.com.cn

Site: <https://jinlongmed.com/>

place of production:

«Changzhou Jinlong Medical Plastic Appliance Co. Ltd», Zhenglu Town Wujin, Jiangsu 213111
China

Authorized representative of the manufacturer on the territory of the Russian Federation
Sana Co., Ltd.

Registered address: 302008, Orel Oblast, Orel, st. Vysokovoltnaya, h. 2 «G»

Postal address: 302008, Orel Oblast, Orel, st. Vysokovoltnaya, h. 2 «G»

Telephone: +7 (4862) 44-02-92, +7 (4862) 44-02-93

E-mail: sanaorel@mail.ru

Code of the All-Russian classifier of products by type of economic activity OKPD 2:

32.50.13.110

Type of medical device in accordance with the nomenclature classification of medical devices: 349160, 260600

Class depending on the degree of potential risk of use – 2a.

Type of contact with the human body: indirect contact with blood and the internal environment of the body through drugs, short-term contact with the skin.

2. Introduction

Disposable injection syringes with and without needles (*hereinafter referred to as syringes, disposable injection syringes with a needle, disposable injection syringes with two needles, disposable injection syringes without needle*) are intended for subcutaneous, intramuscular and intravenous administration of various liquid medicines into the body, as well as for blood collection and suction of various fluids from the body.

3. Indications for use

The product is recommended for use for all types of injections in hospital, outpatient and home conditions.

4. Medical device specifications

The syringe materials are compatible with injection fluids. The cylinder is made of polypropylene, which makes it possible to control the absence of air bubbles (excluding embolism of small vessels) when liquid drugs are injected into the body and to evaluate the appropriate type of drug. At the distal end of the cylinder there is a connector (Luer-lock / Luer-slip) for connecting a hypodermic needle. For the most accurate adherence to the dosage, the syringe has a clearly printed scale (insulin / tuberculin). The syringe piston seal is made of

natural rubber, which ensures a tight contact between the piston and the cylinder during the selection and administration of drugs. The needles used in the syringes are made of high quality stainless austenitic steel, have a triangular laser sharpening, which minimizes pain during injection. The needle is subjected to repeated chemical treatment during the production process, which makes the outer surface of the needle shiny, smooth, without cracks, scratches and other defects. Sizes of needles: 0.4 x 12 mm; 0.45 x 12 mm; 0.5 x 16 mm; 0.6 x 30 mm; 0.6 x 38 mm; 0.63 x 32 mm; 0.63 x 45 mm; 0.7 x 38 mm; 0.7 x 40 mm; 0.8 x 38 mm; 0.8 x 40 mm; 0.8 x 50 mm; 0.9 x 50 mm; 1.2 x 38 mm, 1.2 x 40 mm. The needle guard is made of polypropylene. The cannulas of the needles are hypoallergenic and painted in the colors established by international standards. Controlled administration of a dose of medical substance is provided by a clearly printed scale applied with indelible ink.

The main characteristics of the versions of the syringes are shown in Table 1.


Table 1 - Syringe options

Type	Volume	Needle size
Disposable injection syringes with needle	1 ml	0,4 x 12 mm (27G);
Disposable injection syringes with needle	2 ml	0,6 x 30 mm (23G);
Disposable injection syringes with needle	2 ml	0,63 x 32 mm (23G);
Disposable injection syringes with needle	2 ml	0,63 x 45 mm (23G);
Disposable injection syringes with needle	3 ml	0,6 x 30 mm (23G);
Disposable injection syringes with needle	3 ml	0,63 x 32 mm (23G);
Disposable injection syringes with needle	3 ml	0,63 x 45 mm (23G);
Disposable injection syringes with needle	3 ml	0,7x40 mm / 0,7 x 38 mm (22G);
Disposable injection syringes with needle	5 ml	0,7x40 mm / 0,7 x 38 mm (22G);
Disposable injection syringes with needle	5 ml	0,8 x 50 mm (21G);

Disposable injection syringes with needle	10 ml	0,7x40 mm / 0,7 x 38 mm (22G);
Disposable injection syringes with needle	10 ml	0,8 x 40 mm / 0,8 x 38 mm (21G)
Disposable injection syringes with needle	10 ml	0,8 x 50 mm (21G);
Disposable injection syringes with needle	20 ml	0,8 x 40 mm / 0,8 x 38 mm (21G)
Disposable injection syringes with needle	20 ml	0,9 x 50 mm (20G);
Disposable injection syringes with needle	50 ml	1,2 x 40 mm / 1,2 x 38 mm (18G);
Disposable injection syringes with two needles	1 ml	0,45 x 12 mm (26G); 0,5 x 16 mm (25G);
Disposable injection syringes with two needles	2 ml	0,63 x 32 mm (23G); 0,7 x 38 mm (22G);
Disposable injection syringes with two needles	2 ml	0,63 x 32 mm (23G); 0,63 x 45 mm (23G);
Disposable injection syringes with two needles	5 ml	0,6 x 38 mm (23G); 0,7 x 38 mm (22G);
Disposable injection syringes with two needles	5 ml	0,7 x 40 mm (22G); 0,8 x 50 mm (21G);
Disposable injection syringes with two needles	10 ml	0,7 x 38 mm (22G); 0,8 x 38 mm (21G);
Disposable injection syringes with two needles	10 ml	0,7 x 40 mm (22G); 0,8 x 50 mm (21G);
Disposable injection syringes with two needles	20 ml	0,8 x 40 mm (21G); 0,8 x 50 mm (21G);
Disposable injection syringes with two needles	20 ml	0,8 x 40 mm (21G); 0,9 x 50 mm (20G);
Disposable injection syringes without needle	1 ml	Not available
Disposable injection syringes without needle	2 ml	Not available
Disposable injection syringes without needle	3 ml	Not available
Disposable injection syringes without needle	5 ml	Not available
Disposable injection syringes without needle	10 ml	Not available
Disposable injection syringes without needle	20 ml	Not available
Disposable injection syringes without needle	50 ml	Not available

The design of a disposable injection syringe with a needle is shown in Figure 1.

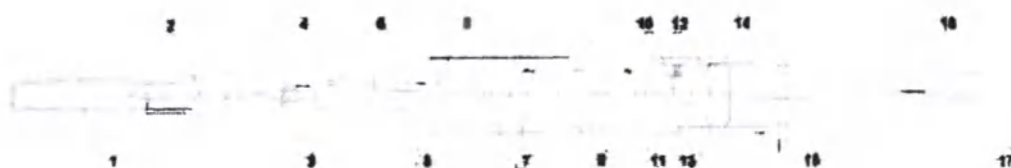
Figure 1- Schematic representation of a disposable injection syringe with a needle

- | | | |
|--------------------------|---|------------------|
| 1. Protective needle cap | 7. Graduated line | 13. Sealant |
| 2. Needle | 8. Zero line | 14. Piston |
| 3. Compound | 9. Cylinder | 15. Finger rests |
| 4. Tip protection cap | 10. Graduation line of nominal capacity | 16. Stock |
| 5. Taper tip | 11. Full Graduated Capacity Line | 17. Stem stop |
| 6. Tip hole | 12. Reference line | |

The design of a disposable injection syringe without a needle is shown in Figure 2.

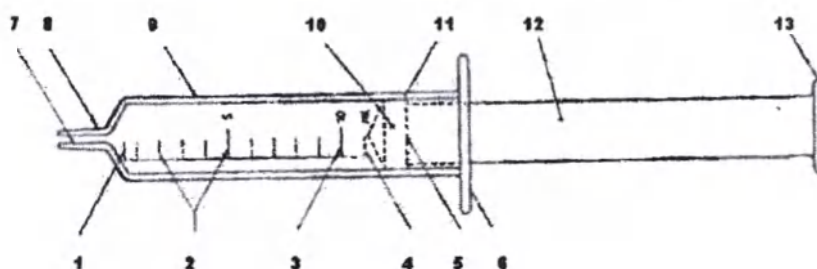Figure 2 - Schematic representation of a disposable injection syringe without a needle

- | | |
|--|---------------|
| 1. Zero line | 7. Tip hole |
| 2. Graduated lines | 8. Taper tip |
| 3. Graduation line of nominal capacity | 9. Cylinder |
| 4. Full Graduated Capacity Line | 10. Piston |
| 5. Reference line | 11. Sealant |
| 6. Finger rests | 12. Stock |
| | 13. Stem stop |

sterilization method - gas, sterilized with ethylene oxide.

5. The order (algorithm) of intended use (application)

Before the procedure, perform hygienic treatment of hands.

Use gloves during the procedure.

Prepare the syringe (check the expiration date, tightness of the package).

Remove the syringe from the packaging, remove the protective cap from the needle.

Type in the syringe drug.

- Make sure that when you set the drug needle is always in solution: in this case, air is excluded from entering the syringe.

- Make sure there is no air in the syringe. If there are air bubbles on the walls of the cylinder, you should slightly pull the plunger of the syringe and several times "turn" the syringe in a horizontal plane. Then it is necessary to expel air, holding a syringe over a sink or in an ampoule. The syringe is used immediately after filling.

Carefully treat the area of the skin at the injection site with a swab with a disinfectant solution.

Puncture the area of the skin at the injection site and slowly enter the drug.

After injection, hold the puncture site with a swab with disinfectant solution. Close the needle with the cap.

Dispose of the syringe.

Avoid contact with the needle!

6. Storage conditions

Products are stored in packaged form in a dry and ventilated room at a temperature from +5 ° C to +40 ° C with a relative humidity of not more than 80%. During storage, syringes should be protected from direct sunlight, at a distance of at least 1 m from heat-emitting devices.

7. Manufacturer warranty

7.1 The warranty storage period for syringes is five years from the date of manufacture.

7.2 «Changzhou Jinlong Medical Plastic Appliance Co., Ltd» guarantees the compliance of syringes with the necessary requirements under the conditions of storage and transportation.

7.3 For questions related to the warranty, contact the manufacturer:

«Changzhou Jinlong Medical Plastic Appliance Co., Ltd»;

Address: Changzhou Jinlong Medical Plastic Appliance Co. Ltd, Zhenglu Town Wujin, Jiangsu 213111 China.

authorized representative of the manufacturer in the Russian Federation:

«Sana» Co., Ltd.

302008, Orel Oblast, Orel, st. Vysokovolt'naya, h. 2 «G»

(4862) 44-02-92, (4862) 44-02-93

sanaorel@mail.ru

8. Complaints address:

If side effects are detected that are not specified in the instructions for use of vacuum tubes, adverse reactions during its use, facts and circumstances that pose a threat to the life and health of citizens and medical workers when using vacuum tubes, it is necessary to send messages containing the specified information to the authorized representative of the manufacturer:

«Sana» Co. Ltd.

302008, Orel Oblast, Orel, st. Vysokovolt'naya, h. 2 «G»

(4862) 44-02-92, (4862) 44-02-93

sanaorel@mail.ru

9. Transportation

Syringes are transported by all types of transport in covered vehicles in accordance with the rules for the carriage of goods in force for this type of transport at temperatures from + 5 ° C to 40 ° C with a relative humidity of not more than 80%.

10. Disposal procedures

Measures for the destruction and disposal of single-use injection syringes should be carried out in accordance with the requirements of sanitary regulations and other regulatory legal acts of the Russian Federation in all health care facilities, regardless of their profile.

Changzhou Jinlong Medical Plastic Appliance Co., Ltd

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