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Alinity i C-Peptide Controls

For the estimation of test precision and the detection of systematic analytical deviations of the Alinity i analyzer when used for the quantitative determination of C-peptide in human serum, plasma and urine.

• **CONTROL L**, **CONTROL M**, **CONTROL H** contain human C-peptide (synthetic) in PBS buffer with heat inactivated horse serum and stabilizers.

Preservatives: ProClin 300 and ProClin 950.

Distributed in the USA by Abbott Laboratories, Abbott Park, IL 60064 USA

09P3610

REF ONLY

C-Peptide Ctrl

www.abbottdiagnostics.com/IFU

09P3610

Exp. 2020-06-12

LOT 00412L819

(01) 00380740153526 (17) 200612
(10) 00412L819 (240) 09P3610

Information for USA only

Alinity i C-Peptide Controls

For the estimation of test precision and the detection of systematic analytical deviations of the Alinity i analyzer when used for the quantitative determination of C-peptide in human serum, plasma and urine.

• **CONTROL L**, **CONTROL M**, **CONTROL H** contain human C-peptide (synthetic) in PBS buffer with heat inactivated horse serum and stabilizers.

Preservatives: ProClin 300 and ProClin 950.

Distributed in the USA by Abbott Laboratories, Abbott Park, IL 60064 USA

09P3610

REF ONLY

C-Peptide Ctrl


www.abbottdiagnostics.com/IFU


09P3610

Exp. 2020-06-12

LOT 00412L819

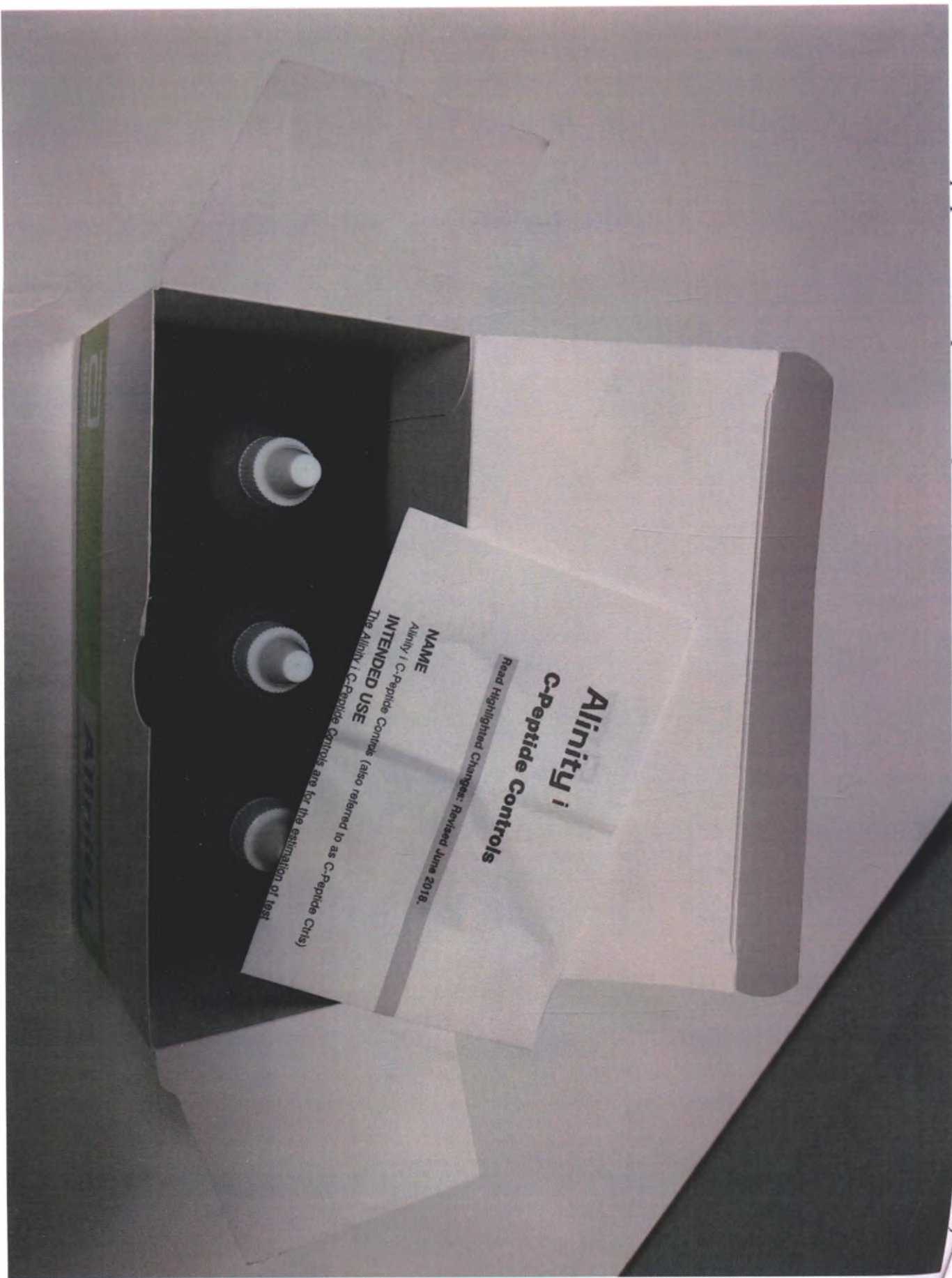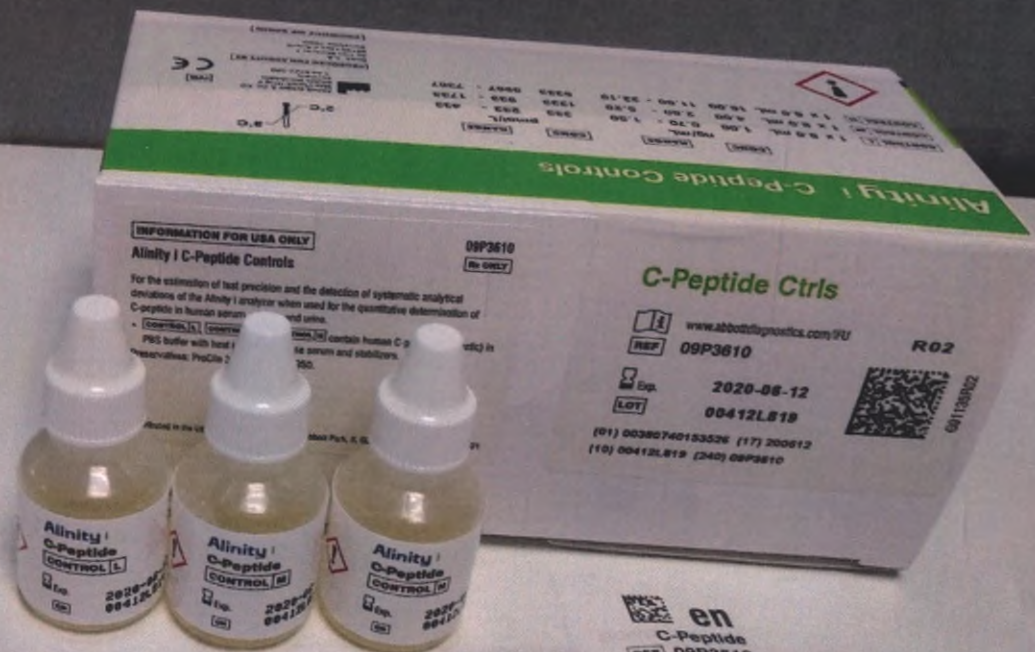(01) 00380740153526 (17) 200612
(10) 00412L819 (240) 09P3610

Alinity i C-Peptide Controls

Read Highlighted Changes: Revised June 2018.

NAME

Alinity i C-Peptide Controls (also referred to as C-Peptide Ctrl's)

INTENDED USE

The Alinity i C-Peptide Controls are for the estimation of test precision and the detection of systematic analytical deviations of the Alinity i analyzer when used for the quantitative determination of C-peptide in human serum, plasma and urine.

For additional information, refer to the Alinity i C-Peptide reagent insert and the Alinity ci-series Operations Manual.

Response	
P302+P352	IF ON SKIN: Wash with plenty of water.
P333+P313	If skin irritation or rash occurs: Get medical advice / attention.
P362+P364	Take off contaminated clothing and wash it before reuse.
Disposal	
P501	Dispose of contents / container in accordance with local regulations.

Safety Data Sheets are available at www.abbottdiagnostics.com or contact your local representative.

For a full discussion of safety precautions during system installation, refer to the Alinity i C-Peptide reagent insert and the Alinity ci-series Operations Manual, Section 8.

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