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Место производства: Fisher Diagnostics, A Division of Fisher Scientific Company, LLC, A Part of Thermo Fisher Scientific, Inc., 8365 Valley Pike, Middletown, Virginia, 22645, USA

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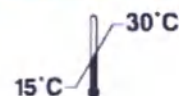
CONTAINS: AZIDE

Abbott GmbH & Co. KG
Max-Planck-Ring 2
65205 Wiesbaden
Germany
+49-6122-580

PRODUCED FOR ABBOTT BY
Fisher Diagnostics,
A Div. of Fisher Scientific Company, LLC
A Part of Thermo Fisher Scientific, Inc.
8365 Valley Pike
Middletown VA 22645 USA

PRODUCT OF USA

INFORMATION FOR USA ONLY

Rx ONLY

For the quantitation of protein in human urine or cerebrospinal fluid (CSF) on the Alinity c analyzer.

- **R1** Active ingredients: Carbonate Buffer (100 mmol/L), Sodium Chloride (140 mmol/L), Preservative: sodium azide (0.05%).
- **R2** Benzethonium Chloride (20 g/L).

Distributed in the USA by Abbott Laboratories, Abbott Park, IL 60064 USA

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U/CSF Prot

Alinity c Urine/CSF Protein Reagent Kit

Abbott

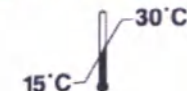
U/CSF Prot

REF 07P5930 10 x 250

www.abbottdiagnostics.com/IFU

Exp.

LOT

R1 10 x 59.2 mL
R2 10 x 15.6 mL

CONTAINS: AZIDE

Abbott GmbH & Co. KG
Max-Planck-Ring 2
65205 Wiesbaden
Germany
+49-6122-580

IVD

PRODUCED FOR ABBOTT BY

Fisher Diagnostics,
A Div. of Fisher Scientific Company, LLC
A Part of Thermo Fisher Scientific, Inc.
8365 Valley Pike
Middletown, VA 22645 USA

PRODUCT OF USA

INFORMATION FOR USA ONLY **Rx ONLY**
For the quantitation of protein in human urine or cerebrospinal fluid (CSF) on the Alinity c analyzer.
• **R1** Active ingredients: Carbonate Buffer (100 mmol/L), Sodium Chloride (140 mmol/L), Preservative: sodium azide (0.05%).
• **R2** Benzethonium Chloride (20 g/L).
Distributed in the USA by Abbott Laboratories, Abbott Park, IL 60064 USA

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Abbott

G71055R01

Alinity[®] Urine/CSP Protein Reagent Kit

Read Important Changes: Revised March 2018

Instructions must be carefully followed. Misuse or error results cannot be guaranteed. It does not replace the user manual.

NAME
Alinity[®] Urine/CSP Protein Reagent Kit (also referred to as U/CSP Kit)

INTENDED USE
The Alinity[®] Urine/CSP Protein Reagent Kit is used for the quantitative analysis of urine and cerebrospinal fluid (CSF) for the Alinity[®] analyzer.

CSF protein measurements are used in the diagnosis and treatment of conditions such as meningitis, brain tumors, and infections of the central nervous system.¹

SUMMARY AND EXPLANATION OF THE TEST
The rate of the renal system in the concentration of plasma proteins has been assigned for some time. Under normal physiological conditions, small molecular weight proteins, such as albumin, pass through the glomerular barrier in varying large amounts. Intermediate size proteins, such as transferrin and albumin, also pass through in relatively small amounts. Most of these proteins are reabsorbed in the renal tubules.^{2,3}

Most CSF protein originates by diffusion from plasma across the blood-CSF barrier. Elevated levels occur as a result of increased permeability of the blood-CSF barrier or with increased local synthesis of macroglobulins.⁴

PRINCIPLES OF THE PROCEDURE
The Alinity[®] Urine/CSP Protein Reagent Kit is a turbidimetric procedure in which benzothiazolyl chloride is used as the protein denaturing agent. Proteins present in the urine or CSF are denatured by benzothiazolyl chloride, resulting in the formation of a fine turbid suspension.

Warnings and Precautions
• Risk
• Use of this reagent kit
• Storage

Safety Precautions

CAUTION: This product requires the handling of human specimens. It is recommended that all human specimen materials be considered potentially infectious and handled in accordance with the OSHA standard on Bloodborne Pathogens, Biosafety Level 2 or other appropriate biohazard precautions should be used for materials that contain or are suspected of containing infectious agents.^{5,6}

The following warnings and precautions apply to this kit.

Reagents	Contains potassium carbonate, benzothiazolyl chloride, and sodium azide.
Stability	Caution: avoid skin contact.
Storage	Caution: avoid skin contact.
Precautions	Caution: avoid skin contact.
Response	• IN USE: Please carefully with water for several minutes. Remove contact lenses, if present, and allow to dry. Continue testing. • STOP: Stop testing. Rinse thoroughly with water. • DISPOSAL: Dispose of waste in accordance with local, state, and federal regulations.

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Limited liability company