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«APPROVE»

Managing Director / Director Operations
(position)

Jochen Dreymann

(name)

Dentsply DeTrey GmbH
De-Trey-Straße 1
78467 Konstanz
Germany

(signature)

30.06.2023 Stamp
Date

Instruction for use for a medical device

Neo Spectra ST flow Kits for anterior and posterior teeth restoration in design
variants

1 INTRODUCTION

ATTENTION! For safety when using the "Neo Spectra ST Flow Kits for anterior and posterior restorations in design variants", as intended, all users should carefully study these instructions for use before using the product. The instructions for use are written to familiarize the consumer with the design of the medical device, its functions, and also describe the conditions and rules of operation. The instructions for use are intended for healthcare professionals.

- Always follow these Instructions for Use.
- The instructions for use should be kept in the immediate vicinity of the product.

2 NAME OF THE MEDICAL DEVICE

Neo-Spectra ST flow Kits for anterior and posterior teeth restoration in design variants.

3 COMPLETENESS

Neo Spectra ST flow Kits for anterior and posterior teeth restoration in design variants:

1. Neo Spectra ST flow Syringe Discovery Kit, consisting of:
 - 1) Light-cured flowable dental restorative material Neo Spectra ST flow in syringes 1.8 g each, shade: A2 – 1 pce.
 - 2) Applicator for syringe - 25 pcs;
 - 3) Illustrated technical guide - 1 pce;
 - 4) Instruction for use on paper and/or in electronic form - 1 pce
2. Neo Spectra ST flow Compula® Discovery Kit, consisting of:
 - 1) Light-cured flowable dental restorative material Neo Spectra ST flow Compula® tips 0.25 g each, shades: A2 (4 pcs), A3 (4 pcs) – 8 pcs;
 - 2) Illustrated technical guide - 1 pce;
 - 3) Instruction for use on paper and/or in electronic form - 1 pce
3. Neo Spectra ST flow Compula® Intro Kit, consisting of:
 - 1) Light-cured flowable dental restorative material Neo Spectra ST flow Compula® tips 0.25 g, shades: A1 (4 pcs), A2 (4 pcs), A3 (8 pcs), A3.5 (4 pcs), A4 (4 pcs) - 24 pcs;
 - 2) Prime & Bond® universal adhesive in bottle, 2.5 ml - 1 pce;
 - 3) Applicator tips - 50 pcs;
 - 4) CliXdish mixing well - 1 pce;
 - 5) Compules® Tips Gun - 1 pce;
 - 6) Order form for Neo Spectra ST Shade Guide - 1 pce (if necessary);
 - 7) Illustrated technical guide - 1 pce;
 - 8) i-Shade label - 1 pce.;

- 9) Instruction for use on paper and/or in electronic form - 2 pcs.
- 10) Information leaflet - 1 pce (if necessary)

4. Neo Spectra ST flow Syringe Intro Kit, consisting of:

- 1) Light-cured flowable dental restorative material Neo Spectra ST flow in syringes 1.8 g each, shades: A1 (1 pce), A2 (1 pce), A3 (1 pce), A3.5 (1 pce), A4 (1 pce) – 5 pcs
- 2) Prime & Bond® universal adhesive in bottle, 2.5 ml - 1 pce.;
- 3) Applicator for syringe - 50 pcs;
- 4) CliXdish mixing well - 1 pce.;
- 5) Applicator tips – 50 pcs;
- 6) Order form for Neo Spectra ST Shade Guide - 1 pce (if necessary);
- 7) Illustrated technical guide - 1 pce.;
- 8) i-Shade label - 1 pce;
- 9) Instruction for use on paper and/or in electronic form - 2 pcs
- 10) Information leaflet - 1 pce (if necessary)

4 INFORMATION ABOUT THE MEDICAL DEVICE MANUFACTURER

Dentsply De Trey GmbH

Address: De-Trey-Strasse 1, 78467 Konstanz, GERMANY

E-mail: service-konstanz@dentsplysirona.com

5 INFORMATION ABOUT THE AUTHORIZED REPRESENTATIVE OF THE MANUFACTURER

Dentsply Sirona Limited Liability Company

Address: premises 3N, bld.2, Ovchinnikovskaya embankment, 18/1, ZamoscvoRechie municipal district, intra-city area, Moscow, 115035

+7 (495) 725-10-87

E-mail: MoscowRus-Reception@dentsplysirona.com

6 PURPOSE AND OPERATING PRINCIPLES OF THE MEDICAL DEVICE

Neo Spectra ST flow Kits for anterior and posterior teeth restoration in design variants are intended for restoration of anterior and posterior teeth.

7 INDICATIONS

Neo Spectra ST flow Kits are indicated for the following procedures:

- Direct restoration of cavities (except occlusal stress bearing Class II cavities), lesions.
- Filling of defects and undercuts in crowns, inlays, and onlays.
- Blockouts.

- Repair of defects.
- Pit and fissure sealants.
- Cementation of light transmissible indirect restorations.

8 CONTRAINDICATIONS

The medical device is contraindicated for patients who have previously had allergic reactions to methacrylate resins or to any other component of the composite material.

9 SAFETY NOTES

Be aware of the following general safety notes and the special safety notes in other chapters of these Directions for Use.

Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury..

It is recommended to use the medical device in clean, disinfected gloves.

10 WARNINGS

The material contains methacrylates and polymerizable monomers which may be irritating to skin, eyes and oral mucosa and may cause sensitization by skin contact and allergic contact dermatitis in susceptible persons.

• **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes rinse with plenty of water and seek medical attention.

• **Avoid skin contact** to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, remove material with cotton and alcohol and wash thoroughly with soap and water. In case of skin sensitization or rash, discontinue use and seek medical attention.

Avoid contact with oral soft tissues/mucosa to prevent inflammation. If accidental contact occurs, remove material from the tissues. Flush mucosa with plenty of water and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.

11 PRECAUTIONS

This product is intended to be used as specifically outlined in the Directions for Use. Any use of this product inconsistent with the Directions for Use is at the discretion and sole responsibility of the practitioner.

- If refrigerated, allow material to reach room temperature prior to use.
- Contact with saliva and blood during composite placement may cause failure of the restoration. Use of rubber dam or adequate isolation is recommended.
- Use protective measures for the dental team and patients such as glasses and rubber dam in accordance with local best practice.
- Devices marked "single use" on the labeling are intended for single use only. Discard after use. Do not reuse in other patients in order to prevent cross-contamination.
- The syringes cannot be reprocessed. To prevent syringes from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the syringes are handled with clean/disinfected gloves. Discard syringes if contaminated.
- As additional precautionary measure, syringes may be protected from gross debris but not from all contamination by applying a protective barrier.
- Use of Compula® Tips with the Dentsply Sirona Compules® Tips Gun is recommended.
- The Compules® Tips Gun is only intended to be used with Dentsply Sirona Compules® or Compula® Tips.
- For further information please refer to the Dentsply Sirona Compules® Tips Gun Directions for Use.
- The shade guide and the individual shade guide tabs cannot be reprocessed. To prevent the shade guide and the individual shade guide tabs from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the shade guide and the individual shade guide tabs are handled with clean/disinfected gloves.
- The material should extrude easily. **DO NOT USE EXCESSIVE FORCE.** Excessive pressure may result in unanticipated extrusion of the material or cause the Compula® Tip to rupture or to eject from the Dentsply Sirona Compules® Tips Gun.
- Tightly close syringes immediately after use. Do not allow dispensing tip to remain as a syringe cap. Replace original cap tightly after each use. Discard and properly dispose dispensing tips after use, as the tips may clog if material is allowed to dry or set inside.

- Dentsply Sirona supplies the appropriate dispensing tip for the Neo Spectra ST flow composite restorative material syringe. This tip is the only tip that should be utilized for placement of the material.

- Insufficient data exist to support the use in Class I cavities with isthmus widths beyond central groove areas, or when replacing a functional cusp. Excessive wear or restoration failure may result.

- Insufficient data exist to support the use in Root canal fillings. Restoration failure may result.

- Interactions:

- Do not use eugenol- or hydrogen peroxide-containing materials in conjunction with this product because they may interfere with hardening and cause softening of the polymeric components of the material.

- The Neo Spectra ST flow composite restorative is light cured material. Therefore, it should be protected from ambient light. Proceed immediately once material has been placed.

- If mineral-impregnated (e.g., ferric compounds) retraction cords and/or hemostatic solutions are used in conjunction with adhesive procedures, marginal seal may be adversely affected, allowing microleakage, subsurface staining and/or restoration failure. If gingival retraction is necessary, use of plain, non-impregnated cord is recommended.

12 ADVERSE REACTIONS

- Eye contact: Irritation and possible corneal damage.
- Skin contact: Irritation or possible allergic response. Reddish rashes may be seen on the skin.
- Contact with mucous membranes: Inflammation (see Warnings).

13 DESCRIPTION

Neo Spectra ST flow composite restorative is a visible light cured, radiopaque composite resin restorative material with flow characteristics that make it ideal for use in anterior and posterior teeth.

Neo Spectra ST flow composite restorative adapts to the cavity wall without the use of hand instruments. The unique "stackable" flow characteristics allow intimate adaptation without slumping.

Neo Spectra ST flow is a hybrid composite possessing the high strength and wear resistance required by anterior and certain posterior restorations, combined with an easily achieved high surface luster and smoothness.

Prime & Bond® universal is a combination of Etch & Rinse, Selective Etch and Self Etch adhesive. It provides easy adhesive application for direct and indirect restorations and is capable of adhering to enamel, dentin, composites, zirconia and metals. Prime & Bond® universal is compatible with conventional methacrylate light-cured composite restorations and cements.

Registration Certificate No. RZN 2018/7377 dated July 18, 2018.

- CliXdish plastic mixing well.

Registration Certificate No. RZN 2018/7377 dated July 18, 2018.

- Compules® Tips Gun.

Registration Certificate No. RZN 2017/5954 dated July 10, 2017.

Illustrated technical guide of Neo Spectra ST flow dental material in pictures with explanations.

The text of the document is contained in Annex 2 to the technical file for the medical device.

Fig.1 Illustrated technical guide

If the Neo Spectra ST Shade Guide is not available, we recommend sticking the i-Shade label to the back of the VITA® Classic Scale. Essential for determining the shade of the tooth and using the correct shade of Neo Spectra ST flow Dental Material.

The text of the document is contained in Annex 1 to the technical file for the medical device

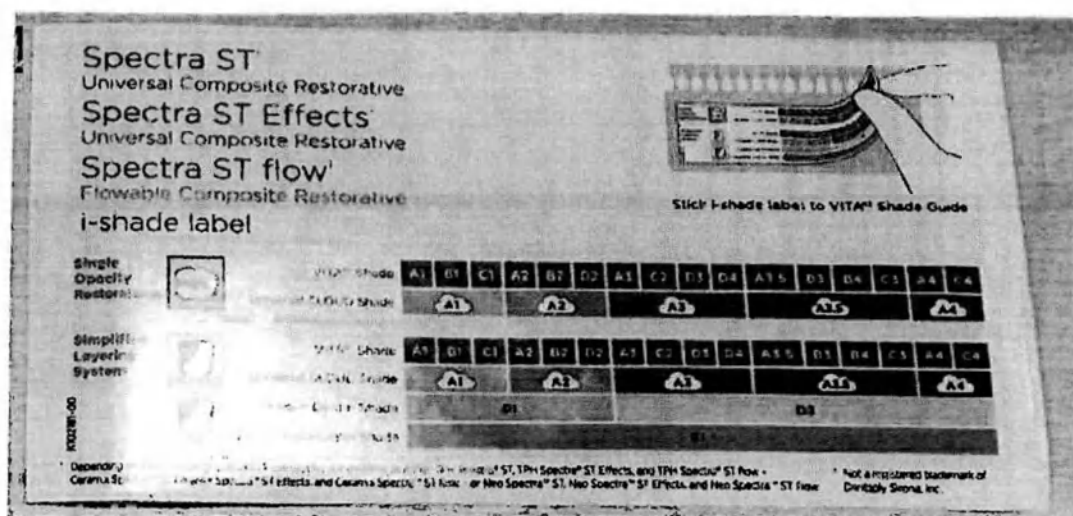
Fig. 2 i-Shade label

Order form for Neo Spectra ST Shade Guide is required to order the Neo Spectra ST Shade Guide. Supplied if necessary.

The text of the document is contained in Annex 3 to the technical file for the medical device.

Fig.3 Order form for shade guide

The information leaflet contains general information about Neo Spectra ST dental material, including Neo Spectra ST flow modification and design variants.

The text of the document is contained in Annex 4 to the technical file for the medical device.

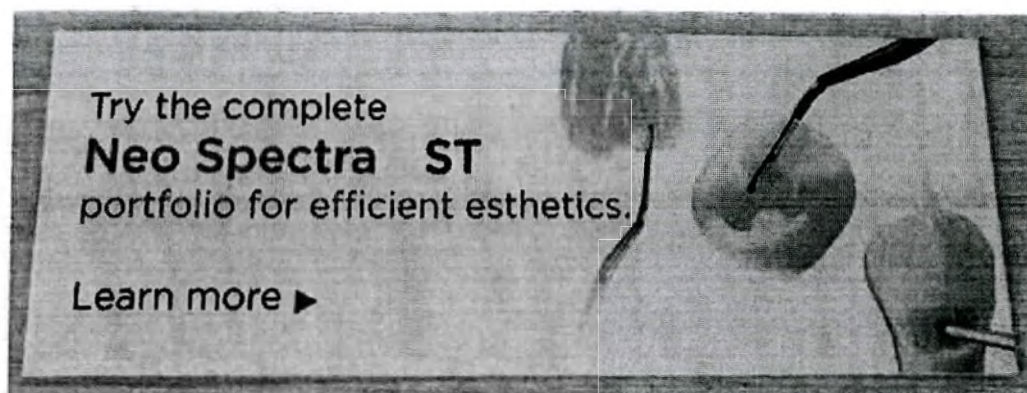
Fig.4 The information leaflet

13.1 Delivery forms

Neo Spectra ST flow composite flowable restorative is available in:

- Predosed Compula® Tips
- Syringes

14 METHOD OF APPLICATION OF THE MEDICAL DEVICE

14.1 Direct Application – Cavities, Lesions, Repair

Shade selection

1. Prior to shade selection, clean the tooth surface with prophylaxis paste to remove plaque or surface stain.
2. Select the shade while teeth are hydrated.

Neo Spectra ST flow composite restorative has been scientifically designed to allow the tooth structure enamel and dentin to blend together with the composite with a life-like translucent result. The universal CLOUD shades of Neo Spectra ST flow composite restorative adequately reproduce the traditional 16 VITA® shades.

- A1 (= VITA® shades A1, B1, C1)
- A2 (= VITA® shades A2, B2, D2)
- A3 (= VITA® shades A3, C2, D3, D4)
- A3.5 (= VITA® shades A3.5, B3, B4, C3)
- A4 (= VITA® shades A4, C4)

A Bleach White shade (BW), as well as opaque dentin shades D1 and D3 and translucent enamel shade E1 are also available.

For shade selection, there are two tools that may be used:

• The VITA® classical shade guide in combination with the i-shade label. The Spectra ST i-shade label is applicable for the Neo Spectra ST flow composite restorative and facilitates shade selection by assigning one of five universal CLOUD shades to each of the 16 VITA® shades (see table below).

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Fig.5 Correspondence of shades of Neo Spectra ST to VITA® shades

*VITA® - is not a registered trademark of Dentsply Sirona. Not a product of Dentsply De Trey GmbH.

The VITA® Shade Guide (RC No. FSZ 2011/09519, No. FSZ 2010/07252) is designed to precisely match the shade of the restored tooth. The VITA® Shade Guide is essential for a high-quality and correct selection of shades for filling or prosthetics. The shade guide helps to choose the color of the tooth / prosthesis so that it is in harmony with the shade of the adjacent teeth, the skin tone of the patient's face.

• Neo Spectra ST Shade Guide.

The Neo Spectra ST Shade Guide consists of shaded inlays made of Neo Spectra ST composite material. To match the shade, remove the individual tabs from the holder. To determine the material shade required for the restoration using the Neo Spectra ST shade, select the shade that best matches the surface to be restored on the shaded insert.

Fig. 7 Neo Spectra ST Shade Guide

It is a scale for the selection of the shade of a human tooth for the selection of a suitable shade for restoration.

Avoid contamination of the shade guide.

To prevent the shade guide from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the shade guide is handled offside the dental unit with clean/disinfected gloves. For shade selection, remove individual tabs from shade guide holder.

14.2 Cavity preparation

1. Prepare the cavity so that no residual amalgam or restorative is left (unless repairing an otherwise sound restoration).
2. Use adequate isolation such as rubber dam.
3. Rinse surface with water spray and carefully remove rinsing water. Do not desiccate the tooth structure.

14.2.1 Matrix placement and wedging

Place a matrix (e.g. Mylar, AutoMatrix® Matrix System or Palodent® Matrix System) and wedge. Burnishing of the matrix band will improve contact and contour. Pre-wedging or BiTine® ring placement is recommended.

14.3 Pulp protection, tooth conditioning/dentin pre-treatment, adhesive application

Refer to adhesive manufacturer's directions for pulp protection, tooth conditioning and/or adhesive application. Once the surfaces have been properly treated, they must be kept uncontaminated. Proceed immediately to placement of restorative material.

14.4 Application – Delivery

Using Compula® Tips

- 1 Insert Compula® Tip into the notched opening of the Compules® Tips Gun. Be certain that the collar on the Compula® Tip is inserted first.
2. Remove the colored cap from the Compula® Tip. The Compula® Tip may be rotated 360° to gain the proper angle of entrance into the cavity.
3. Dispense the material into the cavity preparation using a slow, steady pressure.

DO NOT USE EXCESSIVE FORCE.

4. To remove the used Compula® Tip, be sure that the Compules® Tips Gun plunger is pulled back completely by allowing the handle to open to its widest position. Apply a downward motion to the front end of the Compula® Tip and remove.

Excessive force.

Injury.

1. Apply slow and steady pressure on the Compules® Tips Gun.
2. Do not use excessive force. Compula® Tip rupture or ejection from Compules® Tips Gun may result.

14.5 Using syringes

Avoid contamination of the syringe.

To prevent the syringe from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the syringe is handled offside the dental unit with clean/disinfected gloves.

1. Remove cap from the end of the syringe. To assure free flow of material from syringe, express a small amount onto pad, away from the patient field.

2. Attach disposable dispensing tip to end of the syringe. Turn tip clockwise 1/4 to 1/2 turn to assure that it is fully seated. Tug on tip to be sure that it is locked into the collar of the syringe.

3. Neo Spectra ST flow composite restorative should flow freely with gentle pressure. **DO NOT USE EXCESSIVE FORCE.** If more than gentle pressure is required, remove from patient field and check for obstruction.

4. **DISCARD AND PROPERLY DISPOSE OF DISPENSING TIP IMMEDIATELY AFTER USE. REPLACE ORIGINAL CAP.** Do not store syringe with gray dispensing tip in place. **STORE ONLY WITH ORIGINAL CAP.**

Note: It is recommended to pull back slightly on the syringe plunger after use to prevent excessive flow of material.

14.6 Applying restorative

Dispense Neo Spectra ST flow composite restorative directly into cavity preparation/tooth surface from the Compula® Tip or syringe tip using slow, steady pressure. Avoid lifting the tip while dispensing to minimize air entrapment. Any visible air bubbles should be pierced with a clean, sharp explorer prior to curing. Material may be placed and light cured in increments up to 2 mm. In deeper preparations it is recommended to double the curing time to compensate for lower irradiance (see Curing, Step 14.7). For maximum light transmission and to minimize finishing, avoid overfilling cavity.

14.7 Light curing

Light cure each area of the restoration surface with a suitable visible light curing unit designed to cure materials containing camphorquinone (CQ) initiator, i.e. peak spectral output 440-480 nm.

Cure universal CLOUD shades for 10 seconds when using a curing light that has a minimum irradiance of 800 mW/cm².

Cure universal CLOUD shades for 20 seconds when minimum irradiance is between 550 mW/cm² and 800 mW/cm².

Refer to curing light manufacturer's Directions for Use for compatibility and curing recommendations.

Curing time for 2 mm increments

Neo Spectra ST flow Curing Recommendations		
Shades	Irradiance mW/cm ²	Curing Time
A1-A4	≥ 550	20 seconds
	≥ 800	10 seconds

	Insufficient curing.
	Inadequate polymerization.
	1. Check compatibility of curing light.
	2. Check curing cycle.
	3. Check minimum irradiance.
	4. Cure each area of each increment for the recommended curing time.
	5. Check distance to surface to be cured.

14.8 Finishing and polishing

1. Contour the restoration using finishing burs or diamonds.
2. Use Enhance[®] finishing devices for additional finishing.
3. Prior to polishing, check that surface is smooth and free of defects.
4. For achieving a very high luster of the restoration, we recommend Enhance[®] and Enhance[®] PoGo[®], finishing and polishing instruments, Prisma[®] Gloss[™] Regular and Extrafine polishing paste.

For contouring, finishing, and/or polishing, follow the manufacturer's Directions for Use.

All shades of Neo Spectra ST flow composite restorative are radioopaque, with a 1 mm radio-opacity equivalent to 1.8 mm radio-opacity of aluminum. Aluminum has a radio-opacity equivalent to that of dentin. Thus 1 mm of material having a radio-opacity equivalent to 1 mm of aluminum has a radio-opacity equivalent to that of dentin.

14.9 Cementation of light transmissible indirect restorations

- Following removal of the temporary restoration and any remaining temporary cement, clean enamel and dentin as directed by the adhesive manufacturer's directions.
- Check the fit and esthetics of the restoration. Technique Tip: Occlusal adjustment of veneers and onlays is best accomplished after final cementation.

14.10 Treatment of the restoration

Ceramic/Composite Restorations

Follow the dental laboratory or restoration manufacturer's instructions for pre-treatment, if required. Restorations designed to be silanated or if the internal silanated surface has been disturbed during try-in, apply Calibra® Silane Coupling Agent (available separately) according to the manufacturer's instructions.

14.11 Tooth conditioning/dentin pre-treatment/adhesive application

1. Proceed with adhesive application as directed in the complete Directions for Use.

Delayed or reduced adhesion.

Adhesive incompatibility.

1. Select compatible light cure adhesive.
2. Follow selected adhesive Directions for Use.

2. Always refer to and follow adhesive manufacturer's complete Directions for Use for selection and application of the appropriate light cure adhesive. Once the surfaces have been properly treated, they must be kept uncontaminated. Proceed immediately following adhesive application to cementation with Neo Spectra ST flow material.

14.12 Cementation (light cured)

Before proceeding refer to above sections for Treatment of the restoration and Tooth conditioning/dentin pre-treatment/adhesive application.

- Adhesives compatible with light cured, CQ initiated methacrylates are recommended.

- If restorations exceed 1.0 mm thickness, are heavily opaqued, or otherwise impede the transmission of light, operator is urged to select Calibra® Esthetic Resin Cement and follow the Dual-Cured cementation technique, mixing base with catalyst (available separately).

- Apply adhesive to the internal bonding surface of the restoration if directed by adhesive manufacturer's Directions for Use.

1. Dispense the desired shade of Neo Spectra ST flow material from the syringe directly onto the restoration. Protect cement from exposure to light.

2. Seat the loaded restoration in place. Remove excess from the margins with a blunt instrument. Briefly (10 seconds or less) light cure the gingival portion only to tack restoration in place. Remove any remaining excess from margins. Technique Tip: Placement of mylar strips between preparation and adjacent teeth prior to seating veneers aids in isolation and in excess cement clean up. After removing gingival excess and tacking veneer in place, remove interproximal excess cement by pulling mylar strip towards the facial, i.e., from tooth to restoration surface.

14.13 Curing

Insufficient curing.

Inadequate polymerization.

1. Check compatibility of curing light.
2. Check curing cycle.
3. Check minimum irradiance.
4. Cure each area of each increment for the recommended curing time.
5. Check distance to surface to be cured.

- Light cure with a suitable visible light curing unit designed to cure materials containing camphorquinone (CQ) initiator, i.e. peak spectral output 440-480 nm. Cure each area of the restoration surface (buccal, lingual, and interproximal) for at least 10 seconds at an irradiance of minimum 800 mW/cm². To ensure proper irradiance follow curing light instructions (e.g. increase exposure time to compensate lower irradiance due to increased distance between light guide/tip and restorative surface or curing lights with lower radiant emittance).

- Following the light cure check and adjust occlusion as necessary. Proceed to Finishing and Polishing as outlined in section 14.8 above.

14.14 Hygiene

**Cross-contamination.**

Infection.

- Do not reuse single use products. Dispose of in accordance with local regulations.
- Syringes cannot be reprocessed. Dispose of contaminated syringes in accordance with local regulations.
- Reprocess reusable products according to the instruction.

a. Compules® Tips Gun

For reprocessing instructions please refer to the Directions for Use of the Compules® Tips Gun, which is available on our webpage at www.dentsplysirona.com and www.dentsply.eu/IFU.

b. Compula® Tips – cross-contamination**Cross-contamination.**

Infection.

- Do not reuse Compula® Tips. Dispose of Compula® Tips in accordance with local regulations.

c. Syringes – cross-contamination**Cross-contamination.**

Infection.

- The syringes cannot be reprocessed.
- To prevent the syringes from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the syringes are handled with clean/disinfected gloves. Do not reuse syringes if contaminated.
- Dispose of contaminated syringes in accordance with local regulations.

To prevent syringes from exposure to spatter or spray of body fluids or contaminated hands, or oral tissues, use of a protective barrier is recommended. The use of protective barriers is an additional precautionary measure against gross debris but not against all contamination.

Incidental contact of the syringe with water, soap or a water-based hospital-level disinfection solution will not damage syringe body. Do not allow any solution contact with contained material. Discard composite material that has been in contact with any fluid or non-sterile instrument.

Repeated liquid contact may damage label. Dry the syringe with a lint-free single use cloth.

NOTE: Vigorous wiping can destroy the label. Wipe syringe gently.

d. Shade guide and individual shade guide tabs – cross-contamination

**Cross-contamination.**

Infection.

- The shade guide and the individual shade guide tabs cannot be reprocessed.
- To prevent the shade guide and the individual shade guide tabs from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the shade guide and the individual shade guide tabs are handled with clean/disinfected gloves. Do not reuse shade guide and the individual shade guide tabs if contaminated.
- Dispose of contaminated shade guide and the individual shade guide tabs in accordance with local regulations.

15 INFORMATION ON POTENTIAL USERS OF THE MEDICAL DEVICE AND CONDITIONS OF USE

The field of application is dentistry.


Application conditions

"Neo Spectra ST flow kits for restoration of anterior and posterior teeth in versions" can only be used in dental clinics, clinics, hospitals and laboratories by specialized personnel who have been trained and have sufficient experience in dentistry.


16 DESCRIPTION OF MAIN FUNCTIONAL ELEMENTS, PARTS AND ACCESSORIES OF THE MEDICAL DEVICE

Table 1 - Description of the main functional elements, components and accessories

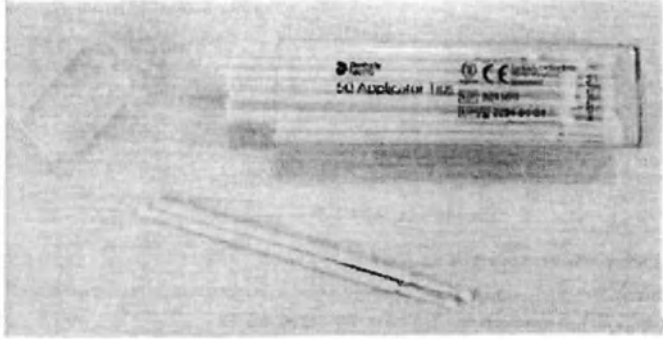Neo Spectra ST flow Kits for anterior and posterior teeth restoration in design variants	
Light-cured flowable dental restorative material Neo Spectra ST flow in syringes 1.8 g each 	Dimensions: Length of syringe, mm: $102,1 \pm 5\%$ Width, mm: $26 \pm 5\%$ Height, mm: $26 \pm 5\%$ Recoverable volume, g: $1,8 \pm 5\%$ Piston length, mm: $66,6 \pm 5\%$ Weight, g: $5,0 \pm 5\%$
Light-cured flowable dental restorative material Neo Spectra ST flow Compula® tips 0.25 g each 	Dimensions: Outer diameter of the tip, mm: $1,0 \pm 1\%$ Flange diameter, mm: $8,0 \pm 1\%$ Height of the compula, mm: $20,5 \pm 1\%$ Outer diameter of Compula® Tips, mm: $5,60 \pm 1\%$ Weight, g: $0,8 \pm 1\%$
Prime&Bond® universal adhesive in bottle, 2.5 ml	Dimensions: Height, mm: $39.6 \pm 5\%$ Diameter, mm: $20.7 \pm 1\%$ Volume, ml: $2.5 \pm 1\%$ Weight, g: $2.3 \pm 1\%$ Purpose: Provides easy adhesive application for direct and indirect

	restorations and adheres to enamel, dentin, composites, zirconia and metals
CliXdish mixing well 	Dimensions: Length, mm: $41,3 \pm 5\%$ Width, mm: $26,0 \pm 5\%$ Weight, g: $7,2 \pm 1\%$ Purpose: Mixing well / adhesive protection
Applicator for syringe 	Dimensions: Inner diameter, mm: $1 \pm 1\%$ Connected part diameter, mm: $7,6 \pm 1\%$ Weight, g: $19 \pm 1\%$ Designed for introducing material into the tooth cavity.
Applicators 	Dimensions: Length, mm: $84 \pm 5 \%$ Diameter, mm: $3.0 \pm 1 \%$ Weight, g: $0.28 \pm 1 \%$ Purpose: designed for economical and convenient application of adhesive to the tooth surface

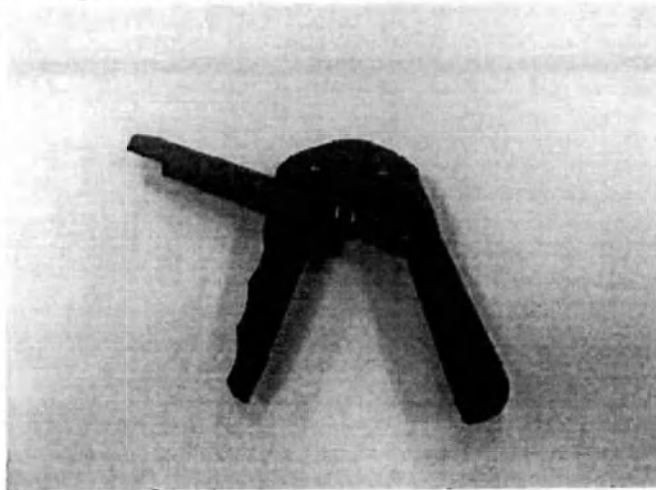Compules® Tips Gun

Dimensions:

Length, mm: $99.0 \pm 5\%$

Width, mm: $157.0 \pm 5\%$

Weight, g: $48.0 \pm 1\%$

Purpose: designed to work with material in unidose/compules

Illustrated technical guide

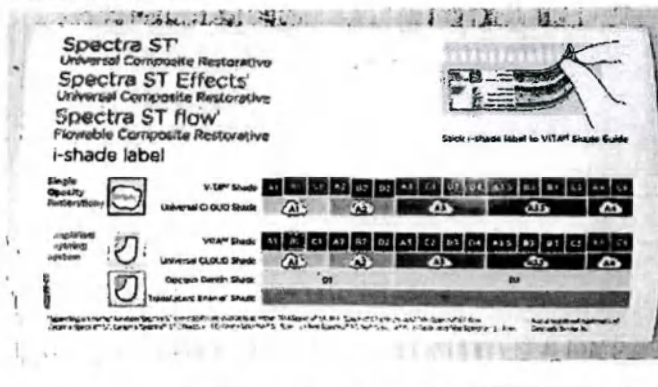
Dimensions:

Length, mm: $149 \pm 5\%$

Width, mm: $80 \pm 5\%$

i-Shape label

Dimensions:

Length, mm: $149 \pm 5\%$

Width, mm: $80 \pm 5\%$

Order form for Neo Spectra ST Shade Guide
(if necessary)

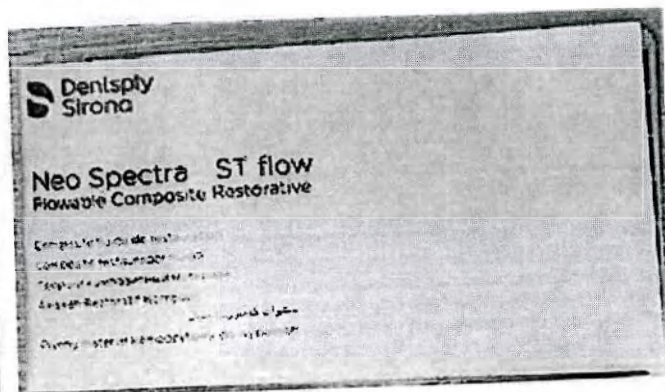
Dimensions:

Length, mm: 170± 5%

Width, mm: 100± 5%

Instruction for use on paper and/or in
electronic form

Dimensions:

Length, mm: 150± 5%

Width, mm: 80± 5%

Information leaflet (if necessary)

Dimensions:

Length, mm: 149± 5%

Width, mm: 55± 5%

17 TECHNICAL CHARACTERISTICS

Table 2.

Characteristic	Neo Spectra ST flow, shade: A1	Neo Spectra ST flow, shade: A2	Neo Spectra ST flow, shade: A3	Neo Spectra ST flow, shade: A3,5	Neo Spectra ST flow, shade: A4
Appearance	Colored paste without impurities	Colored paste without impurities	Colored paste without impurities	Colored paste without impurities	Colored paste without impurities
Sensitivity to ambient light, s	≥60	≥60	≥60	≥60	≥60
Cure time	With power ≥550 mW/cm ² .-20 sec. With power ≥800 mW/cm ² .-10 sec.	With power ≥550 mW/cm ² .-20 sec. With power ≥800 mW/cm ² .-10 sec.	With power ≥550 mW/cm ² .-20 sec. With power ≥800 mW/cm ² .-10 sec.	With power ≥550 mW/cm ² .-20 sec. With power ≥800 mW/cm ² .-10 sec.	With power ≥550 mW/cm ² .-20 sec. With power ≥800 mW/cm ² .-10 sec.
Transparency coefficient, %	41 ± 2	41 ± 2	41 ± 2	41 ± 2	41 ± 2
Water absorption, µg / mm ³ , no more than	13,3	13,3	13,3	13,3	13,3
Water solubility, µg / mm ³ , no more than	1,3	1,3	1,3	1,3	1,3
Viscosity, Pa·s	55-130	55-130	55-130	55-130	55-130
Elastic modulus, kPa	0,7-1,5	0,7-1,5	0,7-1,5	0,7-1,5	0,7-1,5
Depth of cure for 10 s, mm.	≥1,5	≥1,5	≥1,5	≥1,5	≥1,5
Flexural strength, MPa	≥80	≥80	≥80	≥80	≥80
Radiopacity	should be equal to or greater than the radiopacity of an aluminum sample of the same thickness and not less than 0.2 mm lower than the value declared by the manufacturer	should be equal to or greater than the radiopacity of an aluminum sample of the same thickness and not less than 0.2 mm lower than the value declared by the manufacturer	should be equal to or greater than the radiopacity of an aluminum sample of the same thickness and not less than 0.2 mm lower than the value declared by the manufacturer	should be equal to or greater than the radiopacity of an aluminum sample of the same thickness and not less than 0.2 mm lower than the value declared by the manufacturer	should be equal to or greater than the radiopacity of an aluminum sample of the same thickness and not less than 0.2 mm lower than the value declared by the manufacturer

18 LIST AND DESCRIPTION OF MEDICAL DEVICE MATERIALS FOR DIRECT OR SUBMITTED CONTACT WITH THE PATIENT'S BODY (HUMAN BODY)

Chemical composition according to the shade

EOBPADMA; Urethane resin EBPADMA; 1,12-Dodecanediol-dimethacrylate; Trimethylpropane trimethacrylate; Camphorquinone (CQ); Ethyl 4 (dimethylamino) benzoate (DMABE); BHT (butylhydroxytoluene stabilizer); 2-hydroxy-4-methoxybenzophenone (UV – 9); Lumilux (fluorescent substance); Fluoride-containing barium-aluminum and borosilicate glass, unsilanized; Fluoride-containing barium-aluminum and borosilicate glass, silanized (Ceram.XX 0.4 granulated); Amorphous highly dispersed silicon dioxide, silanized surface; Amorphous silica with a surface treated with polydimethylsiloxane; Ytterbium fluoride; Barium-aluminum and borosilicate glass, silanized; Titanium dioxide; Iron oxide black pigment; Iron oxide yellow powdery; Iron oxide red.

Atmosphere pressure	from 70 kPa to 106 kPa
---------------------	------------------------

Medical devices are regularly operated inside the oral cavity, if there are biological fluids in it at nominal temperatures: from + 32°C to + 42°C, for accessories that are not used in the oral cavity: from + 10°C to + 35°C.

Table 11 - Storage conditions.

Parameter	Values
Temperature	from +2°C to +28 °C
Humidity	no more than 70%
Atmosphere pressure	from 70 kPa to 106 kPa

If the device is stored in a refrigerator, and as after its transportation at subzero temperatures, it is necessary to allow warming it up to room temperature before use.

During the entire shelf life, the device in the manufacturer's packaging should be stored in a covered, dry, ventilated, heated warehouse, protected from light. Do not freeze.

Improper storage conditions will shorten the service life and may damage the device.

- Do not expose the product to sunlight and store in a well-ventilated place at temperatures between +2°C and +28°C.

- The composite material should be at room temperature by the time of use.

- Protect against water penetration.

- Do not use beyond the expiration date.

Products should be transported by all types of transport in covered vehicles in accordance with the transport rules in force for this type of transport.

Boxes with medical devices should be placed and fixed in vehicles so as to ensure their stable position, excluding the possibility of displacement of boxes and their impacts against each other and against the walls of vehicles.

Table 12 - Transportation conditions

Parameter	Values
Temperature	from +5°C to +30°C
Humidity	no more than 70%
Atmosphere pressure	from 70 kPa to 106 kPa

Medical devices must be transported in accordance with the rules applicable for each mode of transport at temperatures between +5°C and +30°C and relative humidity <70%.

When transported to countries with unacceptable ambient temperatures, products are transported in special containers that maintain the required temperature range and relative humidity.

20 LABELLING

20.1 Consumer packaging labelling

Consumer packaging labelling of Neo Spectra ST Flow dental materials for anterior and posterior teeth restoration in design variants contains:

- Name and/or model and/or type of medical device;
- Manufacturer's name and address;
- Do not reuse;
- Batch code;
- Catalog number;
- Temperature Range;
- Avoid exposure to sunlight;
- Carefully! Refer to instructions for use;
- Refer to the instructions for use;
- The product complies with the requirements of the European Directive on Medical Devices 93/42 / EEC;
- Use by dental practitioners only.

Table 13 - Symbols used on labelling:

Symbol / model	Meaning
	Manufacturer
	Batch code
	Number according to the manufacturer's catalog
	Carefully! Refer to instructions for use
	Refer to Instructions for use/ Instruction for use in electronic form
	The product complies with the requirements of the European Directive on Medical Devices 93/42 / EEC
	Do not reuse

	Temperature Range
	Use by dental practitioners only
	Avoid exposure to sunlight
	Keep away from moisture
	Symbol of the trade mark "Green dot"
	Use by....
	Medical device
	QR-code

Labelling of consumer packaging of applicator for syringe contains:

- Name and/or model and/or type of medical device;
- Manufacturer's name and address;
- Do not reuse;
- The product complies with the requirements of the European Directive on Medical Devices 93/42 / EEC;
- Batch code;
- Use by.....

Table 14 - Symbols used on labelling:

Symbol / model	Meaning
	Manufacturer
	The product complies with the requirements of the European Directive on Medical Devices 93/42/EEC

	Do not reuse
	Use by....
	Batch code

20.2 Syringe labelling

Syringe labelling contains:

- Batch code;
- Use by...;
- The product complies with the requirements of the European Directive on Medical Devices 93/42/EEC.

Table 15 - Symbols used on labelling:

Symbol / model	Meaning
	Batch code
	Use by....
	The product complies with the requirements of the European Directive on Medical Devices 93/42/EEC

20.3 Labelling of the individual package for compules

Labelling of the individual package for compules contains:

- The product complies with the requirements of the European Directive on Medical Devices 93/42/EEC;
- Do not reuse.

Table 26 – Symbols used on labelling:

Symbol / model	Meaning
	The product complies with the requirements of the European Directive on Medical Devices 93/42/EEC

Do not reuse.

20.4 Labelling of the transporting package

Labelling of the transporting package contains:

- Name of the company

21 PROCEDURE AND CONDITIONS FOR THE DISPOSAL OF THE MEDICAL DEVICE

Dispose the product in accordance with SanPiN 2.1.3684-21 for class B waste (epidemiologically hazardous waste). After hardware disinfection methods using physical methods and changing the appearance of the waste, excluding the possibility of their reuse, class B waste can accumulate, temporarily stored, transported, destroyed and disposed of together with class A waste (epidemiologically safe waste, in composition close to solid household waste). The packaging of disinfected medical waste of class B must have a label indicating that the waste has been disinfected.

22 MANUFACTURER'S WARRANTY

The manufacturer guarantees the safety of the medical device, as well as all their parameters and characteristics, during the entire shelf life - subject to the integrity of the individual packaging and compliance with the conditions of storage, transportation and use in accordance with the instructions for use.

23 EXPIRATION DATE OF THE MEDICAL DEVICE

Expiration date of kits:

1. Neo Spectra ST flow Syringe Discovery Kit – 2 years;
2. Neo Spectra ST flow Compula® Discovery Kit - 2 years;
3. Neo Spectra ST flow Compula® Intro Kit – 2 years;
4. Neo Spectra ST flow Syringe Intro Kit – 2 years;

The shelf life of a medical device is 2 years and is counted from the date of manufacture of the composite material itself, and not from the date of its manufacture (packaging), encrypted in the first four digits of the batch code (LOT).

The date of manufacture (packaging) of a medical device is the date of completion of the last stage of production of a medical device - packaging.

Therefore, the expiration date indicated for a medical device is usually longer than the period of time between the date of manufacture and the expiration date indicated on the packaging of the medical device.

Expiration of Prime&Bond® universal adhesive in bottle - 2 years.

Expiration for Compules® Tips Gun – 5 years.

Expiration of CliXdish plastic mixing well – 5 years.

Expiration of applicators – 5 years.

Expiration of applicator for syringe – 5 years.

24 INFORMATION ABOUT DESINFECTION, CLEANING AND STERILIZATION OF THE MEDICAL DEVICE

The medical device is not subject to cleaning or disinfection. The medical device is not sterile or sterilizable.

Refer to the Compules® Tips Gun Instructions for Use at www.dentsplysirona.com and www.dentsply.eu/IFU for cleaning or disinfection instructions.

25 ENVIRONMENTAL REQUIREMENTS

This medical device, when used, transported and stored, does not have a negative impact on humans and the environment.

26 RECLAMATION

Organization authorized by the manufacturer to accept claims related to product quality on the territory of the Russian Federation:

Dentsply Sirona Limited Liability Company (Dentsply Sirona, LLC)

Address: premises 3N, bld.2, Ovchinnikovskaya embankment, 18/1, ZamoscvoRechie municipal district, intra-city area, Moscow, 115035

Tel.: +7 (495) 725-10-87

E-mail: MoscowRus-Reception@dentsplysirona.com

ANNEX 1 i-SHADE

i-shade

Spectra ST
Universal Composite Restorative

Spectra ST Effects
Universal Composite Restorative

Spectra ST flow
Flowable Composite Restorative

i-shade label

Stick i-shade label to VITA® Shade Guide

Single Opacity Restorations

Simplified Layering System

VITA® Shade

A1	D1	C1	A2	B2	D2	A3	C2	D3	D4	A3.5	B3	B4	C3	A4	C4
Universal CLOUD Shade															
VITA® Shade															
A1	D1	C1	A2	B2	D2	A3	C2	D3	D4	A3.5	B3	B4	C3	A4	C4
Universal CLOUD Shade															
Opaque Dentin Shade															
D1															
D3															
Translucent Enamel Shade															

Depending on market location Spectra ST composites are available as either TPH Spectra® ST, TPH Spectra® ST Effects, and TPH Spectra® ST flow - Caramx Spectra® ST, Caramx Spectra® ST Effects, and Caramx Spectra® ST flow - or Neo Spectra® ST, Neo Spectra® ST Effects, and Neo Spectra® ST flow.

* Not a registered trademark of Dentsply Sirona, Inc.

ANNEX 2 ILLUSTRATED TECHNICAL GUIDE

Spectra ST flow[®] Flowable Composite Restorative

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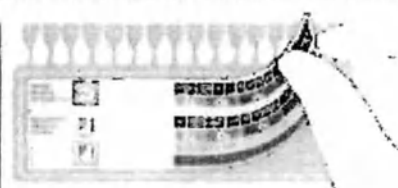
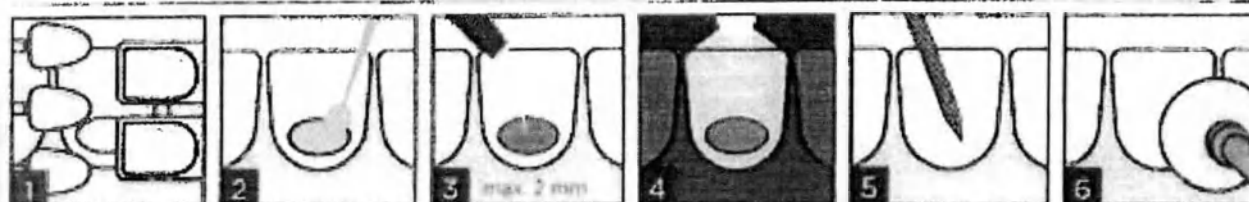
Spectra ST flow[®] Flowable Composite Restorative

Shade Selection

Use the VITA Shade Guide and shade label on the Spectra ST flow fingers

Conditioning and/or bonding

After shade selection apply adhesive according to the respective manufacturer's directions for use

Placement of Restorative

Dispense material in up to 2 mm increments using slow steady pressure. Avoid lifting the tip while dispensing to minimize air entrapment

Light curing

Cure each 2 mm increment with a minimum irradiance of 150 mW/cm². Refer to curing chart for appropriate curing time for each shade.

Shaping and contouring

With finishing diamond or carbide burs

Finishing and polishing

Shape and contour finish and polish (e.g. with Enhance Finishing System and Enhance FoGu[®] Polishing System)

Representative of Spectra ST flow composite restorative. Spectra ST flow is a trademark of Dentsply Sirona. Spectra ST flow is a registered trademark of Dentsply Sirona. © 2018 Dentsply Sirona, Inc. www.dentsplysirona.com

Caution: consult directions for use

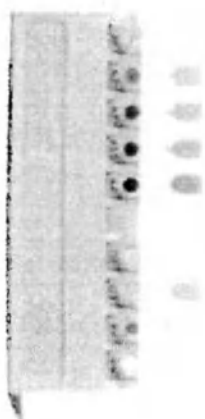
ANNEX 3 ORDER FORM FOR NEO SPECTRA ST SHADE GUIDE

Easy and reliable shade match

The Neo Spectra ST shade guide is made of a special material that allows you to match the color of your teeth with the color of the composite restoration.

Neo Spectra™ ST Composites

Neo Spectra™ ST Composites

Order your free shade guide

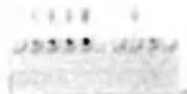
Order your free Neo Spectra™ ST shade guide

online www.dentsplysirona.com/freespectrastshadeguide

per fax +49 (0)7531 583-660116

per mail explored@dentsplysirona.com

☐ Yes, I want to order
☒ a free shade guide

Practice stamp

Affix
stamp
here

Dentsply DeTrey GmbH
Marketing
De-Trey Str. 1
78467 Konstanz
GERMANY

ANNEX 4 THE INFORMATION LEAFLET

Try the complete
Neo Spectra™ ST
portfolio for efficient esthetics.

Learn more ►

Try the complete
Neo Spectra™ ST
portfolio for efficient esthetics.

Learn more ►

PREFERRED HANDLING AND SIMPLIFIED ESTHETICS FOR EVERY COMPOSITE CASE.

From common Class II restorations to esthetically demanding anterior cases, there's a Neo Spectra™ ST composite solution.

Our portfolio offers a complete, universal handling and placement system for all types of Class II and Class III restorations. The Neo Spectra™ ST composite portfolio simplifies the process of achieving efficient esthetics in every form of composite restorations. There's a Neo Spectra™ ST composite that has you covered.

The key advantages offered by the Neo Spectra™ ST composite portfolio are made possible by the unique design of our proprietary SphereTEC® filler technology.

Optimized handling for
easy placement

Enhanced chameleon blending
ability for simplified shading

Excellent durability, polish, and stain
resistance for lasting esthetics

Neo Spectra™ ST Effects

Universal Class II Composite Restorations

COMPLEX ANTERIOR CASES SIMPLIFIED

Universal Class II Composite Restorations

Neo Spectra™ ST flow

Flowable composite

ESTHETIC FLOWABLE VENEER LITE, MADE EFFICIENT

- Flowable composite
- Flowable composite
- Flowable composite
- Flowable composite
- Flowable composite

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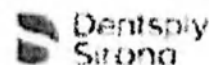
TO LEARN MORE ABOUT THE
NEO SPECTRA™ ST PORTFOLIO, VISIT
www.dentsplysirona.com/NeoSpectraST-composites

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
© 2018 Dentsply Sirona, Inc.
www.dentsplysirona.com

K102135-00

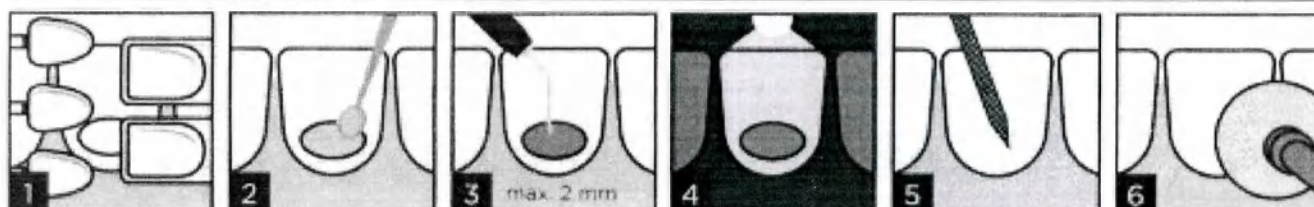Spectra ST flow¹ Flowable Composite Restorative

Shade Selection

Use the VITA[®] Shade Guide and i-shade label or the Spectra ST shade fingers.

Conditioning and/or bonding

After shade selection apply adhesive according to the respective manufacturer's directions for use.

Placement of Restorative

Dispense material in up to 2 mm increments using slow, steady pressure. Avoid lifting the tip while dispensing to minimize air entrapment.

Light curing

Cure each 2 mm increment with a minimum irradiance of 550 mW/cm². Reference curing chart for appropriate curing time for each shade.

Shaping and contouring

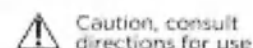
With finishing diamond or carbide burs.

Finishing and polishing

Shape and contour, finish and polish (e.g. with Enhance[®] Finishing System and Enhance[®] PoGo[®] Polishing System).

¹ Depending on market location, Spectra ST flow is available as either TPH Spectra[®] ST flow, Ceramix Spectra[™] ST flow, or Neo Spectra[™] ST flow.

² Not a registered trademark of Dentsply Sirona, Inc.

Caution, consult directions for use

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