More than

75
Years

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**Important note for users of
KARL STORZ devices and
instruments**

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It is recommended to check the suitability of the products for the planned intervention prior to use.

Thank you for your expression of confidence in the KARL STORZ brand name. Like all of our other products, this product is the result of years of experience and great care in manufacture. You and your organization have decided in favor of a modern, high-quality item of equipment from KARL STORZ.

This instruction manual is intended to serve as an aid in the proper handling, cleaning, disinfection and sterilization of the Video-Uretero-Renoscope FLEX-X[©]. All essential details of the equipment and all actions required on your part are clearly presented and explained. We thus ask that you read this manual carefully before proceeding to work with the instrument. Keep this manual available for ready reference.

WARNING: KARL STORZ instruments are delivered unsterilized, and must be cleaned, disinfected and/or sterilized prior to the initial use and before each subsequent use.

KARL STORZ is constantly working on the further development of all products. Please appreciate that changes to the scope of supply in form, equipment and technology are possible for this reason. Therefore, no claims may be derived from the information, figures and descriptions in this manual.

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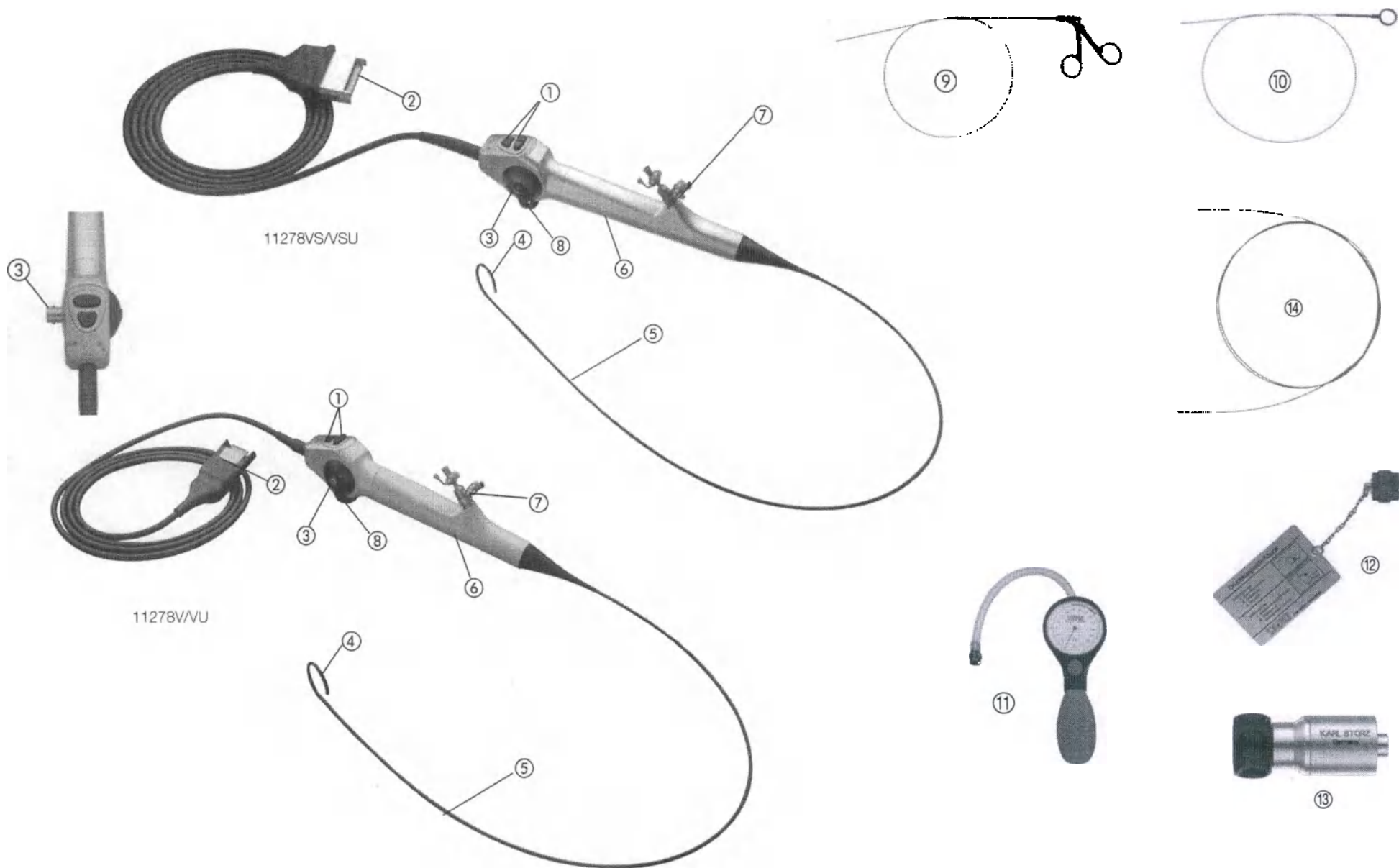
Optionales Zubehör:

- 11275 FE
- 11275 ZE
- 110931-50

- ① Control buttons
- ② Supply connector
- ③ Connection for leakage tester or pressure compensation cap
- ④ Distal tip
- ⑤ Sheath
- ⑥ Handpiece
- ⑦ Working channel with LUER-Lock connectors
- ⑧ Deflection lever
- ⑨ Biopsy forceps 27023 ZE
- ⑩ Cleaning brush 27651 AL
- ⑪ Leakage tester 13242 XL
- ⑫ Pressure compensation cap 11025 E
- ⑬ LUER adaptor
- ⑭ Single-use cleaning brush 110931-50

11278 VSK/VSUK comprises:

- 11025 E Pressure compensation cap
- 13242 XL Leakage tester
- 27651 AL Cleaning brush
- 27014 Y LUER adaptor, with seal

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Important note for users of KARL STORZ devices and instruments	I
Pictures of the instruments.....	II
Controls, displays, connectors and their uses.....	III
Safety information	3
Intended Use.....	3
Indications	4
Contraindications.....	5
User qualification	5
Safety precautions when using the Videoendoscopes	6
Warnings and cautions	6
Handling	10
Unpacking the device	10
Basic equipment	10
Operating conditions	10
Electromagnetic Compatibility.....	10
Handling	11
Operating	11
Holding the Videoendoscope correctly	11
Handling the sheath	11
Handling the deflection mechanism	12
Handling the accessories (forceps, cleaning brushes, HF electrodes etc.) ..	14
Compatible camera control units	15
Connection to the camera control unit	16
Control buttons	17
Preparation prior to use.....	18
Inspecting the surfaces.....	18
Checking the deflection	19
Inspecting the LED function.....	20
Inspecting the camera control unit.....	20
Inspecting the instruments and the instrument channel	21

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Cleaning, disinfection, care and sterilization.....	22
Explanation of Warnings and Cautions.....	22
General warnings	23
Accessories	25
Disassembly	25
Precleaning directly after use.....	25
Leakage test.....	26
Performing the leakage test – dry.....	26
Performing the leakage test – in liquid.....	26
Manual cleaning	27
Manual disinfection.....	28
Machine cleaning and disinfection	28
Manual preparation.....	29
Machine cleaning/chemical-thermal disinfection	29
Assembly, inspection and care.....	30
Sterilization	31
Maintenance	36
Servicing and repair.....	36
Maintenance.....	36
Limitation of liability.....	36
Warranty.....	37
Directive compliance	37
Important information	38
Disposal	38
Appendix	39
Troubleshooting	39
Subsidiaries.....	42

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Please read these safety instructions carefully before using the instrument. Before using the instrument on the patient it is imperative that you be acquainted with how the instrument operates and is controlled.

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Intended Use

11278V/VS/VU/VSU

Flexible uretero-renaloscopes (fiber endoscopes and videoscopes) are used for endoscopic imaging of the renal collecting system for the purposes of diagnosis and treatment. Flexible uretero-renaloscopes with an integrated working channel allow for the use of flexible auxiliary instruments such as forceps, stone baskets etc., coagulation electrodes and laser fibers for lithotripsy or for the ablation of benign/malignant tissue. The flexible uretero-renaloscope can be used for the imaging/treatment of all renal calices, as well as the pyeloureteral junction or the ureter. The special feature of the flexible uretero-renaloscope lies in the fact that it can also be used percutaneously via an operating sheath, and can be used in antegrade mode to treat all renal calices as well as the pyeloureteral junction or the ureter.

Flexible uretero-renaloscopes are designed for short-term use in endoscopic interventions performed through natural orifices or surgical invasive interventions.

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Indications

11278 V/VS/VU/VSU:

The use of ureterscopes/uretero-renoscopes is indicated if uretero(reno)scopy is indicated in the opinion of the attending physician.

This includes:

- Further diagnostic measures in the event of conspicuous imaging/filling defects in the upper urinary tract shown in the imaging
- Pathological bladder-wash cytology without reference to bladder tumors or from the upper urinary tract
- Unclear haematuria (macro/microhaematuria)
- Diagnosis and treatment of urinary stones in the ureter or renal collecting system.
- Diagnosis and treatment of ureteral strictures or tumors
- Iatrogenically introduced foreign material
- Narrow pyeloureteral junction
- Possibly the percutaneous antegrade treatment of kidney stones, tumors and ureter outflow constrictions.
- UUT (Upper Urinary Tract) tumors or UTUC (Upper Tract Urothelial Cancer)
- Nephrolithiasis of lower calyceal group
- Positive bladder-wash cytology from the upper urinary tract
- Tumors in the upper urinary tract
- Indication retrograde treatment of a transitional cell carcinoma
 - impaired renal function/single kidney
 - bilateral tumor development
 - multiple concomitant diseases/limited life expectancy
 - Tumor stages in which only palliation is possible

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Contraindications**11278V/VS/VU/VSU:**

The use of uretero(reno)scopes is contraindicated if, in the opinion of the attending physician, the surgical method as such is contraindicated or if the patient is not able to undergo surgery or anesthesia due to his or her general condition.

Uretero(reno)scopes must not be used for interventions in direct contact with the central nervous system (CNS) and central cardiovascular system.

The following contraindications also apply:

- Urinary tract infection
- Urosepsis
- Untreated/unresolved coagulation disorder
- Contraindication for lithotomy position
- Pregnancy

User qualification

The KARL STORZ Videoendoscopes may only be used by persons with an appropriate medical qualification and who are acquainted with the technique of uretero-renaloscopy. The information given in these instructions only serves to instruct in the correct handling and reprocessing of the Videoendoscopes.

It is not intended as an introduction to the technique of renaloscopy.

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Safety precautions when using the Videoendoscopes

The Video-Uretero-Renoscopes FLEX-X[®] must be used according to the recognized medical rules and procedures of endoscopy.

NOTE: The patient leakage current limit of the CF applied parts may be exceeded if CF applied parts (application on heart) are used in combination with other BF applied parts.

In this case, the patient leakage current should be reduced by using only CF applied parts as far this is possible.

NOTE: The video endoscope does not offer any electric insulation against high frequency (HF) currents. Protection against high frequency currents should be ensured through the use of correct HF instruments. Please read the instructions for use in the instruction leaflet supplied with the HF applied part for more information.

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Warnings and cautions

Please read this manual carefully and follow the instructions exactly. The words **Warning**, **Caution**, and **Note** convey special meanings. Wherever they are used in this manual, they should be carefully reviewed to ensure the safe and effective operation of this product. To further emphasize the words **Warning** and **Caution**, they are preceded by a pictogram.

WARNING: A Warning indicates that the personal safety of the patient or physician may be involved. Disregarding a Warning could result in injury to the patient or physician.

CAUTION: A Caution indicates that particular service procedures or safety precautions must be followed to avoid any damage to the product.

NOTE: A Note indicates special information about operating the product, or clarifies important information.

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WARNING: Read the following safety instructions carefully before using the Videoendoscope to avoid putting your patients, personnel or yourself at risk.

WARNING: Inspect the videoendoscope prior to each surgical procedure to ensure that it functions correctly and that it has been correctly cleaned, disinfected and/or sterilized.

WARNING: Every time the Videoendoscope is used, you must first check that it, and any accessories used in combination with it, is in perfect condition. Damaged Videoendoscopes or damaged accessories must not be used.

WARNING: Combinations of medical devices are only then assured to be safe if

- they are identified as such in the respective instruction manual or
- the intended purpose and the interface specifications of the devices used in combination permit this.

Pay careful attention to the instructions and interface specifications of medical devices used in combination.

WARNING: The Videoendoscope is delivered unsterilized and must be cleaned and disinfected/sterilized before the initial use as well as before any further application.

WARNING: The **pressure compensation cap** must be **removed** from the vent port for:

- endoscopic interventions
- manual/machine reprocessing

WARNING: Only use accessories (guide probes, cleaning brushes etc.) that are specifically intended for use with Videoendoscopes and can be operated in line with generally accepted technical standards and occupational safety guidelines.

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WARNING: Do not allow any live components or instruments to come into contact with the Videoendoscope.

WARNING: The Videoendoscope must not be used during the discharge of a cardiac defibrillator. Remove the Videoendoscope from the operation area before discharge takes place.

WARNING: Danger of explosion if flammable gases are used in the immediate proximity of the Videoendoscope. The Videoendoscope must not be used in the presence of flammable or combustible media.

WARNING: As a result of the light emitted by the endoscope, the tip of the latter may heat up.

Prolonged direct contact of light-emitting surfaces on tissue can lead to burns, and must therefore be avoided at all costs.

WARNING: The **pressure compensation cap** must be **in place** on the vent port for:

- hydrogen peroxide gas sterilization
- venting
- shipping

WARNING: The instructions 'Cleaning, Disinfection, Care and Sterilization of KARL STORZ Instruments' must be followed. There the procedures for cleaning, disinfection and sterilization are explained in detail.

WARNING: The Videoendoscope may only be repaired by personnel authorized by KARL STORZ. Unauthorized repairs may impair functioning and safety, and are therefore strictly forbidden. KARL STORZ gives no warranty for Videoendoscopes that are repaired by unauthorized personnel.

WARNING: Damage to the Videoendoscope and injury to the patient or user resulting from incorrect operation are not covered by the manufacturer's warranty.

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WARNING: Always unplug the Videoendoscope before carrying out any maintenance and cleaning work on it.

WARNING: Risk of infection during shipment! Contaminated Videoendoscopes may cause serious or life-threatening illnesses during transportation or on delivery to the recipient. Defective Videoendoscopes with attached pressure compensation cap must be wiped down with disinfectant and, if necessary, sterilized before shipment. Complete the repairs form.

WARNING: If a malfunction should occur during application on a patient, stop the application immediately. Place the distal tip of the Videoendoscope in the neutral, nondeflected position and remove the video endoscope slowly and carefully from the patient.

CAUTION: Do not expose the Videoendoscopes to direct sunlight or excessive heat, to avoid damage to the device.

CAUTION: Do not steam sterilize (autoclave) the Videoendoscopes.

CAUTION: The maximum permissible temperature for decontamination, sterilization, transport and storage is 65 °C.

CAUTION: Do not hit the distal tip of the Videoendoscope against hard objects, and never use a sharp object to remove dirt.

CAUTION: Never lay heavy objects on the Videoendoscope.

NOTE: Constant changing between various reprocessing methods is especially hard on the materials and must therefore be avoided. However, if the reprocessing method is changed just once this can be regarded as harmless.

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Electromagnetic Compatibility

This instrument has been tested and complies with the EMC limits as per EN/IEC 60601-1-2: 2001 CISPR 11 Class B.

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Handling

! **NOTE:** Damage to the Videoendoscope resulting from incorrect operation is not covered by the manufacturer's warranty.

! **WARNING:** The Videoendoscope is delivered unsterilized, so it has to be cleaned, disinfected/sterilized prior to initial use and before each subsequent reuse.

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Operating

The following instructions outline the correct operation of the Videoendoscope and the corresponding accessories. This is not an introduction to the techniques of renoscopy. Instructions on endoscopic technique can be found in the relevant medical literature.

Suitable accessories for your Video-uretero-Renoscope FLEX-X^c, such as biopsy forceps, can be found in the UROLOGY catalog or extracts from this catalog.

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Holding the Videoendoscope correctly

Grip the handle of the Videoendoscope in your left or right hand, as shown in the picture. With this kind of control grip, the Videoendoscope can be operated with one hand.

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Handling the sheath

! **CAUTION:** Do not carry the Videoendoscope by the sheath. Do not pull, clamp or twist the sheath.

! **CAUTION:** The sheaths of the Videoendoscope must never be kinked or coiled up too tightly. It is best to suspend the Videoendoscopes for storage.

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WARNING: Do not apply lubricant to the lens, to avoid clouding the image and thereby prevent risks for the patient!

Use a suitable water-soluble lubricant for the distal section of the insertion sheath. Observe the lumen the whole time during use. If you lose your orientation, pull the sheath back slightly in the neutral position until vision is restored.

If, during use, the image fails, is disturbed, or becomes blurred, or if vision in general is impeded, put the deflection mechanism into the neutral position and slowly pull the sheath of the Videoendoscope out of the patient's body.

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Handling the deflection mechanism

NOTE: The neutral position of the distal tip is straight, without any deflection whatsoever.

WARNING: Suitable accessories, such as laser fibers and biopsy forceps, must only be introduced into the instrument channel of the Videoendoscope with the distal tip in the neutral, non-deflected position. Once the instrument has been introduced, actuate the deflection lever gently and carefully as far as the limit stop. The deflection lever must never be operated with force or abruptly to avoid injuring the patient or damaging the device!

WARNING: Before removing the Videoendoscope from the patient, the distal tip of the Videoendoscope must be put into the neutral position, as otherwise injuries to the patient or damage to the instrument may occur.

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WARNING: If the distal tip can no longer be properly controlled (sticking etc.) via the deflection lever, or if the deflection mechanism develops any other fault, the Videoendoscope must not be used.

WARNING: If a functional defect occurs during use on the patient, discontinue use immediately. Put the distal tip of the Videoendoscope into the neutral, non-deflected position and remove the Videoendoscope slowly and carefully from the patient.

WARNING: The lens at the distal end of the Videoendoscope may become detached, e.g. due to mechanical forces. For this reason, it is essential to check that the lens is undamaged and present before and after every cleaning.

CAUTION: The resistance that can be felt when deflecting the tip using the deflection lever increases as the angle of deflection increases. Actuate the deflection gently and carefully, without any sudden or jerky movements, as far as the limit stop.

CAUTION: Do not use force to turn the deflection lever, to avoid either injuring the patient or damaging the instrument.

The distal tip can be deflected 270° upward or downward by continuously actuating the deflection lever slowly and carefully. Before use, the user must always have full control over the deflection and must develop a sure feel for the deflection mechanism and in particular for the neutral position.

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Handling the accessories (forceps, cleaning brushes, HF electrodes etc.)

! **CAUTION:** Only use flexible accessories (forceps, cleaning brushes etc.) from KARL STORZ. Accessories from other manufacturers may damage the Videoendoscope.

! **CAUTION:** Only introduce the instruments from the proximal port of the Videoendoscope.

! **WARNING:** When HF electrodes or laser probes are used, it must be ensured that power is not applied until the active parts have emerged from the instrument channel and are in the user's field of vision, and the site of the operation is clearly recognizable. Activation in the instrument channel may lead to serious injury to the patient and damage to the Videoendoscope.

! **WARNING:** When HF electrodes or laser probes are used, it must be ensured that power is only applied when there are no flammable gases in the operating field or in the proximity of the Videoendoscope.

! **CAUTION:** When laser probes are used, it must be ensured that the laser beam cannot hit the distal tip of the Videoendoscope.

! **CAUTION:** Do not force accessories, such as biopsy forceps, through the working channel, to avoid damaging the device. It should be possible to move the accessory easily, without any hindrance and with closed jaws through the entire channel.

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- !** **CAUTION:** When the distal tip is deflected, do not introduce any instrument into the Videoendoscope. Return the distal tip to the neutral position before introducing the instrument and then move the deflection lever gently and carefully to the limit stop.
- !** **CAUTION:** Inserting sharp-tipped accessories into the instrument channel may damage the Videoendoscope.
- !** **CAUTION:** Foreign bodies or parts to be extracted must not be removed through the instrument channel using instruments. Instead, the entire Videoendoscope must be removed from the patient and the foreign body extracted.

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! **WARNING:** The KARL STORZ Videoendoscope should not be used in combination with camera control units from other manufacturers. The use of non-compatible units may cause injury to the patient or damage to the instrument. It must be ensured that parts of the endoscope that are not electrically insulated do not come into contact with conductive surfaces or live parts of other units, at any time.

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Connection to the camera control unit

 **CAUTION:** The Videoendoscope can be used with the above mentioned camera control units. The instruction manual of the camera control unit must be followed. The complete operating instructions for the camera control unit can be found in the instruction manual.

 **WARNING:** Risk of burns and glare! Never look directly into the light emitted by the Videoendoscope. The high-intensity light emitted by the Videoendoscope at the distal light outlet can cause glare or high temperatures and thereby result in burns.

 **WARNING:** Risk of burns and fire hazard! Never place the distal tip of the Videoendoscope on or under a sterile drape. The intensity of the light can cause burns to the patient and/or the sterile drape.

 **WARNING:** Reduce the light intensity to the extent that optimal illumination of the operation field is still ensured. The higher the intensity of the light, the greater the heat emitted at the tip of the Videoendoscope.

 **WARNING:** The keyboard of the IMAGE1 S must not come into contact with patients or be placed in the vicinity of patients.

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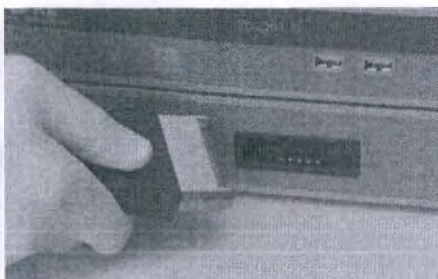
CAUTION: Take care with the Videoendoscope connections! The Videoendoscope connections can be damaged if connected incorrectly. Make sure that the connector is correctly aligned.

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
1. Install the camera control unit on a firm, level surface.
2. Plug the power cord into the specified power socket.
3. Plug the Videoendoscope supply cable into the HD socket of the camera control unit.
4. Activate the image and illumination function (integrated LED) by switching on the camera control unit.

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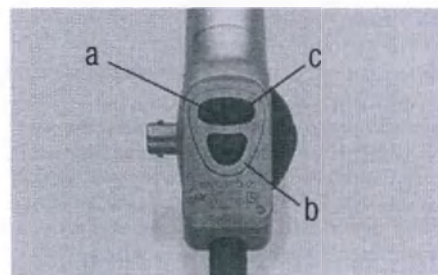Control buttons

The Videoendoscope has three control buttons which can be assigned specific functions in the camera control unit's device menu.

- a Scroll up: Scrolls up the options in the menu.
- b Menu/Select: Displays the live menu. As soon as this menu is displayed, this button activates the highlighted option (identical to ENTER on a computer keyboard).
- c Scroll down: Scrolls down the options in the menu.

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The assignment is retained after switching off and on again.
Instructions on programming the control keys can be found in the instruction manual of the camera control unit.

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Preparation prior to use

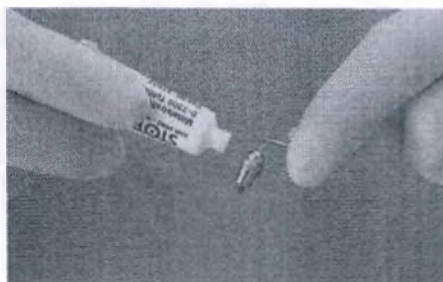
NOTE: Before the device is used for the first time, it must be subjected to a complete reprocessing cycle (excluding preliminary cleaning) in accordance with the risk classification.

WARNING: Every time the Videoendoscope is used, you must first check that it, and any accessories used in combination with it, are in perfect condition. Damaged Videoendoscopes or damaged accessories must not be used.

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Lightly grease the running surfaces of the cock plug (special grease for stopcocks; Art. no. 27657).

NOTE: This thin layer of grease not only serves to lubricate the metal surfaces which are in contact with one another, but also makes the stopcocks leakproof. Oil is not suitable as a sealant.

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Inspecting the surfaces

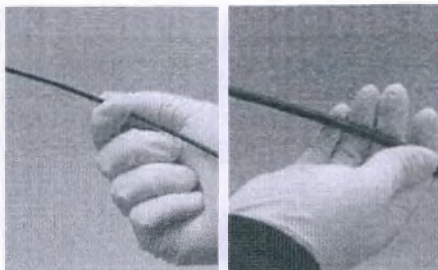
WARNING: The surface of the flexible sheath must be inspected visually and by carefully feeling it, to detect any cracks, cuts, dents, foreign bodies or other forms of damage.

CAUTION: Do not squeeze the distal tip of the Videoendoscope too tightly.

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Every time the Videoendoscope is used, you must first check that its surface, and above all the surface of the sheath and distal tip, is in perfect condition and that there are no sharp edges. Videoendoscopes with sharp edges must not be used. Otherwise serious injuries may be caused to the patient or the user.

Inspect the supply cable for damage (such as breaks, cracks, twisted or squashed sections...) and the supply connector for loose connections.

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Checking the deflection

WARNING: Actuate the deflection lever gently and carefully as far as the limit stop. The deflection lever must never be operated with force or abruptly, to avoid injuring the patient or damaging the device! Move the deflection lever smoothly and slowly!

WARNING: Before use, always check that the deflection lever and the distal tip connected to it function correctly. Defective Videoendoscopes must not be used.

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The distal tip is deflected by means of the deflection lever. Turn the deflection lever slowly; the end piece should move downward. Turn the deflection lever in the opposite direction to produce an upward movement.

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Inspecting the LED function

Prepare the IMAGE1 S as described above and connect to the Videoendoscope. Emission of light at the distal end should be recognizable. Light intensity is automatically regulated by the camera control unit.

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Inspecting the camera control unit

WARNING: If the image fails or there is interference during use in the patient, put the distal tip into the neutral position and remove the Videoendoscope from the patient's body gently and carefully. If the Videoendoscope is used in the patient after the image has failed or when there is interference, this may result in serious injury to the patient.

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Prepare the camera control unit as described above and connect to the Videoendoscope.

CAUTION: The camera control unit must be switched off before connecting the Videoendoscope!

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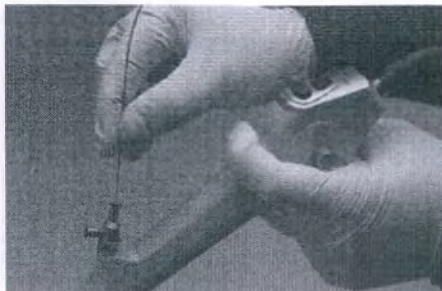
The control buttons can be individually programmed.

Programming of the control buttons is described in the instructions for the camera control unit.

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Inspecting the instruments and the instrument channel

CAUTION: It should be possible to move the forceps with closed jaws easily and without any hindrance through the entire channel.

CAUTION: Accessories, such as biopsy forceps, which are bent or twisted may no longer function properly, are difficult to pass through the channel, and may cause serious damage to the channel. They must therefore be replaced!

Check the instrument channel by introducing the appropriate instrument (max. diameter of instruments: 3 Fr.).

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**Explanation of Warnings and
Cautions**

WARNING: A Warning indicates a potential hazard. Failure to observe a Warning may result in injury to the patient, user or third parties.

CAUTION: A Caution indicates that particular procedures or precautions must be followed to avoid possible damage to the medical devices.

NOTE: A Note contains special information regarding reprocessing or important explanations.

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General warnings

 **WARNING:** Risk of infection: Incorrectly reprocessed medical devices expose patients, users and third parties to a risk of infection as well as the risk that the medical device may malfunction. Observe the manual 'Cleaning, Disinfection, Care and Sterilization of KARL STORZ Instruments' and the accompanying documentation.

 **WARNING:** Risk of infection: These medical devices are not sterile when delivered. The use of non-sterile medical devices poses a risk of infection for patients, users and third parties. Inspect medical devices for visible contamination. Visible contamination is an indication that reprocessing has not been carried out or has been carried out incorrectly. Reprocess the medical devices before initial use and before and after every subsequent use using validated procedures.

 **WARNING:** When carrying out any work on contaminated medical devices, the guidelines of the Employers' Liability Insurance Association and equivalent organizations striving to ensure personal safety must be observed.

 **WARNING:** Before reusing the device, replace any defective parts or send the flexible endoscope for repair.

 **CAUTION:** When preparing and using solutions, follow the chemical manufacturer's instructions, paying close attention to proper concentration, exposure time and service life. Prolonged immersion and incorrect concentration may result in damage. Bear in mind the microbiological range of action of the chemicals used.

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- !** **CAUTION:** Danger of damage to medical devices: The use of chemicals that have not been approved by KARL STORZ or chemical manufacturers may cause damage to the medical devices. Only use chemicals approved for the medical device for reprocessing. A helpful guide is available on the Internet at www.karlstorz.com.
- !** **CAUTION:** National laws and regulations must be observed.
- !** **CAUTION:** The pressure compensation cap must be in place on the vent port for sterilization and transport to ensure pressure compensation.
- !** **CAUTION:** Never clean flexible endoscopes with metal brushes.
- !** **CAUTION:** Constant changing between various reprocessing methods is especially hard on the materials and must therefore be avoided. However, a one-off change of method, due to procurement of new cleaning equipment, for example, is possible.
- !** **CAUTION:** Flexible endoscopes must never be steam sterilized (autoclaved).
- !** **CAUTION:** To prevent damage to the flexible endoscope and accessories, it must be ensured that all connections are completely dry prior to use.
- i** **NOTE:** The manual 'Cleaning, Disinfection, Care, and Sterilization of KARL STORZ Instruments' (Art. no. 96216003) can be downloaded or requested by visiting www.karlstorz.com.
- i** **NOTE:** The valves must be removed in their individual parts prior to cleaning and disinfection.

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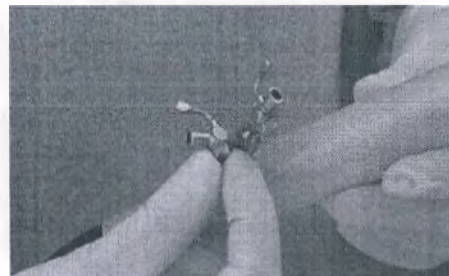
Accessories

Accessories required for carrying out reprocessing:

Accessories: pressure compensation cap, manual leakage tester

Place of application	Art. number brush/adaptor
Inlet and outlet nozzle	27650 B, 39107 ALK
Working channel	27651 AL
Clean surface	27652
Ventilation connector	13242 XA, 11025 E

Disassembly

The valves must be removed in their individual parts prior to cleaning and disinfection.

Precleaning directly after use

Heavy soiling, corrosive solutions and pharmaceuticals must be removed from the endoscope immediately after use.

Wipe-clean the exterior surfaces of the endoscope with a damp cloth.

Rinse out the working channel in order to prevent drying.

NOTE: To avoid damaging the sheath, do not use pushing and pulling movements.

NOTE: Do not use disinfectants for precleaning; these cause fixation of proteins and organic residues.

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Leakage test

The endoscope should be tested to ensure it is leak-proof after use and before reprocessing.

Performing the leakage test – dry

The connection tube of the leakage tester and the vent port on the endoscope must be dry. Connect the leakage tester connector securely to the supply connector vent port and lock by turning the bayonet catch 25° clockwise.

NOTE: The rubber seal in the connection adaptor of the leakage tester should be treated regularly with instrument oil.

Inflate the leakage tester until the maximum test pressure of 160–180 mmHg (blue area) is reached. The deflection rubber on the distal end may expand slightly. Using the deflection mechanism on the control body, move the deflection part in all possible directions as far as possible and observe the pointer on the manometer. The test pressure can drop by up to 10 mmHg until the endoscope is completely filled with air.

After stabilizing the pressure on the manometer, wait at least 2 minutes to see whether the pressure remains constant or drops.

Performing the leakage test – in liquid

NOTE: To avoid damage from the penetration of liquid, a test in the dry state must be performed in advance.

NOTE: Any lumina must be covered with no air bubbles by filling in a targeted manner.

Immerse the device in solution so that it is completely covered. It must be ensured that the leakage tester does not become wet. Perform the leakage test as described in 'Leakage test – dry' and note whether bubbles emerge along the device.

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If, after an observation period of at least 3 minutes, no bubbles can be seen emerging (continuously) from the device and, in addition, the manometer does not show a fall in pressure, the device is leak-proof.

Remove the endoscope from the liquid and press the vent button on the manometer. The test pressure must drop to 0 before the leakage tester can be removed.

NOTE: Never release the pressure in the liquid.

Cleaning can then begin.

If a pressure loss occurs, this means that the endoscope leaks and may no longer be used, immersed in liquid or reprocessed by machine.

CAUTION: In this case, fill out the accompanying repairs form, mark the endoscope, which has been cleaned by wiping to remove organic soiling/residue, as not disinfected and return it to the Service Department for repair or, alternatively, contact KARL STORZ. Line the storage/carrying case with the transport protection sheet 13990 SFN.

Manual cleaning

The endoscope must be completely immersed in a cleaning solution. The working channel must be covered with no air bubbles by filling in a targeted manner. After the necessary exposure time as specified by the manufacturer, clean the instrument mechanically with the aid of a suitable brush. The channels are brushed until the brush head emerges and is visibly clean. Before retracting the brush head that emerges at the distal end, clean it. Never brush back and forth within the channel. Finally, it must be rinsed with cold water to ensure neutralization.

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CAUTION: Never remove dirt with a needle or other sharp objects, since these can cause severe damage.

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Manual disinfection

The endoscope must be completely immersed in a cleaning solution. The working and suction channel must be covered with no air bubbles by filling in a targeted manner. At the end of the exposure time as specified by the manufacturer, the endoscope must be rinsed several times in order to remove all chemical residues. Sterile filtered water must be used for the final rinsing.

Drying

Finally, the surfaces and channels are dried completely with (preferably sterile) compressed air in accordance with the national regulations. The cleaning gun with accessories (Art. no. 27660) is ideal for this purpose.

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Machine cleaning and disinfection

The following methods for machine decontamination have been validated and approved subject to compliance with the process parameters described in the manual, 'Cleaning, Disinfection, Care and Sterilization of KARL STORZ Instruments' (Art. no. 96216003).

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Manual preparation

The endoscope must be completely immersed in a cleaning solution. The working and suction channel must be covered with no air bubbles by filling in a targeted manner. After the necessary exposure time as specified by the manufacturer, clean the instrument mechanically with the aid of a suitable brush. The channels are brushed until the brush head emerges and is visibly clean. Before retracting the brush head that emerges at the distal end, clean it. Never brush back and forth within the channel.

Finally, it must be rinsed with cold water to ensure neutralization.

Connecting

In order to ensure effective machine cleaning and disinfection, the endoscope's working and suction channel must be connected to the washer and disinfectant using the recommended adaptors.

CAUTION: The pressure compensation cap must not be in place on the vent port for cleaning and disinfection.

Machine cleaning/chemical-thermal disinfection

A maximum temperature of 65°C must not be exceeded. The relevant national requirements and the chemical manufacturer's instructions must be taken into account when using this method. The selection of a suitable endoscope holder, which should ensure that the medical device is thoroughly rinsed out or through, must take place in consultation with the manufacturer of the device.

NOTE: If necessary, the instrument must be dried off manually afterwards.

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Assembly, inspection and care

The cleaned and disinfected medical device must be visually inspected for cleanliness, completeness, damage and dryness:

- If residues or contamination are still present, the medical device must be manually cleaned and subjected to a full cleaning and disinfection procedure once more.

NOTE: Clean the distal lens with a cotton tip applicator soaked in lens cleaner or 70% isopropyl alcohol. When cleaning the distal lens, always wipe with a circular movement.

- Damaged or corroded medical devices must be withdrawn from use.

NOTE: Inspect plastic parts for discoloration, porosity and flexibility. If the parts have changed substantially from their original condition, they must be replaced. This especially applies to seals and rubber caps.

- If there is still residual moisture on or in the endoscope, it must be dried again manually. The channels must be completely dried using (preferably sterile) compressed air in accordance with the national regulations. The cleaning gun with accessories (Art. no. 27660) is ideal for this purpose.

CAUTION: The pressure must not exceed 100 kPa (1 bar / 14.5 psi). It may therefore be necessary to operate the cleaning gun with a pressure reducer.

- Afterwards, a functional check must be carried out.

NOTE: Check for a clear view and steerability.

NOTE: During care procedures, use items from the catalog 'Care, Sterilization and Storage Techniques'.

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Sterilization

The procedures as well as the process-relevant parameters for the individually validated methods are described in detail in the manual 'Cleaning, Disinfection, Care and Sterilization of KARL STORZ Instruments' (Art. no. 96216003). The method must be selected taking into account the respective applicable national requirements and in consultation with the device and product manufacturers.

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CAUTION: Flexible endoscopes must never be steam sterilized (autoclaved).

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NOTE: All deviations from the recommended sterilization parameters must be validated by the user.

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The following sterilization procedures have been validated and approved by KARL STORZ for this medical device:

**Hydrogen peroxide (H₂O₂)
sterilization - ASP STERRAD®**

 **WARNING:** Medical devices must be dismantled for sterilization.

 **WARNING:** Before sterilization, all parts of the endoscope must be thoroughly cleaned, disinfected, rinsed and then dried as otherwise sterility cannot be achieved.

 **WARNING:** Please note that there are restrictions concerning the instruments which can be sterilized in the different STERRAD® sterilization systems in respect of lumen dimensions and material.

 **CAUTION:** For detailed information, please consult the user handbook of the respective device.

 **CAUTION:** The pressure compensation cap must be in place on the vent port for sterilization and transport to ensure pressure compensation.

 **NOTE:** Refer to the 'STERRAD® Sterility Guide' to ensure that the medical device concerned can be sterilized using the different STERRAD® devices.

 **NOTE:** Sterilization is not possible on surfaces which have been lubricated and oiled.

The following STERRAD® sterilization methods have been validated and approved for this medical device by KARL STORZ:

- STERRAD® 100S Long Cycle with Boosters
- STERRAD® NX® Advanced Cycle
- STERRAD® 100NX® Flex Cycle
- STERRAD® 100NX® DUO Cycle

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**Hydrogen peroxide (H₂O₂)
sterilization - STERIS® AMSCO®V-
PROTM 1**

WARNING: Medical devices must be dismantled for sterilization.

WARNING: Before sterilization, all parts of the endoscope must be thoroughly cleaned, disinfected, rinsed and then dried as otherwise sterility cannot be achieved.

CAUTION: For detailed information, please consult the user handbook of the respective device.

CAUTION: The pressure compensation cap must be in place on the vent port for sterilization and transport to ensure pressure compensation.

NOTE: The 'STERIS Low Temperature Reprocessing Device Compatibility Matrix' can be used to verify whether the medical device can be sterilized using the various STERIS V-PRO® devices.

NOTE: Sterilization is not possible on surfaces which have been lubricated and oiled.

The following STERIS V-PRO® sterilization methods have been validated and approved for this medical device by KARL STORZ:

- V-PRO® maX Flexible Cycle
- V-PRO® 60 Flexible Cycle

Ethylene oxide sterilization (EO)

The ethylene oxide procedure is validated with 100% ethylene oxide at 55°C and a hold time of 30-45 minutes.

WARNING: Before sterilization, all parts of the endoscope must be thoroughly cleaned, disinfected, rinsed and then dried as otherwise sterility cannot be achieved.

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CAUTION: For gas sterilization with ethylene oxide, the airing times stipulated by the device manufacturer must be observed for instruments, due to gas absorption of the materials. The airing times depend on the processing technology of the EO sterilizer (concentration, process control).

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Reprocessing of flexible accessories (Forceps, scissors etc)

Separate reprocessing instructions are available for the preparation of flexible accessories. These can be downloaded from www.karlstorz.com.

Limits of reprocessing

The end of the product's service life is largely determined by wear, reprocessing methods, the chemicals used and any damage resulting from use.

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Temperatur: 0 °C – 55 °C
Luftfeuchtigkeit: 15 % – 90 %

Storage

The storage method must be selected taking into account the respective applicable national requirements.

KARL STORZ only recommends dust-protected, suspended storage for decontaminated endoscopes.

Horizontal storage is possible for sterilized endoscopes. KARL STORZ recommends the use of a specially designed tray for sterilization and storage.

CAUTION: Do not expose flexible endoscopes to direct sunlight or excessive heat.

CAUTION: The distal tip of the flexible endoscope must be in the neutral position during storage.

CAUTION: Do not store the flexible endoscope in the case.

CAUTION: The sheath of the flexible endoscope must never be kinked or coiled up too tightly.

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Servicing and repair

Defective items of equipment must be serviced and repaired exclusively by persons authorized by us; all repair work must employ original parts.

Maintenance

Preventive maintenance is not essential. Regular maintenance can, however, contribute to identifying potential problems before they become serious, enhancing the instrument's reliability and extending its useful operating life.

Maintenance services can be obtained from your local representative or from the manufacturer. Regardless of the accident prevention regulations or testing intervals for medical devices prescribed in different countries, we recommend a functional or safety test of the unit at least once a year.

Limitation of liability

KARL STORZ SE & Co. KG shall be liable for failure or deterioration in the safe operation, operational reliability, and performance of this equipment only subject to the conditions that all assembly, operations, system expansions, readjustments, modifications, or repairs to same have been performed by KARL STORZ, and that the instrument has been used in accordance with its operating instructions at all times.

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Warranty

The guarantees provided can be found in the Standard Conditions of Business of KARL STORZ.

The medical device must always be sent to your local subsidiary (see "Subsidiaries" section), even during the warranty period.

Opening the equipment or performance of any repairs or modifications to the equipment by unauthorized persons shall relieve us of any liability for its performance. Any such opening, repair, or modification performed during the warranty period shall void all warranty.

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Directive compliance

This medical device bears the CE mark in accordance with the Medical Device Directive (MDD) 93/42/EEC. A code number after the CE mark indicates the responsible notified body.

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Important information

To avoid the spread of infectious illnesses among hospital staff or among KARL STORZ employees, telescopes, instruments and equipment must be cleaned and sterilized/disinfected before they are sent for repair. We reserve the right to return contaminated instruments/equipment to the sender.

Repairs, changes or expansions which are not carried out by KARL STORZ or by experts authorized by KARL STORZ will invalidate all warranty rights.

KARL STORZ gives no warranty on the correct functioning of equipment or instruments which have been repaired by unauthorized third parties.

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Disposal

To dispose of Videoendoscopes, no special measures are necessary. National laws and regulations must be observed.

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KARL STORZ Endoscopy Canada Ltd.
7171 Millcreek Drive, Mississauga, Ontario
L5N 3R3 Canada
Phone: +1 905 816-4500, Fax: +1 905 816-4599
Toll free phone: 1-800-268-4880 (Canada only)
Toll free fax: 1-800-482-4198 (Canada only)
E-Mail: info-canada@karlstorz.com

KARL STORZ Endoscopy -America, Inc.
2151 East Grand Avenue
El Segundo, CA 90245-5017, USA
Phone: +1 424 218-8100, Fax: +1 424 218-8525
Toll free phone: 800 421-0837 (USA only)
Toll free fax: 800 321-1304 (USA only)
E-Mail: communications@kseas.com

KARL STORZ Veterinary Endoscopy-America, Inc.
1 South Los Cameros Road
Goleta, CA 93117, USA
Phone: +1 805 968-7776, Fax: +1 805 685-2588
E-Mail: info@karlstorzvet.com

KARL STORZ Endoscopia Latino-America, Inc.
815 N. W. 57th Avenue, Suite 480
Miami, FL 33126-2042, USA
Phone: +1 305 262-8980, Fax: +1 305 262-8986
E-Mail: info@kseas.com

KARL STORZ Endoscopia México S.A. de C.V.
Edificio Atlantic, Oficina 3G
Calle D e/ 1ra y 3ra
10400 Vedado, Havana, Cuba
Phone: +537 836 95 06, Fax: +537 836 97 76
E-Mail: kstorzcuiba@gmail.com

KARL STORZ Endoscopia México S.A. de C.V.
Av. Ejercito Nacional No. 453 Piso 2,
Colonia Granada, Alcaldia Miguel Hidalgo
C.P. 11520 Ciudad de México
Phone: +52 (55) 1101 1520
E-Mail: mx-info@karlstorz.com

KARL STORZ Marketing América Do Sul Ltda.
Rua Joaquim Floriano, nº. 413, 20º andar – Itaim Bibi,
CEP-04534-011 São Paulo, Brasil
Phone: +55 11 3526-4600, Fax: +55 11 3526-4680
E-Mail: br-info@karlstorz.com

KARL STORZ Endoscopia Argentina S.A.
Zufriategui 627 6º Piso, B1638 CAA – Vicente Lopez
Provincia de Buenos Aires, Argentina
Phone: +54 11 4718 0919, Fax: +54 11 4718 2773
E-Mail: info@karlstorz.com.ar

KARL STORZ Endoskopi Norge AS
Stamveien 1
1483 Hagan, Norway
Phone: +47 6380 5600, Fax: +47 6380 5601
E-Mail: post@karlstorz.no

KARL STORZ Endoskop Sverige AB
Storsåtragränd 14
127 39 Skärholmen, Sweden
Phone: +46 8 505 648 00
E-Mail: kundservice@karlstorz.se

KARL STORZ Endoscopy Suomi Oy
Taivaltie 5
01610 Vantaa, Finland
Phone: +358 (0)96824774, Fax: +358 (0)96824775
E-Mail: asiakaspalvelu@karlstorz.fi

KARL STORZ SE & Co. KG
Representation Office
Kestutis st. 59 / Lenktolij st. 27
08124 Vilnius, Lithuania
Phone: +370 5 272 0448
Mobile: +370 685 67 000
E-Mail: info-lt-lv@karlstorz.com

KARL STORZ Endoskopi Danmark A/S
Skovlydten 33
2840 Holte, Denmark
Phone: +45 45162600, Fax: +45 45162609
E-Mail: marketing@karlstorz.dk

KARL STORZ Endoscopy (UK) Ltd.
415 Perth Avenue, Slough
Berkshire, SL1 4TQ, United Kingdom
Phone: +44 1753 503500, Fax: +44 1753 578124
E-Mail: info-uk@karlstorz.com

KARL STORZ Endoscopia Nederland B. V.
Displayweg 2
3821 BT Amersfoort, Netherlands
Phone: +31 (0)33 4545890
E-Mail: info-nl@karlstorz.com

KARL STORZ Endoscopy Belgium N. V.
Phone: +31 (0)33 4545890
E-Mail: info-be@karlstorz.com

KARL STORZ Endoscopia France S. A. S.
12, rue Georges Guynemer, Quartier de l'Europe
78280 Guyancourt, France
Phone: +33 1 30484200, Fax: +33 1 30484201
E-Mail: marketing-fr@karlstorz.com

KARL STORZ Endoskop Austria GmbH
Landstraßer Hauptstr. 148/1/G1
1030 Wien, Austria
Phone: +43 1 71 56 0470, Fax: +43 1 71 56 0479
E-Mail: storz-austria@karlstorz.com

KARL STORZ Endoscopia Iberica S. A.
Parque Empresarial San Fernando
Edificio Munich – Planta Baja
28830 Madrid, Spain
Phone: +34 91 6771051, Fax: +34 91 6772981
E-Mail: info-es@karlstorz.com

KARL STORZ Endoscopia Italia S. r. l.
Via dell'Artigianato, 3
37135 Verona, Italy
Phone: +39 045 8222000, Fax: +39 045 8222001
E-Mail: info-ita@karlstorz.com

KARL STORZ Croatia d.o.o.
Capraška 6
10000 Zagreb, Croatia
Phone: +385 1 6406 070, Fax: +385 1 6406 077
E-Mail: info@karlstorz.hr

KARL STORZ Endoskopija d.o.o.
Cesta v Gorice 34b
1000 Ljubljana, Slovenia
Phone: +386 1 620 5880, Fax: +386 1 620 5882
E-Mail: pisarna@karlstorz.si

KARL STORZ Polska Sp. z o.o.
ul. Bojkowska 47
44-100 Gliwice, Poland
Phone: +48 32 706 13 00, Fax: +48 32 706 13 07
E-Mail: info-pl@karlstorz.com

KARL STORZ Endoskop Magyarország Kft.
Toberek utca 2. fsz. 17/b
HU-1112 Budapest, Hungary
Phone: +36 195 096 31, Fax: +36 195 096 31
E-Mail: info-hu@karlstorz.com

KARL STORZ Endoscopia Romania srl
Str. Prof. Dr. Anton Colorian, nr. 74, Sector 4
041393 Bukarest, Romania
Phone: +40 (0)31 4250800, Fax: +40 (0)31 4250801
E-Mail: info-ro@karlstorz.com

KARL STORZ Endoscopy Greece M.E.P.E.*
Patriarchou Grigoriou E' 34
54248 Thessaloniki, Greece
Phone: +30 2310 304868, Fax: +30 2310 304862
E-Mail: info-gr@karlstorz.com

*Repair & Service Subsidiary

KARL STORZ Industrial**
Gedik Is Merkezi B Blok
Kat 5, D 38-39, Bagdat Cad. No: 162
Maltepe Istanbul, Turkey
Phone: +90 216 442 9500, Fax: +90 216 442 9030

**Sales for Industrial Endoscopy

000 KARL STORZ Endoscopy – WOSTOK
Derbenyevskaya nab. 7, building 4
115114 Moscow, Russia
Phone: +7 495 983 02 40, Fax: +7 495 983 02 41
E-Mail: info-ru@karlstorz.com

TOV LLC KARL STORZ Ukraine
Avenue Geroyiv Stalingrada Str. 2D, office 717
Kyiv, 04210/Ukraine
Phone: +38 095 000-895-0, +38-097-000-895-0,
+38 073 000-895-0
E-Mail: marketing@karlstorz.com.ua

KARL STORZ SE & Co. KG
Representation Office
Sabit Orudschow 1184, apt. 23
1025 Baku, Azerbaijan
Phone: +99 450 613 30 60
E-Mail: info-az@karlstorz.com

KARL STORZ ENDOSKOPE –
East Mediterranean and Gulf (Offshore) S.A.L.
Spark Tower 1st floor
Charles Helou St., Horch Tabet – Sin El Fil
Beirut, Lebanon
Phone: +961 1 501105, Fax: +961 1 501950
E-Mail: info@karlstorz-emg.com

KARL STORZ Endoscopy (South Africa) (Pty) Ltd.
P.O. 6061, Roggebaai 8012
Cape Town, South Africa
Phone: +27 21 417 2600, Fax: +27 21 421 5103
E-Mail: info@karlstorz.co.za

TOO KARL STORZ Endoscopy Kasachstan
Saryarka, 6, BC "Arman", off. 910
010000 Astana, Republic of Kazakhstan
Phone: +7 7172 552-549, 552-788, Fax: -444
E-Mail: info@karlstorz.kz

KARL STORZ ENDOSKOPE
East Mediterranean & Gulf (branch)
Building West Side 7A – Unit 7WA – 3008
Dubai Airport Free Zone, P.O. Box 54983
Dubai – United Arab Emirates
Phone: +971 (0)4 2958887, Fax: +971 (0)4 3205282
Service Hotline: +971 (0)4 3415882
E-Mail: info-gne@karlstorz-emg.com

KARL STORZ Endoscopy India Private Limited
11th Floor, Dr. Gopal Das Bhawan
28, Barakhamba Road
New Delhi 110001, India
Phone: +91 11 4374 3000, Fax: +91 11 4374 3010
E-Mail: infoindia@karlstorz.in

KARL STORZ SE & Co. KG
Interchange 21 Tower, Level 33
399 Sukhumvit Road,
North Klongtoey, Wattana,
10110 Bangkok, Thailand
Phone: +84 28 3823 8000
Fax: +84 28 3823 8039
E-Mail: infovietnam@karlstorz.com

KARL STORZ SE & Co. KG
Resident Representative Office
14th Floor, MPLaza Saigon
39 Le Duan, District 1
Ho Chi Minh City, Vietnam
Phone: +84 28 3823 8000, Fax: +84 28 3823 8039
E-Mail: infovietnam@karlstorz.com

KARL STORZ Endoscopy China Ltd.
Room 2503-05, 25F AXA Tower, Landmark East,
No. 100 How Ming Street, Kwun Tong, Kowloon,
Hong Kong, People's Republic of China
Phone: +852 28 65 2411, Fax: +852 28 65 4114
E-Mail: inquiry@karlstorz.com.hk

KARL STORZ Endoscopy (Shanghai) Ltd., Beijing Branch
Room 1805-1807, Building B, 18F Beijing IFC
No. 8, Jianguomenwai Street, Chaoyang District,
100022, Beijing, People's Republic of China
Phone: +86 10 56381188, Fax: +86 10 56381199
E-Mail: info@karlstorz.com.cn

KARL STORZ Endoscopy (Shanghai) Ltd., Shanghai Branch
Room 701A Building 5 & Room 501 Building 7,
No. 3000 Longdong Avenue, Pilot Free Trade Zone, 201203,
Shanghai, People's Republic of China
Phone: +86 21 60339888, Fax: +86 21 60339808
E-Mail: info@karlstorz.com.cn

KARL STORZ Endoscopy (Shanghai) Ltd., Chengdu Branch
Room 803-805, 8F Jin Jiang International Building
No. 1 West Linjiang Road, Wuhou District,
610044, Chengdu, People's Republic of China
Phone: +86 28 86587977, Fax: +86 28 86587975
E-Mail: info@karlstorz.com.cn

KARL STORZ Endoscopy (Shanghai) Ltd., Shenyang Branch
Room 2001-2005, 20F N-MEDIA International Center,
No. 167 Youth Avenue, Shenhe District,
110014,
g, People's Republic of China
Phone: + 3181118, Fax: +86 24 23181119
E-Mail: ir storz.com.cn

KARL STORZ Endoscopy (Shanghai) Ltd., Guangzhou Branch
Room 02B & 03 & 04A, 35F Teem Tower,
No. 208 Tianhe Road, Tianhe District,
510620, Guangzhou, People's Republic of China
Phone: +86 20 87321281, Fax: +86 20 87321286
E-Mail: info@karlstorz.com.cn

KARL STORZ Endoscopy Asia Marketing Pte Ltd.
No. 8 Commonwealth Lane #03-02
Singapore 149555, Singapore
Phone: +65 69229150, Fax: +65 69229155
E-Mail: infoasia@karlstorz.com

KARL STORZ Endoscopy Singapore Sales Pte Ltd
No. 8 Commonwealth Lane #03-02
Singapore 149555, Singapore
Phone: +65 69229150, Fax: +65 69229155
E-Mail: infoasia@karlstorz.com

KARL STORZ SE & Co. KG
Representative Office Indonesia
Sinarmas MSG Tower Level 37
Jl. Jend. Sudirman No. Kav. 21
Jakarta Selatan
DKJ Jakarta 12920
E-Mail: infoindonesia@karlstorz.com

KARL STORZ Endoscopy Korea Co. Ltd.
9F Hyowon-Building
97, Jungdae-ro, Songpa-gu
05719 Seoul, Korea
Phone: +82-70-4350-7474, Fax: +82-70-8277-3299
E-Mail: infokorea@karlstorz.com

KARL STORZ Endoscopy Taiwan Ltd.
12F, No. 192, Sec. 2, Chung Hsin Rd.,
Sindian District, New Taipei City, Taiwan
Phone: +886 933 014 160, Fax: +886 2 8672 6399
E-Mail: info-tw@karlstorz.com

KARL STORZ SE & Co. KG
Representative Office Philippines
1901 Picadilly Star Bldg., 4th Avenue, BGC
Taguig City 1636, Philippines
Phone: +63 2 317 45 00, Fax: +63 2 317 45 11
E-Mail: philippines@karlstorz.com

KARL STORZ Endoscopy Japan K. K.
Stage Bldg. 8F, 2-7-2 Fujimi
Chiyoda-ku, Tokyo 102-0071, Japan
Phone: +81 3 6380-8622, Fax: +81 3 6380-8633
E-Mail: info@karlstorz.co.jp

KARL STORZ Endoscopy Australia Pty. Ltd.
68 Waterloo Road, Macquarie Park NSW 2113
P O Box 50 Lane Cove NSW 1595, Australia
Phone: +61 (0)2 9490 6700
Toll free: 1800 996 562 (Australia only)
Fax: +61 (0)2 9420 0695
E-Mail: karlstorz@karlstorz.com.au
www.karlstorz.com

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ADDENDUM TO THE INSTRUCTION MANUAL OF THE MEDICAL DEVICE

«Flexible Flex-XC Video Uretherorenoscope in versions, with accessories »

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