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Tel. +49 61 22 58 0
Fax +49 61 22 58 1244

I, Zaman Khan, Associate Director Regulatory Affairs EMEA and Pakistan, Abbott GmbH & Co. KG., hereby certify that the attached documents that keep the number of pages indicated below are the operational documentation. These documents are in a full accordance with actual manuals of manufacturer.

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#	Title	Number of pages
1.	Instruction for use for Alinity с Carbon Dioxide Calibrator Kit with ADDENDUM for Russia.	4

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Signed: _____

Zaman Khan,
Associate Director Regulatory
Affairs EMEA and Pakistan,
Abbott GmbH & Co. KG.

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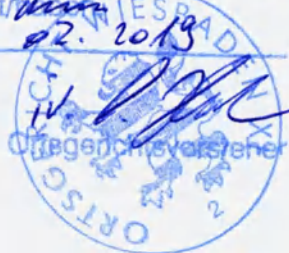
Read Highlighted Changes: Revised February 2018.

Package insert instructions must be carefully followed. Reliability of assay results cannot be guaranteed if there are any deviations from the instructions in this package insert.

NAME

Alinity c Carbon Dioxide Calibrator Kit (also referred to as CO2 Cals)

INTENDED USE

For use in the calibration of the Alinity c Carbon Dioxide assay on the Alinity c analyzer.

For additional information, refer to the Alinity c Carbon Dioxide reagent assay package insert and the Alinity ci-series Operations Manual.

CONTENTS

The **CAL 1-2** are prepared in an aqueous solution.

Preservative: sodium azide (0.10%).

Lot-specific calibrator values are listed in the Alinity c Carbon Dioxide Calibrator Kit value sheet, packaged with the calibrator. Verify that the lot number listed on each calibrator carton agrees with the lot number printed on the value sheet. The last 2 digits of the lot numbers can vary.

Calibrator	Quantity
CAL 1	3 x 3.0 mL
CAL 2	3 x 3.0 mL

MATERIALS REQUIRED BUT NOT PROVIDED

- 04R1001 Alinity ci-series Calibrator/Control Replacement Caps

STANDARDIZATION

The Alinity c Carbon Dioxide Calibrator uses sodium bicarbonate/sodium carbonate (A.C.S. > 99% pure) and has been standardized against an internal standard which is traceable to a National Institute of Standards and Technology Standard Reference Material (NIST SRM 351).

PRECAUTIONS

- IVD**
- For *In Vitro* Diagnostic Use
- Rx ONLY**

Safety Precautions

The following warnings and precautions apply to: CAL 1-2	
Contains sodium azide.	
EUH032	Contact with acids liberates very toxic gas.
P501	Dispose of contents / container in accordance with local regulations.

Safety Data Sheets are available at www.abbottdiagnostics.com or contact your local representative.

For a detailed discussion of safety precautions during system operation, refer to the Alinity ci-series Operations Manual, Section 8.

PREPARATION FOR USE

- This product is liquid ready-to-use.
- Prior to each use, mix by gentle inversion.

STORAGE

- Do not use past expiration date.

	Storage Temperature	Maximum Storage Time	Additional Storage Instructions
Unopened	2 to 8°C	Until expiration date	
Opened	2 to 8°C	30 days after opening	Store tightly capped with new replacement cap. Return to refrigerated storage after use.

The analyzer will track In-use Stability, which is the time the calibrator is outside of refrigerated storage while on the analyzer. The analyzer will not allow the use of the calibrator if the In-use Stability has been exceeded. Maximum In-use Stability can be found in the Assay Parameter Report. For additional information on calibrator In-use Stability, refer to the Alinity ci-series Operations Manual, Section 5.

For additional information on printing assay parameters, refer to the Alinity ci-series Operations Manual, Section 5.

INSTRUMENT PROCEDURE

- Calibrator values may be configured using e-files accessed and imported from www.abbottdiagnostics.com, or from Abbott Mail.
- Verify that the correct calibrator values have been entered into the calibration file.
- Calibration is performed by running a water blank and Alinity c Carbon Dioxide Calibrator Kit.
- Water for the blank is provided by the instrument.
- Verify control results are within acceptable limits before reporting patient results.
- For information on configuring calibrator data, refer to the Alinity ci-series Operations Manual, Section 2.
- For instructions on ordering and loading calibrators on the instrument, refer to the Alinity ci-series Operations Manual, Section 5.

INDICATIONS OF INSTABILITY OR DETERIORATION

Instability or deterioration should be suspected if there are visible signs of leakage, if calibration does not meet the appropriate package insert and/or Alinity ci-series Operations Manual criteria, or if controls do not meet the appropriate criteria.

LIMITATIONS OF THE PROCEDURE

Accurate and reproducible results are dependent upon properly functioning instruments and reagents, storage of product as directed, and good laboratory technique.

Notes

Note for number formatting:

- A space is used as thousands separator (example: 10 000 specimens).
- A period is used to separate the integer part from the fractional part of a number written in decimal form (example: 3.12%).

Key to Symbols

ISO 15223 Symbols

	Consult instructions for use
	Manufacturer
	Temperature limitation
	Use by/Expiration date
IVD	<i>In Vitro</i> Diagnostic Medical Device
LOT	Lot Number
REF	List Number
SN	Serial number

Other Symbols

ALTERNATE UNITS	Alternate Units
ASSAY NAME	Assay Name
ASSAY NO.	Assay Number
CAL 1	Calibrator 1
CAL 1-2	Calibrators 1 and 2
CAL 2	Calibrator 2
CN	Control Number
CONTAINS: AZIDE	Contains Sodium Azide. Contact with acids liberates very toxic gas.
DEFAULT UNITS	Default Units
DISTRIBUTED IN THE USA BY	Distributed in the USA by
INFORMATION FOR USA ONLY	Information needed for United States of America only
PRODUCED FOR ABBOTT BY	Produced for Abbott by
PRODUCT OF USA	Product of USA
Rx ONLY	For use by or on the order of a physician only (applicable to USA classification only).

Alinity is a trademark of Abbott Laboratories in various jurisdictions.
All other trademarks are property of their respective owners.

Abbott GmbH & Co. KG
Max-Planck-Ring 2
65205 Wiesbaden
Germany
+49-6122-580

PRODUCED FOR ABBOTT BY

Microgenics Corporation
46500 Kato Road
Fremont, CA 94538 USA

DISTRIBUTED IN THE USA BY

Abbott Laboratories
Abbott Park, IL 60064 USA

**Customer Service: Contact your local representative
or find country-specific contact information on
www.abbottdiagnostics.com**

Revised February 2018.

©2016, 2018 Abbott Laboratories

ADDENDUM

to the Instruction for Use Alinity c Carbon Dioxide Calibrator Kit for Russia

SHIPPING AND STORAGE CONDITIONS

Recommended shipping conditions for devices are 2-8°C. Medical devices must be stored in refrigerators/refrigerator chambers to ensure regulated temperature conditions with a daily temperature record.

Medical devices must be carried out in single-use thermal containers containing coolant elements, or in reusable thermo containers with automatically maintained temperature or in refrigerated trucks using thermal indicators.

Device is stable until the expiration date indicated on the package. Device is no longer suitable for use if shipping temperature limits have been violated. Ship in any kind of enclosed carrier vehicle in accordance with shipping rules, applicable to the respective type of transport.

WARRANTY

Manufacturer guarantees that device remains stable until the expiration date when stored and handled as directed. Manufacturer sets the maximum shelf life of

Alinity c Carbon Dioxide Calibrator Kit – 12 months from date of manufacturing.

Device must not be used if stored with deviations from manufacturer's requirements (GOST R 51088-2013).

MAINTENANCE

Not required.

REPAIRS

Not required.

LIST OF APPLICABLE STANDARDS

EN ISO 9001

ISO 13485:2003

ISO 14971 (equivalent to EN ISO 14971:2012)

EN ISO 18113-1

EN ISO 18113-2

EN ISO 18113-3

EN 13641

EN 980

EN ISO 15223-1

EN ISO 15223-2

ISO 80000-1

POTENTIAL USER

The Alinity c Carbon Dioxide Calibrator Kit is intended for use in a variety of hospital clinical laboratory environments including, but not limited to; hospitals and commercial laboratories, whose personnel have been trained to use the Alinity i analyzer.

INFORMATION ABOUT PRESENCE OF DRUGS IN COMPOSITION OF MEDICAL DEVICE

Does not contain any drugs

STERILIZATION

Medical device is not required to be sterile.

INFORMATION ABOUT POSSIBILITY OF MULTIPLE APPLICATION

Medical device cannot be re-applied

TECHNICAL CHARACTERISTICS

pH-value at 20 °C:

- Calibrator 1, Calibrator 2: 10

Density at 20 °C:

- Calibrator 1: 1,018 g/cm³
- Calibrator 2: 1,016 g/cm³

COMPATIBILITY RESTRICTIONS OF MEDICAL DEVICE

No available data.

MATERIALS OF ANIMAL/HUMAN ORIGIN

Materials of animal/human origin are tested and risks of using MD are indicated in SDS. Safety Data Sheets are available at www.abbottdiagnostics.com or contact your local representative.

INDICATIONS

Not applicable.

CONTRAINDICATIONS

Not applicable.

CALIBRATION

For instructions on performing a calibration, refer to the Alinity ci-series Operations Manual, Section 5.

Calibration is stable for approximately 14 days (336 hours), but is required with each change in reagent lot. Verify calibration with at least 2 levels of controls according to the established quality control requirements for your laboratory.

If control results fall outside acceptable ranges, recalibration may be necessary. This assay may require recalibration after maintenance to critical parts or subsystems or after service procedures have been performed.

MATERIALS REQUIRED BUT NOT PROVIDED

- 04R1001 Alinity ci-series Calibrator/Control Replacement Caps

COLOR AND ODOR

Colorless and Odorless

cal_CO2C_	LOT	Exp.	REF 08P7201
-----------	-----	------	-------------

ASSAY NAME	REF	ASSAY NO.	DEFAULT UNITS		ALTERNATE UNITS	
			CAL 1	CAL 2	CAL 1	CAL 2
CO2	07P72	1267	mEq/L	mEq/L	mmol/L	mmol/L

CAUTION: Verify the assay name ((ASSAY NAME)) before entering calibrator values.

en Refer to the package insert for information on accessing and importing calibrator values electronically.
Calibrator values were generated using the ARCHITECT cSystem.

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