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19, rue de la Presse
CS 60132

42003 ST-ETIENNE CEDEX 1
Tél. 04 77 74 51 11

Operational Documentation of Medical Product:

KITETT KOLOR PLAY Manual breast pump with KITETT KOLOR funnel

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La Diffusion Technique Française considers information contained in this document as confidential. The text of this document cannot be edited or distributed without the company's authorization.

2020

KOLOR PLAY

by **kitett** 

GB – Instructions for use, page 3

KITETT KOLOR PLAY Manual breast pump with KITETT KOLOR funnel

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CE 0459 (2016)

93/42/EEC Directive amended by the 2007/47/EC Directive – Medical device of Class IIa
K.NU.MU.185.0720 – Version 1 – Non contractual pictures

DTF MEDICAL

LA DIFFUSION TECHNIQUE FRANCAISE
19 rue de la Presse, CS 60132, 42003 SAINT-ETIENNE Cedex 1 France
Tel: +33.(0)4.77.74.51.11 – Fax: +33.(0)4.77.79.67.72
Email: dtf@dtf.fr – www.dtf.fr

DTF[®]
medical

Contents

KOLOR PLAY

by kitett 

Instructions for use of KITETT KOLOR PLAY Manual breast pump with KITETT KOLOR funnel

1. Intended Use	3
a. Indications	3
b. Contraindications and adverse effects.....	3
2. Product description	3
a. Expression principle (principle of operation).....	4
b. Product name, models or design variants.....	4
c. List of components	5
d. Delivery kit	6
e. Choice of funnel size.....	6
3. Cleaning/Disinfecting of the breast pump	7
a. Cleaning	7
b. Disinfecting at home	7
4. Use	9
a. Preparing the breast pump to use	9
b. Operating the breast pump	10
c. After each use.....	10
d. Disassembling the breast pump.....	10
5. Breastfeeding	11
a. Storing breast milk	11
b. Physiology of baby breastfeeding	11
6. Cautions and Important recommendations	11
7. Troubleshooting	12
8. Data on compatible products.....	12
9. Warranty	12
10. Technical specifications.....	13
a. Description of the breast pump	13
b. Materials, quality of surfaces and leak tightness of the breast pump components.....	14
c. Color design and organoleptic properties.....	15
11. Operation conditions	15
12. Packaging and Transportation	15
13. Symbols.....	16

1. Intended Use

a. Indications

In cases where breastfeeding is insufficient or impossible, breast pumps help mothers to ensure their babies always have the benefit of breast milk.

Breast pumps can be used at different stages of lactation. At the beginning, they can help to stimulate the production of milk and later, help to maintain long-term production.

Women may wish to express their milk for a number of reasons, including:

- Medical reasons related to their baby, such as sucking problems, premature birth, neonatal pathologies or cleft lip.
- Medical reasons related to motherhood, such as cracks, engorgement, painful breastfeeding, inverted nipples or medical treatment incompatible with breastfeeding.
- Comfort reasons, for example mothers who wish to store their milk due to temporary absence.
- Reasons related to returning to work, for mothers wishing to continue to feed their baby with breast milk.

b. Contraindications and adverse effects

There is no known contraindication to the use of KITETT KOLOR PLAY manual breast pumps

2. Product description

These instructions are designed to inform readers about KITETT KOLOR PLAY manual breast pumps.

KITETT KOLOR PLAY is a manual breast pump. It is particularly indicated for occasional use. It may be used to supplement an electrical breast pump, in cases of temporary separation or for any other reason making breastfeeding insufficient or impossible.

The ergonomics of the KITETT KOLOR PLAY breast pump have been especially designed for mothers, making it easy to handle and simplifying the pumping action. The handle and the collection container are comfortable, designed with the shape of mothers' hands in mind.

All sizes of KITETT KOLOR funnels fit the breast pump. Mothers may therefore use a KOLOR funnel size that suits their breast shape and enjoy optimal comfort for prolonged, sustainable breastfeeding. You may also use a KITETT KOLOR funnel already in your possession, for example from having rented KITETT FISIO or KITETT FISIO PRO breast pumps.

KITTT KOLOR PLAY breast pump benefits from «**Clean Valve**» technology, limiting the risk of bacterial contamination from fingers onto the parts in contact with the breast milk.

KITTT KOLOR PLAY reproduces the **2 physiological phases** of breastfeeding:

- stimulation phase,
- expression phase,

a. Expression principle (principle of operation)

The principle of operation is to generate a vacuum between the membrane with base and the handle of the breast pump. The vacuum is formed by movement of the handle of the breast pump, generating the intermittent negative pressure needed to express the breast milk. At the same time, the Clean Valve opens and closes by turn.

When the handle is pressed (the handle is forced against the container), the valve closes and enables the transfer of the low pressure (vacuum) generated by the pump into the funnel.

When the handle is returned to its initial position, the valve opens and allows air to enter the system, thereby eliminating the low pressure generated by the pump, resulting in milk expression.

b. Product name, models or design variants

KITETT KOLOR PLAY Manual Breast Pump with KITETT KOLOR funnel
Design variants:

1) KITETT KOLOR PLAY Manual Breast Pump with KITETT KOLOR 21 S funnel
includes

1. KITETT KOLOR funnel, size: 21 S (1 pc)
2. Vacuum Module Body (1 pc)
3. Membrane with base (1 pc)
4. Handle (1 pc)
5. Clean Valve (1 pc), including:
 - transparent valve
 - positioning ring (taupe coloured)
6. Collection container for breast milk collecting and storing, 100 ml (1 pc)
7. Cap for the collection container for breast milk collecting and storing (1 pc)
8. Container Stand (1 pc)
9. Bag for accessories storage (1 pc)
10. Instructions for Use (1 pc)

2) KITETT KOLOR PLAY Manual Breast Pump with KITETT KOLOR 24 L funnel
includes

1. KITETT KOLOR funnel for KITETT Breast Pump, size: 24 L (1 pc)
2. Vacuum Module Body (1 pc)
3. Membrane with base (1 pc)
4. Handle (1 pc)
5. Clean Valve (1 pc), including:
 - transparent valve
 - positioning ring (taupe coloured)
6. Collection container for breast milk collecting and storing, 100 ml (1 pc)
7. Cap for the collection container for breast milk collecting and storing (1 pc)
8. Container Stand (1 pc)
9. Bag for accessories storage (1 pc)
10. Instructions for Use (1 pc)

3) KITETT KOLOR PLAY Manual Breast Pump with KITETT KOLOR 26 L funnel
includes

1. KITETT KOLOR funnel for KITETT Breast Pump, size: 26 L (1 pc)
2. Vacuum Module Body (1 pc)
3. Membrane with base (1 pc)

4. Handle (1 pc)
5. Clean Valve (1 pc), including:
 - transparent valve
 - positioning ring (taupe coloured)
6. Collection container for breast milk collecting and storing, 100 ml (1 pc)
7. Cap for the collection container for breast milk collecting and storing (1 pc)
8. Container Stand (1 pc)
9. Bag for accessories storage (1 pc)
10. Instructions for Use (1 pc)

4) KITETT KOLOR PLAY Manual Breast Pump with KITETT KOLOR 30 L funnel includes

1. KITETT KOLOR funnel for KITETT Breast Pump, size: 30 L (1 pc)
2. Vacuum Module Body (1 pc)
3. Membrane with base (1 pc)
4. Handle (1 pc)
5. Clean Valve (1 pc), including:
 - transparent valve
 - positioning ring (taupe coloured)
6. Collection container for breast milk collecting and storing, 100 ml (1 pc)
7. Cap for the collection container for breast milk collecting and storing (1 pc)
8. Container Stand (1 pc)
9. Bag for accessories storage (1 pc)
10. Instructions for Use (1 pc)

c. List of components

1 KITETT KOLOR funnel

2 "Clean Valve" composed of a transparent valve and a taupe-coloured positioning ring

3 Collection container for breast milk collecting and storing, 100 ml

4 Vacuum Module Body

5 Membrane with base

6 Handle

7 Container stand

8 Cap for the collection container for breast milk collecting and storing

9 Bag for storing KOLOR PLAY breast pump accessories (funnel, collection container)

The «Clean Valve» to be used with the KITETT KOLOR PLAY manual breast pump should be taupe-coloured. Should not be used with a KITETT electric breast pump.

d. Delivery kit

KITETT KOLOR PLAY Manual Breast Pump with KITETT KOLOR funnel

Delivery kit:

1. KITETT KOLOR 21 S / 24 L/ 26L/ 30 L funnel (1 pc) - depending on the MD design variant
2. Vacuum module body (1 pc)
3. Membrane with base (1 pc)
4. Handle (1 pc)
5. Clean Valve (1 pc), including:
 - transparent valve
 - positioning ring (taupe coloured)
6. Collection container for breast milk collecting and storing, 100 ml (1 pc)
7. Cap for the collection container for breast milk collecting and storing (1 pc)
8. Container Stand (1 pc)
9. Bag for accessories storage (1 pc)
10. Instructions for Use (1 pc)

Design variant	REF*
KOLOR PLAY Manual Breast Pump with KITETT KOLOR 21 S funnel	KPLAY21ES
KOLOR PLAY Manual Breast Pump with KITETT KOLOR 24 L funnel	KPLAY24EL
KOLOR PLAY Manual Breast Pump with KITETT KOLOR 26 L funnel	KPLAY26EL
KOLOR PLAY Manual Breast Pump with KITETT KOLOR 30 L funnel	KPLAY30EL

* it is allowed to use two additional letters separated by a hyphen after the main catalog number and regulating the region (of the country) of destination

Identification: each breast pump is provided with a lot number that indicates the date of its manufacture and an operator's code.

Example: LOT 12-2016 46


12: operator's code; 2016: year; 46: week of manufacture

e. Choice of funnel size

A funnel (21 S, 24 L, 26 L or 30 L) is included in the standard KITETT KOLOR PLAY breast pump delivery kit.

Funnels are available in 4 inside diameters and 2 «comfort» sizes, that fit perfectly onto all shapes of breast, ensuring the highest level of comfort. The funnels has no massage protrusions.

- 4 inside diameters: 4 colour codes

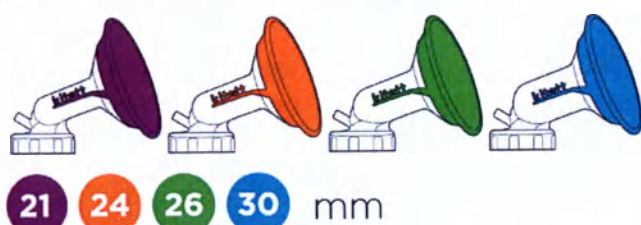
- 2 comfort sizes
S = 76 mm and L = 90 mm

KITETT helps you in choosing the right size of funnel with:

- the Kolorama size ruler to help choose your size.
- download

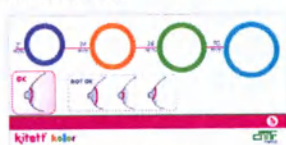- KOLOR YOU application available for free

KOLOR funnels are guaranteed Phthalate and BPA free.

They can be reused throughout the period of breast pump use.


For optimal hygiene, funnel is a single unit, to avoid touching the internal parts of the funnel or the screw thread of the collection container.

Measurements:

Measurements should be taken after breastfeeding or expressing your milk. If the nipples are measured in calm state, add additional 2 mm to the result in order to choose the correct size of the internal diameter.

- ⚠ Your nipple should not be too free. You should see space around your nipple in order to let it move easily in the funnel during pumping.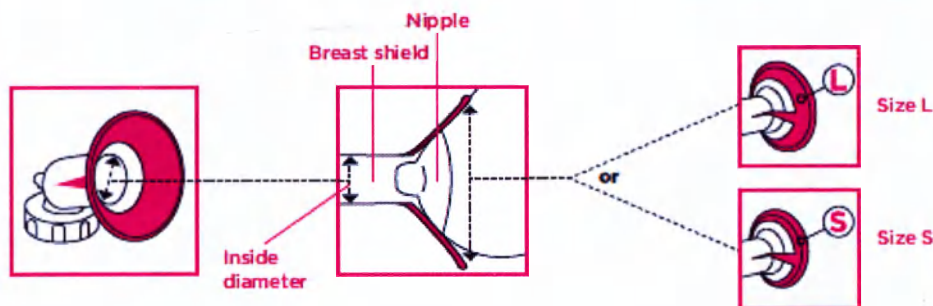

- If the funnel is too tight: lesions, compressed ducts
- If the funnel is too wide: less milk will be expressed


A correct comfort size (S or L outer fitting) will improve breast pump pressure (optimal airtightness).

3. Cleaning/Disinfecting of the breast pump

- Before using for the first time and after each use thereafter (and long periods of disuse)

a. Cleaning

- Rinse out all the elements comprising the breast pump in cold water, then wash in hot soapy water.
- Rinse all the elements and assemble the two components comprising the "Clean Valve": the transparent valve and the positioning ring (making sure that the valve is positioned correctly on the ring).
- Drain on a piece of kitchen paper, then leave to dry in the air.

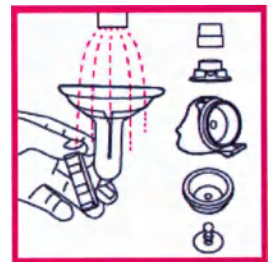! You are advised to wash all parts in contact with breast milk immediately after breast pump use. This makes cleaning easier and prevents the development of bacteria.

- Before using for the first time and regularly during use:

b. Disinfecting at home

Sterilization in boiling water

All parts of the breast pump are resistant to boiling water (up to 100°C)

Disinfect all parts in boiling water (10 minutes minimum), in the dishwasher or in a microwave sterilizer.*

* Consult the instructions provided by the microwave sterilizer manufacturer and do not exceed 1000 W.


If disinfection takes place in boiling water and your water is hard, you may add a few drops of white vinegar (2 or 3 drops) to avoid deposits forming on the components.

Do not use any disinfecting agents or solutions for cleaning and disinfection of breast pump. The components of breast pump may be damaged by chemical disinfecting agents or solutions.

4. Use

a. Preparing the breast pump to use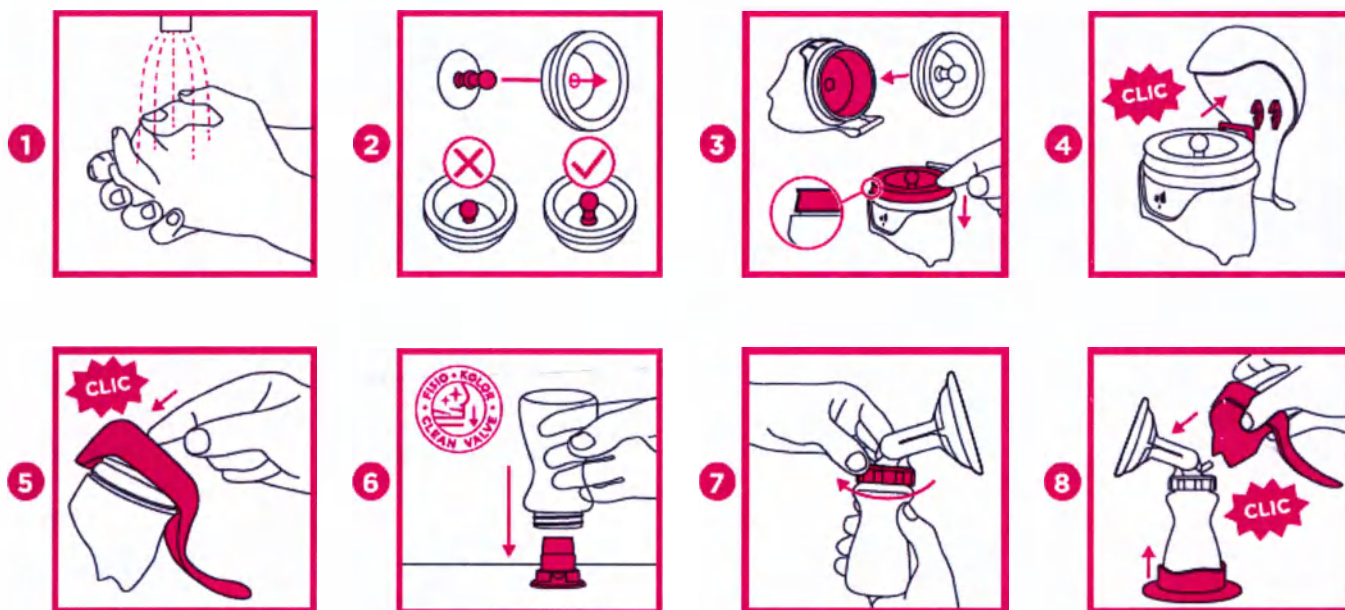

1 - Wash your hands in soapy water before handling breast pump components.

Assemble the Vacuum module:

2 - Assemble the membrane and the base with the stem. Make sure the base fits correctly against the membrane.

3 - Insert the whole assembly into the body of the vacuum module and squeeze the rim tightly to ensure a good watertight fit around the membrane.

4/5 - Clip the handle onto the body via the notches and horizontal bar (click).

Assemble the breast pump:

6 - Place the collection container for breast milk collecting and storing directly onto the taupe-coloured "Clean Valve"* (the valve should be fixed to a smooth, clean surface). To ensure hygiene, both parts of the valve should be pre-assembled after cleaning: valve and positioning ring. (see chapter 3)

If necessary, take care to handle the "Clean Valve" by the rim (taking care not to touch parts that may be contact with the breast milk).


The innovative «Clean Valve» technology, exclusive to KITETT, prevents the risk of contamination through contact with fingers in the funnel.

7 - Screw the KOLOR funnel onto the collection container for breast milk collecting and storing (ensuring that the taupe-coloured "Clean Valve" is correctly positioned in the collection container)

8 - Fit the assembled vacuum module onto the KITETT KOLOR funnel and container. Make sure everything is interlocked (click). Place the assembled KITETT KOLOR PLAY breast pump on the container stand for optimal stability.

b. Operating the breast pump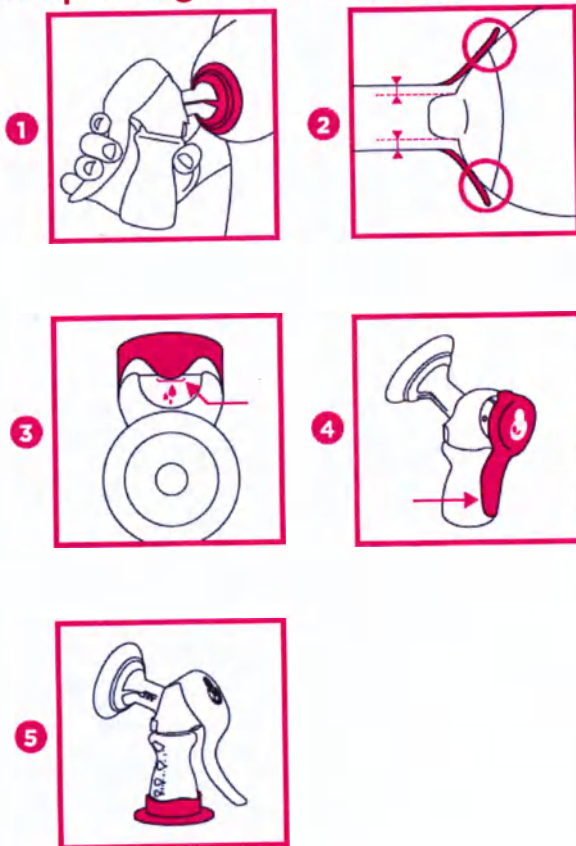

1 - Place the funnel correctly over your breast, ensuring the milk can flow freely into the collection container.

2 - Ensure your nipple is centred inside the duct. The flexible part of the funnel must also be properly positioned on your breast to ensure complete sealing.

3 - Start to pump.

Stimulation phase:

Pump with short, rapid movements to initiate milk let-down.

The handle should be pressed down as far as the grey mark on the body of the breast pump (fig. 3).

4 – Expression phase:

When the first drops of milk start to flow, start pumping with longer, less frequent movements.


The handle should be pressed down until it touches the collection container (fig. 4).

The speed and length of movements will help you to adjust the strength of suction, according to how you feel.

5 - When milk no longer flows or when the collection container is full, stop pumping.

Place the breast pump on the container stand for optimal stability.

c. After each use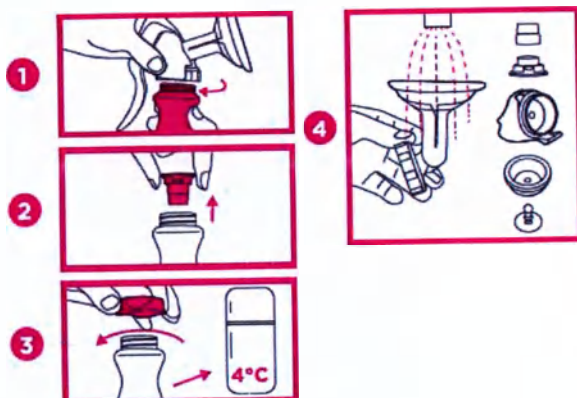

1 - Unscrew the collection container from the breast pump with funnel.


2 Remove the taupe coloured "Clean Valve". Use the container stand to avoid any risk of upsetting the collection container.

3 - Place the cap firmly on the collection container.

Please refer to the following section regarding the preservation/de-freezing of breast milk. Breast milk can be kept in a refrigerator at +4°C for 48 hours.

4 - After use, rinse all parts in contact with milk in cold water. Then wash them in hot and soapy water. Rinse and leave to dry in the open-air on a piece of paper towel.

d. Disassembling the breast pump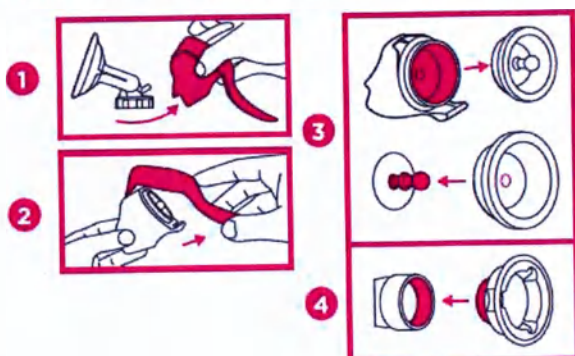

1 - Disconnect the handle/body/membrane assembly of the funnel by twisting slightly on the side.

2 - Twist the handle upwards to release it from the body of the breast pump.

3 - Remove the membrane and its base. Carefully remove the stem from the base, making sure you do not damage the membrane orifice.

4 - Dismantle the two parts comprising the "Clean Valve": the transparent valve and taupe-coloured positioning ring.

5. Breastfeeding

a. Storing breast milk

International recommendations **

BREAST MILK	ROOM TEMPERATURE (25°C)	REFRIGERATOR (4°C)	FREEZER (-18°C)
Freshly expressed	<6 hours (preferably stored in the refrigerator)	8 days (in the coldest part of the refrigerator)	< 12 months
Defrosted in the refrigerator but not reheated	< 4 hours	< 24 hours	Do not refreeze
Defrosted and heated in a hot water bath	Not indicated	Not indicated	Do not refreeze
Transport conditions: chill the milk as quickly as possible for a maximum of 24 hours in an insulated compartment (15°C)			

** Data from the Human Milk Banking Association of North America (www.hmbana.org) that are also used by the International Lactation Consultant Association - www.ilca.org - on storing breast milk for an infant in good health.

b. Physiology of baby breastfeeding

KITETT KOLOR PLAY breast pump provides mothers with the natural sensations of breastfeeding by recreating the way in which babies feed.

It reproduces the 2 physiological phases of breastfeeding:

- The stimulation phase to generate milk let-down (short, rapid movements),
- The expression phase to collect milk (longer, less frequent movements).

6. Cautions and Important recommendations

Use - Functionnality

- This device must not be left within the reach of unsupervised children or animals (risks caused by detachable components).
- The device should be installed and started up in accordance with the information supplied in the accompanying documents.
- The device must only be used under the operating conditions indicated by the manufacturer.
- If you notice that the device is not working correctly, contact your supplier.
- This is a fragile medical device, to be handled with care.
- No specific knowledge or training is required to use this device.
- Take care to keep these instructions with the device as they contain the information needed on how to use the equipment correctly.
- The Export version funnel (with Export screw thread) can only be used with a corresponding collection container (same screw thread).

CAUTION, in all cases:

- WARNING: No modifications permitted.
- WARNING: Only use the components supplied and listed in the instructions for use.

7. Troubleshooting

Problem	Solution
Low or non-existent suction	Check that the funnel is fully screwed onto the collection container (screw tightly). Check for presence of taupe-coloured «Clean Valve» specific to manual breast pump. Check the right positioning of the transparent valve on the ring of the 'Clean Valve'. Check the membrane is fixed on its base. Check the membrane and the handle are fixed onto the vacuum module body. Check the state of the seal between the funnel and the collection container for breast milk collecting and storing.

Maintenance and repair: not applied

8. Data on compatible products

KITETT KOLOR funnels for breast pumps*

Ref. numbers WB21ES/WB21EL - WB24ES/WB24EL - WB26ES/WB26EL - WB30ES/WB30EL, WB21ESH - WB24ESH - WB26ELH - WB30ELH

KITETT KOLOR Expression Kits for breast pumps*

Including KITETT KOLOR funnel (one of 21 S, 21 L, 24 S, 24 L, 26 S, 26 L, 30 S, 30 L size range, collection container for breast milk collecting and storing, 100 ml (1 pc), cap for the collection container for breast milk collecting and storing (1 pc), PVC tube (1 pc), interface bottle (1 pc)

Internal diameter (mm)	Expression Kit with one funnel	
	KOLOR® funnel (S)	KOLOR® funnel (L)
21	K21ES	K21EL
24	K24ES	K24EL
26	K26ES	K26EL
30	K30ES	K30EL

All parts of the kit (except for the PVC tube and the interface bottle) are compatible and must be used with the KITETT KOLOR PLAY breast pump.

* not registered as separate medical devices in the territory of the Russian Federation

9. Warranty

This device is guaranteed for 1 year as from date of purchase against all manufacturing defects.

This warranty does not cover the consequences of accidents or any use that does not comply with the manufacturer's instructions.

To be accepted, invoices must include the vendor's stamp and be dated on the day of purchase.




For warranty claims, please contact your supplier.

10. Technical specifications

Potential risk class	Ila
Vacuum generated	-300 hPa

a. Description of the breast pump

Dimensions and weight

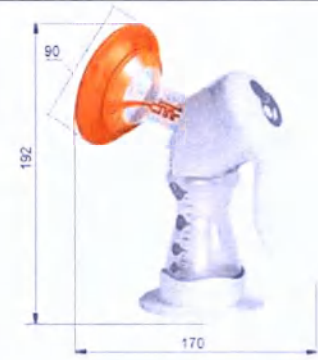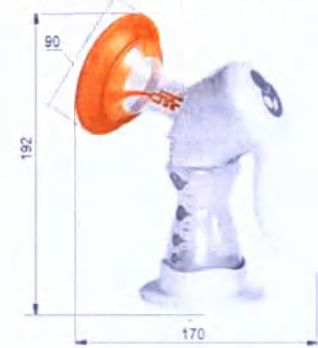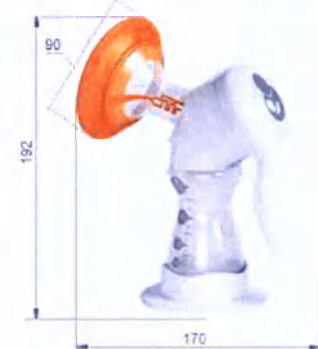Component	Dimensions, mm (+/- 5%)	Weight, g (+/- 5%)	Schematic view with dimensions
KITETT KOLOR PLAY Manual Breast Pump with KITETT KOLOR 21 S funnel (assembled, with a stand)	height 190 width 168 depth 88	165	
KITETT KOLOR PLAY Manual Breast Pump with KITETT KOLOR 24 L funnel (assembled, with a stand)	height 192 width 170 depth 90	167	
KITETT KOLOR PLAY Manual Breast Pump with KITETT KOLOR 26 L funnel (assembled, with a stand)	height 192 width 170 depth 90	169	
KITETT KOLOR PLAY Manual Breast Pump with KITETT KOLOR 30 L funnel (assembled, with a stand)	height 192 width 170 depth 90	171	

Additional information concerning the collection container for breast milk collecting and storing

Requirements to the measuring scale of the collection container for breast milk collecting and storing	Two measuring scales on opposite sides of the container: In milliliters: Graduation by numbers 5, 20, 40, 60, 80, 100 ml Intermediate graduation by lines every 5 ml In ounces: Graduation by numbers 1, 2, 3 oz Intermediate graduation by lines every 0.5 oz
Locking ring type of the collection container for breast milk collecting and storing	Thread
Wall thickness of the collection container for breast milk collecting and storing	0.5 mm
Nominal and total capacity of the collection container for breast milk collecting and storing	Nominal capacity 100 ml Total capacity 130 ml
Type of the collection container for breast milk collecting and storing	shaped
Information on quality of the container surface	The outer and inner surfaces of all parts of the breast pump must be clean, smooth and free of through holes, bubbles, burrs and cracks. Ripple effect, nebulosity, foreign inclusions are also not allowed on the surface. The surface of the container locking ring thread should be smooth, without chips or ridges. Defects in the thread of the locking ring are not allowed

b. Materials, quality of surfaces and leak tightness of the breast pump components

Materials and quality of surfaces

The outer and inner surfaces of all parts of the breast pump must be clean, smooth and free of through holes, bubbles, burrs and cracks. Ripple effect, nebulosity, foreign inclusions are also not allowed on the surface.

Vacuum module body, handle, container stand and cap for the collection container for breast milk collecting and storing are made of white nontransparent polypropylene.

Collection container for breast milk collecting and storing and hard part of KITETT KOLOR funnel are made of semi-transparent uncolored polypropylene.

Soft part of KITETT KOLOR funnel (thermoplastic elastomer) has different colors (purple, orange, green or blue, depending on the funnel size). Surface of the thermoplastic elastomer should be soft, smooth, warm and nice on the touch, uniformly colored, free of turbidity, dirt, bubbles, burrs and cracks.

The two-component Clean Valve consists of a transparent silicone valve and a polyethyleneterephthalate positioning ring (taupe coloured).

Membrane with base are made of silicone (membrane) and polypropylene (base).

Leak tightness

All connected parts of the breast pump are leak-tight to the extent necessary to ensure the breast pump functioning.

The threaded connection of the collection container for breast milk collecting and storing and the vacuum module body, as well as the cap for the collection container for breast milk collecting and storing, are completely leak-tight to prevent milk leakage and provide temporary storage of breast milk.

c. Color design and organoleptic properties

Color design

Component	Color design
KITETT KOLOR funnel, size 21 S	Semi-transparent polypropylene, purple thermoplastic elastomer
KITETT KOLOR funnel, size 24 L	Semi-transparent polypropylene, orange thermoplastic elastomer
KITETT KOLOR funnel, size 26 L	Semi-transparent polypropylene, green thermoplastic elastomer
KITETT KOLOR funnel, size 30 L	Semi-transparent polypropylene, blue thermoplastic elastomer
Vacuum module body	White polypropylene
Membrane with base	Membrane – taupe silicone Base - white polypropylene
Handle	White polypropylene
Clean Valve, including: - transparent valve - positioning ring (taupe coloured)	Transparent silicone, Taupe polyethyleneterephthalate
Collection container for breast milk collecting and storing, 100 ml	Semi-transparent polypropylene
Cap for the collection container for breast milk collecting and storing	White polypropylene
Container stand	White polypropylene
Bag for accessories storage	Gray polyester with pink manufacturer's logo

Organoleptic properties

The outer and inner surfaces of all parts of the breast pump, that are made of plastic and silicone, are smooth to the touch, homogeneous, without burrs, bubbles, cracks and dirt. The thermoplastic elastomer of which the soft part of the KITETT KOLOR funnel is made is a soft, warm and nice on the touch material.

The breast pump components are free of odor.

11. Operation conditions

The breast pump can be used indoors and outdoors in the following conditions:

Ambient temperature (T)	$+10^{\circ}\text{C} < T < +40^{\circ}\text{C}$
Relative humidity (H)	$15\% < H < 93\%$

12. Packaging and Transportation

Individual packing

The breast pump is packed in an individual cardboard box (225.0 x 92.5 x 252.0 mm, weight 500 g, $\pm 5\%$) with transparent PET (polyethyleneterephthalate) inserts.

Inside the box, there are transparent PET (polyethyleneterephthalate) dividers with shaped cutouts for placing the individual parts of the breast pump to prevent them from moving inside the box.

Transport package

4, 6, 10 or 32 individual boxes with the breast pump can be placed in the transport package.

Packaging material: corrugated cardboard box

Quantity of individual boxes, pcs	Dimensions (w x d x h), cm (± 1 cm)	Weight, kg ($\pm 0,2$ kg)
4	42.5 x 31.5 x 27	2.8
6	58 x 34 x 28	3.5
10	58.5 x 38 x 37	6.5
32	90 x 58.5 x 41.5	16.5

Transportation and Storage Conditions

Ambient temperature (T) during transportation and storage	$-25^{\circ}\text{C} < T < +50^{\circ}\text{C}$
Relative humidity (H) during transportation and storage	$15\% < H < 93\%$

13. Symbols

Symbol used	
	Compliance with directive 93/42/EEC of 14 June 1993 on medical devices
	Manufacturer
	Refer to the instructions for use
	Device reference
	Lot number
	Store away from direct sunlight
	This symbol indicates the limit temperature which should not be exceeded for operation
	European screw thread
	BPA free material
	This symbol indicates the moisture limit which must not be exceeded in the case of operation, transport, and storage.

Disposal procedure

For Russian Federation:

The device should be disposed of in accordance with the rules approved by "SanPiN 2.1.7.2790-10 (Sanitary and Epidemiological Requirements for Medical Waste Management), Class A (Non-hazardous Waste)".

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Authorized representative in the Russian Federation: BioSystems, LLC

197046 St. Petersburg, 3 Pinsky lane, let. A
Tel.: +7 (812) 320 49 49, Fax: +8 (812) 320 49 40

Authorized representative in the Republic of Belarus: BioLine-BS, LLC

220123 Minsk, 32a V. Khoruzhei str.
tel./fax: +375 (17) 399 43 79

Authorized representative in Ukraine: BioLine-Ukraine, LLC

03062 Kiev, 67 Peremogi pr., let. Д
Tel.: +380 (44) 200 89 37, Fax: +380 (44) 200 89 20

Authorized representative in the Republic of Kazakhstan: BioLine-Kazakhstan, LLP

010000 Astana, 14 Beibitshilik str., office 908
Tel.: +7 (7172) 27 37 26, Fax: +7 (7172) 27 68 84

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