“APPROVED”

By
Jean-Philippe MASSARDIER,
Co-CEO

Signature, stamp

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**OPERATIONAL DOCUMENTATION
for the Medical device**

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La Diffusion Technique Française considers information contained in this document as confidential. The text of this document cannot be edited or distributed without the company's authorization.

2020

Common information:

Instructions for use for medical device Electric Breast Pump KITETT in versions with accessories contain all necessary and detailed information about proper and safe usage, conditions of use, features and co-use of medical device with its accessories. Instructions for use booklets are placed inside the package of medical devices.

Nevertheless every kit of accessories is supplied with brief instructions for use, containing information about kit composition and main features. Kit instructions for use are represented by short leaflets inside the package or leaflets posted on the package from outside.

Instructions for use of the medical device and Kit instructions for use are available in the different languages, including Russian language.

Content

- Instructions for use of the medical device: **Electric Breast Pump KITETT version FISIO with accessories**
- Attachment № 1 to the Instructions for use: **Electric Breast Pump KITETT version FISIO with accessories**
- Instructions for use of the medical device: **Electric Breast Pump KITETT version FISIO PRO with accessories**
- Attachment № 1 to the Instructions for use: **Electric Breast Pump KITETT version FISIO PRO with accessories**
- Kit instructions for use: *KITETT FISIOBIB kit for breast milk collecting and storing*
- Kit instructions for use: *KITETT KOLOR Kit Expression (single and double) for KITETT Breast Pump*
- Kit instructions for use: *KITETT FISIOCOOL kit for breast milk cooling*
- Kit instructions for use: *KITETT FISIOFRESH Cooling and Transportation Kit*

Instructions for use

kitett

FISIO

Electric breast pump
with accessories

Associated with KITETT KOLOR KIT EXPRESSION

CE 0459 (2003)

K.NU.MU.156.0619 – Version 4, June 2019

BREASTFEEDING BY DTF MEDICAL

LA DIFFUSION TECHNIQUE FRANCAISE
19 rue de la Presse, CS 60132, 42003 SAINT-ETIENNE Cedex 1 France
Phone number: +33.(0)4.77.74.51.11 – Fax :+33.(0)4.77.79.67.72
Email: dtf@dtf.fr – www.dtf.fr

000 ЦИРМИ INFO@CIRMI.RU WWW.GOSZDRAVNADZOR.RU

1. Intended use	3
a) Indications.....	3
b) Contraindications and adverse events	3
2. Product description	3
a) Detailed view and description	3
b) Selecting the breast shield size.....	4
3. Cleaning and disinfection	4
a) Breast pump cleaning and disinfection	4
b) Components cleaning at home.....	5
c) Disinfection at home.....	5
d) Cleaning and sterilizationby healthcare establishments	6
4. Use	6
a) Breast pump functioning	6
b) Components preparation	6
c) Using the breast pump	6
d) After each use	7
5. Breastfeeding	7
a) Storing breast milk.....	7
b) Baby's breastfeeding physiology	7
6. Safety precautions.....	7
7. Problem solving.....	8
8. List of components and spare parts available	9
9. Warranty / Recycling	10
10. Technical specifications	10
11. Electromagnetic compatibility (EMC).....	11
12. Pictograms	12

1. Intended use

a) Indications

Breast pumps are designed to be used by lactating women in order to extract their maternal milk. They help them to relieve their mastitis or breast engorgement and facilitate their healing. They also help mothers to relieve cracked or painful nipples or when breastfeeding is insufficient or impossible (inverted or painful nipples, etc.). Furthermore, they give the possibility to mothers to breastfeed their baby if they cannot be directly breastfed (preterm birth or neonatal pathologies (cleft lip and/or palate), difficulty of the baby to suck, etc.). Finally, they allow to initiate or maintain lactation, especially in case of a medical treatment incompatible with breastfeeding.

b) Contraindications and adverse events

There are no known contraindications for the use of the KITETT FISIO electric breast pump.

Adverse event: a risk of allergy with the materials (thermoplastic elastomer) in contact with the body parts can appear.

2. Product description

The KITETT FISIO breast pump is an electric pump. It can be used in case of temporary separation or any other reason making breastfeeding impossible or not sufficient. Easy to use, it is ideal for active mothers.


The FISIO pump can be used in single or double pumping and should be plugged into the power supply. It combines a suction power adjustment with a set-up rhythm. The rhythm and the suction power are **simultaneously** adjusted. It tends to reproduce the baby's sucking rhythm: the mother experiences the stimulation and expression phases.

All KITETT KOLOR breast shield sizes connect to the breast pump. The mother can thus have a suitable sized breast shield for her breast morphology to provide optimal comfort for an effective, long-lasting breastfeeding experience.

The breast shield is a one-piece which favours an optimized hygiene, and makes it possible not to touch the thread nor the internal parts of the feeding collection container and the breast shield.

a) Detailed view and description

The breast pump design is presented on the first page. The user interface is presented as below:

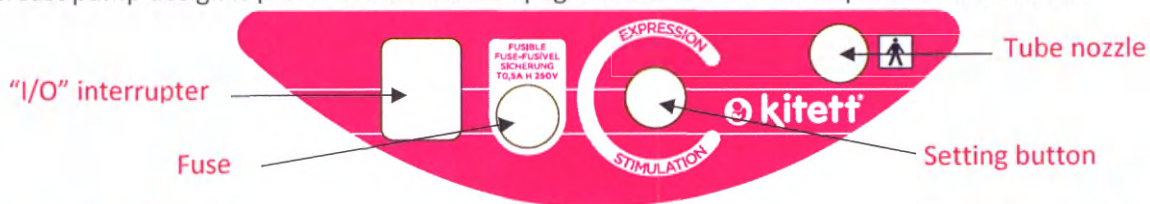Single KITETT KOLOR KIT EXPRESSION: contains 1 KOLOR breast shield, 1 collection container for breast milk collecting and storing, 100 mL, 1 cap for the collection container for breast milk collecting and storing, 1 interface bottle and 1 PVC tube.

Available in autoclavable version on request.

Double KITETT KOLOR KIT EXPRESSION: contains 2 KOLOR breast shields, 2 collection containers for breast milk collecting and storing, 100 mL, 2 caps for the collection containers for breast milk collecting and storing, 1 interface bottle and 2 PVC tubes.

Every KIT EXPRESSION and spare parts available and compatible with the FISIO breast pump are presented in Part 8.


NOTE: KOLOR breast shields, collection containerfor breast milk collecting and storing, capfor the collection container for breast milk collecting and storing and interface bottle are in polypropylene and thermoplastic elastomer. KOLOR breast shields are guaranteed BPA and Phthalate free.


b) Selecting the breast shield size

The FISIO breast pump is used with theKITETT KOLORKIT EXPRESSION.TheKOLOR breast shield is available in 4 inner diameters and 2 «comfort» sizes that adapt perfectly to the breast morphology and guarantee a high level of comfort.

4 inner diameters: 4 colour codes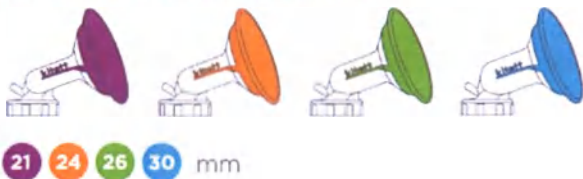

2 comfort sizes: S = 76 mm and L = 90 mm

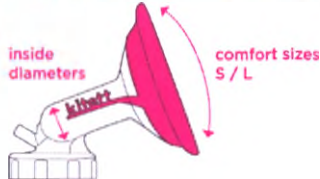
KITETT helps you choose the breast shield using:

- theKolorama size ruler


- the KOLORYOU application available for free download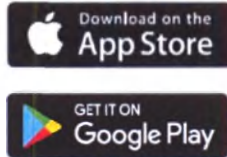

Measurements:

Measurement should be taken after breastfeeding or expressing your milk. Alternatively, if your nipple is at rest, add an extra 2mm around it to select the correct inner diameter size.

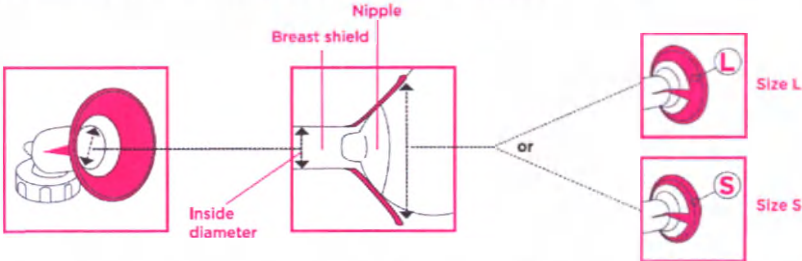The nipple should not feel too constricted neither too free. You should see space around your nipple in order to let it move easily in the breast shield during pumping.If the breast shield is too tight, it could causepain, lesions or compressed ducts. If the breast shield is too wide, less milk will be expressed. A correct comfort size (S-Small- or L-Large- comfort shape) will improve breast pump efficiency (optimal airtightness).

3. Cleaning and disinfection

a) Breast pump cleaning and disinfection

Disinfection is done with a clean and damp cloth that is impregnated with a surface agent disinfectant for medical devices. Follow the instructions provided by the supplier of the surface disinfectant agent. Do not use abrasive cleaning agents. Unplug the device before cleaning.

b) Components cleaning at home

Components concerned:

- KOLOR breast shield
- Collection container for breast milk collecting and storing
- Cap for the collection container for breast milk collecting and storing

Before using for the first time and after each use:

- ❶Disassemble each element.
- ❷Rinse every part in contact with the milk in cold water (about 20°C).
- ❸Then clean with hot and soapy water.
- ❹Rinse every part.
- ❺Let them dry in an open air on a paper towel or a clean linen.

- NOTES:**
- It is better to clean all parts in contact with the breast milk immediately after using the breast pump. It facilitates the cleaning and avoids the development of bacteria.
 - If condensation appears inside the tube or when milk enters inside, rinse the tube in order to remove liquid residues.
 - The interface bottle prevents any liquid penetration into the breast pump. It is therefore important to clean it with soapy water and disinfect it in boiling water (10 minutes), as soon as it has been soiled.Check if the breast pump is not wet or damp. If it is, do not use it.
 - Do not use the milk that penetrated into the interface bottle or tubes. The tubes should be briefly dipped into boiling water (10 minutes) and then thoroughly dried. The tubing colour may change but it will not alter its good functioning.

c) Disinfection at home

Components concerned:

- KOLOR breast shield
- Collection container for breast milk collecting and storing
- Cap for the collection container for breast milk collecting and storing

For full-term babies (without medical problems), disinfect before the first use and once a day. For premature babies or those with weakened immune systems, disinfect before the first use and after each use.

- ❶ Disinfect every part in boiling water (10 minutes), in a dishwasher or in a microwave steriliser*.
- *Refer to the manufacturer’s data for the microwave sterilizer without exceeding 1000 Watt.*
- ❷ Let them dry in an open air on a paper towel or clean linen.

- NOTES:**
- In the case of intensive disinfection (premature infants or witha weakened immune system), boiling water is the only method that guarantees an optimal lifetime of the components.
 - If the disinfection takes place in boiling water and in case of hard water, 2 or 3 drops of white vinegar can be added in the water to avoid whitish deposits on the parts.
 - In the case of disinfection with a dishwasher, the plastic parts may be discoloured by food pigments. It does not affect their functioning.
 - Regularly check the condition of your breast shields during use, cleaning and disinfection.

	Breast shield	Collection container	Cap	Tubes& interface bottle	Breast pump
Before the first use and after each use					
Cleaning	✓	✓	✓	x	x
Before the first use and once a day					
Disinfection	✓	✓	✓	x	x
When it has been soiled					
Cleaning & disinfection	x	x	x	✓	x
Before passing on to the next mother					
Cleaning & disinfection (Cloth with disinfectant)	x	x	x	x	✓

d) Cleaning and sterilization by healthcare establishments

The autoclavable KITETT KOLOR KIT EXPRESSION contains 1 autoclavable KOLOR breast shield, 1 collection container for breast milk collecting and storing, 1 cap for collection container for breast milk collecting and storing, 1 interface-bottle and 1 silicone tube.

Autoclavable KOLOR breast shields, collection containers for breast milk collecting and storing, caps for the collection containers for breast milk collecting and storing and silicone tubes have been designed to be sterilized at 125°C during 30 minutes or 134°C during 18 minutes. The total number of cycles should not exceed 50 cycles. Do not use the breast shield in case of damage or failure. The interface bottle is not autoclavable. It is therefore important to discard it as soon as it has been soiled. For more information, please refer to the autoclavable KITETT KOLOR KIT EXPRESSION instruction for use.

4. Use

Note: Follow the exact order of the steps below to get the correct use of the breast pump.

a) Breast pump functioning

The FISIO breast pump should be plugged into the power supply.

- Check that the values on the breast pump (volts and Hz) corresponds to the values of your distribution network.
- Connect the breast pump to the power supply.

NOTES:

- Check if the breast pump is suitable for the wall outlet.

b) Components preparation


NOTES:

- Only use the components provided in the kit or those manufactured by DTF MEDICAL.
- Before use, check that the KIT EXPRESSION components are free from defect, damage or sign of wear. If this is the case, replace them.
- All the components must be completely dry before use.


Before the first use, please refer to chapter 3 for cleaning and disinfection.

- ❶ Wash your hands with hot and soapy water before handling the breast pump parts.
- ❷ Screw the KOLOR breast shield onto the collection container for breast milk collecting and storing.
- ❸ Connect the suction tube to the inlet hose of the KOLOR breast shield and to the interface bottle on the unit.

Set up of the single KITETT KOLOR KIT EXPRESSION: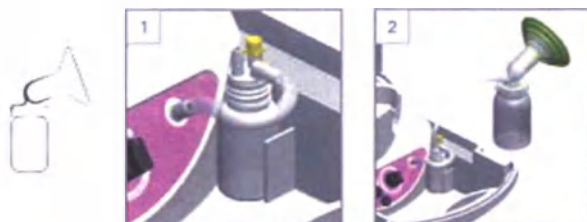


Set up of the double KITETT KOLOR KIT EXPRESSION (can also be used as single one):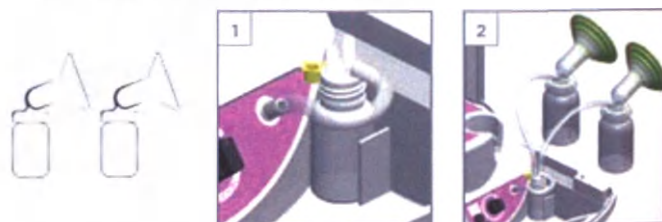

WARNING: Correctly close the interface bottle cap when the breast pump is used on simple pumping.

c) Using the breast pump

- ❶ Place the KOLOR breast shield correctly over your breast ensuring the milk can flow freely into the collection container for breast milk collecting and storing.

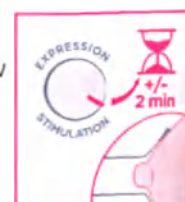❶ Ensure your nipple is centred inside the KOLOR breast shield. The flexible part of the KOLOR breast shield must also be properly positioned on your breast to ensure complete sealing.

❷ The regulation button allows adjusting the cycle and the suction power. To stimulate the lactation, set the button on "stimulation", minimum position, (rapid cycle, low suction).

❸ Switch on the breast pump (position "I").

It is necessary to start with this stimulation phase in order to stimulate the milk arrival and to activate the milk ejection reflex. This phase should last about 2 minutes.

❹ After the ejection reflex, set the button progressively on the "expression" position (slower cycle and higher suction level) to obtain a good expression according to your own level of comfort. Milk expression is optimal at the level of maximum depression and which should not be painful.

❺ Check the proper flow of milk inside the collection container for breast milk collecting and storing.

❻ After expressing the milk, switch off the breast pump (position "O").

NOTES:

- Always unplug the breast pump from the power supply after use.
- Do not fill the collection container for breast milk collecting and storing beyond the marking.
- Be careful not to pinch the tubes during use.

d) After each use

❶ Disconnect the KOLOR breast shield tube(s) and unscrew the KOLOR breast shield(s) from the collection container(s) for breast milk collecting and storing.

❷ Secure the cap(s) onto the collection container(s).

❸ Disconnect the tube(s) from the interface bottle. Check that no milk has penetrated into the interface bottle or inside the tubes.

❹ After each use, rinse and clean every part as described in chapter 3.

❺ Store the breast pump and its components in a dry place away from sunlight.

5. Breastfeeding

a) Storing breast milk

Information on the preservation of breast milk is available on the website www.dtf.fr and on the application KOLORYOU.

b) Baby's breastfeeding physiology

The FISIO breast pump provides mothers with the natural sensation of breastfeeding by recreating the way in which the baby feeds.

It reproduces the 2 physiological phases of the breastfeeding:

- The stimulation phase to generate the milk let-down (short, rapid movements),
- The expression phase to collect milk (longer, less frequent movements).

6. Safety precautions

Use of electricity

- Before connecting the breast pump to the power supply, check the values on the breast pump (Volt –V- and Hertz –Hz-) with your distribution network.
- The warranty does not cover faulty connections.
- This medical device is designed to operate at altitudes less than or equal to 3 000 meters.
- Do not open the breast pump when connected to the power supply.
- Warning: this medical device is directly connected to the electric power supply, do not use it in bath or under the shower, do not immerse.

- Keep this medical device away from any high heat source.
- Connect the breast pump to an easy-to-access socket to allow quick disconnection of the network in the event of a problem.
- The plug is the disconnection device and must always be accessible.

Use of breast pump

- The user of this medical device is the breastfeeding mother.
- This medical device must not be left within the reach of unsupervised children or animals (potential hazards with tubing and detachable components, risk of electrification).
- The breast pump should be installed and started up in accordance with the information supplied in the accompanying documents.
- Properly store the breast pump and its components after use.
- An animal may gnaw at the power cord. Be sure to check the condition of the equipment before use.
- Do not use the appliance outdoors, near a heat source or near a water source (bathtub, sink ...).
- Make sure the unit is not exposed to direct sunlight or excessive heat.
- This medical device must not be used in presence of a mixture of inflammable anaesthetic with air or oxygen or nitrous oxide.
- After use, unplug the appliance from the wall outlet.
- Fragile medical device, to be handled with care.
- No special knowledge or training is required to operate this equipment. Please read the instructions carefully before use.
- Keep these instructions in a safe place with the appliance as it contains the necessary information for the proper use of the equipment.
- This medical device is to be used either in health care establishment or at home.
- The user can safely use all the functions of the breast pump.
- For hygiene reasons, a new complete KITETT KOLOR KIT EXPRESSION must be sold to each mother.

WARNINGS:

- Do not touch the breast pump box for more than 1 second during use.
- The tube connecting the collection container to the breast pump can reach the temperature of 43.5°C. The user should not touch the tube for more than 10 minutes during the breast pump use.

Notes on the functionality of the breast pump

- The KITETT KOLOR breast shield with an international screw thread cap (right) can be used only with a matching collection container (same screw thread cap).
- Switch off the breast pump when the collection container is filled in order to prevent milk from getting into the interface bottle.
- If the device has to be rinsed, entrust the operation to a maintenance professional (risk of bad handling).


CAUTION, in all cases:

- WARNING: Always disconnect the plug from the power supply before opening it.
- WARNING: Only an approved customer service is authorized to repair the breast pump.
- WARNING: Modifications are forbidden.
- WARNING: Only use the components supplied and mentioned in the instructions for use.
- WARNING: The warranty does not cover connection errors with non-compatible components.
- WARNING: Do not undertake service or maintenance operations while the breast pump is used.

7. Problem solving

Problem	Solution
No operating noise can be heard from the pump	Make sure that the power cord is connected to the power supply. Check whether the switch is on the operating position. Check that the current delivered by the power supply corresponds to that of the breast pump (Volt Hz). Check with another breast pump if power supply is available in the wall outlet. If the problem continues, please contact DTF MEDICAL or your local distributor.

Problem	Solution
There is low or no suction	Check that the KOLOR breast shield is fully screwed onto the collection container (screw tightly). Check that every tubes properly connected and not pinched. Check that the cap of interface bottle correctly fits. Check the condition of the O-ring between the KOLOR breast shield and the collection container. Check that there is no residue in the tube. If the problem continues, please contact DTF MEDICAL or your local distributor.
The suction is too strong	Switch off then switch on the breast pump. If the problem continues, please contact DTF MEDICAL or your local distributor.
The breast pump stops abruptly when used	Disconnect the tube from the interface bottle to vent it. In case of alternating stop, please contact DTF MEDICAL or your local distributor.
The pump sucks continuously	Do not tear off the breast shield from your breast (could cause pain and/or damage). Insert your finger between your breast and the breast shield to let the air come in and remove it more easily or disconnect the tube from the breast shield.
A piece or an accessory is missing	Please contact DTF MEDICAL or your local distributor.
The breast pump is too noisy or malfunctioning	Please contact DTF MEDICAL or your local distributor.
Milk entered inside the interface bottle	Clean it as soon as it has been soiled or discard it if you are in a healthcare establishment. Check if the breast pump is not wet or damp. If it is, do not use the breast pump. Do not use the milk that penetrated into the interface bottle or tubes. The tubes should be briefly dipped into boiling water and then thoroughly dried. Switch off the breast pump.
Milk was spilt or entered inside the breast pump	Unplug the breast pump for the power supply. Let the breast pump in a dry and warm place during 24 hours. Do not expose the breast pump to direct sunlight. Do not accelerate the drying process.

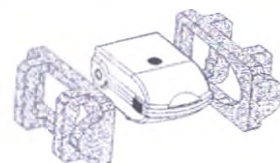Please contact DTF MEDICAL or your local distributor:

- for assistance, if necessary, in the assembly, use or maintenance of the breast pump, or,
- to report any unexpected operation or event.

WARNING: Keep the specific packaging and foam set for sending back the breast pump.

8. List of components and spare parts available

Part designation	Item code
KITETT KOLOR KIT EXPRESSION	See below
Autoclavable KITETT KOLOR KIT EXPRESSION	See below
KITETT KOLOR breast shield	See below
Autoclavable KITETT KOLOR breast shield	See below
KITETT FISIOBIB – Batch of 2 collection containers for breast milk collecting and storing 100mL and 2 caps for the collection containers for breast milk collecting and storing (international screw thread)	FIBIBN-E
KITETT FISIOCOOL – Set of 2 cooling elements KITETT	FICOOOL
KITETT FISIOFRESH – 4 collection containers for breast milk collecting and storing, 4 caps for the collection containers for breast milk collecting and storing, 4 cooling elements KITETT, 1 KITETT thermobag and 1 bag for accessories storing	FIFRESH-E
PVC tube, 1 m-length, for KITETT KOLOR KIT EXPRESSION	Y1091
Silicone tube, 1 m-length, for autoclavable KITETT KOLOR KIT EXPRESSION	TU32
Interface bottle	W2
KITETT soft transport bag for the KITETT electric breast pump	MBOXK
Foam filter	YR36

KITETT KOLOR KIT EXPRESSION:

Kit Ref.	Breast shield size	Shape size	Single / Double	Kit Ref.	Breast shield size	Shape size	Single / Double
K21ES	21mm	Small	Single	K24ES	24mm	Small	Single
K21EL	21mm	Large	Single	K24EL	24mm	Large	Single
K21ESD	21mm	Small	Double	K24ESD	24mm	Small	Double
K21ELD	21mm	Large	Double	K24ELD	24mm	Large	Double
K26ES	26mm	Small	Single	K30ES	30mm	Small	Single
K26EL	26mm	Large	Single	K30EL	30mm	Large	Single
K26ESD	26mm	Small	Double	K30ESD	30mm	Small	Double
K26ELD	26mm	Large	Double	K30ELD	30mm	Large	Double

Autoclavable KITETT KOLOR KIT EXPRESSION:

Kit Ref.	Breast shield size	Shape size	Kit Ref.	Breast shield size	Shape size
K21ESH	21mm	Small	K24ESH	24mm	Small
K26ELH	26mm	Large	K30ELH	30mm	Large

KITETT KOLOR breast shield:

Kit Ref.	Breast shield size	Shape size	Kit Ref.	Breast shield size	Shape size
WB21ES	21mm	Small	WB24ES	24mm	Small
WB21EL	21mm	Large	WB24EL	24mm	Large
WB26ES	26mm	Small	WB30ES	30mm	Small
WB26EL	26mm	Large	WB30EL	30mm	Large

Autoclavable KITETT KOLOR breast shield:

Kit Ref.	Breast shield size	Shape size	Kit Ref.	Breast shield size	Shape size
WB21ESH	21mm	Small	WB24ESH	24mm	Small
WB26ELH	26mm	Large	WB30ELH	30mm	Large

NOTE: There are country variants. These end with the two letters of the country code. For more information, please contact DTF MEDICAL or your local distributor.

9. Warranty / Recycling

Warranty:

- The breast pump is guaranteed for 3 years from the date of purchase against any manufacturing defect, except case of misuse.
- For after sales service under warranty, please contact DTF MEDICAL or your local distributor.
- The technical documentation is available on request.

Recycling:

The device must be disposed of in accordance with local regulation on the disposal of electrical appliances. So you contribute to the human health and environmental protection. Product is on the market since August the 13th 2005.

10. Technical specifications

Device class (breast pump)	Ila
Device class in electrical safety (breast pump)	II
Type of applied parts	BF
Protection level	IP 30
Expected shelf life of components	1 year
Expected shelf life of autoclavable components	50 cycles
Expected shelf life of the breast pump	5 years
Collection container capacity	100ml
Weight	4 kg
Dimensions	L360 x l250 x h180mm

Voltage	Available in 230V – 50Hz; 220V – 50Hz or 220V – 60Hz	
Consumption	100VA	
Fuses	T 0,5A H 250V	
Suction values	Variable from -90 to 300hPa	
Cycles	40 to 140 cycles/minutes	
Sound level	<57dBa	
Running mode	Continuous	
Power supply	Alternative current 230V – 50Hz	
Ambient temperature (T) - For transport and storage		-25°C < T < +70°C
Ambient temperature (T) - When in functioning		+5°C < T < +40°C
Relative humidity (H) For transport, storage and functioning		15% < H < 93%
Atmospheric pressure (P) For transport, storage and functioning		700 hPa< P < 1060 hPa

- Provide adequate suction so that mothers can express their milk.
- Control the rate of suction to best meet the baby's sucking rhythm.

Applied part: KITETT KOLOR breast shield.

Identification:

Each KITETT breast pump has a unit identification (reference and number) which indicates, among other things, the exact date of manufacture and the serial number of the breast pump. The first two digits show the year of manufacture (e.g. 68: 1950 + 68 = 2018) and the next three digits indicate the day of the year of manufacture (e.g. 285 = 12th October). The last two digits indicate the serial order number in the series. In addition, each component is identified by a lot number mentioned. For more information, refer to the instructions for use of the KITETT KOLOR KIT EXPRESSION.

11. Electromagnetic compatibility (EMC)

Electro medical devices require special precautions against EMC and must be installed and set to operate in accordance with the EMC information provided in this manual.

WARNING: Avoid using this unit near or stacked onto other equipment as this may cause a malfunction. If this is necessary, this appliance and the other appliances should be monitored for normal operation.

WARNING: Use of components, transducers and cables other than those specified or supplied by the manufacturer of this device may cause an increase in electromagnetic emissions or a decrease in the immunity of this device and cause improper operation.

WARNING: Portable RF communication devices (including peripherals such as antenna cables and external antennas) should not be used closer than 12 inches (30 cm) to any part of the FISIO breast pump, including cables specified by the manufacturer. Otherwise, performance may be impaired.

WARNING: If the device malfunctions due to electromagnetic disturbances, the performance of the device may be impaired and result in delayed patient care.

Wireless communication devices such as wireless home networking devices, mobile phones, cordless phones and their bases, walkie-talkies can affect the device, and it is recommended to keep them at a distance of at least 0.5 m from the device.

Guidance and manufacturer's declaration – electromagnetic emissions

Emissions test	Compliance
RF emissions (CISPR 11)	Group 1
RF emissions (CISPR 11)	Class B
Harmonic emissions (IEC 61000-3-2)	Class A
Voltage fluctuations / flicker emissions (IEC 61000-3-3)	Complies

Guidance and manufacturer's declaration – electromagnetic immunity		
Immunity test	IEC 60601 test level	Compliance level
Electrostatic discharge (ESD) (IEC 61000-4-2)	± 8 kV contact ± 15 kV air	± 8 kV ± 15 kV
Electrical fast transient / burst (IEC 61000-4-4)	± 2 kV for power supply lines ± 1 kV for input/output lines	± 2 kV ± 1 kV
Surge (IEC 61000-4-5)	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth
Voltage dips, short interruptions and voltage variations on power supply input lines (IEC 61000-4-11)	0% U _T ; 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0% U _T ; 1 cycle AND 70% U _T ; 25/30 cycles Single phase: at 0°	At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0% U _T ; 1 cycle AND 70% U _T ; 25/30 cycles Single phase: at 0°
Voltage interruptions (IEC 61000-4-11)	0% U _T ; 250/300 cycles	0% U _T ; 250/300 cycles
Power frequency (50/60 Hz) magnetic field (IEC 61000-4-8)	30 A/m	30 A/m
Radiated RF (IEC 61000-4-3)	10 V/m 80 MHz to 2.7 GHz 385 MHz at 27 V/m 450 MHz at 28 V/m 710, 745, 780 MHz at 9 V/m 810, 870, 930 MHz at 28 V/m 1720, 1845, 1970 MHz at 28 V/m 2450 MHz at 28 V/m 5240, 5500, 5785 MHz at 9 V/m	10 V/m 80 MHz to 2.7 GHz 385 MHz at 27 V/m 450 MHz at 28 V/m 710, 745, 780 MHz at 9 V/m 810, 870, 930 MHz at 28 V/m 1720, 1845, 1970 MHz at 28 V/m 2450 MHz at 28 V/m 5240, 5500, 5785 MHz at 9 V/m
Conducted RF (IEC 61000-4-6)	3V 0,15 MHz – 80 MHz 6V in ISM bands and amateur radio bands between 0.15 MHz and 80 MHz 80% MA at 1kHz	3V 0,15 MHz – 80 MHz 6V in ISM bands and amateur radio bands between 0.15 MHz and 80 MHz 80% MA at 1kHz

12. Pictograms

Symbol on the breast pump, its components and on the packaging

In accordance with Directive 93/42/CEE of 14th June 1993 regarding medical devices

Manufacturer

Reference

Serial number (breast pump)

Batch number (components)

Device class II

IP30

Level of protection:

3:protected against ingress of solid foreign objects with a diameter ≥ 2.5 mm
0:not protected (against water penetration)

Applied part type BF

Alternative current

Follow instructions for use

Caution/Warning

Keep dry; Fear of moisture

Fragile; Handle with Care

Store away from light

This symbol indicates the limit of atmospheric pressure not to be exceeded during operation, transport or storage.

This symbol indicates the moisture limit which must not be exceeded in the case of operation, transport, and storage.

This symbol indicates the temperature limit which must not be exceeded in the case of operation, transport, and storage.

«Off» (power off)

«On» (power on)

Do not throw away. Product on the market since 13th August 2005.

BPA free

International screw thread

Electric breast pump with intermittent medium suction and low flow.

Attachment № 1 to the Instructions for use: ELECTRIC BREAST PUMP KITETT, VERSION FISIO WITH ACCESSORIES

FOR RUSSIAN FEDERATION ONLY

The Instruction for use for electric breast pump KITETT, version FISIO, developed by DTF Medical in accordance with local and international requirements, is standard for all countries. In accordance to Russian legislation related to Medical devices, supplied to the Russian Federation, this Instruction for use for electric breast pump KITETT, version FISIO is supplemented by this Attachment № 1, which is an integral part of this instruction.

Following chapters of the Instruction for use are supplemented:

Chapter 2 "Product description"

Full name of medical device:

Electric breast pump KITETT, versions: FISIO, FISIO PRO with accessories

Electric breast pump KITETT version FISIO, composed of:

1. Electric breast pump KITETT FISIO - 1 pcs.
2. Single KITETT KOLOR KIT EXPRESSION for KITETT breast pump – up to 10 pcs. , where applicable, composed of:
 - KITETT KOLOR breast shield for KITETT breast pump, size range: 21 S, 24 S, 26 S, 30 S, 21 L, 24 L, 26 L, 30 L - 1 pcs. from size range;
 - Collection container for breast milk collecting and storing, 100 ml - 1 pcs .;
 - Cap for the collection container for breast milk collecting and storing - 1 pcs .;
 - PVC tube - 1 pcs .;
 - Interface bottle - 1 pcs.
 - Kit instructions for use - 1 pcs.
3. Double KITETT KOLOR KIT EXPRESSION for KITETT breast pump - up to 10 pcs. , where applicable, composed of:
 - KITETT KOLOR breast shield for KITETT breast pump, size range: 21 S, 24 S, 26 S, 30 S, 21 L, 24 L, 26 L, 30 L – 2 pcs. from size range;
 - Collection container for breast milk collecting and storing, 100 ml - 2 pcs .;
 - Cap for the collection container for breast milk collecting and storing - 2 pcs .;
 - PVC tube - 2 pcs.;
 - Interface bottle - 1 pcs .;
 - Kit instruction for use - 1 pcs.
4. Autoclavable single KITETT KOLOR KIT EXPRESSION for KITETT breast pump - up to 10 pcs., where applicable, composed of:
 - Autoclavable KITETT KOLOR breast shield for KITETT breast pump, size range: 21 S, 24 S, 26 L, 30 L - 1 pcs. from size range;
 - Collection container for breast milk collecting and storing, 100 ml - 1 pcs .;
 - Cap for the collection container for breast milk collecting and storing - 1 pcs .;
 - Silicone tube - 1 pcs .;
 - Interface bottle - 1 pcs .;
 - Kit instruction for use - 1 pcs.
5. KITETT FISIOCOOL kit for breast milk cooling - up to 30 pcs., where applicable, composed of:
 - Cooling element KITETT - 2 pcs.
 - Kit instruction for use - 1 pcs.

6. KITETT FISIOBIB kit for breast milk collecting and storing - up to 30 pcs., where applicable, composed of:

- Collection container for breast milk collecting and storing, 100 ml - 2 pcs.;
- Cap for the collection container for breast milk collecting and storing - 2 pcs.;
- Kit instruction for use - 1 pcs.

7. KITETT FISIOFRESH kit for breast milk cooling and transporting - up to 10 pcs., where applicable, composed of:

- Collection container for breast milk collecting and storing, 100 ml - 4 pcs.
- Cap for the collection container for breast milk collecting and storing - 4 pcs
- Thermobag KITETT - 1 pcs.;
- Cooling element KITETT - 4 pcs.;
- Bag for accessories storing - 1 pcs.;
- Kit instruction for use - 1 pcs.

8. Instruction for use - 1 pcs

9. Kolorama size ruler - 1 pcs.

Accessories:

1. Foam filter - up to 10 pcs.

2. KITETT soft transport bag for the KITETT electric breast pump - 1 pcs.

Application scope of the medical device:

The electric breast pump KITETT, version FISIO is applied in medical centers as well as is used at home.

Short names used in this Instruction for use:

Full name	Short names
Electric breast pump KITETT version FISIO	Breast pump FISIO breast pump KITETT FISIO breast pump KITETT electric breast pump KITETT breast pump KITETT FISIO electric breast pump
KITETT KOLOR breast shield for KITETT breast pump	KITETT KOLOR breast shield KOLOR breast shield Breast shield
Autoclavable KITETT KOLOR breast shield for KITETT breast pump	Autoclavable KITETT KOLOR breast shield Autoclavable KOLOR breast shield
Single KITETT KOLOR KIT EXPRESSION for KITETT breast pump	KITETT KOLOR KIT EXPRESSION Single KITETT KOLOR KIT EXPRESSION
Double KITETT KOLOR KIT EXPRESSION for KITETT breast pump	KITETT KOLOR KIT EXPRESSION Double KITETT KOLOR KIT EXPRESSION
Autoclavable single KITETT KOLOR KIT EXPRESSION for KITETT breast pump	Autoclavable KITETT KOLOR KIT EXPRESSION Autoclavable KIT EXPRESSION
KITETT FISIOBIB kit for breast milk collecting and storing	KITETT FISIOBIB
KITETT FISIOCOOL kit for breast milk cooling	KITETT FISIOCOOL
KITETT FISIOFRESH kit for breast milk cooling and transporting	KITETT FISIOFRESH
Collection container for breast milk collecting and storing	Container Collection and storing container
Cap for the collection container for breast	Cap

milk collecting and storing	Cap for container
PVC tube	Tube Suction tube
Silicone tube	Tube Suction tube

Chapter 4 «Use»

Installation of the breast pump before use

1. Wash your hands with hot and soapy water.
2. Put the interface bottle into the special place with a holder in the right corner of the breast pump.
3. Connect the tube of the interface bottle with the output nozzle on the breast pump.
4. Connect one extremity of the suction tube (PVC/silicon) with the interface bottle and the second extremity - with the inlet hose on the breast shield.

Chapter 5 «Breastfeeding»

Information of breast milk storage conditions is available on the websites www.dtf.fr, www.kitett.com, www.koloryou.com and KOLORYOU mobile application.

Chapter 9, «Warranty / Recycling»

Warranty

Authorized representative of the manufacturer in the Russian Federation, providing warranty and post-warranty service and support:

BioSystems LLC.,

197046, Saint-Petersburg, Pinsky lane, 3, lit. A

Phone: (812) 320-49-49 Fax: (812) 320-49-40, e-mail: main@bioline.ru

Web site: www.bioline.ru

Hotline 8-800-333-00-49

Recycling of the electric breast pump KITETT FISIO

According to the legal provisions, the KITETT FISIO breast pump refers to waste electrical and electronic equipment.

Decommissioning or disposal of the medical device is carried out in accordance with European Directive 2002/96/CE.

The device should be handed over to a special waste collection center. Do not throw it in the environment or with household waste in accordance with WEEE directive 2002/96/CE.

The disposal should be carried out in accordance with applicable local, regional, and national laws — removal to an approved waste storage space or incineration under approved controlled conditions.

In accordance with rules for medical waste of class A (Non-hazardous Waste) specified in "SanPiN 2.1.7.2790-10 (Sanitary and Epidemiological Requirements for Medical Waste Management)", the breast pumps should be disposed as the waste of class A.

Materials, of which the components (KITETT KOLOR KITs EXPRESSION, KITETT FISIOBIB kits, KITETT FISIOCOOL kits, KITETT FISIOFRESH kits, collection containers for breast milk collecting and storing,

caps for collection containers and tubes) are made, do not contain chlorine or its derivatives. Therefore, they can be disposed of as household waste, including by incineration without dioxin formation.

Recycling of the accessories

The accessories should be disposed of in accordance with applicable local, regional, and national laws — removal to an approved waste storage space or incineration under approved controlled conditions and in accordance with the rules for medical waste of class A (Non-hazardous Waste) specified in “SanPiN 2.1.7.2790-10 (Sanitary and Epidemiological Requirements for Medical Waste Management)”.

Materials of which the accessories are made do not contain chlorine or its derivatives. Therefore, they can be disposed of as household waste, including by incineration without dioxin formation.

Chapter 11 «Electromagnetic Compatibility (EMC)»

Medical device meets the requirements of Technical Regulation of Customs Union (TRCU) 020/2011 “Electromagnetic compatibility of technical devices” (this information will be included in the instruction for use after confirmation of compliance and receipt of the Declaration of conformity TRCU)

Chapter 13 «Maintenance of the medical device»

Periodic preventive maintenance

Periodic preventive maintenance (once a year) and access to internal parts of the breast pump should be carried out only by authorized technical specialists of the service department.

If necessary, apply for warranty and post-warranty maintenance in Russia to the center authorized by DTF Medical.

Periodicity of checks

Periodicity	After use	After correct maintenance	Before use	Trim.	Year
Check points					
External visual control	●	●	●		●
Internal visual control		●			
Electrical safety test		●			●
Hygiene (internal)	●	●	●		●
Hygiene (foam filter)					●
Hygiene (external)	●		●	●	●
Performance tests		●	●		●
Hygiene (disinfection of the kit)	DEFINE ACCORDING TO NEEDS				

Independently can be performed:

1. Cleaning/disinfection of external parts of the breast pump.

2. Replacement of a foam filter.
3. Internal circuit cleaning.

Cleaning/disinfection of external parts of the breast pump are particularly described in the Chapter 3 "Cleaning and disinfection".

Replacement of a foam filter and internal circuit cleaning are recommended to perform in following circumstances:

1. If you observe liquid in the tube between the interface bottle and the breast pump,
2. If suction does not work correctly,
3. During routine checks or after annual preventive maintenance.


Replacement of a foam filter:

There is a filter on the rear of the device that is designed exclusively to protect the breast pump from liquid penetration that may damage the internal elements of the system.

Warnings:

- Do not turn the device on if you see liquid between the interface bottle and the breast pump.
- Remove the interface bottle before storage.
- Empty and clean the interface bottle before storage.

Internal circuit cleaning:

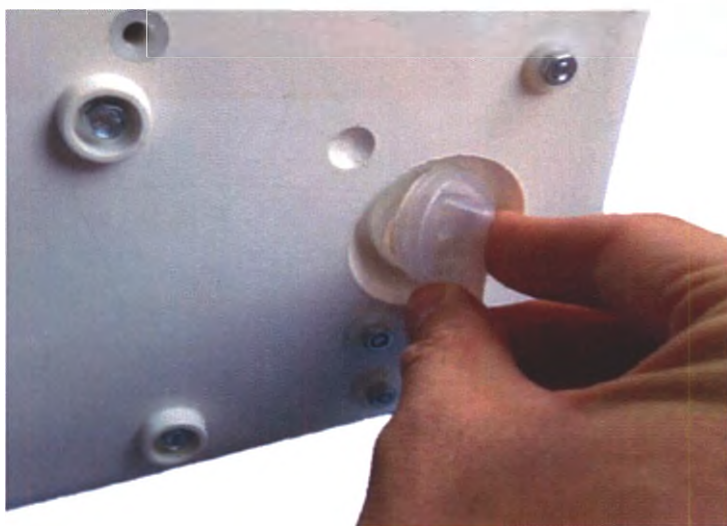STEP 1

Remove the plug located on the rear side of the breast pump, get the foam filter from the outlet.

STEP 2

Set the device over a sink or any other reservoir, be sure that the outlet (on the rear side of the breast pump) is still above.

STEP 3

Put an extremity of the silicone tube on the breast pump output nozzle and place the other in a container with warm water (0.5L at 40-60°C).

STEP 4

Switch on the device. Water from the container will go through the internal circuit of the breast pump and leak away from the drain hole. Let it rinse until the water comes out clean.

At the end, remove the tube from the container. Let the device on until the ejection of the water. Leave the device out of service on, at least 20 minutes to dry the internal circuits. After that, place the new foam and close the plug.

Chapter 14 «Transportation and storage conditions»

Depending on stability to climatic effects, the medical device is applicable to the climate version UHL 4.2.

Depending on perceptible mechanical effects, medical device is applicable to group 2 – wearable, transportable and removable, not intended to work during transportation or movement within a healthcare facility.

Transportation by all types of enclosed transport and storage at warehouses is allowed under the following conditions:

Parameters	Value
Temperature at transportation and storage	-25°C - +70°C
Relative humidity at transportation and storage	15%...93%
Atmosphere pressure at transportation and storage	700 hPa – 1060 hPa

Instructions for use

kitett

FISIO PRO

Electric breast pump
with accessories

Associated with KITETT KOLOR KIT EXPRESSION

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Phone number: +33.(0)4.77.74.51.11 – Fax :+33.(0)4.77.79.67.72

INFO@CIRMI.RU WWW.GOSZDRAVNADZOR.RU

1. Intended use	3
a) Indications.....	3
b) Contraindications and adverse events	3
2. Product description	3
a) Detailed view and description	3
b) Selecting the breast shield size.....	4
3. Cleaning and disinfection.....	4
a) Breast pump cleaning and disinfection	4
b) Components cleaning at home.....	5
c) Disinfection at home.....	5
d) Cleaning and sterilization by healthcare establishments	6
4. Use	6
a) Breast pump functioning	6
b) Components preparation	6
c) Using the breast pump	6
d) After each use	7
5. Breastfeeding	7
a) Storing breast milk.....	7
b) Baby's breastfeeding physiology	7
6. Safety precautions.....	7
7. Problem solving.....	8
8. List of components and spare parts available	9
9. Warranty / Recycling	10
10. Technical specifications	10
11. Electromagnetic compatibility (EMC).....	11
12. Pictograms	12

1. Intended use

a) Indications

Breast pumps are designed to be used by lactating women in order to extract their maternal milk. They help them to relieve their mastitis or breast engorgement and facilitate their healing. They also help mothers to relieve cracked or painful nipples or when breastfeeding is insufficient or impossible (inverted or painful nipples, etc.). Furthermore, they give the possibility to mothers to breastfeed their baby if they cannot be directly breastfed (preterm birth or neonatal pathologies (cleft lip and/or palate), difficulty of the baby to suck, etc.). Finally, they allow to initiate or maintain lactation, especially in case of a medical treatment incompatible with breastfeeding.

b) Contraindications and adverse events

There are no known contraindications for the use of the KITETT FISISIO PRO electric breast pump.

Adverse event: a risk of allergy with the materials (thermoplastic elastomer) in contact with the body parts can appear.

2. Product description

The KITETT FISISIO PRO breast pump is an electric pump. It can be used in case of temporary separation or any other reason making breastfeeding impossible or not sufficient. Easy to use, it is ideal for active mothers.

The FISISIO PRO pump can be used in single or double pumping and should be plugged into the power supply. It combines a suction power adjustment with a set-up rhythm. The rhythm and the suction power can be **independently** adjusted. It tends to reproduce the baby's sucking rhythm: the mother experiences the stimulation and expression phases.

All KITETT KOLOR breast shield sizes connect to the breast pump. The mother can thus have a suitable sized breast shield for her breast morphology to provide optimal comfort for an effective, long-lasting breastfeeding experience.

The breast shield is a one-piece which favours an optimized hygiene, and makes it possible not to touch the thread nor the internal parts of the feeding collection container and the breast shield.


a) Detailed view and description

The breast pump design is presented on the first page. The user interface is presented as below:

Single KITETT KOLOR KIT EXPRESSION: contains 1 KOLOR breast shield, 1 collection container for breast milk collecting and storing, 100 mL, 1 cap for the collection container for breast milk collecting and storing, 1 interface bottle and 1 PVC tube.

Available in autoclavable version on request.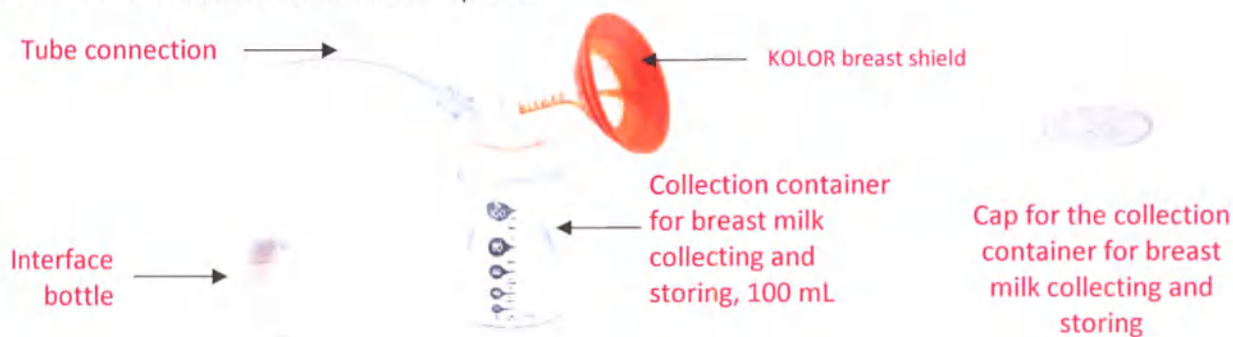

Double KITETT KOLORKIT EXPRESSION: contains 2 KOLOR breast shields, 2 collection containers for breast milk collecting and storing, 100 mL, 2 caps for the collection containers for breast milk collecting and storing, 1 interface bottle and 2 PVC tubes.

Every KIT EXPRESSION and spare parts available and compatible with the FISIO PRO breast pump are presented in Part 8.


NOTE: KOLOR breast shields, collection container for breast milk collecting and storing, cap for the collection container for breast milk collecting and storing and interface bottle are in polypropylene and thermoplastic elastomer. KOLOR breast shields are guaranteed BPA and Phthalate free.


b) Selecting the breast shield size

The FISIO PRO breast pump is used with the KITETT KOLORKIT EXPRESSION. The KOLOR breast shield is available in 4 inner diameters and 2 «comfort» sizes that adapt perfectly to the breast morphology and guarantee a high level of comfort.

4 inner diameters: 4 colour codes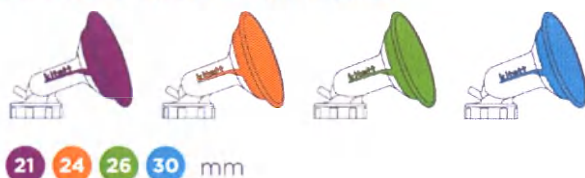

2 comfort sizes: S = 76 mm and L = 90 mm

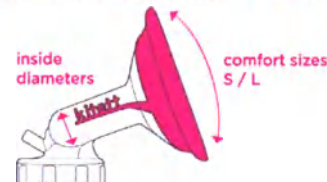
KITETT helps you choose the breast shield using:

- the Kolorama size ruler


- the KOLORYOU application available for free download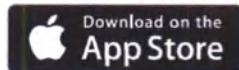

Measurements:

Measurement should be taken after breastfeeding or expressing your milk. Alternatively, if your nipple is at rest, add an extra 2mm around it to select the correct inner diameter size.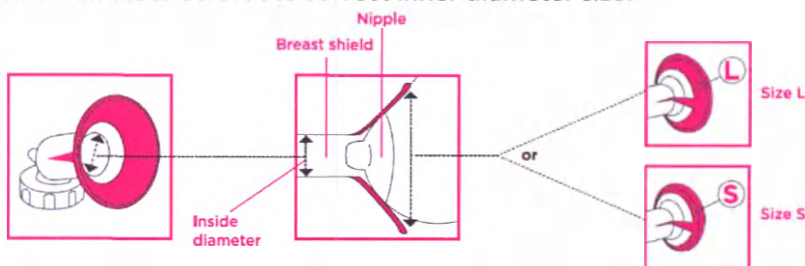

The nipple should not feel too constricted neither too free. You should see space around your nipple in order to let it move easily in the breast shield during pumping. If the breast shield is too tight, it could cause pain, lesions or compressed ducts. If the breast shield is too wide, less milk will be expressed. A correct comfort size (S -Small- or L -Large- comfort shape) will improve breast pump efficiency (optimal airtightness).

3. Cleaning and disinfection

a) Breast pump cleaning and disinfection

Disinfection is done with a clean and damp cloth that is impregnated with a surface agent disinfectant for medical devices. Follow the instructions provided by the supplier of the surface disinfectant agent. Do not use abrasive cleaning agents. Unplug the device before cleaning.

b) Components cleaning at home

Components concerned:

- KOLOR breast shield
- Collection container for breast milk collecting and storing
- Cap for the collection container for breast milk collecting and storing

Before using for the first time and after each use:

- ① Disassemble each element.
- ② Rinse every part in contact with the milk in cold water (about 20°C).
- ③ Then clean with hot and soapy water.
- ④ Rinse every part.
- ⑤ Let them dry in an open air on a paper towel or a clean linen.

NOTES:

- It is better to clean all parts in contact with the breast milk immediately after using the breast pump. It facilitates the cleaning and avoids the development of bacteria.
- If condensation appears inside the tube or when milk enters inside, rinse the tube in order to remove liquid residues.
- The interface bottle prevents any liquid penetration into the breast pump. It is therefore important to clean it with soapy water and disinfect it in boiling water (10 minutes), as soon as it has been soiled. Check if the breast pump is not wet or damp. If it is, do not use it.
- Do not use the milk that penetrated into the interface bottle or tubes. The tubes should be briefly dipped into boiling water (10 minutes) and then thoroughly dried. The tubing colour may change but it will not alter its good functioning.

c) Disinfection at home

Components concerned:

- KOLOR breast shield
- Collection container for breast milk collecting and storing
- Cap for the collection container for breast milk collecting and storing

For full-term babies (without medical problems), disinfect before the first use and once a day. For premature babies or those with weakened immune systems, disinfect before the first use and after each use.

- ① Disinfect every part in boiling water (10 minutes), in a dishwasher or in a microwave steriliser*.

*Refer to the manufacturer's data for the microwave sterilizer without exceeding 1000 Watt.

- ② Let them dry in an open air on a paper towel or clean linen.

NOTES:

- In the case of intensive disinfection (premature infants or with a weakened immune system), boiling water is the only method that guarantees an optimal lifetime of the components.
- If the disinfection takes place in boiling water and in case of hard water, 2 or 3 drops of white vinegar can be added in the water to avoid whitish deposits on the parts.
- In the case of disinfection with a dishwasher, the plastic parts may be discoloured by food pigments. It does not affect their functioning.
- Regularly check the condition of your breast shields during use, cleaning and disinfection.

	Breast shield	Collection container	Cap	Tubes & interface bottle	Breast pump
Before the first use and after each use					
Cleaning	✓	✓	✓	x	x
Before the first use and once a day					
Disinfection	✓	✓	✓	x	x
When it has been soiled					
Cleaning & disinfection	x	x	x	✓	x
Before passing on to the next mother					
Cleaning & disinfection (Cloth with disinfectant)	x	x	x	x	✓

d) Cleaning and sterilization by healthcare establishments

The autoclavable KITETT KOLORKIT EXPRESSION contains 1 autoclavable KOLOR breast shield, 1 collection container for breast milk collecting and storing, 1 cap for collection container for breast milk collecting and storing, 1 interface bottle and 1 silicone tube.

Autoclavable KOLOR breast shields, collection containers for breast milk collecting and storing, caps for the collection containers for breast milk collecting and storing and silicone tubes have been designed to be sterilized at 125°C during 30 minutes or 134°C during 18 minutes. The total number of cycles should not exceed 50 cycles. Do not use the breast shield in case of damage or failure. The interface bottle is not autoclavable. It is therefore important to discard it as soon as it has been soiled. For more information, please refer to the autoclavable KITETT KOLORKIT EXPRESSION instruction for use.

4. Use

Note: Follow the exact order of the steps below to get the correct use of the breast pump.

a) Breast pump functioning

The FISIO PRO breast pump should be plugged into the power supply.

- Check that the values on the breast pump (volts and Hz) corresponds to the values of your distribution network.
- Connect the breast pump to the power supply.

NOTES:

- Check if the breast pump is suitable for the wall outlet.

b) Components preparation

NOTES:



- Only use the components provided in the kit or those manufactured by DTF MEDICAL.
- Before use, check that the KIT EXPRESSION components are free from defect, damage or sign of wear. If this is the case, replace them.
- All the components must be completely dry before use.

Before the first use, please refer to chapter 3 for cleaning and disinfection.

- 1 Wash your hands with hot and soapy water before handling the breast pump parts.
- 2 Screw the KOLOR breast shield onto the collection container for breast milk collecting and storing.
- 3 Connect the suction tube to the inlet hose of the KOLOR breast shield and to the interface bottle on the unit.

Set up of the single KITETT KOLOR KIT EXPRESSION:


Set up of the double KITETT KOLOR KIT EXPRESSION (can also be used as single one):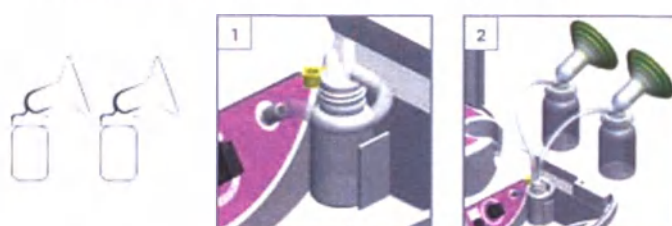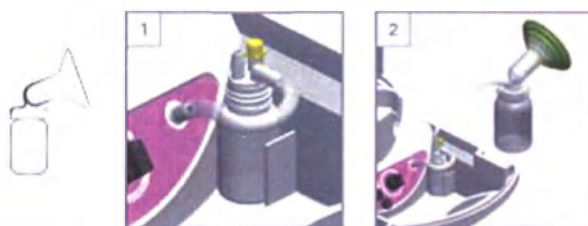

WARNING: Correctly close the interface bottle cap when the breast pump is used on simple pumping.

c) Using the breast pump

- 1 Place the KOLOR breast shield correctly over your breast ensuring the milk can flow freely into the collection container for breast milk collecting and storing
- 2 Ensure your nipple is centred inside the KOLOR breast shield. The flexible part of the KOLOR breast shield must also be properly positioned on your breast to ensure complete sealing.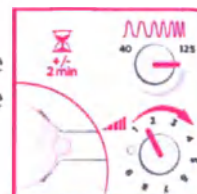

④ The cycle button allows adjusting the suction cycles. The suction button graduated from 1 to 9 allows adjusting the suction from minimum to maximum. To stimulate the lactation, set the suction button to 1 (lowest suction) and the cycle button to 125 (quick cycles).



④ Switch on the breast pump (position "I").

It is necessary to start with this stimulation phase in order to stimulate the milk arrival and to activate the milk ejection reflex. This phase should last about 2 minutes. According to your own comfort level, you may adapt these settings by increasing the suction force and/or decreasing the cycle.

⑤ After the ejection reflex, select progressively a slower cycle by turning the cycle button clockwise and increase the suction button to obtain a good expression according to your own level of comfort. Milk expression is optimal at the level of maximum depression and which should not be painful.

⑥ Check the proper flow of milk inside the collection container for breast milk collecting and storing.

⑦ After expressing the milk, switch off the breast pump (position "O").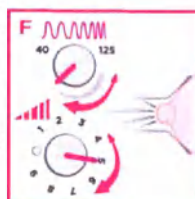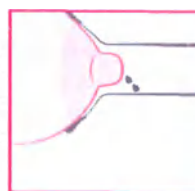

NOTES:

- Always unplug the breast pump from the power supply after use.
- Do not fill the collection container for breast milk collecting and storing beyond the marking.
- Be careful not to pinch the tubes during use.

d) After each use

① Disconnect the KOLOR breast shield tube(s) and unscrew the KOLOR breast shield(s) from the collection container(s) for breast milk collecting and storing.

② Secure the cap(s) onto the collection container(s).

③ Disconnect the tube(s) from the interface bottle. Check that no milk has penetrated into the interface bottle or inside the tubes.

④ After each use, rinse and clean every part as described in chapter 3.

⑤ Store the breast pump and its accessories in a dry place away from sunlight.

5. Breastfeeding

a) Storing breast milk

Information on the preservation of breast milk is available on the website www.dtf.fr and on the application KOLORYOU.

b) Baby's breastfeeding physiology

The FISIO PRO breast pump provides mothers with the natural sensation of breastfeeding by recreating the way in which the baby feeds.

It reproduces the 2 physiological phases of the breastfeeding:

- The stimulation phase to generate the milk let-down (short, rapid movements),
- The expression phase to collect milk (longer, less frequent movements).

6. Safety precautions

Use of electricity

- Before connecting the breast pump to the power supply, check the values on the breast pump (Volt –V- and Hertz –Hz-) with your distribution network.
- The warranty does not cover faulty connections.
- This medical device is designed to operate at altitudes less than or equal to 3 000 meters.
- Do not open the breast pump when connected to the power supply.
- Warning: this medical device is directly connected to the electric power supply, do not use it in bath or under the shower, do not immerse.
- Keep this medical device away from any high heat source.

- Connect the breast pump to an easy-to-access socket to allow quick disconnection of the network in the event of a problem.
- The plug is the disconnection device and must always be accessible.

Use of breast pump

- The user of this medical device is the breastfeeding mother.
- This medical device must not be left within the reach of unsupervised children or animals (potential hazards with tubing and detachable components, risk of electrification).
- The breast pump should be installed and started up in accordance with the information supplied in the accompanying documents.
- Properly store the breast pump and its components after use.
- An animal may gnaw at the power cord. Be sure to check the condition of the equipment before use.
- Do not use the appliance outdoors, near a heat source or near a water source (bathtub, sink ...).
- Make sure the unit is not exposed to direct sunlight or excessive heat.
- This medical device must not be used in presence of a mixture of inflammable anaesthetic with air or oxygen or nitrous oxide.
- After use, unplug the appliance from the wall outlet.
- Fragile medical device, to be handled with care.
- No special knowledge or training is required to operate this equipment. Please read the instructions carefully before use.
- Keep these instructions in a safe place with the appliance as it contains the necessary information for the proper use of the equipment.
- This medical device is to be used either in health care establishment or at home.
- The user can safely use all the functions of the breast pump.
- For hygiene reasons, a new complete KITETT KOLOR KIT EXPRESSION must be sold to each mother.

WARNINGS:

- Do not touch the breast pump box for more than 1 second during use.
- The tube connecting the collection container to the breast pump can reach the temperature of 43.5°C. The user should not touch the tube for more than 10 minutes during the breast pump use.

Notes on the functionality of the breast pump

- The KITETT KOLOR breast shield with an international screw thread cap (right) can be used only with a matching collection container (same screw thread cap).
- Switch off the breast pump when the collection container is filled in order to prevent milk from getting into the interface bottle.
- If the device has to be rinsed, entrust the operation to a maintenance professional (risk of bad handling).


CAUTION, in all cases:

- WARNING: Always disconnect the plug from the power supply before opening it.
- WARNING: Only an approved customer service is authorized to repair the breast pump.
- WARNING: Modifications are forbidden.
- WARNING: Only use the components supplied and mentioned in the instructions for use.
- WARNING: The warranty does not cover connection errors with non-compatible components.
- WARNING: Do not undertake service or maintenance operations while the breast pump is used.

7. Problem solving

Problem	Solution
No operating noise can be heard from the pump	Make sure that the power cord is connected to the power supply. Check whether the switch is on the operating position. Check that the current delivered by the power supply corresponds to that of the breast pump (Volt Hz). Check with another breast pump if power supply is available in the wall outlet. If the problem continues, please contact DTF MEDICAL or your local distributor.
There is low or no suction	Check that the KOLOR breast shield is fully screwed onto the collection container (screw tightly). Check that every tubes properly connected and not pinched.

Problem	Solution
	Check that the cap of interface bottle correctly fits. Check the condition of the O-ring between the KOLOR breast shield and the collection container. Check that there is no residue in the tube. If the problem continues, please contact DTF MEDICAL or your local distributor.
The suction is too strong	Switch off then switch on the breast pump. If the problem continues, please contact DTF MEDICAL or your local distributor.
The breast pump stops abruptly when used	Disconnect the tube from the interface bottle to vent it. In case of alternating stop, please contact DTF MEDICAL or your local distributor.
The pump sucks continuously	Do not tear off the breast shield from your breast (could cause pain and/or damage). Insert your finger between your breast and the breast shield to let the air come in and remove it more easily or disconnect the tube from the breast shield.
A piece or an accessory is missing	Please contact DTF MEDICAL or your local distributor.
The breast pump is too noisy or malfunctioning	Please contact DTF MEDICAL or your local distributor.
Milk entered inside the interface bottle	Clean it as soon as it has been soiled or discard it if you are in a healthcare establishment. Check if the breast pump is not wet or damp. If it is, do not use the breast pump. Do not use the milk that penetrated into the interface bottle or tubes. The tubes should be briefly dipped into boiling water and then thoroughly dried. Switch off the breast pump.
Milk was spilt or entered inside the breast pump	Unplug the breast pump for the power supply. Let the breast pump in a dry and warm place during 24 hours. Do not expose the breast pump to direct sunlight. Do not accelerate the drying process.

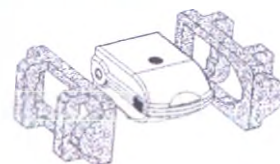Please contact DTF MEDICAL or your local distributor:

- for assistance, if necessary, in the assembly, use or maintenance of the breast pump,
- or,
- to report any unexpected operation or event.

WARNING: Keep the specific packaging and foam set for sending back the breast pump.

8. List of components and spare parts available

Part designation	Item code
KITETT KOLOR KIT EXPRESSION	See below
Autoclavable KITETT KOLOR KIT EXPRESSION	See below
KITETT KOLOR breast shield	See below
Autoclavable KITETT KOLOR breast shield	See below
KITETT FISOBIB – Batch of 2 collection containers for breast milk collecting and storing 100mL and 2 caps for the collection containers for breast milk collecting and storing (international screw thread)	FIBIBN-E
KITETT FSIOCOOL – Set of 2 cooling elements KITETT	FICOOL
KITETT FSIOFRESH – 4 collection containers for breast milk collecting and storing, 4 caps for the collection containers for breast milk collecting and storing, 4 cooling elements KITETT, 1 KITETT thermobag and 1 bag for accessories storing	FIFRESH-E
PVC tube, 1 m-length, for KITETT KOLOR KIT EXPRESSION	Y1091
Silicone tube, 1 m-length, for autoclavable KITETT KOLOR KIT EXPRESSION	TU32
Interface bottle	W2
KITETT soft transport bag for the KITETT electric breast pump	MBOXK
Foam filter	YR36

KITETT KOLOR KIT EXPRESSION:

Kit Ref.	Breast shield size	Shape size	Single / Double	Kit Ref.	Breast shield size	Shape size	Single / Double
K21ES	21mm	Small	Single	K24ES	24mm	Small	Single
K21EL	21mm	Large	Single	K24EL	24mm	Large	Single
K21ESD	21mm	Small	Double	K24ESD	24mm	Small	Double

K21ELD	21mm	Large	Double	K24ELD	24mm	Large	Double
K26ES	26mm	Small	Single	K30ES	30mm	Small	Single
K26EL	26mm	Large	Single	K30EL	30mm	Large	Single
K26ESD	26mm	Small	Double	K30ESD	30mm	Small	Double
K26ELD	26mm	Large	Double	K30ELD	30mm	Large	Double

Autoclavable KITETT KOLOR KIT EXPRESSION:

Kit Ref.	Breast shield size	Shape size	Kit Ref.	Breast shield size	Shape size
K21ESH	21mm	Small	K24ESH	24mm	Small
K26ELH	26mm	Large	K30ELH	30mm	Large

KITETT KOLOR breast shield:

Kit Ref.	Breast shield size	Shape size	Kit Ref.	Breast shield size	Shape size
WB21ES	21mm	Small	WB24ES	24mm	Small
WB21EL	21mm	Large	WB24EL	24mm	Large
WB26ES	26mm	Small	WB30ES	30mm	Small
WB26EL	26mm	Large	WB30EL	30mm	Large

Autoclavable KITETT KOLOR breast shield:

Kit Ref.	Breast shield size	Shape size	Kit Ref.	Breast shield size	Shape size
WB21ESH	21mm	Small	WB24ESH	24mm	Small
WB26ELH	26mm	Large	WB30ELH	30mm	Large

NOTE: There are country variants. These end with the two letters of the country code. For more information, please contact DTF MEDICAL or your local distributor.

9. Warranty / Recycling

Warranty:

- The breast pump is guaranteed for 3 years from the date of purchase against any manufacturing defect, except case of misuse.
- For after sales service under warranty, please contact DTF MEDICAL or your local distributor.
- The technical documentation is available on request.

Recycling:

The device must be disposed of in accordance with local regulation on the disposal of electrical appliances. So you contribute to the human health and environmental protection. Product is on the market since August the 13th 2005.

10. Technical specifications

Device class (breast pump)	Ila
Device class in electrical safety (breast pump)	II
Type of applied parts	BF
Protection level	IP 30
Expected shelf life of components	1 year
Expected shelf life of autoclavable components	50 cycles
Expected shelf life of the breast pump	5 years
Collection container capacity	100ml
Weight	4 kg
Dimensions	L360 x l250 x h180mm
Voltage	Available in 230V – 50Hz; 220V – 50Hz or 220V – 60Hz
Consumption	100VA
Fuses	T 0,5A H 250V
Suction values	Variable from -50 to -300hPa
Cycles	40 to 125 cycles/minutes

Sound level	<57dBa
Running mode	Continuous
Power supply	Alternative current 230V – 50Hz
Ambient temperature (T) - For transport and storage	 $-25^{\circ}\text{C} < T < +70^{\circ}\text{C}$
Ambient temperature (T) - When in functioning	$+5^{\circ}\text{C} < T < +40^{\circ}\text{C}$
Relative humidity (H) For transport, storage and functioning	 $15\% < H < 93\%$
Atmospheric pressure (P) For transport, storage and functioning	 $700\text{ hPa} < P < 1060\text{ hPa}$

Essential performance features:

- Provide adequate suction so that mothers can express their milk.
- Control the rate of suction to best meet the baby's sucking rhythm.

Applied part:KITETT KOLOR breast shield.

Identification:

Each KITETT breast pump has a unit identification (reference and number) which indicates, among other things, the exact date of manufacture and the serial number of the breast pump. The first two digits show the year of manufacture (e.g. **68**: 1950 + **68** = 2018) and the next three digits indicate the day of the year of manufacture (e.g. 285 = 12th October). The last two digits indicate the serial order number in the series. In addition, each component is identified by a lot number mentioned. For more information, refer to the instructions for use of the KITETT KOLOR KIT EXPRESSION.

11. Electromagnetic compatibility (EMC)

Electro medical devices require special precautions against EMC and must be installed and set to operate in accordance with the EMC information provided in this manual.

WARNING: Avoid using this unit near or stacked onto other equipment as this may cause a malfunction. If this is necessary, this appliance and the other appliances should be monitored for normal operation.

WARNING: Use of accessories, transducers and cables other than those specified or supplied by the manufacturer of this device may cause an increase in electromagnetic emissions or a decrease in the immunity of this device and cause improper operation.

WARNING: Portable RF communication devices (including peripherals such as antenna cables and external antennas) should not be used closer than 12 inches (30 cm) to any part of the FISIO PRO breast pump, including cables specified by the manufacturer. Otherwise, performance may be impaired.

WARNING: If the device malfunctions due to electromagnetic disturbances, the performance of the device may be impaired and result in delayed patient care.

Wireless communication devices such as wireless home networking devices, mobile phones, cordless phones and their bases, walkie-talkies can affect the device, and it is recommended to keep them at a distance of at least 0.5 m from the device.

Guidance and manufacturer's declaration – electromagnetic emissions

Emissions test	Compliance
RF emissions (CISPR 11)	Group 1
RF emissions (CISPR 11)	Class B
Harmonic emissions (IEC 61000-3-2)	Class A
Voltage fluctuations / flicker emissions (IEC 61000-3-3)	Complies

Guidance and manufacturer's declaration – electromagnetic immunity		
Immunity test	IEC 60601 test level	Compliance level
Electrostatic discharge (ESD) (IEC 61000-4-2)	± 8 kV contact ± 15 kV air	± 8 kV ± 15 kV
Electrical fast transient / burst (IEC 61000-4-4)	± 2 kV for power supply lines ± 1 kV for input/output lines	± 2 kV ± 1 kV
Surge (IEC 61000-4-5)	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth
Voltage dips, short interruptions and voltage variations on power supply input lines (IEC 61000-4-11)	0% U _T ; 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0% U _T ; 1 cycle AND 70% U _T ; 25/30 cycles Single phase: at 0°	0% U _T ; 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0% U _T ; 1 cycle AND 70% U _T ; 25/30 cycles Single phase: at 0°
Voltage interruptions (IEC 61000-4-11)	0% U _T ; 250/300 cycles	0% U _T ; 250/300 cycles
Power frequency (50/60 Hz) magnetic field (IEC 61000-4-8)	30 A/m	30 A/m
Radiated RF (IEC 61000-4-3)	10 V/m	10 V/m
	80 MHz to 2.7 GHz	80 MHz to 2.7 GHz
	385 MHz at 27 V/m	385 MHz at 27 V/m
	450 MHz at 28 V/m	450 MHz at 28 V/m
	710, 745, 780 MHz at 9 V/m	710, 745, 780 MHz at 9 V/m
	810, 870, 930 MHz at 28 V/m	810, 870, 930 MHz at 28 V/m
	1720, 1845, 1970 MHz at 28 V/m	1720, 1845, 1970 MHz at 28 V/m
	2450 MHz at 28 V/m	2450 MHz at 28 V/m
	5240, 5500, 5785 MHz at 9 V/m	5240, 5500, 5785 MHz at 9 V/m
	3V	3V
Conducted RF (IEC 61000-4-6)	0,15 MHz – 80 MHz 6V in ISM bands and amateur radio bands between 0.15 MHz and 80 MHz 80% MA at 1kHz	0,15 MHz – 80 MHz 6V in ISM bands and amateur radio bands between 0.15 MHz and 80 MHz 80% MA at 1kHz

12. Pictograms

Symbol on the breast pump, its components and on the packaging

In accordance with Directive 93/42/CEE of 14th June 1993 regarding medical devices

Manufacturer

Reference

Serial number (breast pump)

Batch number (components)

Device class II

IP30

Level of protection:

3: protected against ingress of solid foreign objects with a diameter ≥ 2.5 mm

0: not protected (against water penetration)

Applied part type BF

Alternative current

Follow instructions for use

Caution/Warning

Keep dry; Fear of moisture

Fragile; Handle with Care

Store away from light

This symbol indicates the limit of atmospheric pressure not to be exceeded during operation, transport or storage.

This symbol indicates the moisture limit which must not be exceeded in the case of operation, transport, and storage.

This symbol indicates the temperature limit which must not be exceeded in the case of operation, transport, and storage.

«Off» (power off)

«On» (power on)

Do not throw away. Product on the market since 13th August 2005.

BPA free

International screw thread

Electric breast pump with intermittent medium suction and low flow.

Attachment № 1 to the Instructions for use: ELECTRIC BREAST PUMP KITETT, VERSION FISIO PRO WITH ACCESSORIES

FOR RUSSIAN FEDERATION ONLY

The Instruction for use for electric breast pump KITETT, version FISIO PRO, developed by DTF Medical in accordance with local and international requirements, is standard for all countries. In accordance to Russian legislation related to Medical devices, supplied to the Russian Federation, this Instruction for use for electric breast pump KITETT, version FISIO PRO is supplemented by this Attachment № 1, which is an integral part of this instruction.

Following chapters of the Instruction for use are supplemented:

Chapter 2 "Product description"

Full name of medical device:

Electric breast pump KITETT, versions: FISIO, FISIO PRO with accessories

Electric breast pump KITETT version FISIO PRO, composed of:

1. Electric breast pump KITETT FISIO PRO - 1 pcs.
2. Single KITETT KOLOR KIT EXPRESSION for KITETT breast pump – up to 10 pcs., where applicable, composed of:
 - KITETT KOLOR breast shield for KITETT breast pump, size range: 21 S, 24 S, 26 S, 30 S, 21 L, 24 L, 26 L, 30 L - 1 pcs. from size range;
 - Collection container for breast milk collecting and storing, 100 ml - 1 pcs.;
 - Cap for the collection container for breast milk collecting and storing - 1 pcs.;
 - PVC tube - 1 pcs.;
 - Interface bottle - 1 pcs.
 - Kit instructions for use - 1 pcs.
3. Double KITETT KOLOR KIT EXPRESSION for KITETT breast pump - up to 10 pcs., where applicable, composed of:
 - KITETT KOLOR breast shield for KITETT breast pump, size range: 21 S, 24 S, 26 S, 30 S, 21 L, 24 L, 26 L, 30 L – 2 pcs. from size range;
 - Collection container for breast milk collecting and storing, 100 ml - 2 pcs.;
 - Cap for the collection container for breast milk collecting and storing - 2 pcs.;
 - PVC tube - 2 pcs.;
 - Interface bottle - 1 pcs.;
 - Kit instruction for use - 1 pcs.
4. Autoclavable single KITETT KOLOR KIT EXPRESSION for KITETT breast pump - up to 10 pcs., where applicable, composed of:
 - Autoclavable KITETT KOLOR breast shield for KITETT breast pump, size range: 21 S, 24 S, 26 L, 30 L – 1 pcs. from size range;
 - Collection container for breast milk collecting and storing, 100 ml - 1 pcs.;
 - Cap for the collection container for breast milk collecting and storing - 1 pcs.;
 - Silicone tube - 1 pcs.;
 - Interface bottle - 1 pcs.;
 - Kit instruction for use - 1 pcs.
5. KITETT FISIOCOOL kit for breast milk cooling - up to 30 pcs., where applicable, composed of:
 - Cooling element KITETT - 2 pcs.
 - Kit instruction for use - 1 pcs.

6. KITETT FISIOBIB kit for breast milk collecting and storing - up to 30 pcs., where applicable, composed of:
- Collection container for breast milk collecting and storing, 100 ml - 2 pcs.;
 - Cap for the collection container for breast milk collecting and storing - 2 pcs.;
 - Kit instruction for use - 1 pcs.
7. KITETT FISIOFRESH kit for breast milk cooling and transporting - up to 10 pcs., where applicable, composed of:
- Collection container for breast milk collecting and storing, 100 ml - 4 pcs.;
 - Cap for the collection container for breast milk collecting and storing - 4 pcs.;
 - Thermo bag KITETT - 1 pcs.;
 - Cooling element KITETT - 4 pcs.;
 - Bag for accessories storing - 1 pcs.;
 - Kit instruction for use - 1 pcs.
8. Instruction for use- 1 pcs.
9. Kolorama size ruler - 1 pcs.

Accessories:

1. Foam filter - up to 10 pcs.
2. KITETT soft transport bag for the KITETT electric breast pump - 1 pcs.

Application scope of the medical device:

The electric breast pump KITETT, version FISIO PRO is applied in medical centers as well as is used at home.

Short names used in this Instruction for use:

Full name	Short names
Electric breast pump KITETT version FISIO PRO	Breast pump FISIO PRO breast pump KITETT FISIO PRO breast pump KITETT electric breast pump KITETT breast pump KITETT FISIO PRO electric breast pump
KITETT KOLOR breast shield for KITETT breast pump	KITETT KOLOR breast shield KOLOR breast shield Breast shield
Autoclavable KITETT KOLOR breast shield for KITETT breast pump	Autoclavable KITETT KOLOR breast shield Autoclavable KOLOR breast shield
Single KITETT KOLOR KIT EXPRESSION for KITETT breast pump	KITETT KOLOR KIT EXPRESSION Single KITETT KOLOR KIT EXPRESSION
Double KITETT KOLOR KIT EXPRESSION for KITETT breast pump	KITETT KOLOR KIT EXPRESSION Double KITETT KOLOR KIT EXPRESSION
Autoclavable single KITETT KOLOR KIT EXPRESSION for KITETT breast pump	Autoclavable KITETT KOLOR KIT EXPRESSION Autoclavable KIT EXPRESSION
KITETT FISIOBIB kit for breast milk collecting and storing	KITETT FISIOBIB
KITETT FISIOCOOL kit for breast milk cooling	KITETT FISIOCOOL
KITETT FISIOFRESH kit for breast milk cooling and transporting	KITETT FISIOFRESH
Collection container for breast milk collecting and storing	Container Collection and storing container
Cap for the collection container for	Cap

breast milk collecting and storing	Cap for container
PVC tube	Tube Suction tube
Silicone tube	Tube Suction tube

Chapter 4 "Use"

Installation of the breast pump before use

1. Wash your hands with hot and soapy water.
2. Put the interface bottle into the special place with a holder in the right corner of the breast pump.
3. Connect the tube of the interface bottle with the output nozzle on the breast pump.
4. Connect one extremity of the suction tube (PVC/silicon) with the interface bottle and the second extremity - with the inlet hose on the breast shield.

Chapter 5 "Breastfeeding"

Information of breast milk storage conditions is available on the websites www.dtf.fr, www.kitett.com, www.koloryou.com and KOLORYOU mobile application.

Chapter 9, "Warranty / Recycling"

Warranty

Authorized representative of the manufacturer in the Russian Federation, providing warranty and post-warranty service and support:

BioSystems LLC,

197046, Saint-Petersburg, Pinsky lane, 3, lit. A

Phone: (812) 320-49-49 Fax: (812) 320-49-40, e-mail: main@bioline.ru

Web site: www.bioline.ru

Hotline 8-800-333-00-49

Recycling of the electric breast pump KITETT FISIO PRO

According to the legal provisions, the KITETT FISIO PRO breast pump refers to waste electrical and electronic equipment.

Decommissioning or disposal of the medical device is carried out in accordance with European Directive 2002/96/CE.

The device should be handed over to a special waste collection center. Do not throw it in the environment or with household waste in accordance with WEEE directive 2002/96/CE.

The disposal should be carried out in accordance with applicable local, regional, and national laws — removal to an approved waste storage space or incineration under approved controlled conditions. In accordance with rules for medical waste of class A (Non-hazardous Waste) specified in "SanPiN 2.1.7.2790-10 (Sanitary and Epidemiological Requirements for Medical Waste Management)", the breast pumps should be disposed as the waste of class A.

Materials, of which the components (KITETT KOLOR KITs EXPRESSION, KITETT FISIOBIB kits, KITETT

FISIOCOOL kits, KITETT FISIOFRESH kits, collection containers for breast milk collecting and storing, caps for collection containers and tubes) are made, do not contain chlorine or its derivatives. Therefore, they can be disposed of as household waste, including by incineration without dioxin formation.

Recycling of the accessories

The accessories should be disposed of in accordance with applicable local, regional, and national laws — removal to an approved waste storage space or incineration under approved controlled conditions and in accordance with the rules for medical waste of class A (Non-hazardous Waste) specified in "SanPiN 2.1.7.2790-10 (Sanitary and Epidemiological Requirements for Medical Waste Management)".

Materials of which the accessories are made do not contain chlorine or its derivatives. Therefore, they can be disposed of as household waste, including by incineration without dioxin formation.

Chapter 11 "Electromagnetic Compatibility (EMC)"

Medical device meets the requirements of Technical Regulation of Customs Union (TRCU) 020/2011 "Electromagnetic compatibility of technical devices" (this information will be included in the instruction for use after confirmation of compliance and receipt of the Declaration of conformity TRCU)

Chapter 13 «Maintenance of the medical device»

Periodic preventive maintenance

Periodic preventive maintenance (once a year) and access to internal parts of the breast pump should be carried out only by authorized technical specialists of the service department. If necessary, apply for warranty and post-warranty maintenance in Russia to the center authorized by DTF Medical.

Periodicity of checks

Periodicity	After use	After correct maintenance	Before use	Trim.	Year
Check points					
External visual control	●	●	●		●
Internal visual control		●			
Electrical safety test		●			●
Hygiene (internal)	●	●	●		●
Hygiene (foam filter)					●
Hygiene (external)	●		●	●	●
Performance tests		●	●		●
Hygiene (disinfection of the kit)	DEFINE ACCORDING TO NEEDS				

Independently can be performed:

1. Cleaning/disinfection of external parts of the breast pump.
2. Replacement of a foam filter.
3. Internal circuit cleaning.

Cleaning/disinfection of external parts of the breast pump are particularly described in the Chapter 3 "Cleaning and disinfection".

Replacement of a foam filter and internal circuit cleaning are recommended to perform in following circumstances:


1. If you observe liquid in the tube between the interface bottle and the breast pump,
2. If suction does not work correctly,
3. During routine checks or after annual preventive maintenance.

Replacement of a foam filter:

There is a filter on the rear of the device that is designed exclusively to protect the breast pump from liquid penetration that may damage the internal elements of the system.

Warnings:

- Do not turn the device on if you see liquid between the interface bottle and the breast pump.
- Remove the interface bottle before storage.
- Empty and clean the interface bottle before storage.

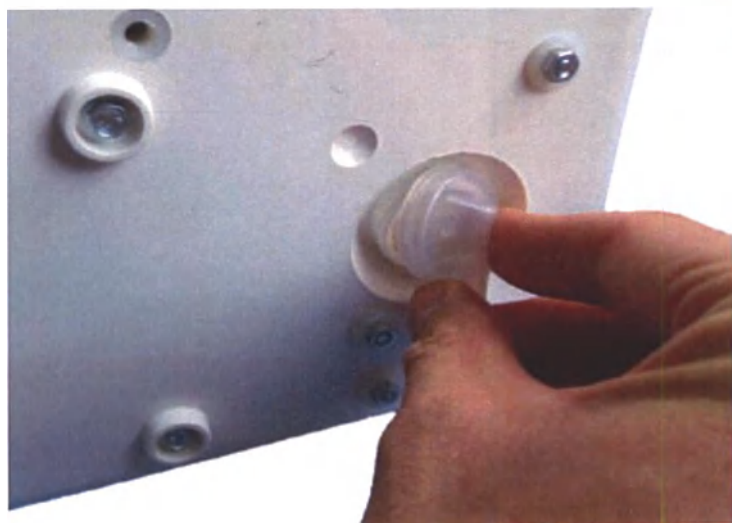Internal circuit cleaning:**STEP 1**

Remove the plug located on the rear side of the breast pump, get the foam filter from the outlet.

STEP 2

Set the device over a sink or any other reservoir, be sure that the outlet (on the rear side of the breast pump) is still above.

STEP 3

Put an extremity of the silicone tube on the breast pump output nozzle and place the other in a container with warm water (0.5L at 40-60°C).

STEP 4

Switch on the device. Water from the container will go through the internal circuit of the breast pump and leak away from the drain hole. Let it rinse until the water comes out clean.

At the end, remove the tube from the container. Let the device on until the ejection of the water. Leave the device out of service on, at least 20 minutes to dry the internal circuits. After that, place the new foam and close the plug.

Chapter 14 «Transportation and storage conditions»

Depending on stability to climatic effects, the medical device is applicable to the climate version UHL 4.2.

Depending on perceptible mechanical effects, medical device is applicable to group 2 – wearable, transportable and removable, not intended to work during transportation or movement within a healthcare facility.

Transportation by all types of enclosed transport and storage at warehouses is allowed under the following conditions:

Parameters	Value
Temperature at transportation and storage	-25°C - +70°C
Relative humidity at transportation and storage	15%...93%
Atmosphere pressure at transportation and storage	700 hPa – 1060 hPa

Kit instructions for use:
KITETT FISIOBIB kit for breast milk collecting and storing

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GB - Breastmilk bottles for collecting and storing

Instructions for use:

- Rinse with cold water all parts in contact with milk.
- Cleaning and sterilization methods (before first use too):
Clean in soapy hot water and then rinse with clear water. Sterilize in boiling water (10 min. minimum), or in dishwasher, or in a breastmilk bottle steam sterilizer. *(Refer to the breastmilk bottle steam sterilizer's manufacturer instructions; for microwave steam sterilizers do not exceed 1000 Watt)*
- Avoid touching the inside of breastmilk bottles.
- Leave to dry naturally on a paper towel.

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Kit instructions for use:

KITETT FISIOCOOL kit for breast milk cooling

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GB - Cooling packs help maintain breastmilk at low temperatures during transportation.

Instructions for use:

- They are intended to wrap the bottle of breast milk previously stored in the refrigerator. For optimal efficiency, place cooling packs in the freezer for 12 hours before use.
- The cooling pack is not intended to be used at hot temperature.

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