INSTRUCTIONS FOR USE OF A MEDICAL PRODUCT

Cure for nose cavity lavage «NATUMA Aquaflu nasal 125 ml» for children and adults,
manufactured by YS LAB, France

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1. Main Characteristics

Description: Cure for nose cavity lavage «NATUMA Aquaflu nasal 125 ml» for children and adults is a aerosol product in pressurized metal container with internal antiseptic coating, with anatomy nose sprayer and plastic cap.

It contains a clear colorless and odorless with slightly salty taste, which is a sterile seawater solution with hypertonic NaCl concentration.

Hypertonic solution of sea water reduces swelling of the mucous membrane of the nasal cavity, due to the removal of excess fluid from the intercellular space by the difference in osmotic pressure.

Sterilization method: radiation.

The product doesn't contain any conservants. It is not a drug.

Cure for nose cavity lavage «NATUMA Aquaflu nasal 125 ml» intended for children from 1 year and adults

2. Technical Specifications

Technical and functional characteristics of the product are provided in the table below:

Name of characteristic	Description
Product description and design	Aerosol in pressurized metal container. A four layer bag with content and a special nonreturn valve maintaining sterility of the content during the whole term of use by preventing backflow of content and penetration of ambient air into the bag is placed inside the container.
Package materials	Container – aluminum (ALU 8); bag with nonreturn valve: four-layer bag – polypropylene (PP 75)-polyamide (OPA 15)-aluminum foil (ALU 8) – polyester (PETP 12); valve body – polypropylene (HJ 325MO); valve: piston – polyacetal resin (POM Kepital F2503); spring – stainless steel (AISI 316); inner liner – natural rubber (DB 219); external liner – butyl (U 133); valve cup – aluminum (ALU 8); nose piece – polyethylene (PE HE7541); cap – polypropylene (PP RF830MO).
Product form	Nasal spray
Size (mm) h-high; d-diameter	Balloon (h-148±1; d-45±1), cap (h-33±1; d-33±1), diffuser (h-38±1; d-34±1)
Balloon volume (ml)	125
Weight with cap (g)	167±5
Description of contents	Clear colorless and odorless liquid with a slight salty taste, being a sterile sea water solution, with hypertonic NaCl concentration
Contents composition	46,5 % natural sea water, 53,5 % purified water
The content of chemical elements in seawater	Sodium, Na (mg/l) - 10750.0 Manganese, Mn (µg/l) - 9 Chloride, Cl (mg/l) - 19780 Bromine, Br (mg/l) - 67 Magnesium, Mg (mg/l) - 1290 Calcium, Ca (mg/l) - 404

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**YSLAB**

TECHNOLOGIE MARINE

	Potassium, K (mg/l) - 403 Iodine, I (µg/l) 60 Zinc, Zn (µg/l) 0.5 Copper, Cu (µg/l) 0.2 Selenium, Se (µg/l) 0.3
Content of chemical elements in solution of a medical device, (%)	Sodium, Na - 0,49 Manganese, Mn - 0,0000004 Chloride, Cl - 0,89 Bromine, Br - 0,003 Magnesium, Mg - 0,058 Calcium, Ca - 0,018 Potassium, K - 0,018 Iodine, I - 0,0000003 Zinc, Zn - 0,00000002 Copper, Cu - 0,000000009 Selenium, Se - 0,000000009
	6.0-8.5
Content (g/l)	19-23
Storage conditions	Keep in a dry place protected from direct sunlight at +5 °C to +25 °C. Keep out of the reach of children.
Stability	Sterile. Sterilization method: radiation. Maintains sterility during the whole period of use.
Relative density at 20 °C (density solution)	1.008 - 1.016
Quantity of extracted substances (ml)	>120.0
Source of raw materials for the control, of which the manufacturer allows the preparation of a medical device	Nitrogen, Sea water from the North of Brittany on the Pink Granite Coast (Cote de Granit Rose), in the Cotes d'Armor department, in Pleumeur-Bodou, on the island "Ile Grande" (22560), more than 22 meters deep. Purified water
Maximum pressure during closing of control valve	should not exceed 40 N (4 kp)
Permissible steam leakage through seal	should not exceed 0,1±0,05 cm ³ during 6 hours under pressure (1,20*0,05) MPa
Resistance to axial force during compression (opening/closing)	Aerosol valve should withstand axial force of compression not less 150 N (15 kp) during 500 work cycles (opening/closing)
Permissible case	Allowable deformity of valve (case) under overpressure no more

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**YSLAB**

BIOTECHNOLOGIE MARINE

deformity under overpressure	0,3±0,05 mm under overpressure of vapor: (1,00±0,05) MPa – for aluminum case
Force of tube removal off pocket shank	Not less 60 N (6 kp). Allowable gap between tube's back and pocket - no more 0,8 mm.
Spray head's nozzle pressure	Spray head's nozzle should withstand pressure without falling out: Plastic nozzle - (1,00±0,05) MPa
Volume of a single dose	100-300 µL

3. Intended use

Cure for nose cavity lavage «NATUMA Aquaflu nasal 125 ml» for children and adults is intended for the prevention and treatment of acute respiratory viral infections, influenza, acute rhinitis (including allergic during exacerbation of chronic rhinitis, and also for the prevention and comprehensive treatment of sinusitis. Cure for nose cavity lavage «NATUMA Aquaflu nasal 125 ml» for children and adults has the following properties:

- thoroughly flush all parts of the nasal cavity and nasal pharynx actively clearing them from bacteria, viruses, allergens, crusts, mucus, dust particles;
- reduces swelling of the nasal mucosa and nasal pharynx, helps to restore nasal breath;
- makes breathing easier in cases of rhinitis;
- induce the moisturizing effect, relieves irritation;
- reduces the risk of complications after surgery in the nasal cavity and paranasal sinuses;
- helps to maintain normal physiological condition of the nasal mucosa and nasal pharynx;
- enhances the therapeutic efficacy of drugs applied to the nasal mucosa, reduces the need for their use and side effects.

Conditions of use: for individual use and in health care institutions.

4. Indications and contraindications:

Indications for use:

It is indicated for children older than 1 year and adults.

Prevention and treatment of acute and chronic inflammatory diseases of the nasal cavity, paranasal sinuses and nasal pharynx (infectious, allergic, atrophic):

- acute and chronic rhinitis (including allergic and vasomotor),
- acute and chronic sinusitis (including antritis),
- acute and chronic adenoiditis,
- dryness of the mucous membrane of the nasal cavity (rhinitis subatrophic and atrophic);

Prevention and treatment of acute respiratory viral infections and influenza,

Preparation of the nasal cavity to the application of drugs,

Contraindications:

Individual intolerance to the components of the container contents.

Special instructions:

Pregnancy and breastfeeding are not contraindications.

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YSLAB

OTOLOGIE MARINE

Preparation for work and use of the product

Children from the 1 to 2 years:

Nasal lavage is performed in the supine position (Fig.1):

the child's head to one side.

Insert the can's nozzle into the nasal passage on the top. Rinse the nasal cavity for several seconds.

Remove residual rinsing liquid with an aspirator. Repeat the procedure if necessary.

Perform the procedure with another nasal passage.

Fig.1

Children older than 2 years and adults:

Nasal lavage for children older than 2 years and adults (Fig.2):

the head on one side.

Insert the can's nozzle into the nasal passage on the top. Rinse the nasal cavity for several seconds.

Remove residual rinsing liquid with an aspirator/ for a child to blow his nose. Repeat the procedure if necessary.

Perform the procedure with another nasal passage.

Fig.2

Irrigate each nasal passage 4-6 times a day in case of severe nasal congestion.

The duration and frequency of application is not limited.

Sanitization indications: before using medical devices to treat anatomical head with boiling water or ethanol solution.

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BIOTECHNOLOGIE MARINE

7. Packaging and labelling

Metal container with anatomy nose sprayer and plastic cap in carton box with instruction for use.

The following information is specified on the container:

The name, address and country of the manufacturer;
The name of the medical device on Russian and English;
Intended use;
Application;
Rules of storage;
Indication of sterility;
Date of manufacture;
Serial number;
Shelf life;
Conditions of supply from pharmacies;

The following information is specified on the package:

The name, address and country of the manufacturer;
The name of the medical device on Russian and English;
Intended use;
Application;
Rules of storage;
Indication of sterility;
Serial number;
Date of manufacture;
Shelf life;
Conditions of supply from pharmacies;
Address for submission of claims.

8. Rules of storage, transportation, waste disposal

Shelf life of the product – 3 years. Use before the date labeled on the package.

Store the product at the temperature from +5°C to +25°C, in a dry place protected from direct sunlight.

Keep out of the reach of children.

Cure for nose cavity lavage « NATUMA Aquaflu nasal 125 ml » for children and adults can delivered by any kind of transport in covered vehicles in accordance with the current rules of this mode of transport at a temperature within the range of from +1 °C to +40 °C.

Cure for nose cavity lavage « NATUMA Aquaflu nasal 125 ml » at the end of shelf life is disposed with other industrial waste in accordance with applicable national legislation.

9. Manufacturing organization, warranty

The product is manufactured by YS LAB, Le Forum, 2 rue Felix Le Dantec, QUIMPER 29000, France.

Quality of Cure for nose cavity lavage « NATUMA Aquaflu nasal 125 ml » for children and adults and production facilities of YS LAB, France comply with all international quality standards, which is confirmed by the ISO 13485:2012 certificate №12656 Revision 5 and declaration of compliance for the product.

The manufacturer guarantees the quality of the product during the shelf life.

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YSLAB

TECHNOLOGIE MARINE

On the territory of the Russian Federation all the claims to quality must be sent to FarmaStar Co., Ltd.,
14005, Moscow, 2-ya Baumanskaya street, building 7, str.1A, tel. 8 495 969 21 84.

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